

# Bracing & Support

2024 Catalog

---



# BRACING & SUPPORT SOLUTIONS CATALOG

## TABLE OF CONTENTS

ÖSSUR OA SOLUTIONS.....	4
INTRODUCTION TO SOLUTIONS.....	5
UNLOADER® HIP .....	6
UNLOADER ONE® X.....	8
UNLOADER ONE® .....	14
UNLOADER ONE® CUSTOM .....	16
UNLOADER® CUSTOM .....	19
CTI® OA.....	21
FORMFIT® PRO KNEE OA .....	23
OA EASE .....	24
ÖSSUR INJURY SOLUTIONS.....	26
MIAMI LSO™ .....	28
MIAMI TLSO™ 464 .....	30
MIAMI TLSO 456 .....	32
MIAMI LUMBAR® POSTEO .....	34
FORMFIT® PRO BACK .....	35
MIAMI J® SELECT .....	36
MIAMI J® .....	37
MIAMI JTO® THORACIC EXTENSION .....	39
OCCIAN® BACK & OCCIAN® BACK JR .....	40
MIAMI Jr® .....	41
PAPOOSE® .....	42
PHILADELPHIA® ADJUSTABLE TRACHEOTOMY COLLAR.....	43
PHILADELPHIA® TRACHEOTOMY COLLAR.....	44
UNIVERSAL CERVICAL COLLAR .....	45
FOAM CERVICAL COLLAR.....	45
NECLOC® .....	46
NECLOC® KIDS .....	46
PATRIOT® COLLAR.....	47
OAM RIGID LUMBAR .....	48
FORMFIT® BACK SUPPORT, AIR .....	50
FORMFIT® BACK SUPPORT .....	51
UNIVERSAL ELASTIC RIB BELT .....	52
UNIVERSAL ABDOMINAL BINDER.....	52
ELASTIC ABDOMINAL BINDER.....	53
UPPER EXTREMITY .....	54
FORMFIT® PRO ELBOW .....	55
FORMFIT® SHOULDER BRACE .....	56
SHOULDER BRACE WITH AIRPLANE KIT .....	57

REBOUND® POST-OP ELBOW .....	58
SHOULDER ABDUCTION SLING .....	60
PREMIUM SHOULDER IMMOBILIZER.....	60
PADDED STRAP SHOULDER IMMOBILIZER.....	61
PREMIUM CONTACT CLOSURE ARM SLING .....	61
BUCKLE CLOSURE ARM SLING.....	62
BERGSCHULTZ UNIVERSAL ARM SLING.....	62
PREMIUM PADDED ARM SLING .....	63
FRONT CLOSURE CLAVICLE SUPPORT.....	63
FIGURE-8 CLAVICLE SPLINT.....	64
SLING AND SWATHE .....	64
PEDIATRIC UNIVERSAL ARM SLING .....	65
TENNIS ELBOW SUPPORT WITH HOT/COLD GEL THERAPY .....	65
FORMFIT® TENNIS ELBOW .....	66
AIRFORM® TENNIS ELBOW SUPPORT .....	66
FORMFIT® PRO WRIST .....	67
EXOFORM® WRIST .....	68
EXOFORM® CARPAL TUNNEL WRIST .....	68
FORMFIT® WRIST / FORMFIT® WRIST & FOREARM .	69
6" WRIST .....	70
FORMFIT® WRIST & THUMB, UNIVERSAL.....	71
FORMFIT® THUMB SPICA.....	72
6" THUMB.....	73
UNIVERSAL THUMB SPLINT .....	74
HIP.....	75
REBOUND® HIP .....	76
KNEE.....	77
REBOUND® CARTILAGE .....	78
REBOUND® ACL .....	81
REBOUND® ACL CUSTOM .....	84
REBOUND® PCL.....	87
CTI® CUSTOM .....	90
CTI® .....	93
CTI® 3.....	96
REBOUND® DUAL .....	98
REBOUND® DUAL .....	100
REBOUND® DUAL .....	101
REBOUND® DUAL ST .....	102
REBOUND® DUAL ST CUSTOM.....	105
PARADIGM® .....	108

## BRACING & SUPPORT SOLUTIONS CATALOG

---

EXTREME® .....	109	AIRFORM® UNIVERSAL INFLATABLE	
FLEX™ .....	110	ANKLE STIRRUP .....	149
C180™ ROCKET .....	112	AIRFORM® PRE-INFLATED ANKLE STIRRUP .....	149
REBOUND® POST OP KNEE.....	113	<b>FORMFIT® HONEYCOMB ANKLE STIRRUP .....</b>	<b>150</b>
FORMFIT® POST-OP KNEE.....	114	EXOFORM® DORSAL NIGHT SPLINT .....	151
EXOFORM® KNEE IMMOBILIZER.....	115	<b>FORMFIT® NIGHT SPLINT .....</b>	<b>152</b>
REBOUND® KNEE IMMOBILIZER UNIVERSAL .....	116	AIRFORM® NIGHT SPLINT .....	153
<b>FORMFIT® HINGED LATERAL J.....</b>	<b>117</b>	SQUARE TOE POST-OP SHOE .....	154
<b>FORMFIT® KNEE, HINGED.....</b>	<b>118</b>	SOFT TOP POST-OP SHOE.....	154
<b>FORMFIT® KNEE ROM .....</b>	<b>119</b>	MESH TOP POST-OP SHOE .....	155
FORMFIT® TRACKER .....	120	CANVAS ROCKER BOTTOM CAST SHOE .....	155
REBOUND® KNEE BRACE .....	121	<b>SPECIALITY CARE,</b>	
NEOPRENE HINGED KNEE SUPPORT .....	122	<b>CASTING &amp; TRACTION .....</b>	<b>156</b>
NEOPRENE WRAPAROUND HINGED		COLD RUSH® .....	157
KNEE SUPPORT .....	122	COLD RUSH® COMPACT .....	157
NEOPRENE KNEE SUPPORT WITH		COLD RUSH® PADS.....	158
STABILIZED PATELLA .....	123	CAST PADDING.....	159
FORMFIT® PRO KNEE .....	124	STOCKINETTE .....	159
FORMFIT® PRO KNEE FLITE .....	125	TECHFORM® PREMIUM .....	160
NEOPRENE KNEE SLEEVE .....	126	TECHFORM® ORTHO-ROLL® .....	161
PREMIUM ELASTIC KNEE SUPPORT .....	127	TECHFORM® PRE-CUT SPLINTS.....	161
ELASTIC KNEE SUPPORT .....	127	DUALPULSE™ COMPACT .....	162
<b>FOOT AND ANKLE.....</b>	<b>128</b>		
REBOUND® DIABETIC WALKER.....	129		
REBOUND® AIR WALKER .....	131		
AFO DYNAMIC .....	133		
AFO LEAF SPRING .....	134		
AFO LIGHT.....	135		
FOOT-UP® .....	136		
REBOUND FOOT-UP® .....	137		
<b>FORMFIT® WALKER, AIR .....</b>	<b>138</b>		
<b>FORMFIT® WALKER .....</b>	<b>139</b>		
ROM WALKER .....	140		
DH OFFLOADING WALKER® .....	141		
DH OFFLOADING POST-OP STYLE SHOE .....	141		
PEDIATRIC WALKER .....	142		
METAL UPRIGHT AIR WALKER .....	143		
FORMFIT® PRO ANKLE .....	144		
REBOUND® ANKLE BRACE.....	145		
<b>FORMFIT® ANKLE BRACE.....</b>	<b>146</b>		
<b>FORMFIT® ANKLE WITH SPEEDLACE.....</b>	<b>147</b>		
<b>FORMFIT® ANKLE STIRRUPS .....</b>	<b>148</b>		



## ÖSSUR OA SOLUTIONS

---

### Enhance quality of life and mobility for people living with osteoarthritis.

These product solutions have been built on decades of experience and are designed to enhance the quality of life of people living with osteoarthritis by reducing pain and improving mobility. Through innovation leadership, OA expertise and partnerships with medical professionals, we offer comprehensive, proven and effective treatment options.

## UNLOADER®

The Össur Unloader® brand name has become synonymous with Osteoarthritis bracing, representing OA Solutions' technology as the industry standard, with more than 15 clinical studies demonstrating its efficacy.

Össur strives to maintain its position as the innovation leader for OA Solutions by continuing to develop simple, yet effective clinically proven treatment solutions that increase people's quality of life.







HIP OA



UNLOADER® HIP

PAGE 6

KNEE OA



UNLOADER ONE® X

PAGE 8



FORMFIT® PRO KNEE OA

PAGE 24



OA EASE

PAGE 25

## UNLOADER<sup>®</sup> HIP

Unloader Hip is designed to reduce pain by optimizing load dispersion for patients suffering from mild and moderate osteoarthritis of the hip, thus contributing to hip stability and improved mobility.




### INDICATIONS FOR USE

- Mild and moderate osteoarthritis of the hip
- Conditions benefitting from repositioning of the hip and/or compression around the greater trochanters

### FEATURES AND BENEFITS

- The Dynamic Rotation Straps with SmartDosing dial allows individual adjustment of the external rotation force, to achieve optimum pain relief for each patient
- Easy donning and doffing thanks to pocket wrap closure system and zipper
- Pulley system for improved stability
- Premium quality lycra for breathability and a snug fit

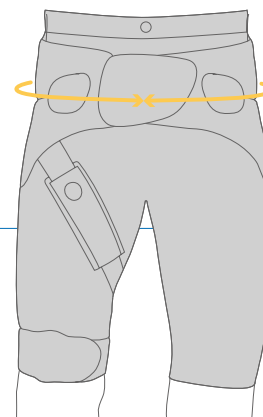

### REQUIRED MEASUREMENTS

- Measure the hip circumference at the anterior superior iliac spine



# UNLOADER<sup>®</sup> HIP

## ORDERING INFORMATION

### UNLOADER HIP

Size	Hip Circumference	Woman	Men
Small	31.5 – 35" (80 – 88 cm)	B-465350030	B-465350031
Medium	35 – 38" (88 – 96 cm)	B-465350032	B-465350033
Large	38 – 41" (96 – 104 cm)	B-465350034	B-465350035
X-Large	41 – 45" (104 – 114 cm)	B-465350036	B-465350037
XX-Large	45 – 49" (114 – 124 cm)	B-465350038	B-465350039

## ACCESSORIES/ OPTIONS

### REPLACEMENT TROUSERS

Size	Measurements	Woman	Men
Small	31.5 – 35" (80 – 88 cm)	B-465350077	B-465350078
Medium	35 – 38" (88 – 96 cm)	B-465350079	B-465350080
Large	38 – 41" (96 – 104 cm)	B-465350081	B-465350082
X-Large	41 – 45" (104 – 114 cm)	B-465350083	B-465350084
XX-Large	45 – 49" (114 – 124 cm)	B-465350085	B-465350086

## OTHER ACCESSORIES

Description	Part#
Rotation strap	B-465350064
Washbag	B-465350065
Trochanter pad	B-465350066
Thigh strap S	B-465350067
Thigh strap M	B-465350068
Thigh strap L	B-465350069
Thigh strap XL	B-465350070
Thigh strap XXL	B-465350071
Bilateral kit* S	B-465350072
Bilateral kit* M	B-465350073
Bilateral kit* L	B-465350074
Bilateral kit* XL	B-465350075
Bilateral kit* XXL	B-465350076
Unloader Hip front closure	B-112600020

\*The bilateral kit consists of a rotation control strap, a thigh strap and a trochanter pad



## UNLOADER ONE® X

Unloader One X improves mobility by providing pain relief from knee osteoarthritis, degenerative meniscal tears, and other unicompartmental knee conditions that may benefit from unloading.

It features Össur's unique, proprietary, and clinically proven 3-point Leverage System that generates unicompartmental unloading of the knee joint.

Available in custom and OTS versions to help you fit your patient's different leg anatomies.

### INDICATIONS FOR USE

For knee conditions that may benefit from unicompartmental load reduction, such as:

- Mild to severe unicompartmental knee osteoarthritis
- Degenerative meniscal tears
- Articular cartilage defect repair
- Meniscal repair
- Avascular necrosis
- Condylar bone marrow lesions (bone bruises)
- Tibial plateau fractures

### FEATURES AND BENEFITS

- Relieves Pain - 3 Point Leverage System with Dynamic Force System® straps, rigid shells, and a single upright hinge biomechanically unloads the affected knee compartment.
- Control - Tool-free SmartDosing® dials enable the patient to fine-tune the strap tension to control pain.
- Tool-Free - Tool-free polycentric hinge with optional ROM stops for range-of-motion restriction.
- Comfortable - Skin-friendly liners feel great against the skin and reduce pressure points. Anti-migration strap technology design captures the widest part of the calf to reduce brace migration.
- Easy-to-Use - Alignment guide on hinge facilitates correct positioning of the brace on the leg and color coded touch points aid in easy donning and doffing.
- Easy-to-Wear - Slender frame with soft edges easily contours to individual anatomies and remains discreet whether worn under or over clothing.
- Easy Care - Entire brace is machine washable (washbag included).



### REQUIRED MEASUREMENTS: UNLOADER ONE X CUSTOM

Custom made to measure: Smartmeasure, 3D Scan (.AOP, .STL, .OBJ, .CPV) or Patient Cast

#### SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilizes a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.




Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-3 12000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee center M/L required


# UNLOADER ONE® X

## REQUIRED MEASUREMENTS: UNLOADER ONE X OTS

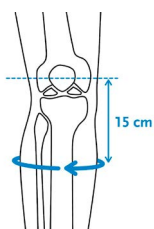

Measure calf circumference 15 cm (6") below mid-patella. Leg in extension.

Measure leg length from floor to groin.

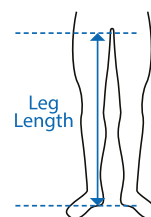

Size	Calf circumference 15 cm (6") below mid patella	
	cm	inch
XS - M	25 - 43	9¾ - 16¾
L - XL	43 - 62	16¾ - 24¾

Model	Leg Length
Standard	> 85 cm (33.5")
Short	≤ 85 cm (33.5")

## COLORS

### CUSTOM STANDARD COLORS



Black Brace /  
Black Components



Blue Brace /  
Black Components



Charcoal Grey Brace /  
Black Components



Red Brace /  
Black Components



White Brace /  
Black Components



Yellow Brace /  
Black Components

### OTS COLOR



Black Brace /  
Black Components

Actual colors may vary.

## ORDERING INFORMATION

Hinge is placed on the side of the affected compartment.

Short version recommended for patients with a leg length of ≤ 85 cm (33.5").

### CUSTOM

Description	Part#
Right	UOX0R
Left	UOX0L

### OTS

Standard	Right Medial	Right Lateral	Left Medial	Left Lateral
XS - M	UOX0518713	UOX0528713	UOX0618713	UOX0628713
L - XL	UOX0518715	UOX0528715	UOX0618715	UOX0628715

Short	Right Medial	Right Lateral	Left Medial	Left Lateral
XS - M	UOX1518713	UOX1528713	UOX1618713	UOX1628713
L - XL	UOX1518715	UOX1528715	UOX1618715	UOX1628715


# UNLOADER ONE® X

## OTS - NO BOX

Standard	Right Medial	Right Lateral	Left Medial	Left Lateral
XS - M	UOX0518718	UOX0528718	UOX0618718	UOX0628718
L - XL	UOX0518719	UOX0528719	UOX0618719	UOX0628719

Short	Right Medial	Right Lateral	Left Medial	Left Lateral
XS - M	UOX1518718	UOX1528718	UOX1618718	UOX1628718
L - XL	UOX1518719	UOX1528719	UOX1618719	UOX1628719


## REPLACEMENT PARTS

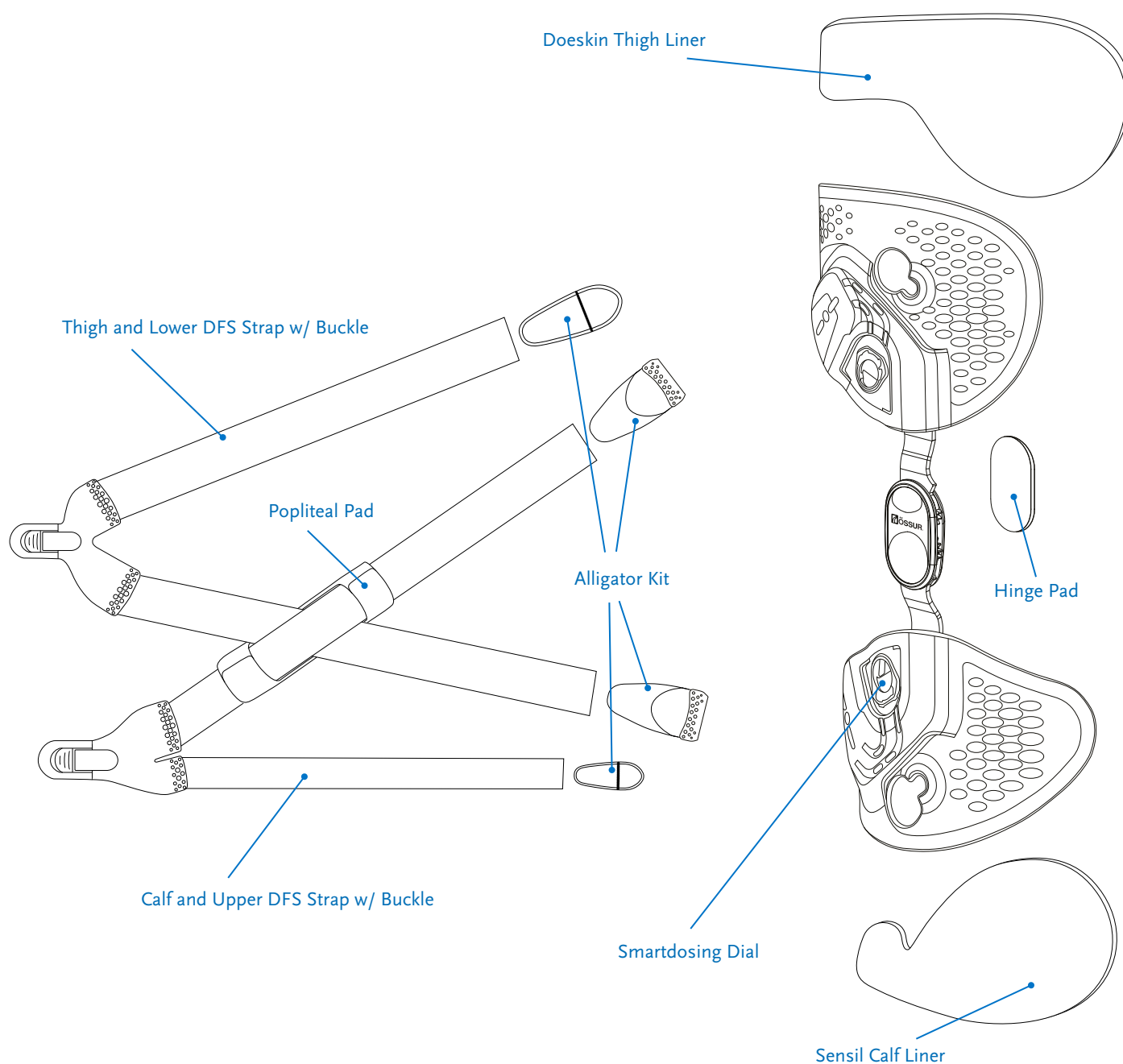



# UNLOADER ONE® X

## ACCESSORIES

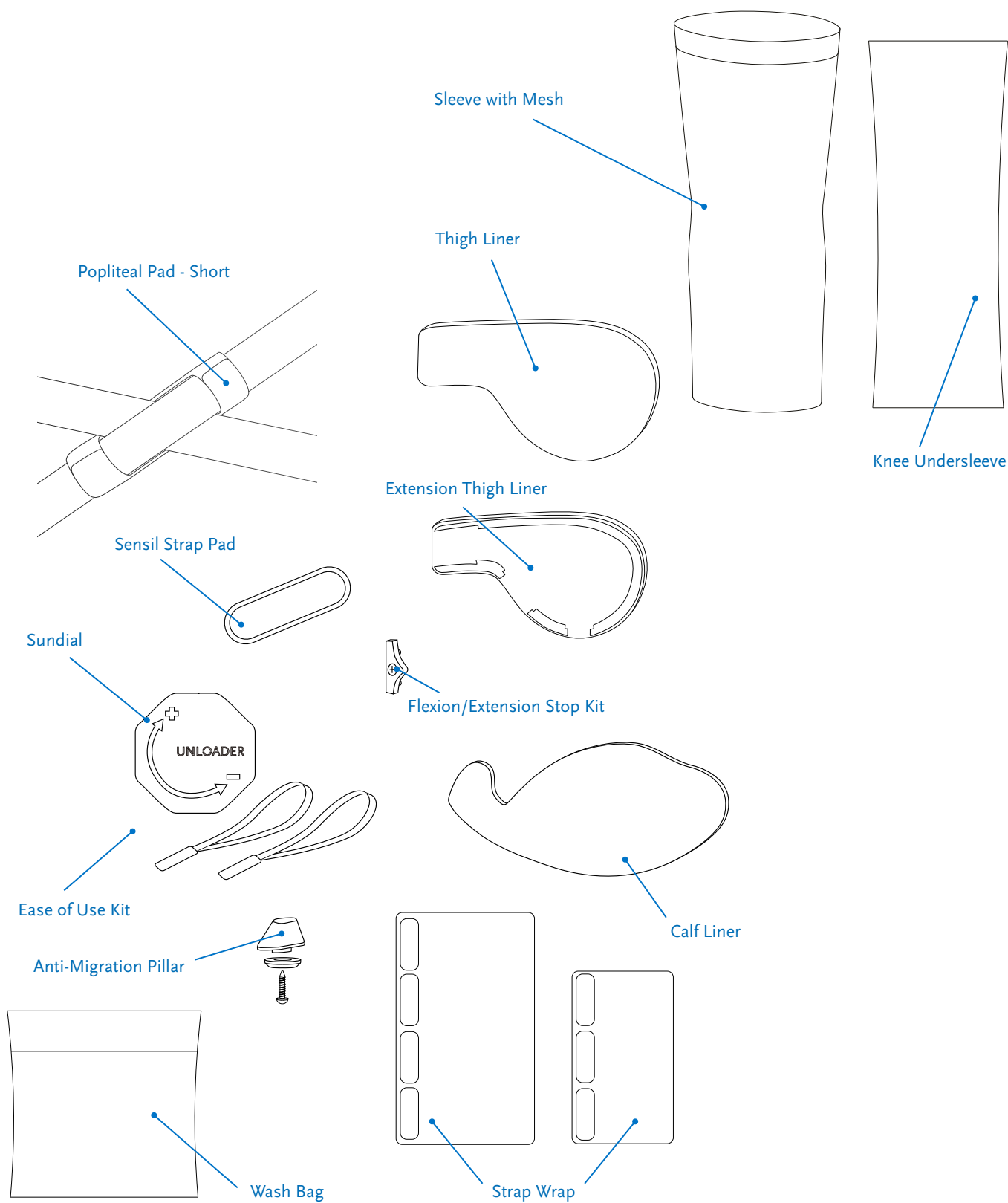

# UNLOADER ONE® X

THIGH LINERS	THIGH, Right Medial, Left Lateral	THIGH, Right Lateral, Left Medial	THIGH EXTENSION, Right Medial, Left Lateral	THIGH EXTENSION, Right Lateral, Left Medial
Black, Sensil	UOX526801	UOX525801	NA	NA
Black, Doeskin	UOX526901	UOX525901	UOX546901	UOX545901

CALF LINERS	Right Medial, Left Lateral	Right Lateral, Left Medial
Black, Sensil XS-M	UOX516952	UOX515952
Black, Sensil L-XL	UOX516962	UOX515962
Black, Doeskin XS-M	UOX516951	UOX515951
Black, Doeskin L-XL	UOX516961	UOX515961

THIGH AND LOWER DFS STRAP*	Right Medial, Left Lateral	Right Lateral, Left Medial
Black	UOX406902	UOX405902

\*Includes Thigh and DFS Strap with buckle

CALF AND UPPER DFS STRAP*	Right Medial, Left Lateral	Right Lateral, Left Medial
Black	UOX406901	UOX405901

\*Includes Calf and DFS Strap with buckle

UOX SERVICE KIT*	Right Medial, Left Lateral	Right Lateral, Left Medial
Black, XS-M	UOX416901	UOX415901
Black, L-XL	UOX416911	UOX415911

\* Includes Doeskin Thigh Liner, Sensil Calf Liner, Thigh and Lower DFS Strap with Buckle, Calf and Upper DFS Strap with Buckle, Hinge Pad, Popliteal Pad, and Alligator Kit

ANTI-MIGRATION PILLAR	Right Medial, Left Lateral	Right Lateral, Left Medial
Black	UOX700015	UOX700035

# UNLOADER ONE® X

OTHER REPLACEMENT PARTS AND ACCESSORIES	Part #
Alligator Kit*	UOX900001
Hinge Pad	UOX700091
Popliteal Pad	UOX700012
Popliteal Pad - Short	UOX700022
Sensil Strap Pad	UON0181
Sensil Strap Pad - 10 Pieces	UOX700113
Smartdosing Dial	UOX1000001
Strap Wrap Set	UOX700014
Ease of Use Kit**	UOX800001
OA Hinge Extension Stop Tree (5°, 10°, 15°, 30°)	OA40029
OA Hinge Flexion Stop Tree (0°, 30°, 60°, 90°)	OA40039
Sundial	UOX1200001
Wash Bag	WB00001

\* Includes 4 Alligator Clips: 2 DFS, 1 Thigh and 1 Calf

\*\* Includes Sundial and 2 buckle finger-loops

SLEEVES	Black - Sleeve with Mesh*	Black - Össur Knee Undersleeve**
XS	SLV60011	B-250150101
S	SLV60012	B-250150102
M	SLV60013	B-250150103
L	SLV60014	B-250150104
XL	SLV60015	B-250150105
XXL	SLV60016	B-250150106

\*Sleeve with Mesh: Main fabric: 92% Cotton/8% Spandex - Mesh: 100% Polyester - Band: 80% Polyester/20% Spandex

\*\*Össur Knee Undersleeve: 80% Nylon/20% Spandex



# UNLOADER ONE®

The Unloader One is a comfortable, lightweight, low-profile brace that provides excellent suspension and pain relief. This lightweight brace preserves the clinically proven Össur Unloader 3-Points of Leverage design by incorporating Dual Dynamic Force Straps that are easily fine-tuned with the SmartDosing® dial. Color-coded Quick Fit Buckles simplify application and removal, while flexible shells and breathable tibial liners, coated with Össur Sensil® Silicone, improve comfort and reduce migration, helping to ensure patient compliance. Recent research demonstrates that the Unloader One reduces pain, improves function and decreases the use of pain medication.



## INDICATIONS FOR USE

- Moderate to severe unicompartmental knee osteoarthritis
- Degenerative meniscal tears
- Other knee conditions that may benefit from unicompartmental unloading

## FEATURES AND BENEFITS

- Dual Dynamic Force Straps designed to reduce pressure on sensitive bony surfaces of the knee, resulting in reduction of pain
- Tool-free SmartDosing dial enables fine-tuned pain management and can be adjusted even when wearing the brace under clothing
- Tool-free polycentric hinge with optional ROM stops for range-of-motion restriction
- Color coded quick fit buckles enable easy application and removal
- Proprietary Sensil® silicone liners designed to reduce migration
- Flexible, low-profile upright contours to patient anatomy and allows use underneath clothing
- Quick Connect Strapping for tool-free removal and replacement
- Lightweight frame is perforated for flexibility and ventilation with custom paint options for the Custom version
- Unloader One is available in pre-fabricated and Custom version

## REQUIRED MEASUREMENTS

- Circumference measurement: 6" (15 cm) below mid-patella

# UNLOADER ONE®

## ORDERING INFORMATION

### UNLOADER ONE WITH SMARTDOSING

Size	Calf circumference	Left medial	Right medial	Left lateral	Right lateral
Small	12.2 – 14" (31 – 36 cm)	B-240618712	B-240518712	B-240628712	B-240528712
Medium	14 – 16.75" (36 – 43 cm)	B-240618713	B-240518713	B-240628713	B-240528713
Large	16.75 – 20" (43 – 51 cm)	B-240618714	B-240518714	B-240628714	B-240528714
X-Large	20 – 24" (51 – 61 cm)	B-240618715	B-240518715	B-240628715	B-240528715

### UNLOADER ONE SHORT WITH SMARTDOSING

Size	Calf circumference	Left medial	Right medial	Left lateral	Right lateral
X-Small	9.8 – 12.2" (25 – 31 cm)	B-241618711	B-241518711	B-241628711	B-241528711
Small	12.2 – 14" (31 – 36 cm)	B-241618712	B-241518712	B-241628712	B-241528712
Medium	14 – 16.75" (36 – 43 cm)	B-241618713	B-241518713	B-241628713	B-241528713
Large	16.75 – 20" (43 – 51 cm)	B-241618714	B-241518714	B-241628714	B-241528714
X-Large	20 – 24" (51 – 61 cm)	B-241618715	B-241518715	B-241628715	B-241528715

### UNLOADER ONE WITH RATCHET

Size	Calf circumference	Left medial	Right medial	Left lateral	Right lateral
Small	12.2 – 14" (31 – 36 cm)	B-240619712	B-240519712	B-240629712	B-240529712
Medium	14 – 16.75" (36 – 43 cm)	B-240619713	B-240519713	B-240629713	B-240529713
Large	16.75 – 20" (43 – 51 cm)	B-240619714	B-240519714	B-240629714	B-240529714
X-Large	20 – 24" (51 – 61 cm)	B-240619715	B-240519715	B-240629715	B-240529715

### UNLOADER ONE SHORT WITH RATCHET

Size	Calf circumference	Left medial	Right medial	Left lateral	Right lateral
X-Small	9.8 – 12.2" (25 – 31 cm)	B-241619711	B-241519711	B-241629711	B-241529711
Small	12.2 – 14" (31 – 36 cm)	B-241619712	B-241519712	B-241629712	B-241529712
Medium	14 – 16.75" (36 – 43 cm)	B-241619713	B-241519713	B-241629713	B-241529713
Large	16.75 – 20" (43 – 51 cm)	B-241619714	B-241519714	B-241629714	B-241529714
X-Large	20 – 24" (51 – 61 cm)	B-241619715	B-241519715	B-241629715	B-241529715

Standard version is recommended for patients between 5'5" and 6'4" (1,65 and 1.92 meters tall).

Short version is recommended for patients under 5'5" (1,65 meters tall).

Hinge is placed on the side of the affected compartment

# UNLOADER ONE®

## UNLOADER ONE® CUSTOM

### UNLOADER ONE CUSTOM WITH SMARTDOSING

Color	Left	Right
Black	B-115600112	B-115500112

### UNLOADER ONE CUSTOM

Color	Left	Right
Black	B-115600111	B-115500111

### CUSTOM STANDARD COLORS



### CUSTOM PAINTED DESIGNS



Additional fees apply for custom colors, custom painted designs, and custom printed graphics.

## SMARTMEASURE®



A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilizes a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.

Description	Part#
SmartMeasure Measuring Kit	B-312000005
Constant Force Caliper	B-312000000

Note: Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required

## ACCESSORIES

### SMARTDOSING ACCESSORY KITS

Description	Color	Right medial/Left Lateral	Right lateral/Left medial
Strap Kit	Grey	B-701270013	
Strap & Strap Pad Kit	Grey	B-705270003	
Strap & Strap Pad & Buckle Clasp Kit	Grey	B-705277003	B-705278003
Buckle Clasp Kit	Grey	B-708217003	B-708218003
Strap Kit	Black	B-701270011	
Strap & Strap Pad Kit	Black	B-705270001	
Strap & Strap Pad & Buckle Clasp Kit	Black	B-705277001	B-705278001
Strap & Strap Pad & Buckle Clasp Kit, Short	Black	UNL075107	UNL076107
Buckle Clasp Kit	Black	B-708217001	B-708218001



# UNLOADER ONE®

## RATCHET ACCESSORY KITS

Description	Color	Right medial/Left Lateral	Right lateral/Left medial
Strap Kit	Grey	B-701210004	
Pad Kit	Grey	B-702210003	
Strap & Strap Pad Kit	Grey	B-701210006	
Strap & Strap Pad & Buckle Clasp Kit	Grey	B-701217008	B-701218008
Ratchet Complete Strap Kit	Grey	B-701210004	
Strap Kit	Black	B-701210005	
Pad Kit	Black	B-702210001	
Strap & Strap Pad Kit	Black	B-701210007	
Strap & Strap Pad & Buckle Clasp Kit	Black	B-701217009	B-701218009
Strap & Strap Pad & Buckle Clasp Kit, Short	Black	UNL085107	UNL086107

## REPLACEMENT PARTS AND ACCESSORIES

Description	Color	Part#
Alligator Tab (Qty. 4)	Grey	B-704210003
Twist-Fit Attachment Kit	Grey	B-721240003
Alligator Tab (Qty. 4)	Black	B-704210001
Twist-Fit Attachment Kit	Black	B-721240001
Hinge Screws	Black	UNL060100
Suspension Strap Kit	Black	UNL00211
Hinge Pad	Black	UNL040100
Popliteal Pad Short	Black	B-304010105
Popliteal Pad Standard	Black	B-304010106
DFS Strap Kit, Standard	Black	B-701240001
DFS Strap Kit, Short	Black	B-701240011
Extension Stop Kit (5°, 10°, 15°, 30°)	Black	OAH40029
Flexion Stop Kit (0°, 30°, 60°, 90°)	Black	OAH40039

## ÖSSUR KNEE UNDERSLEEVE

Size	Color	Thigh Circumference	Part#
X-Small	Grey	12-15" (30.5-38 cm)	B-250150201
Small	Grey	15-18" (38-45.5 cm)	B-250150202
Medium	Grey	18-22" (45.5-56 cm)	B-250150203
Large	Grey	22-26" (56-66 cm)	B-250150204
X-Large	Grey	26-29" (66-73.5 cm)	B-250150205
XX-Large	Grey	29-33" (73.5-84 cm)	B-250150206
X-Small	Black	12-15" (30.5-38 cm)	B-250150101
Small	Black	15-18" (38-45.5 cm)	B-250150102
Medium	Black	18-22" (45.5-56 cm)	B-250150103
Large	Black	22-26" (56-66 cm)	B-250150104
X-Large	Black	26-29" (66-73.5 cm)	B-250150105
XX-Large	Black	29-33" (73.5-84 cm)	B-250150106

Note: When using an Ossur Knee Undersleeve, we advise using the suspension strap kit.

# UNLOADER ONE®

## SENSIL LINERS

		Calf		Thigh	
Size	Color	Right medial/Left lateral	Right lateral/Left medial	Right medial/Left lateral	Right lateral/Left medial
X-Small	Grey	B-419700311	B-419800311	B-409700321	B-409800321
Small	Grey	B-419700312	B-419800312	B-409700322	B-409800322
Medium	Grey	B-419700313	B-419800313	B-409700323	B-409800323
Large	Grey	B-419700314	B-419800314	B-409700324	B-409800324
X-Large	Grey	B-419700315	B-419800315	B-409700325	B-409800325
X-Small	Black	UNL025111	UNL026111	UNL025121	UNL026121
Small	Black	UNL025112	UNL026112	UNL025122	UNL026122
Medium	Black	UNL025113	UNL026113	UNL025123	UNL026123
Large	Black	UNL025114	UNL026114	UNL025124	UNL026124
X-Large	Black	UNL025115	UNL026115	UNL025125	UNL026125

## DOESKIN LINERS

		Calf		Thigh	
Size	Color	Right medial/Left lateral	Right lateral/Left medial	Right medial/Left lateral	Right lateral/Left medial
X-Small	Grey	B-424703721	B-424803721	B-412701721	B-412801721
Small	Grey	B-424703722	B-424803722	B-412701722	B-412801722
Medium	Grey	B-424703723	B-424803723	B-412701723	B-412801723
Large	Grey	B-424703724	B-424803724	B-412701724	B-412801724
X-Large	Grey	B-424703725	B-424803725	B-412701725	B-412801725

## JERSEY LINERS

		Calf		Thigh	
Size	Color	Right medial/Left lateral	Right lateral/Left medial	Right medial/Left lateral	Right lateral/Left medial
X-Small	Black	UNL015111	UNL016111	UNL015121	UNL016121
Small	Black	UNL015112	UNL016112	UNL015122	UNL016122
Medium	Black	UNL015113	UNL016113	UNL015123	UNL016123
Large	Black	UNL015114	UNL016114	UNL015124	UNL016124
X-Large	Black	UNL015115	UNL016115	UNL015125	UNL016125

## SENSIL AMS (ANTI-MIGRATION SYSTEM) WRAPS

Size	Color	Right medial/Left lateral	Right lateral/Left medial
X-Small	Black	UNL035111	UNL036111
Small	Black	UNL035112	UNL036112
Medium	Black	UNL035113	UNL036113
Large	Black	UNL035114	UNL036114
X-Large	Black	UNL035115	UNL036115

# UNLOADER<sup>®</sup> CUSTOM

Unloader Custom braces are clinically proven to significantly reduce osteoarthritis knee pain. With a semi-rigid frame, an adjustable hinge and numerous options to customize, the Unloader Custom accommodates a wide variety of leg shapes and sizes.



## INDICATIONS FOR USE

- Unicompartmental osteoarthritis.
- May also be beneficial for unicompartmental knee conditions that require pain reduction in an affected compartment such as: Articular Cartilage Defect Repair; Avascular Necrosis; Tibial plateau fracture; Bone marrow lesions (bone bruises).

## FEATURES AND BENEFITS

- Custom moulded semi-flexible shells are durable, adaptable to leg shape changes, yet rigid enough to contain tissue while force is applied at the knee.
- Adjustable Dynamic Joint<sup>™</sup> (ADJ) high-carbon steel hinge is strong enough to unload the pressure from the compartment of the knee.
- Dynamic Force System<sup>™</sup> (DFS) strap designed to reduce pressure on sensitive bony surfaces of the knee, resulting in reduction of pain.
- Optional tool-free SmartDosing<sup>®</sup> dial option enables fine-tuned pain management and can be adjusted even when wearing the brace under clothing.

## REQUIRED MEASUREMENTS

- Custom made to measure: SmartMeasure, 3D Scan(.AOP, .STL, .OBJ, .CPV) or Patient Cast

### SMARTMEASURE<sup>®</sup>



A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilizes a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.

Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee center M/L required

## COLORS

### CUSTOM STANDARD COLORS



### CUSTOM PAINTED DESIGNS



No charge for custom standard colors. Additional fees required for other custom colors, custom painted designs and custom printed graphics.

# UNLOADER® CUSTOM

## ORDERING INFORMATION

### UNLOADER CUSTOM

Description	Right	Left
Custom	B-122500020	B-122600020

## ACCESSORIES & REPLACEMENT PARTS

Description	Part#
AMS Wrap	B-426000150
AMS Cuff	B-426000250
ACL Band Kit	B-704000003
Knee Shield	B-250110000
Hinge Cover	B-251030000
Water Ski Strap Lock	B-303010084
Proximal/Distal Non-Elastic Thigh Strap	B-507010000

### LINERS

Description	Part#
Thigh Liner, Jersey Knit	UNL040102
Thigh Liner, Sensil	B-409000004
Calf Liner, Jersey Knit	UNL040101
Calf Liner, Sensil	B-409000003

### ELASTIC CALF SUSPENSION STRAP

Size	Part#
14"	B-504020114
15"	B-504020115
16"	B-504020116
17"	B-504020117
18"	B-504020118
19"	B-504020119
20"	B-504020120
21"	B-504020121
22"	B-504020122
23"	B-504020123
24"	B-504020124

### SLEEVES

Size	Thigh Circumference	Sofsleeve	Neoprene Undersleeve	Neoprene Oversleeve
S	< 15" (38 cm)	20643	20251	20051
M	15 – 18" (38 – 46 cm)	20644	20252	20052
L	18 – 19.5" (46 – 50 cm)	20644	20253	20053
XL	19.5 – 21" (50 – 55 cm)	20645	20254	20054
XXL	21.5 – 24.5" (55 – 62 cm)	20645	20255	20055
3XL	24.5 – 28" (62 – 71 cm)	20646	20256	20056
4XL	28 – 34" (71 – 86 cm)	20647	20257	20057

Size	Thigh Circumference	Black - Sleeve with Mesh*
XS	<17" (43 cm)	SLV60011
S	17 – 18" (43 – 45 cm)	SLV60012
M	18 – 20" (45 – 51 cm)	SLV60013
L	20 – 21" (51 – 53.5 cm)	SLV60014
XL	21 – 22" (43.5 – 56 cm)	SLV60015
XXL	22 – 24" (56 – 61 cm)	SLV60016

Size	Thigh Circumference	Össur Knee Undersleeve**
XS	12 – 15" (30.5 – 38 cm)	B-250150101
S	15 – 18" (38 – 45.5 cm)	B-250150102
M	18 – 22" (45.5 – 56 cm)	B-250150103
L	22 – 26" (56 – 66 cm)	B-250150104
XL	26 – 29" (66 – 73.5 cm)	B-250150105
XXL	29 – 33" (73.5 – 84 cm)	B-250150106

\*Sleeve with Mesh: Main fabric: 92% Cotton/8% Spandex - Mesh: 100% Polyester - Band: 80% Polyester/20% Spandex

\*\*Össur Knee Undersleeve: 80% Nylon/20% Spandex

## CTI® OA

CTI® OA's composite shells can be constructed with up to 7° of varus or valgus correction, offering a high level of protection and stability for clinicians interested in treating osteoarthritis and ligament instability equally. Key features include a Total Support System, Accutrac® hinges that mimic the natural motion of the knee joint and breathable liners, coated with Össur® Sensil® Silicone to improve efficacy and virtually eliminate migration.



### INDICATIONS FOR USE

- Mild to severe unicompartmental osteoarthritis
- ACL, MCL, LCL, PCL, rotary and combined instabilities
- Demanding activities of daily living

### FEATURES AND BENEFITS

- Anatomically-correct Accutrac® hinges with extension stops
- Proprietary Sensil® silicone liners designed to reduce migration
- Total Support System™ for ACL, MCL, LCL, PCL, rotary and combined instabilities
- Carbon composite, rigid frame construction
- Custom-fabricated to each patient's unique measurements with up to 7° varus/valgus correction built into the frame
- Non-corrosive materials, ideal for water sports
- Standard model for medium to high impact levels. Vapor model for low to medium impact levels. Pro Sport model for highest impact levels

### REQUIRED MEASUREMENTS

- Custom-made to measure





# CTI® OA

## ORDERING INFORMATION

### CTI OA

Description	Left	Right
Standard	B-116630010	B-116530010
Pro Sport	B-117630010	B-117530010
Vapor	B-118630010	B-118530010



### SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilizes a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.

Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-3 12000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required

### CUSTOM STANDARD COLORS



### CUSTOM PAINTED DESIGNS



No charge for custom standard colors. Additional fees required for other custom colors, custom painted designs and custom printed graphics.

## ACCESSORIES

Description	Part#
OA Condyle Pad Kit 1/4"	25021
OA Condyle Pad Kit 3/8"	25644

For all other Cti OA Accessories, see CTi Custom Accessories. Page 76.

## FORMFIT® PRO KNEE OA

Össur Formfit® Pro Knee OA is a unique, compressive knee sleeve with a Dynamic Force Strap system, designed to provide pain relief from the early stages of osteoarthritis (OA). The dynamic, 3D MotionTech™ knit is supportive and meets certified medical grade compression requirements, and has been proven to be superior to the leading brands at moisture wicking and breathability, keeping your knee cool during wear. CoolVent™ technology in the popliteal area enhances breathability and comfort, while the distinct patella graphics help with intuitive placing of the sleeve. The integrated StayBilizer™ hinge, constructed from lightweight, flexible EVA material, is suitable for sports, such as football.



MotionTech™



CoolVent™



StayBilizer™



FoldLock™



### INDICATIONS FOR USE

For osteoarthritis and other knee conditions which may benefit from unloading and/or compression, such as:

- Medial or lateral knee pain
- Early or mild unicompartmental osteoarthritis of the knee
- Symptomatic, unicompartmental meniscus degeneration


### CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

### FEATURES AND BENEFITS

- Patented DFS system.
- Certified medical grade compression (20-30 mmHg).
- Lightweight CoolVent™ knit in popliteal area for added breathability and comfort.
- Patented FoldLock™ system for easier and more secure application.
- Sports-compatible StayBilizer™ hinge made from light, flexible EVA material for optimum shape and fit.
- Patella graphics for intuitive placement.
- Available in 5 standard sizes and 3 comfort sizes.

### REQUIRED MEASUREMENTS

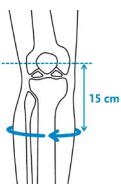

Measure calf circumference 15 cm (6") below mid-patella. Leg in extension.



Measure thigh circumference 15 cm (6") above mid-patella. Leg in extension.

Size	Calf circumference 15 cm (6") below mid patella		Thigh circumference 15 cm (6") above mid patella		Right Medial, Left Lateral	Right Lateral, Left Medial
	cm	inch	cm	inch		
XS	26 - 31	10¼ - 12¼	36 - 41	14¼ - 16¼	FP110401	FP110301
S	31 - 36	12¼ - 14¼	41 - 46	16¼ - 18	FP110402	FP110302
M	36 - 41	14¼ - 16¼	46 - 51	18 - 20	FP110403	FP110303
M Comfort	36 - 41	14¼ - 16¼	51 - 56	20 - 22	FP110413	FP110313
L	41 - 46	16¼ - 18	51 - 56	20 - 22	FP110404	FP110304
L Comfort	41 - 46	16¼ - 18	56 - 60	22 - 23½	FP110414	FP110314
XL	46 - 50	18 - 19¾	56 - 60	22 - 23½	FP110405	FP110305
XL Comfort	46 - 50	18 - 19¾	60 - 64	23½ - 25¼	FP110415	FP110315

## OA EASE

The OA Ease is a single upright brace with a Dynamic Force System™ strap designed to relieve pain from mild-to-moderate unicompartmental knee osteoarthritis (OA). Its wraparound design is easy to apply and remove using simple rubber pull-tabs, color-coded guide dots, and strap tension indicator lines. This low-profile knee brace is comfortable, easy to use, and adapts to a variety of leg anatomies. The OA Ease can also be machine washed for effortless care.




### INDICATIONS FOR USE

- Unicompartmental osteoarthritis of the knee
- Other conditions that may benefit from unicompartmental unloading, such as degenerative meniscal tears

### FEATURES AND BENEFITS

- Easy Wraparound Design - Simple rubber pull-tabs, color-coded guide dots, and a convenient hinge alignment guide make application and removal simple
- Relieves OA Pain - 3-Point Leverage System with Dynamic Force System® strap and shells unload the affected knee compartment
- Machine Washable - Lightweight and breathable fabric feels great against the skin and can be machine washed for effortless care
- Tool-Free - Tool-free polycentric hinge with available ROM stop options for range-of-motion restriction
- Control - Fine-tune strap tension for individual pain relief, with marked dosing indicator lines
- Extension kits are available to further enhance fit and comfort

### REQUIRED MEASUREMENTS

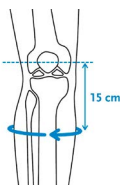

Measure calf circumference  
15 cm (6") below mid-patella.  
Leg in extension.



Measure thigh circumference  
15 cm (6") above mid-patella.  
Leg in extension.

# OA EASE

## ORDERING INFORMATION

Size	Calf circ. 15 cm (6") below mid patella		Thigh circ. 15 cm (6") above mid patella		Right Medial	Left Medial	Right Lateral	Left Lateral
	cm	inch	cm	inch				
XS	28 - 32	11 - 12 <sup>5</sup> / <sub>8</sub>	36 - 41	14 <sup>1</sup> / <sub>8</sub> - 16 <sup>1</sup> / <sub>8</sub>	FOA0518711	FOA0618711	FOA0528711	FOA0628711
S	32 - 36	12 <sup>5</sup> / <sub>8</sub> - 14 <sup>1</sup> / <sub>8</sub>	41 - 46	16 <sup>1</sup> / <sub>8</sub> - 18 <sup>1</sup> / <sub>8</sub>	FOA0518712	FOA0618712	FOA0528712	FOA0628712
M	36 - 41	14 <sup>1</sup> / <sub>8</sub> - 16 <sup>1</sup> / <sub>8</sub>	46 - 53	18 <sup>1</sup> / <sub>8</sub> - 20 <sup>7</sup> / <sub>8</sub>	FOA0518713	FOA0618713	FOA0528713	FOA0628713
L	41 - 46	16 <sup>1</sup> / <sub>8</sub> - 18 <sup>1</sup> / <sub>8</sub>	53 - 60	20 <sup>7</sup> / <sub>8</sub> - 23 <sup>5</sup> / <sub>8</sub>	FOA0518714	FOA0618714	FOA0528714	FOA0628714
XL	46 - 51	18 <sup>1</sup> / <sub>8</sub> - 20 <sup>1</sup> / <sub>8</sub>	60 - 67	23 <sup>5</sup> / <sub>8</sub> - 26 <sup>3</sup> / <sub>8</sub>	FOA0518715	FOA0618715	FOA0528715	FOA0628715
XXL	51 - 56	20 <sup>1</sup> / <sub>8</sub> - 22	67 - 75	26 <sup>3</sup> / <sub>8</sub> - 29 <sup>1</sup> / <sub>2</sub>	FOA0518716	FOA0618716	FOA0528716	FOA0628716
3XL	56 - 61	22 - 24	75 - 81	29 <sup>1</sup> / <sub>2</sub> - 31 <sup>7</sup> / <sub>8</sub>	FOA0518717	FOA0618717	FOA0528717	FOA0628717

Hinge is placed on the side of the affected compartment.

## REPLACEMENT PARTS


Description	Part#
Alligator Clip (Right Medial/Left Lateral)	FOW00501
Alligator Clip (Right Lateral/Left Medial)	FOW00502
Hinge Pad	B-304120101

## ACCESSORIES

Description	Part #
Small Extension Kit 2 Pieces	FOA000301
Small Extension Kit 10 Pieces	FOA000302
Large Extension Kit XS-S (Right Medial/Left Lateral)	FOA000401
Large Extension Kit M-L (Right Medial/Left Lateral)	FOA000402
Large Extension Kit XL-3XL (Right Medial/Left Lateral)	FOA000403
Large Extension Kit XS-S (Right Lateral/Left Medial)	FOA000404
Large Extension Kit M-L (Right Lateral/Left Medial)	FOA000405
Large Extension Kit XL-3XL (Right Lateral/Left Medial)	FOA000406
Wash Bag	WB000001
Suspension Strap Kit	OAW00007
OA Hinge Extension Stop Kit (5°, 10°, 15°, 30°)	OA40029
OA Hinge Flexion Stop Kit (0°, 30°, 60°, 90°)	OA40039

### Small Extension Kit:

Increases calf and/or thigh circumference up to 4.5 cm / 1.8"

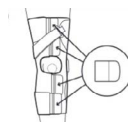


Small Extension Kit

### Large Extension Kit:

XS-S increases thigh circumference up to 4 cm / 1.6"

M-L increases thigh circumference up to 5 cm / 2.0"

XL-3XL increases thigh circumference up to 6 cm / 2.4"

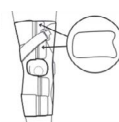

Large Extension Kit

## ÖSSUR INJURY SOLUTIONS

For people recovering from fractures, ligament injuries or in need of post-operative treatment solutions.

These product solutions have been built on decades of experience and are designed to enhance the healing process of bone and soft tissue injuries. Through innovation leadership, proven biomechanical approaches, and partnerships with leading medical professionals, we strive to offer comprehensive and effective treatment options.

Össur Injury Solutions are globally recognized, primarily under the brand names CTi®, Rebound® and Miami®, all of which are developed to assist healing and increase people's quality of life.

We leverage our expertise in Injury Solutions to develop a portfolio of easy to use and reliable products to support patients throughout their healing journey.





SPINE



MIAMI J°

PAGE 41



MIAMI J°  
SELECT

PAGE 41



MIAMI LSO°

PAGE 41



MIAMI TLSO°  
456 & 464

PAGE 41

UPPER EXTREMITIES



BUCKLE CLOSURE  
ARM SLING

PAGE 82



REBOUND  
POST-OP ELBOW

PAGE 82



FORMFIT  
UNIVERSAL WRIST

PAGE 82

KNEES



CTi3°

PAGE 97



REBOUND DUAL

PAGE 97



REBOUND  
POST-OP KNEE

PAGE 97

FEET



REBOUND°  
AIR WALKER

PAGE 139



FORMFIT WALKER

PAGE 139



FORMFIT ANKLE  
WITH SPEEDLACE

PAGE 139



## MIAMI LSO™

Miami LSO is a spinal orthosis designed for pain relief, protection of injured ligaments or muscles and post-surgical immobilization. It provides low-profile, anatomical immobilization, easy fit and wear, options for universal or modular support and an advanced pulley system with a length-adjustable pulley cord.



### INDICATIONS FOR USE

Conditions requiring gross immobilization of the trunk in the lumbar region. Such conditions may include:

- Stable, non-displaced spinal fractures (L1-L5)
- Spinal stenosis
- Herniated discs
- Degenerative spinal pathologies
- Spondylolisthesis
- Spondylolysis

Contra-indication: Unstable, displaced spinal fractures

### FEATURES AND BENEFITS

- Slim, breathable material
- Proven Miami double-pulley system with length-adjustable pulley cord
- Intuitive fitting without trimming with fold-over belt arms
- Optional fit tool for perfect fitting directly on the patient
- Universal belt sizing accommodates waists up to 178 cm/70"
- New Universal Posterior panel to accommodate more patients with less inventory
- Contoured posterior panels sized S-XL for a precise, anatomical fit
- Quick-Fit pocket enclosure and blue highlights for patient ease of use
- Immobilization from L1 – L5 with coverage from S1 – T9


### REQUIRED MEASUREMENTS



- Waist circumference






# MIAMI LSO™ ORDERING INFORMATION

## OPTION 1: Order braces for inventory

MIAMI LSO BELT					
					
SIZE		WAIST CIRC.		PART #	
6" Universal (Slim)		26-50" (66-127 cm)		MLSO-1	
8" Universal (Standard)		26-50" (66-127 cm)		MLSO-2	
6" (Slim) Belt Extension		26-70" (66-177 cm)		MLSO-1EXT	
8" (Standard) Belt Extension		26-70" (66-177 cm)		MLSO-2EXT	

CONTOURED POSTERIOR PANEL			CONTOURED POSTERIOR PANEL WITH LATERAL SUPPORTS		
					
SIZE	WAIST CIRC.	PART #	SIZE	WAIST CIRC.	PART #
Small (S)	26-34" (66-86 cm)	MLSO-P11	Small (S)	26-34" (66-86 cm)	MLSO-P12
Medium (M)	34-38" (86-96 cm)	MLSO-P21	Medium (M)	34-38" (86-96 cm)	MLSO-P22
Large (L)	38-50" (96-127 cm)	MLSO-P31	Large (L)	38-50" (96-127 cm)	MLSO-P32
Extra Large (XL)	50-66" (127-167 cm)	MLSO-P41	Extra Large (XL)	50-66" (127-167 cm)	MLSO-P42

STANDARD POSTERIOR PANEL		UNIVERSAL POSTERIOR PANEL		ANTERIOR PANEL	
					
SIZE	PART #	TYPE	PART #	SIZE	PART #
Standard	MLSO-P56	with Lateral	MLSO-P55	6" Panel (Slim)	MLSO-PA6
		without Lateral	MLSO-P57	8" Panel (Standard)	MLSO-PA8

## OPTION 2: Order a brace in a kit.

KIT	POSTERIOR PANEL	WITHOUT LATERAL SUPPORTS	WITH LATERAL SUPPORTS
		WITH ANTERIOR PANEL	
Slim Belt	S	MLSO-111-1	MLSO-112-1
	M	MLSO-121-1	MLSO-122-1
	L	MLSO-131-1	MLSO-132-1
	XL	MLSO-141-1	MLSO-142-1
	UNIVERSAL	MLSO-157-1	MLSO-155-1
		WITHOUT ANTERIOR PANEL	WITH ANTERIOR PANEL
	STANDARD	MLSO-156-2	MLSO-156-1

KIT	POSTERIOR PANEL	WITHOUT LATERAL SUPPORTS	WITH LATERAL SUPPORTS
		WITH ANTERIOR PANEL	
Standard Belt	S	MLSO-211-1	MLSO-212-1
	M	MLSO-221-1	MLSO-222-1
	L	MLSO-231-1	MLSO-232-1
	XL	MLSO-241-1	MLSO-242-1
	UNIVERSAL	MLSO-257-1	MLSO-255-1
		WITHOUT ANTERIOR PANEL	WITH ANTERIOR PANEL
	STANDARD	MLSO-256-2	MLSO-256-1

## MIAMI TLSO™ 464

The Miami TLSO from Össur directly addresses the needs of patients and practitioners, simplifying preparation without compromising on fit or performance. This easy-to-use pre-assembled Miami TLSO brace offers three configurations in one.

**Compliance:** Easily applied in over the shoulder or underarm configuration, donning and doffing is further simplified with the conveniently located quick release buckles.

**Adjustability:** The universal belt designs allows for setting the belt length on the patient for a more accurate sizing experience.

**Comfort:** Truly shaped to fit the anatomy, the anatomically contoured design better immobilizes the spine while maximizing patient comfort.

**Ease of Use:** The Miami TLSO brace requires minimal preparation and is designed to provide optimal patient comfort.



### INDICATIONS FOR USE

Conditions requiring gross immobilization of the thoracic and lumbar spine. Such conditions may include:

- Post-surgical immobilization
- Spinal Support TLSO attachment is up to T7
- Spinal stenosis
- Herniated disc
- Degenerative spinal fractures
- Spondylolisthesis
- Spondylolysis


### CONTRAINDICATIONS

- Unstable, displaced fractures

### FEATURES AND BENEFITS

- Significantly superior comfort and ease of donning/doffing.
- Fully assembled for fitting with ease.
- Accommodates heights 155–193 cm (5'1"–6'5")
- Posterior thoracic extension option with ergonomic axillary strapping.
- Step-down to an LSO as one progresses through rehabilitation.
- One-touch telescoping anterior-thoracic extension option with single or double-hinge.
- The convenience of a one-size Universal or individualize with a uniquely-sized anatomical design.
- Improve comfort and immobilization with multiple strapping configurations and removable anchors.

### REQUIRED MEASUREMENTS

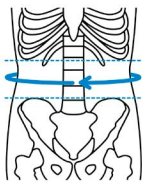

Measure circumference of the trunk at a level midway between the lowest ribs and the upper iliac crest. Standing.

Size	Waist circumference	
	cm	inch
Universal	66 - 177	26 - 70
S	66 - 86	26 - 34
M	86 - 107	34 - 42
L	107 - 127	42 - 50
XL	127 - 177	50 - 70

\*Belt extensions are required for waist circumferences larger than 127 cm / 50"

# MIAMI TLSO™ 464

## ORDERING INFORMATION

### ATE/PTE, LORDOTIC

Size	Part#
Universal	MTLSO-2351
S	MTLSO-2111
M	MTLSO-2121
L	MTLSO-2131
XL	MTLSO-2141

### ATE/PTE, NON-LORDOTIC

Size	Part#
S	MTLSO-2211
M	MTLSO-2221
L	MTLSO-2231
XL	MTLSO-2241

### ATE/PTE, LORDOTIC, SLIM

Size	Part#
Universal	MTLSO-1351
S	MTLSO-1111
M	MTLSO-1121
L	MTLSO-1131
XL	MTLSO-1141

### ATE/PTE, NON-LORDOTIC, SLIM

Size	Part#
S	MTLSO-1211
M	MTLSO-1221
L	MTLSO-1231
XL	MTLSO-1241

## OTHER ACCESSORIES

Description	Part#
5MM Hex Long Wrench	MTLSO-0004
Axillary strap accessory kit	MTLSO-0003
Quick fit tool	B-040100000
Belt extension, standard (Recommended for waist circumferences 127-177 cm (50"-70"). Fits size XL.)	MLSO-2EXT
Belt extension, slim (Recommended for waist circumferences 127-177 cm (50"-70"). Fits size XL.)	MLSO-1EXT

## MIAMI TLSO 456

The Miami TLSO 456 from Össur directly addresses the needs of patients and practitioners, simplifying preparation without compromising on fit or performance. This easy-to-use pre-assembled Miami TLSO brace offers the convenience of a one-size Universal or a uniquely-sized anatomical design.

Compliance: Blue is You™ patient touch-points help send patients home with confidence.

Adjustability: The universal belt design allows for setting the belt length on the patient for a more accurate sizing experience. Heat moldable posterior panels allow a customized fit, while the modular design allows step-down support to an LSO.

Comfort: Featuring the Miami LSO™ Advanced Compression pulley system with lumbar compression you can feel, and an anatomically contoured design that better immobilizes the spine while maximizing patient comfort.

Ease of Use: Easily applied with over-the-shoulder straps, the Miami TLSO brace requires minimal preparation and is designed to provide optimal patient comfort.



### INDICATIONS FOR USE

Conditions requiring gross immobilization of the thoracic and lumbar spine. Such conditions may include:

- Post-surgical immobilization
- Spinal Support TLSO attachment is up to T7
- Spinal stenosis
- Herniated disc
- Degenerative spinal fractures
- Spondylolithesis
- Spondylolysis


### FEATURES AND BENEFITS

- Significantly superior comfort and ease of donning/ doffing (when compared to Miami Lumbar TLSO).
- Fully assembled for fitting with ease.
- Accommodates heights 155–193 cm (5'1"–6'5")
- Step-down to an LSO as one progresses through rehabilitation.
- The convenience of a one-size Universal or individualize with a uniquely-sized anatomical design.

### CONTRAINDICATIONS

- Unstable, displaced fractures

### REQUIRED MEASUREMENTS

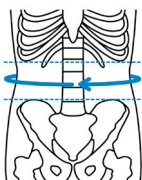

Measure circumference of the trunk at a level midway between the lowest ribs and the upper iliac crest. Standing.

Size	Waist circumference*	
	cm	inch
Universal	66 - 177	26 - 70
S	66 - 86	26 - 34
M	86 - 107	34 - 42
L	107 - 127	42 - 50
XL	127 - 177	50 - 70

\*Belt extensions are required for waist circumferences larger than 127 cm / 50"

# MIAMI TLSO 456

## ORDERING INFORMATION

### PTE LORDOTIC

Size	Part#
Universal	MTLSO-2352
S	MTLSO-2112
M	MTLSO-2122
L	MTLSO-2132
XL	MTLSO-2142

### PTE NON-LORDOTIC, SLIM

Size	Part#
S	MTLSO-1212
M	MTLSO-1222
L	MTLSO-1232
XL	MTLSO-1242

### PTE NON-LORDOTIC

Size	Part#
S	MTLSO-2212
M	MTLSO-2222
L	MTLSO-2232
XL	MTLSO-2242

### PTE LORDOTIC, SLIM

Size	Part#
Universal	MTLSO-1352
S	MTLSO-1112
M	MTLSO-1122
L	MTLSO-1132
XL	MTLSO-1142

## OTHER ACCESSORIES

Description	Part#
Quick fit tool	B-040100000
Belt extension, standard (Recommended for waist circumferences 127-177 cm (50"-70"). Fits size XL.)	MLSO-2EXT
Belt extension, slim (Recommended for waist circumferences 127-177 cm (50"-70"). Fits size XL.)	MLSO-1EXT



## MIAMI LUMBAR® POSTEO

Designed with patient comfort and compliance in mind, the Miami Lumbar Posteo provides functional relief of vertebral fracture pain, and facilitates return to mobility. The lumbar compression system offers both pain relief via active vertebral offloading and better grip of patient's hips for greater ease of use and compliance. The Miami Lumbar Posteo includes an innovative approach to ease of use and consistent donning.

### INDICATIONS FOR USE

- Back pain caused by stable vertebral compression fractures (VCF)

### FEATURES AND BENEFITS

- Active offloading via thoracic extension and lumbar compression
- Integrated pulley system for customized fit
- Trimmable axillary straps with adjustable tension for optimal stabilization
- Cold-moldable aluminum PTE to fit different anatomies
- Extra cushioning around the aluminum strut for minimized skin pressure



### REQUIRED MEASUREMENTS

- Back length measured from the top of the sacrum up to C7 vertebrae, following the spinal curvature

Please note that the Miami Lumbar Posteo includes a universal lumbar belt which can be trimmed down to the appropriate size. Calculation of the appropriate back length size, measured from the tip of the sacrum to the C7 vertebra, following the curvature of the spine.

### ORDERING INFORMATION

#### MIAMI LUMBAR POSTEO

Size	Back Length	Part#
X-Small	14.5" – 17" (37 – 44 cm)	LMB-POSTEO-XS
Small	17" – 20" (44 – 51 cm)	LMB-POSTEO-SM
Medium	19" – 22" (49 – 56 cm)	LMB-POSTEO-MD
Large	21" – 24" (55 – 61 cm)	LMB-POSTEO-LG

### ACCESSORIES

Description	Part#
Strap Pad	LMB-POSTEO ST PD



## FORMFIT® PRO BACK

Össur Formfit® Pro Back provides relief for patients suffering from pains and strains in the lower back. Manufactured with our dynamic, 3D-Motion Tech™ knitting technology, our belts feature superior moisture wicking capabilities and are highly breathable. The patented back panel design utilizes our StayBilizer™ technology to provide optimum shape and fit, while conveniently placed finger loops make donning easy. The SmartPad™ provides a targeted support and cushioning and can be removed if desired.



MotionTech™



StayBilizer™



SmartPad™

### INDICATIONS FOR USE

For lower back conditions that may benefit from compression and gross motion restriction, such as:

- Lower back pain (Lumbago)
- Lumbosacral sprain/strain
- Degenerative joint/disc disease


### CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

### FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit.
- Removable lumbar SmartPad™ for targeted support and cushioning.
- Lightweight, malleable aluminum stays are shaped to the anatomy of the lower back.

### REQUIRED MEASUREMENTS

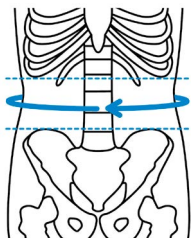

Measure circumference of the trunk at a level midway between the lowest ribs and the upper iliac crest. Standing.



Size	Waist circumference		Black
	cm	inch	Part#
XS	70 - 80	27½ - 31½	FP500001
S	80 - 90	31½ - 35½	FP500002
M	90 - 100	35½ - 39¼	FP500003
L	100 - 115	39¼ - 45¼	FP500004
XL	115 - 130	45¼ - 51¼	FP500005

## MIAMI J® SELECT

The Miami J Select cervical collar by Össur offers the widest range of height adjustability in the market with a single, universal design that replicates the comfort, immobilization and quality you expect from a Miami J product. Featuring a proprietary, first-to-market locking mechanism to encourage patient compliance and anti-microbial Sorbatex padding, Miami J Select is designed to support successful patient outcomes.



### INDICATIONS FOR USE

Situations requiring gross immobilization of the cervical spine, these may include:

- Pre- and post c-spine surgery
- C-spine precaution for trauma patients

### FEATURES AND BENEFITS

- Easy height adjustability with a tamper-proof locking mechanism – ‘Left to Lock’.
- Anti-microbial & biocompatible Sorbatex™ padding prevents bacterial build-up.
- Universal design that doesn’t compromise immobilization or comfort.
- 7 precise fitting sizes with opportunity to fine-tune.
- Miami J® inspired chin tray.
- One-piece, anatomically inspired collar back with Flex Edge® designed to reduce occipital pressure.
- Easy to don and doff with proprietary Reproducible Fit option.
- Intuitive blue Patient Touch-Points to send patients home with confidence – Blue is You™.
- Simplified comfort adjustments with integrated and proprietary Sternal-Relief Dial.
- Large tracheal opening – improved function for ICU patients.
- Easy to clean and replace multi-colored padding.
- X-ray and CT lucent, MR-safe.
- Compatible with Occian™ Back & Miami JTO® accessories.

### ORDERING INFORMATION

Description	Part#	Part#
Miami J Select Collar Set	One-size adjustable (Universal)	MJSR-101
Miami J Select Collar	One-size adjustable (Universal)	MJS-101
Miami J Select Replacement Pads	One-size adjustable (Universal)	MJSP-101
Miami J Select Reproducible Fit Tabs (10 pack)	One-size adjustable (Universal)	MJST-101

## MIAMI J®

Known as the superior c-spine immobilizer through multiple independent studies, the Miami J's patented design meets the specialized needs of all those afflicted with cervical injuries. The Miami J offers features that enhance compliance while providing immobilization, inhibition of skin breakdown and patient comfort.



### INDICATIONS FOR USE

- Post-trauma
- Stable cervical fractures
- Cervical spondylitis
- Cervical spondylosis
- RA and OA cervical spine
- Herniated cervical disc
- Post-operative
- Motor Neurone Disease

### FEATURES AND BENEFITS

- Sorbatex padding is antibacterial and clinically proven to inhibit microbial growth, which enhances comfort and helps assure patient compliance
- Bioengineered to minimize pressure points in key known areas: Chin, occiput, trapezius and clavicle
- Angulation buttons for occiput and sternal pad is removable when supine, lying and swallowing
- MRI, CT and X-ray lucent, MR safe

### REQUIRED MEASUREMENTS

- Sizing based on 6 phenotypes to ensure correct fit and optimum immobilization

Phenotype-driven sizing system makes size selection simple while drastically reducing margin of error in fitting a patient. Each collar has a bioengineered front with a specific back designed to work together for superior immobilization. For pediatric sizing, please see Miami Jr in this Catalog.

#### Specialty Sizes (40% of population)

**Kyphotic (10%)**

- kyphotic
- chin-on-chest
- geriatric
- osteoporitic and/or ankylosing spondylitic

**XSmall (5%)**

- Very short, thin neck
- Principally female
- More prevalent in the Asian population

**Stout (15%)**

- Very large neck circumference
- Obese
- "No neck"
- Massive shoulders

**Tall (10%)**

- Long, tall neck
- "Swan neck"
- Young Women
- Adolescents

#### Standard Sizes (60%)

**Regular (30%)**

- Female
- Standard size for mature women
- Thin, mature male

**Short (30%)**

- Standard size for mature men
- Short-necked women

\*For individuals with neck circumference > 25", the Bariatric collar is recommended.

# MIAMI J®

## ORDERING INFORMATION

### MIAMI J

Size	Collar set*	Collar	Collar Back	Collar Front
Super Short	MJR-200S	MJ-200S	MJ-400BK	MJ-200SFRT
Stout	MJR-200L	MJ-200L	MJ-200LBK	MJ-200LFRT
XS	MJR-250	MJ-250	MJ-250BK	MJ-250FRT
Short	MJR-300	MJ-300	MJ-300BK	MJ-300FRT
Regular	MJR-400	MJ-400	MJ-400BK	MJ-400FRT
Tall	MJR-500	MJ-500	MJ-500BK	MJ-500FRT
Bariatric	MJR-200B	MJ-200B		

\*Collar sets come with one additional set of replacement pads.

Miami J Front and Back Panels may be interchanged to accommodate patient anatomy, e.g. a Short Front may be used with a Tall Back. Choose collar size based on patient neck height, not patient body size. Super Short collars are packaged with Regular Back Panels

## ACCESSORIES

### REPLACEMENT PADS

Size	Part#
Super Short	MJP-100
Stout	MJP-200L
XS	MJP-250
Short	MJP-100
Regular	MJP-100
Tall	MJP-100
Bariatric	MJP-200B

### MIAMI J PATIENT PACK

Description	Part#
Miami J Patient Pack	MJ-PTPK2

## MIAMI JTO® THORACIC EXTENSION

The Miami JTO Thoracic Extension is designed in conjunction with leading spine surgeons, orthotists and nurses. The orthosis is used exclusively with all Miami J® collars when even greater stability at the cervical junctions is indicated.



### INDICATIONS FOR USE

- |  |  |
|--|--|
| <p>Lower C-Spine disruptions</p> <ul style="list-style-type: none"> <li>• Level one instabilities</li> <li>• Isolated compartment fracture</li> <li>• Lateral mass fracture</li> <li>• Spinous Process fracture</li> </ul> | <p>Upper C-Spine disruptions</p> <ul style="list-style-type: none"> <li>• For increased stability over the collar alone</li> <li>• Stable Jefferson or Hangman's Fracture</li> <li>• Type I Odontoid Fracture</li> </ul> |
|--|--|

### FEATURES AND BENEFITS

- Provides more control than a collar alone can provide
- Innovative device designed to treat cervical and high-thoracic injuries
- Patented design with 'floating back' permits maximum immobilization without the risk of compensatory motion
- Snap on and off without tools
- Sorbatex II padding is antibacterial and clinically proven to inhibit microbial growth, which enhances comfort and helps assure patient compliance
- MRI, CT and X-ray lucent, MR compatible

### REQUIRED MEASUREMENTS

- Universal, one size

### ORDERING INFORMATION

#### MIAMI JTO THORACIC EXTENSION

Description	Part#
Miami JTO	MT-900
Miami JTO with replacement softgoods	MTR-900

Compatible with all sizes of the Miami J®, level 1 through 5 of the Miami J® Select, and the P3 Miami Jr®. Any model of Miami J collar is not included with Miami JTO Thoracic Extension and is purchased separately.

### ACCESSORIES

Description	Part#
Replacement softgoods	MTP-950



## OCCIAN® BACK & OCCIAN® BACK JR

The Occian Back is designed exclusively for the specialized needs of the multi-system trauma patient in the ICU. Universally sized, specifically engineered and designed with advanced, Intuitech™ pressure-relieving memory foam, the Occian Back helps caregivers manage c-spine patients who are at high risk of occipital breakdown.



### INDICATIONS FOR USE

- Patients not awake and alert within 24 – 48 hours after admission
- Patients with pre-existing skin injuries and/or lacerations to the head/neck
- Burn patients who must wear a collar
- Elective surgery patients desiring night time post-operative comfort
- Patients who remain in a supine position for long periods of time related to respiratory, neurological, or haemodynamic instability

### FEATURES AND BENEFITS

- Intuitech pressure relieving memory foam virtually eliminates occipital breakdown
- Maintains C spine stability
- Provides unique pressure-relieving comfort by replacing the back section of a Miami J Collar.
- Clinically proven to virtually eliminate occipital breakdown without sacrificing c-spine stability.
- Easy to clean
- MRI, CT & X-ray lucent, MR safe

### REQUIRED MEASUREMENTS

- Universal, one size

### ORDERING INFORMATION

Description	Part#
Occian Back*	ACB-100
Occian Back Junior**	ACB-200

\*Compatible with Miami Jr® size P3 and all sizes of Miami J® and Miami J® Select.

\*\* Compatible with Miami Jr® size P2.

## MIAMI Jr®

The Miami Jr is the breakthrough collar designed exclusively for children of 12 years and under. The Miami Jr line, which spans the growth cycle, is sized according to age. Össur is the only company that has a pediatric line of collars based on the anatomical differences between adults and children. We realize that children are not just “little adults” – they require innovative design.



### INDICATIONS FOR USE

- Post-trauma
- Cervical spondylosis
- Post-operative
- Stable cervical fractures
- RA and OA cervical spine
- Motor Neurone Disease
- Cervical spondylitis
- Herniated cervical disc

### FEATURES AND BENEFITS

- Innovative collar designed exclusively for children 12 years and under anatomy pediatric
- Sorbatex II pads make the collar “skin friendly” and protect skin integrity during extended wear
- MRI, CT and X-ray lucent, MR safe

### REQUIRED MEASUREMENTS

- Age

### ORDERING INFORMATION

#### MIAMI JR

Size	Collar set*	Collar
0 – 6 months	MJR-P0	MJ-P0
6 – 24 months	MJR-P1	MJ-P1
2 – 6 years	MJR-P2	MJ-P2
6 – 12 years	MJR-P3	MJ-P3

\*Collar sets come with one additional set of replacement pads.

### ACCESSORIES

#### REPLACEMENT PADS

Size	Part#
0 – 6 months	MJP-P0
6 – 24 months	MJP-P1
2 – 6 years	MJP-P2
6 – 12 years	MJP-P3

## PAPOOSE®

The Papoose is an innovative design for the youngest and most challenging patients – newborn infants. The Papoose cradles the baby's body securely while gently immobilizing the head and spine.



### INDICATIONS FOR USE

- Immobilization of the spine
- Spinal fractures

### FEATURES AND BENEFITS

- True cervical spine immobilizer for infants up to 3 months
- Allows for continued intimacy between caregiver and child
- MRI, CT and X-ray lucent, MR compatible

### REQUIRED MEASUREMENTS

- Universal, one size

### ORDERING INFORMATION

#### PAPOOSE

Description	Part#
Papoose without front collar	PO-100
Papoose without front with replacement pads	POR-100
Papoose with front collar	PA-100
Papoose with front with replacement pads	PAR-100

### ACCESSORIES

#### REPLACEMENT PADS

Description	Part#
Papoose replacement pads (with front)	PAP-100
Papoose replacement pads (without front)	POP-100

# PHILADELPHIA® ADJUSTABLE TRACHEOTOMY COLLAR

## INDICATIONS FOR USE

- Post-trauma
- Herniated cervical disc
- Cervical spondylitis
- Post-operative
- Cervical spondylosis
- RA and OA cervical spine

## FEATURES AND BENEFITS

- Circumference and height adjustable
- Permanent lock on height adjustment
- Anatomically shaped Plastazote® foam (Not made with natural rubber latex, non-toxic and hypoallergenic to reduce skin irritation), leaving clearance for cervical veins
- Total cervical arch support maintains neutral alignment of the cervical spine
- Water-resistant for bathing and hydro-therapy.
- Well ventilated
- X-ray and CT lucent. MR conditional.

## REQUIRED MEASUREMENTS

- Neck height: 1 inch (or 25mm) of neck length adjustability accommodates varying patient neck length.

## ORDERING INFORMATION

### PHILADELPHIA ADJUSTABLE TRACHEOTOMY COLLAR

Size	Neck circumference
Small (2 tears)	10 – 13" (25.4 – 33 cm)
Medium (1 tear)	13 – 16" (33 – 40.6 cm)
Large (no tear)	16 – 19" (40.6 – 48.3 cm)

NOTE: The table above represents common sizing only. Sizing must be verified by the healthcare provider for each patient individually.

Part#	Size / Description
PHP-AA	Conforms to sizes 2, 3 and 4 of the original Philadelphia Tracheotomy Collar



# PHILADELPHIA® TRACHEOTOMY COLLAR

## INDICATIONS FOR USE

- Post-trauma
- Herniated cervical disc
- Cervical spondylitis
- Post-operative
- Cervical spondylosis
- Motor Neurone Disease
- RA and OA cervical spine

## FEATURES AND BENEFITS

- Not made with natural rubber latex, non-toxic and hypoallergenic  
Plastazote® foam reduces skin irritation
- Total cervical arch support maintains neutral alignment of the cervical spine
- Water-resistant for bathing and hydro-therapy.



## REQUIRED MEASUREMENTS

- Neck circumference and height from tip of chin to sternal notch

## ORDERING INFORMATION

Size	Neck circumference	6 cm / 2.25" height	8 cm / 3.25" height	11 cm / 4.25" height	13 cm / 5.25" height
X-Small	up to 10" (< 25 cm)	PHP-T2XS			
Small	10 – 13" (25 – 33 cm)	PHP-T2S	PHP-T3S	PHP-T4S	PHP-T5S
Medium	13-16" (33 – 41 cm)	PHP-T2M	PHP-T3M	PHP-T4M	PHP-T5M
Large	16-19" (41 – 48 cm)	PHP-T2L	PHP-T3L	PHP-T4L	PHP-T5L
X-Large	19" up (48 cm)	PHP-T2XL	PHP-T3XL	PHP-T4XL	PHP-T5XL

Size	Neck circumference	Height	Part#
Infant	6-8" (15 – 20 cm)	1.25" (3 cm)	PHP-TI
Pediatric	8-11" (20 – 28 cm)	1.75" (4.5 cm)	PHP-TP

## ACCESSORIES

### PHILADELPHIA COOLMAX COTTON LINER

Size	Neck circumference	Part#
Infant	6-8" (15 – 20 cm)	PHP-LCI
X-Small	up to 10" (< 25 cm)	PHP-LCXS
Small	10-13" (25 – 33 cm)	PHP-LCS
Medium	13-16" (33 – 41 cm)	PHP-LCM
Large	16-19" (41 – 48 cm)	PHP-LCL
X-Large	19" up (> 48 cm)	PHP-LCX

### PHILADELPHIA TRACHEOTOMY ATTACHMENTS

Size	Stabilizer	Halo
Pediatric	PHP-OSP	PHP-OHP
Adult	PHP-OS	PHP-OH

## UNIVERSAL CERVICAL COLLAR

### INDICATIONS FOR USE

- Neck extension injuries (Whiplash)
- Cervical spondylitis
- Cervical spondylosis
- RA and OA cervical spine
- Herniated cervical disc

### FEATURES AND BENEFITS

- Medium density foam
- Contact closure for easy fit

### REQUIRED MEASUREMENTS

- Mandible to clavicle height



### ORDERING INFORMATION

Height	Part#
2" (5 cm)	203020
3" (7.5 cm)	203000
4" (10 cm)	203010

## FOAM CERVICAL COLLAR

### INDICATIONS FOR USE

- Neck extension injuries (Whiplash)
- Cervical spondylitis
- Cervical spondylosis
- RA and OA cervical spine
- Herniated cervical disc

### FEATURES AND BENEFITS

- Easy to apply

### REQUIRED MEASUREMENTS

- Neck circumference and mandible to clavicle height
- Ordering Information



Size	Circumference	Neck size	Part#
Small	3" x 18.5" (7 cm x 47 cm)	11-15"	203013
Small/Long	3" x 22" (7 cm x 56 cm)	15-20"	203014
Medium	3.5" x 20" (9 cm x 51 cm)	13-18"	203015
Medium/Long	3.5" x 23" (9 cm x 58 cm)	16-21"	203016
Large	4" x 23" (10 cm x 58 cm)	15-20"	203017
Large/Long	4" x 25" (10 cm x 64 cm)	16-22"	203018



## NECLOC®

### INDICATIONS FOR USE

- Post-trauma

### FEATURES AND BENEFITS

- Ergonomic two-piece clamshell design and retaining strap secure flexion during application
- Five unique phenotype-driven sizes make it easy to select the correct size
- Minimal manipulation of patient during application
- X-ray and CT lucent. MR conditional.



### ORDERING INFORMATION

Size	Part#
Stout	NL-200EL
X-Small	NL-250E
Small	NL-300E
Medium	NL-400E
Large	NL-500E

## NECLOC® KIDS

### INDICATIONS FOR USE

- Post-trauma

### FEATURES AND BENEFITS

- Ergonomic two-piece clamshell design and retaining strap secure flexion during application
- Three pediatric sizes ensure optimal airway and cervical spine immobilization
- Minimal manipulation of patient during application to children
- Meets the unique developing anatomical requirements of children
- X-ray and CT lucent. MR conditional



### REQUIRED MEASUREMENTS

- Age

### ORDERING INFORMATION

Age	Part#
0 – 2 years	NL-P1
2 – 6 years	NL-P2
6 – 12 years	NL-P3

## PATRIOT® COLLAR

### INDICATIONS FOR USE

- Post-trauma

### FEATURES AND BENEFITS

- Multi-height adjustment – Adult (4 positions) and Pediatric (3 positions)
- Open rear ventilation panel for fluid drainage, airflow and cervical spine palpation
- Universal sizing optimizes inventory requirements

### REQUIRED MEASUREMENTS

- Neck circumference



### ORDERING INFORMATION

Size	Neck circumference	Blue
Child	8-18" (20 – 46 cm)	PHP-PP
Adult	11-23" (28 – 58 cm)	PHP-PA

# OAM RIGID LUMBAR

The OAM Rigid Lumbar offers you a discrete look with endless options.

**First, you choose the belt.**

Slim or Standard, 10 pre-cut sizes or universal.

**Then, you choose the panels.**

Different panels for different control, modular to allow you to remove as the patient heals.

**Or, you choose the kit.**

We make it even simpler by providing our most popular sizes pre-configured for you.



## TWO SIMPLE OPTIONS FOR ORDERING




### OPTION 1: ORDER BRACES BY COMPONENTS

OAM RIGID LUMBAR BELT			
SIZE	WAIST CIRC.	STANDARD PART #	SLIM PART #
XS	26-30" (66-76cm)	<a href="#">LMBGEN-001</a>	<a href="#">LMBSLGEN-001</a>
S	30-34" (76-86cm)	<a href="#">LMBGEN-002</a>	<a href="#">LMBSLGEN-002</a>
M	34-38" (86-96cm)	<a href="#">LMBGEN-003</a>	<a href="#">LMBSLGEN-003</a>
L	38-42" (96-106cm)	<a href="#">LMBGEN-004</a>	<a href="#">LMBSLGEN-004</a>
XL	42-46" (106-116cm)	<a href="#">LMBGEN-005</a>	<a href="#">LMBSLGEN-005</a>
2XL	46-50" (116-127cm)	<a href="#">LMBGEN-006</a>	<a href="#">LMBSLGEN-006</a>
3XL	50-54" (127-137cm)	<a href="#">LMBGEN-007</a>	<a href="#">LMBSLGEN-007</a>
4XL	54-58" (137-147cm)	<a href="#">LMBGEN-008</a>	<a href="#">LMBSLGEN-008</a>
5XL	58-62" (147-157cm)	<a href="#">LMBGEN-009</a>	<a href="#">LMBSLGEN-009</a>
6XL	62-66" (157-167cm)	<a href="#">LMBGEN-010</a>	<a href="#">LMBSLGEN-010</a>

ANTERIOR PANELS		
SIZE	WAIST CIRC.	PART #
Standard	<46" <116cm	<a href="#">LMBGEN-ST</a>
Slim	<46" <116cm	<a href="#">LMBGEN-SLST</a>

POSTERIOR PANELS			
SIZE	WAIST CIRC.	WITHOUT LATERAL	WITH LATERAL
S	26-34" (66-86cm)	<a href="#">LMBGEN-NS</a>	<a href="#">LMBGEN-LS</a>
M	34-38" (86-96cm)	<a href="#">LMBGEN-NM</a>	<a href="#">LMBGEN-LM</a>
L	38-50" (96-127cm)	<a href="#">LMBGEN-NL</a>	<a href="#">LMBGEN-LL</a>
XL	50-66" (127-167cm)	<a href="#">LMBGEN-NX</a>	<a href="#">LMBGEN-LX</a>

## OPTION 2: ORDER BRACES BY KIT

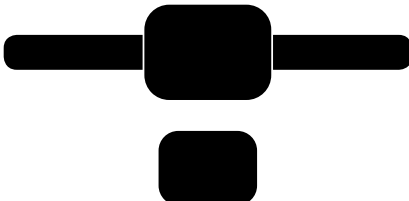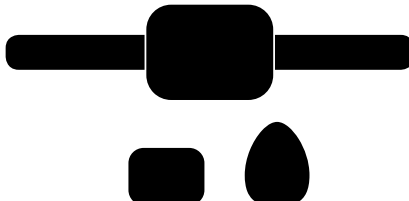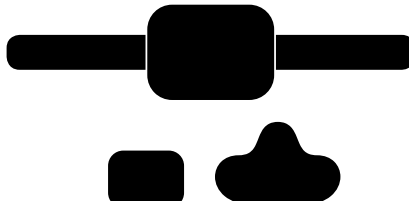PANEL CONFIGURATIONS		
L0627	L0631	L0637
		
ANTERIOR PANEL ONLY (627)	POSTERIOR PANEL WITHOUT LATERAL SUPPORTS (631)	POSTERIOR PANEL WITH LATERAL SUPPORTS (637)

SPINE

SLIM BELT WIDTH										
BELT LENGTH	WAIST CIRC.	ANTERIOR PANEL ONLY (627)	POSTERIOR PANEL <b>WITHOUT</b> LATERAL SUPPORTS (631)				POSTERIOR PANEL <b>WITH</b> LATERAL SUPPORTS (637)			
			S	M	L	XL	S	M	L	XL
Universal	34-50”(86-127cm)	OAM627-01	OAM631-01	OAM631-06	OAM631-16	OAM631-18	OAM637-01	OAM637-06	OAM637-16	OAM637-18
Universal +	34-66”(86-167cm)	OAM627-06	OAM631-02	OAM631-10	OAM631-20	OAM631-22	OAM637-02	OAM637-10	OAM637-20	OAM637-22

STANDARD BELT WIDTH										
BELT LENGTH	WAIST CIRC.	ANTERIOR PANEL ONLY (627)	POSTERIOR PANEL <b>WITHOUT</b> LATERAL SUPPORTS (631)				POSTERIOR PANEL <b>WITH</b> LATERAL SUPPORTS (637)			
			S	M	L	XL	S	M	L	XL
Universal	34-50"(86-127cm)	OAMST627-01	OAMST631-01	OAMST631-06	OAMST631-16	OAMST631-18	OAMST637-01	OAMST637-06	OAMST637-16	OAMST637-18
Universal +	34-66"(86-167cm)	OAMST627-06	OAMST631-02	OAMST631-10	OAMST631-20	OAMST631-22	OAMST637-02	OAMST637-10	OAMST637-20	OAMST637-22

## FORMFIT® BACK SUPPORT, AIR

### INDICATIONS FOR USE

- Muscular low back pain
- Lumbosacral strains
- Chronic low back pain
- Sciatica

### FEATURES AND BENEFITS

- Intuitive integrated pneumatic system for more customized compression and support
- Anti-migration system reduces slippage
- Double wrap design for improved compression
- Finger loops for easy application
- Hot/Cold Gel Pad available separately

### REQUIRED MEASUREMENTS

- Waist circumference



### ORDERING INFORMATION

Size	Waist circumference	Standard
S	25-30" (63 – 76 cm)	209133
M	30-34" (76 – 86 cm)	209135
L	34-38" (86 – 97 cm)	209137
XL	38-42" (97 – 107 cm)	209138
XXL	42-56" (107 – 142 cm)	209139

### ACCESSORIES

Description	Part#
Gel Pad	GB28

Description	Part#
Back Support Suspenders Clip On	10912

# FORMFIT® BACK SUPPORT

## INDICATIONS FOR USE

- Lumbar strains
- General posture support
- Lumbosacral strains
- Sciatica

## FEATURES AND BENEFITS

- Muscular low back pain
- Lumbosacral strains
- Chronic low back pain
- Sciatica

## REQUIRED MEASUREMENTS

- Waist circumference



## ORDERING INFORMATION

Size	Waist Circumference	Standard
Small	25 – 30" (63 – 76 cm)	209143
Medium	30 – 34" (76 – 86 cm)	209145
Large	34 – 38" (86 – 97 cm)	209147
X-Large	38 – 42" (97 – 107 cm)	209148
XX-Large	42 – 56" (107 – 142 cm)	209149
XXX-Large	56 – 60" (142 – 152 cm)	209151

## ACCESSORIES

Description	Part#
Back Support Suspenders Clip On	10912
Hot/Cold Therapy Gel Pad	GB19028



## UNIVERSAL ELASTIC RIB BELT

### INDICATIONS FOR USE

- Pre-or post operative use in the thorax area.

### FEATURES AND BENEFITS

- Flannel-lined
- Contact closure

### REQUIRED MEASUREMENTS

- Measure circumference at waist



### ORDERING INFORMATION

Description	Part#
Male – 6.25" (16 cm) wide x 38.75" (98.5 cm) long	209150
Female – 6.25" (16 cm) wide x 38.75" (98.5 cm) long	209160

## UNIVERSAL ABDOMINAL BINDER

### INDICATIONS FOR USE

- Sacroiliac support
- Low back pain

### FEATURES AND BENEFITS

- Elastic belt has Velcro closures for a perfectly contoured fit
- Available in unisex universal size

### REQUIRED MEASUREMENTS

- Measure circumference at waist



### ORDERING INFORMATION

Description	Fits	Part#
10" Universal binder	26 – 50" (66 – 127 cm)	SS1100

# ELASTIC ABDOMINAL BINDER

## INDICATIONS FOR USE

- Sacroiliac support
- Low back pain

## FEATURES AND BENEFITS

- Durable Elastic
- Abdominal, post-natal and post-surgical application
- Easy adjustment with contact closure
- Provides compression and support for strains and weakness of the abdominal area
- 3 panel and 4 panel options available
- Does not contain natural rubber latex



## REQUIRED MEASUREMENTS

- Measure waist circumference

## ORDERING INFORMATION

Waist size	3-Panel	4-Panel
30 – 42" (76 – 107 cm)	5437691	54376121
42 – 55" (107 – 140 cm)	5437692	54376122
55 – 69" (140 – 175 cm)	5437693	54376123
69 – 84" (175 – 213 cm)	5437694	54376124
84 – 100" (213 – 254 cm)	5437695	54376125





UPPER EXTREMITY





## FORMFIT® PRO ELBOW

Össur Formfit® Pro Elbow provides relief from elbow pain. This 3D compressive elbow sleeve is manufactured using our MotionTech™ technology and features two integrated epicondyle SmartPads™ for more targeted support and cushioning. The RadialStrap™ provides additional compression and support.



MotionTech™



CoolVent™



SmartPad™



FoldLock™



### INDICATIONS FOR USE

For elbow conditions that may benefit from compression, such as:

- Medial and lateral epicondylitis
- Elbow pain and swelling


### CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

### FEATURES AND BENEFITS

- 3D MotionTech™ for compression and dynamic fit keep.
- Epicondyle SmartPads™ for targeted support and cushioning.
- Patented FoldLock™ system for easier and more secure application.
- RadialStrap™ for increased compression and support.

### REQUIRED MEASUREMENTS

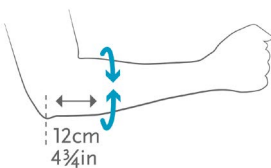

Measure circumference  
12cm (43/4in) below elbow.



Size	Elbow circumference		Blue	Black
	cm	inch	Part#	Part#
XS	17 - 20	6¾ - 7¾	FP301001	FP300001
S	20 - 23	7¾ - 9	FP301002	FP300002
M	23 - 26	9 - 10¼	FP301003	FP300003
L	26 - 29	10¼ - 11½	FP301004	FP300004
XL	29 - 32	11½ - 12½	FP301005	FP300005

## FORMFIT® SHOULDER BRACE

The Formfit Shoulder Brace is designed to provide immobilization following shoulder injury/surgery. With its streamlined fit, sporty look and breathability it offers excellent comfort and ease of use. Same as its predecessor, the SmartSling®, the Formfit Shoulder Brace also provides abduction of the shoulder (when the abduction pillow is attached), or external rotation of the shoulder (when the external rotation pillow is attached).

Optional axilla pillow helps distribute weight over a larger surface area and decreases pressure on the ulnar nerve.

### INDICATIONS FOR USE

- Immobilization for rotator cuff repairs
- Bankart repairs
- Capsular shifts
- Glenohumeral dislocations/subluxations
- Soft tissue repairs/strains
- Shoulder instabilities/injuries
- Post-surgical rehabilitation

### FEATURES AND BENEFITS

- Breathable material
- Quick-release buckles
- Offloading strap
- Extra padding
- Universal thumb loop



Axilla Pillow



External rotation pillow



### REQUIRED MEASUREMENTS

- Olecranon to end of closed fist

### ORDERING INFORMATION

Size	Fits Length	Description	Part
Small	10.5-13" (up to 29 cm)	Formfit Shoulder Brace - sling only	SSS-SO
Medium	13.25-15.75" (29-40 cm)	Formfit Shoulder Brace - sling only	SSR-SO
Large	16-17.5" (40-44.5 cm)	Formfit Shoulder Brace - sling only	SSL-SO
Small	10.5-13" (up to 29 cm)	Formfit Shoulder Brace with Abduction	SSS
Medium	13.25-15.75" (29-40 cm)	Formfit Shoulder Brace with Abduction	SSR
Large	16-17.5" (40-44.5 cm)	Formfit Shoulder Brace with Abduction	SSL
Small	10.5-13" (up to 29 cm)	Formfit Shoulder Brace with External Rotation	SSS-ER
Medium	13.25-15.75" (29-40 cm)	Formfit Shoulder Brace with External Rotation	SSR-ER
Large	16-17.5" (40-44.5 cm)	Formfit Shoulder Brace with External Rotation	SSL-ER

### ACCESSORIES

Description	Part
Axilla Pillow	SSAXP
External Rotation Pillow	SSE

## SHOULDER BRACE WITH AIRPLANE KIT

The Shoulder Brace with Airplane Kit offers multiple configurations and positioning options for various injuries. Capable of both gunslinger and airplane positions with a universally sized brace, this is the go-to option for shoulder stabilization.

### INDICATIONS FOR USE

- For non-surgical or post-operative support and immobilization of the injured shoulder when varying degrees of internal rotation, external rotation, and abduction are required.

### FEATURES AND BENEFITS

- Universal size and side
- True Airplane or Gunslinger alignment
- Adjustable abduction or external rotation angles
- Buckle closures ease donning and doffing



### ORDERING INFORMATION

Description	Part #
Shoulder Brace with Airplane Kit, Black	OSAK3



## REBOUND® POST-OP ELBOW

The Rebound® Post-Op Elbow brace by Össur combines intuitive features for fitters and users with a design that aims to improve the post-op experience. The industry's only elbow immobilizer that combines a drop-lock hinge and four telescoping paddles, individual post-op protocols are easily catered to. The Rebound Post-Op Elbow, lightest of its kind, promotes user comfort with straps that stay in place, anatomical left/right fit with moldable stability cuffs, and blue patient touch-points for easy education.

### INDICATIONS FOR USE

Indications requiring elbow immobilization and/or range of motion restriction. This may include:

- Stable fractures of the elbow, distal humerus, proximal radius or ulna
- Tendon and ligament injuries or repairs
- Collateral ligament reconstructions
- Elbow hyperextension
- Triceps tendon repair
- Chronic elbow injuries
- Tennis elbow release

### FEATURES AND BENEFITS

- Designed to be the most lightweight brace on the market.
- Telescoping paddles relocate to individualize length and keep incision sites clear.
- Easy education for users with intuitive patient touch-points, including Blue is You™ buckles and numbered strapping system.
- Drop-Lock hinge enables secure range of motion with bump-resistant pull adjustment stops and optional ROM lock-outs.
- Moldable stability cuffs designed to be individually customized for improved fit and support.
- Strap guides maintain strap location for added security and comfort.
- Universal fit for both left/right elbows.

### REQUIRED MEASUREMENTS

- Universal, one size



## REBOUND® POST-OP ELBOW

### ORDERING INFORMATION

Side	Part#
Left	B-8391680
Right	B-8391580

### ACCESSORIES

#### ACCESSORY KITS

Description	Part#
Elbow Liner Kit	B-8390002
Elbow Wrist Control Kit	B-8390003
Elbow Sling Kit	B-8390004
Elbow ROM Locks Kit	B-8390005
Strap Tab Kit	B-8390006
Adhesive Hook Sticker Kit	B-8390007
Anti-Migration Pad Kit	B-8390008
Elbow Strap Kit	B-8390009

## SHOULDER ABDUCTION SLING


### INDICATIONS FOR USE

For shoulder conditions that may benefit from shoulder immobilization and/or abduction, such as:

- Pre or post-operative
- Soft tissue repairs/strains
- Shoulder instabilities/injuries

### FEATURES AND BENEFITS

- Provides shoulder abduction at 10° with support
- Cushion rests on waist to support weight of arm
- Breathable cotton provides a comfortable fit
- Fits left and right

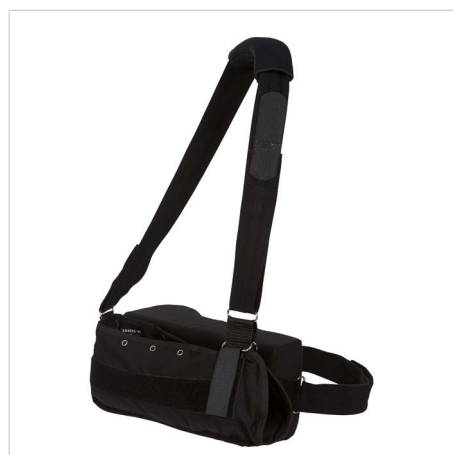

### REQUIRED MEASUREMENTS

- Measure elbow crease to fingertip

### ORDERING INFORMATION

Size	Elbow crease to fingertip	Part#
Small	up to 12.5" (< 32 cm)	204053
Medium	12.5-15.5" (32 – 39 cm)	204055
Large	15.5" and up (39 cm +)	204057

## PREMIUM SHOULDER IMMOBILIZER

### INDICATIONS FOR USE

For shoulder conditions that may benefit from immobilizing the shoulder, such as:

- Pre or post-operative
- Soft tissue repairs/strains
- Shoulder instabilities/injuries

### FEATURES AND BENEFITS

- Comfortable fit
- Easily adjusts
- One size fits all (max circumference 45")



### ORDERING INFORMATION

Description	Part#
Universal	204100

## PADDED STRAP SHOULDER IMMOBILIZER

### INDICATIONS FOR USE

For shoulder conditions that may benefit from immobilizing the shoulder, such as:

- Pre or post-operative
- Soft tissue repairs/strains
- Shoulder instabilities/injuries

### FEATURES AND BENEFITS

- Padded pressure sensitive straps improve comfort
- Waist strap is easily adjusted or can be removed
- Deep pocket design
- Two adjustment points for each strap allows for better fit
- Hook and loop closure make it easy to adjust



### ORDERING INFORMATION

Size	Length	Height	U/M	Part#
X-Small	11" (28 cm)	7.5" (44 cm)	1 ea	204120
Small	13.5" (34 cm)	7.5" (44 cm)	1 ea	204122
Medium	15.5" (39 cm)	7.5" (44 cm)	1 ea	204123
Large	17.5" (44 cm)	7.5" (44 cm)	1 ea	204125
X-Large	20" (51 cm)	7.5" (44 cm)	1 ea	204127

## PREMIUM CONTACT CLOSURE ARM SLING

### INDICATIONS FOR USE

For arm conditions that may benefit from supporting the arm, such as:

- Post trauma and pre or post-operative
- To support a cast or a splint

### FEATURES AND BENEFITS

- Envelope style with thumb support
- Individually boxed
- Includes double "D" ring and slide buckle adjustment
- Canvas Material



### ORDERING INFORMATION

Size	Length	Depth	Strap length	Standard	With Comfort Pad
X-Small	12" (30 cm)	4" (10 cm)	28" (71 cm)	204112	P204112
Small	14" (35 cm)	5" (13 cm)	30" (76 cm)	204113	P204113
Medium	17" (43 cm)	6.75" (17 cm)	35" (89 cm)	204115	P204115
Large	19" (48 cm)	6.75" (17 cm)	38" (97 cm)	204117	P204117
X-Large	19.5" (50 cm)	8" (20 cm)	43" (109 cm)	204118	P204118

## BUCKLE CLOSURE ARM SLING

### INDICATIONS FOR USE

For arm conditions that may benefit from supporting the arm, such as:

- Post trauma and pre or post-operative
- To support a cast or a splint

### FEATURES AND BENEFITS

- Contact closure shoulder strap with clip buckle swathe
- Available in 6 pack only



### ORDERING INFORMATION

Size	Length	Depth	Strap length	Part#
Child	10.5" (27 cm)	5" (13 cm)	24" (61 cm)	204020
X-Small	12.25" (31 cm)	4" (10 cm)	32" (81 cm)	204022
Small	14.5" (37 cm)	5.25" (13 cm)	35" (89 cm)	204023
Medium	17.5" (45 cm)	6.5" (17 cm)	39.5" (100 cm)	204025
Large	19" (48 cm)	7.5" (19 cm)	43.5" (110 cm)	204027

## BERGSCHULTZ UNIVERSAL ARM SLING

### INDICATIONS FOR USE

For arm conditions that may benefit from supporting the arm, such as:

- Post trauma and pre or post-operative
- To support a cast or a splint

### FEATURES AND BENEFITS

- Designed by Jim Bergschultz. R.O.T.
- Blue heavy denim material with contact closure

### REQUIRED MEASUREMENTS

- Universal, one size



### ORDERING INFORMATION

Description	Part#
Bergschulz Universal Arm Sling	204070

## PREMIUM PADDED ARM SLING

### INDICATIONS FOR USE

- Post-trauma
- Post-surgery

### FEATURES AND BENEFITS

- Envelope style
- Canvas Material



### ORDERING INFORMATION

Size	Length	Depth	Strap length	Part#
Small	13" (33 cm)	7" (18 cm)	31" (79 cm)	204313
Medium	16" (41 cm)	8" (20 cm)	35" (89 cm)	204315
Large	19" (48 cm)	9.5" (24 cm)	36" (91 cm)	204317
X-Large	22" (56 cm)	9.5" (24 cm)	40" (102 cm)	204318

## FRONT CLOSURE CLAVICLE SUPPORT

### INDICATIONS FOR USE

- Clavicle fractures and dislocations
- Postural support
- Soft tissue injuries

### FEATURES AND BENEFITS

- Features front closure design for convenience in application
- Comfortable padding over clavicle area

### REQUIRED MEASUREMENTS

- Chest circumference at the sternum area



### ORDERING INFORMATION

Size	Circumference	Part#
X-Small	20 – 24" (51 – 61 cm)	205032
Small	24 – 30" (61 – 76 cm)	205033
Medium	30 – 36" (76 – 91 cm)	205035
Large	36 – 42" (91 – 107 cm)	205037
X-Large	42 – 48" (107 – 122 cm)	205038
XX-Large	48 – 54" (122 – 137 cm)	205039



## FIGURE-8 CLAVICLE SPLINT

### INDICATIONS FOR USE

- Clavicle fractures and dislocations
- Postural support
- Soft tissue injuries

### FEATURES AND BENEFITS

- Sliding buckle offers safe and secure fit

### REQUIRED MEASUREMENTS

- Chest circumference at the sternum area



### ORDERING INFORMATION

Size	Circumference	Part#
X-Small	20 – 24" (51 – 61 cm)	205002
Small	24 – 30" (61 – 76 cm)	205003
Medium	30 – 36" (76 – 91 cm)	205005
Large	36 – 42" (91 – 107 cm)	205007
X-Large	42 – 48" (107 – 122 cm)	205008

## SLING AND SWATHE

### INDICATIONS FOR USE

- Shoulder injuries
- Post-operative

### FEATURES AND BENEFITS

- Comfortable position for patients having the arm next to the body
- One size fits all



### ORDERING INFORMATION

Description	Part#
Sling and Swathe	13074

## PEDIATRIC UNIVERSAL ARM SLING

### INDICATIONS FOR USE

- Post upper limb trauma
- Post upper limb surgery

### FEATURES AND BENEFITS

- Prong buckle closure correct fit without slippage

### REQUIRED MEASUREMENTS

- Universal, one size



### ORDERING INFORMATION

Description	Part#
Pediatric Universal Arm Sling Packed in boxes of 6	204020

## TENNIS ELBOW SUPPORT WITH HOT/COLD GEL THERAPY

### INDICATIONS FOR USE

- Epicondylitis

### FEATURES AND BENEFITS

- Universal 2" / 5 cm wide with loop-lock Velcro closure
- Loop-lock contact closure
- Twin blue cushions provide counter force, reducing traction on the tendons and inflammation

### REQUIRED MEASUREMENTS

- Circumference of the elbow



### ORDERING INFORMATION

Description	Size	Part#
Tennis Elbow Support with hot/cold gel therapy	Universal (fits elbow circumference 8-14"/20 - 36 cm)	E310000

## FORMFIT® TENNIS ELBOW

### INDICATIONS FOR USE

- Epicondylitis

### FEATURES AND BENEFITS

- Patented ergonomically-shaped plastic eliminates tourniquet effect associated with traditional designs.
- Twin blue cushions provide counterforce, reducing traction on the tendons and thus inflammation.
- Ribbed surfaces reduce vibration to the tendon, increasing comfort.
- Durable design and materials enable everyday use for both work and athletic activities.



### REQUIRED MEASUREMENTS

- Circumference of the elbow

### ORDERING INFORMATION

Size	Circumference	Part#
Small	8.7-10.4" (22-26 cm)	B-247000002
Medium	10.4-12.8" (27-32 cm)	B-247000003
Large	12.8-15" (33-38 cm)	B-247000004

## AIRFORM® TENNIS ELBOW SUPPORT

### INDICATIONS FOR USE

- Epicondylitis

### FEATURES AND BENEFITS

- Focused pneumatic compression
- Conforms and comfortably cushions sensitive injured anatomy
- Easy to apply, simple design that fits most, reduces inventory needs

### REQUIRED MEASUREMENTS

- Universal: Fits elbow circumference 20 – 36 cm / 8-14"



### ORDERING INFORMATION

Description	Size	Part#
AirForm Tennis Elbow Support	Universal (fits elbow circumference 8-14" / 20 - 36 cm)	320000

## FORMFIT® PRO WRIST

Össur Formfit Pro Wrist is constructed using our superior signature 3D MotionTech knit, and ideal for injuries that may benefit from compression. Malleable aluminum stays can be adjusted to conform to an individual's anatomy. CoolVent™ technology in the basal thumb area enhances breathability and comfort.



MotionTech™



CoolVent™



StayBilizer™



FoldLock™



### INDICATIONS FOR USE

For wrist conditions that may benefit from compression, such as:

- Rheumatic Arthritis (RA) and other chronic pain conditions
- Wrist sprain and strains
- Overuse and pain conditions
- Wrist tendinitis


### CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

### FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit has been proven to be superior to the leading brands at moisture wicking and breathability, keeping patients cool during wear.
- Malleable aluminum stay for a customized fit.
- CoolVent™ technology in thumb base area for extra breathability and comfort.

### REQUIRED MEASUREMENTS

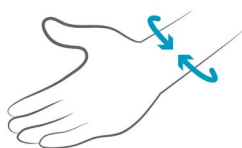

Measure wrist circumference.



Size	Wrist circumference		Blue		Black	
	cm	inch	Left Part#	Right Part#	Left Part#	Right Part#
XS	14 - 15	5½ - 6	FP401201	FP401101	FP400201	FP400101
S	15 - 16	6 - 6¼	FP401202	FP401102	FP400202	FP400102
M	16 - 17	6¼ - 6¾	FP401203	FP401103	FP400203	FP400103
L	17 - 18	6¾ - 7	FP401204	FP401104	FP400204	FP400104
XL	18 - 19	7 - 7½	FP401205	FP401105	FP400205	FP400105
XXL	19 - 20	7½ - 7¾	FP401206	FP401106	FP400206	FP400106

## EXOFORM® WRIST

### INDICATIONS FOR USE

- Mild to Moderate sprains and strains
- Tendonitis
- Post-cast healing
- Other select soft tissue injuries

### FEATURES AND BENEFITS

- Ergonomically contoured padding
- Rigid plastic supports contour to the hand and wrist
- Adjustable aluminum palmar stay allows for fine-tuning when needed

### REQUIRED MEASUREMENTS

- Wrist circumference



### ORDERING INFORMATION

Size	Wrist Circumference	Left	Right
X-Small	5.25 – 6.25" (13 – 16 cm)	507082	507072
Small	6.25 – 7.25" (16 – 18.5 cm)	507083	507073
Medium	7.25 – 8" (18.5 – 20.5 cm)	507085	507075
Large	8 – 9" (20.5 – 23.5 cm)	507087	507077
X-Large	9 – 10" (23.5 – 25.5 cm)	507088	507078

## EXOFORM® CARPAL TUNNEL WRIST

### INDICATIONS FOR USE

- Carpal tunnel syndrome
- Tendonitis

### FEATURES AND BENEFITS

- Allows for full MCP flexion
- Contoured for anatomical fit
- Provides radial and ulnar styloid relief
- Low profile
- Preshaped into a neutral position for comfortable limited motion/ flexibility

### REQUIRED MEASUREMENTS

- Wrist and palm circumference



### ORDERING INFORMATION

Size	Wrist Circumference	Palm Circumference	Left	Right
Small	5.25 – 6.5" (13 – 16.5 cm)	6.75 – 8" (17 – 20.5 cm)	517083	517073
Medium	6.5 – 7.75" (16.5 – 19 cm)	8 – 9.25" (20.5 – 23.5 cm)	517085	517075
Large	7.75 – 10" (19 – 25.5 cm)	9.25 – 10.5" (23.5 – 26 cm)	517087	517077

## FORMFIT® WRIST / FORMFIT® WRIST & FOREARM

Formfit Wrist, and Formfit Wrist & Forearm, provides immobilization and protection for patients suffering from tendonitis, select soft tissue injuries – including mild to moderate sprains and strains, as well as for post-cast healing. The lightweight, breathable material and contour fit make this comfortable product an excellent choice.

### INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Tendonitis
- Post-cast healing
- Other selected soft tissue injuries

### FEATURES AND BENEFITS

- Breathable material
- Contour fit
- Contact Closure Straps
- Customizable Stays

### REQUIRED MEASUREMENTS

- Wrist circumference



## ORDERING INFORMATION

### FORMFIT WRIST & FOREARM

Size	Wrist Circumference	8"		10"	
		Left	Right	Left	Right
XSmall	5.25 – 6.25" (13 – 16 cm)	317082	317072	617082	617072
Small	6.25 – 7.25" (16 – 18.5 cm)	317083	317073	617083	617073
Medium	7.25 – 8" (18.5 – 20.5 cm)	317085	317075	617085	617075
Large	8 – 9" (20.5 – 23.5 cm)	317087	317077	617087	617077
XLarge	9 – 10 (23.5 – 25.5 cm)	317088	317078	617088	617078

## ACCESSORIES

### GEL PAD FOR HOT/COLD THERAPY

Description	Left	Right
Formfit Wrist (8" version)*	GB20L	GB20R
Formfit Wrist & Forearm (10" version)	GB22L	GB22R



\* Fits XSmall/Small. For Medium-XLarge, please use Form Forearm 10" part numbers



# 6" WRIST

INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Tendonitis
- Post-cast healing
- Other selected soft tissue injuries

FEATURES AND BENEFITS

- Contact closure straps
- Contoured fit
- Malleable stays

REQUIRED MEASUREMENTS

- Wrist circumference



ORDERING INFORMATION

Size	Wrist Circumference	Left	Right
Small	6.25 – 7.25" (16 – 18.5 cm)	417083C	417073C
Medium	7.25 – 8" (18.5 – 20.5 cm)	417085C	417075C
Large	8 – 9" (20.5 – 23.5 cm)	417087C	417077C
XLarge	9 – 10 (23.5 – 25.5 cm)	417088C	417078C

## FORMFIT® WRIST & THUMB, UNIVERSAL

Formfit Universal Wrist and Thumb products are designed to restrict motion of the wrist and the thumb. Providing immobilization and protection from select soft tissue injuries, mild to moderate strains and sprains, including gamekeeper's thumb and post cast healing. The lightweight, universal design also reduces inventory needs.

### INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Tendonitis
- Post-cast healing
- Other selected soft tissue injuries

### FEATURES AND BENEFITS

- Quick-Pull close strap
- Positional dorsal stay
- Customizable thumb stay
- Malleable palmar stay

### REQUIRED MEASUREMENTS

- Wrist circumference



## ORDERING INFORMATION

### FORMFIT WRIST

Size	Wrist Circumference	Left	Right
Pediatric	3.5 – 5" (9 – 13 cm)	B-252603301	B-252603300
8" Universal	4.5 – 9.5" (11 – 24 cm)	B-252603109	B-252503109
8" X-Large	7.5 – 11" (19 – 28 cm)	B-252603106	B-252603105

### FORMFIT WRIST & FOREARM UNIVERSAL

Size	Wrist Circumference	Left	Right
10" Universal	4.5 – 9.5" (11 – 24 cm)	B-252603210	B-252603209

### FORMFIT THUMB SPICA UNIVERSAL

Size	Wrist Circumference	Left	Right
Pediatric	3.5 – 5" (9 – 13 cm)	B-253603301	B-253603300
8" Universal	4.5 – 9.5" (11 – 24 cm)	B-253603109	B-253503109
8" X-Large	7.5 – 11" (19 – 28 cm)	B-253603106	B-253603105

# FORMFIT® THUMB SPICA

The Formfit® Thumb Spica is designed to restrict wrist and thumb motion. It provides immobilization and protection for patients suffering from tendonitis (e.g. DeQuervain's Tendonitis), select soft tissue injuries – including mild to moderate sprains and strains and Gamekeeper's Thumb –, as well as for post-cast healing. The lightweight, breathable material and contour fit make this comfortable product an excellent choice.

## INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Tendonitis
  - e.g., DeQuervain's Tendonitis
- Post-cast healing
- Other selected soft tissue injuries
  - e.g., Gamekeeper's Thumb

## FEATURES AND BENEFITS

- Breathable material
- Contour fit
- Contact Closure Straps
- Customizable Stays

## REQUIRED MEASUREMENTS

- Wrist circumference



## ORDERING INFORMATION

Size	Wrist Circumference	Left	Right
XSmall	5 – 6" (13 – 16 cm)	3020	3120
Small	6 – 7" (16 – 18.5 cm)	3030	3130
Medium	7 – 8" (18.5 – 20.5 cm)	3050	3150
Large	8 – 9" (20.5 – 23.5 cm)	3070	3170
XLarge	9 – 10" (23.5 – 25.5 cm)	3080	3180

## ACCESSORIES

### GEL PAD FOR HOT/COLD THERAPY

Description	Left	Right
Formfit Wrist (8" version)*	GB20L	GB20R
Formfit Wrist & Forearm (10" version)	GB22L	GB22R



\* Fits XSmall/Small. For Medium-Xlarge, please use Form Forearm 10" part numbers

# 6" THUMB

INDICATIONS FOR USE

- Gamekeeper’s thumb
- Torn collateral ligaments
- DeQuervain Tendonitis

FEATURES AND BENEFITS

- Contact closure straps
- Contoured fit
- Malleable stays

REQUIRED MEASUREMENTS

- Wrist circumference



## ORDERING INFORMATION

Size	Wrist Circumference	Left	Right
Small	6.25 – 7.25" (16 – 18.5 cm)	30304C	31304C
Medium	7.25 – 8" (18.5 – 20.5 cm)	30504C	31504C
Large	8 – 9" (20.5 – 23.5 cm)	30704C	31704C
XLarge	9 – 10 (23.5 – 25.5 cm)	30804C	31804C

# UNIVERSAL THUMB SPLINT

## INDICATIONS FOR USE

- Gamekeeper's thumb
- Torn collateral ligaments
- DeQuervain Tendonitis

## FEATURES AND BENEFITS

- Elastic contact closures
- Restricts movement of the MP/CMC
- Light weight and breathable

## REQUIRED MEASUREMENTS

- Universal, one size



## ORDERING INFORMATION

Description	Part#
Universal Thumb Splint	30000





ÖSSUR®  
LIFE WITHOUT LIMITATIONS

HIP





## REBOUND® HIP

Össur's Rebound Hip brace has a low-profile, lightweight design that supports patient comfort and compliance. It delivers gentle controlling forces and effective functional ROM restriction in extension/flexion and abduction/adduction to promote optimal post-surgical outcomes. The universal frame and softgood are easy to stock, assemble and quickly fit on patients.



### INDICATIONS FOR USE

Hip conditions that could benefit from motion restriction following surgery:

- Femoral Acetabular Impingement arthroscopic surgery
- Gluteus medius repair surgery
- Hamstring repair surgery

### FEATURES AND BENEFITS

- Low-profile, lightweight design that supports patient comfort and compliance
- Quick-release pegs easily adjust the desired range-of-motion needed for the patient's optimal rehab
- Quick-fit with sliding mechanism on frame to adjust for patient's height
- Optional, adjustable ROM hinge from -10°, 0° 10°, or 20°

### REQUIRED MEASUREMENTS

- Circumference at iliac crest height
- Length from greater trochanter to mid patella

### ORDERING INFORMATION

#### REBOUND HIP

Description	Preset Left	Preset Right
Rebound Hip	980130RB	980135RB
Rebound Hip with adjustable hinge	980120RB	980125RB

### ACCESSORIES

Description	Part#
OR Kit	980301
Belt Extender (Fitting 49-67" hip circumference)	980121RB
Fitting Tool	B-040100000
Rotation Strap	RHK-100024





KNEE





## REBOUND® CARTILAGE

The success of an articular cartilage repair is dependent on patient compliance to the rehabilitation process. The protocol is based on restricted cartilage load combined with gradual increase in activity. Many rehabilitation protocols suggest crutches for partial weight-bearing and it is the standard of care for successful outcomes. However, patient compliance to partial weight-bearing instructions with crutches has been shown to be inconsistent. The Rebound Cartilage is a protective functional solution designed to support regeneration of the knee cartilage after repair procedures such as Microfracture, OATS and (M)ACI. Based on the clinically-proven 3-Point Leverage System and patented Cartilage Protection Straps, joint unloading is maintained in flexion for protection of the cartilage during the healing process.



### INDICATIONS FOR USE

For knee conditions that may benefit from unicompartmental load reduction or range of movement restriction, such as:

- Articular cartilage defect repair
- Meniscal repair
- Avascular necrosis
- Condylar bone marrow lesions (bone bruises)
- Tibial plateau fractures
- Mild to severe unicompartmental knee osteoarthritis

### FEATURES AND BENEFITS

- Easy-to-Stock - Just 4 configurations to fit most of your patients
- Comfortable - Skin-friendly liner feels great against the skin and reduces pressure points.
- Anti-Migration Technology - Strap design captures the thickest part of the calf to reduce brace migration during activity.
- Control - SmartDosing® dials put you in control of your pain by allowing you to fine-tune strap tension, while Cartilage Protection Straps maintain joint unloading in flexion.
- Tool-Free - Tool-free polycentric hinge with optional ROM stops for range-of-motion restriction.

### REQUIRED MEASUREMENTS

- Circumference measurement: 6" (15 cm) below mid-patella

## REBOUND® CARTILAGE

### ORDERING INFORMATION

Rebound Cartilage	Left medial	Right medial	Left lateral	Right lateral
Universal 31 - 61.5 cm (12.2 - 24")	B-254610006	B-254510006	B-254620006	B-254520006

Rebound Cartilage - No Box	Left medial	Right medial	Left lateral	Right lateral
Universal 31 - 61.5 cm (12.2 - 24")	B-254610007	B-254510007	B-254620007	B-254520007

Rebound Cartilage - Custom	Left	Right
Calf circumference 25 - 61.5 cm (9.9 - 24")	B-154600001	B-154500001

### SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilizes a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required

#### CUSTOM STANDARD COLORS

Red	Pink	Silver
Orange	Black	White
Yellow	Navy Blue	Champagne
Lime Green	Ocean Blue	Charcoal

#### CUSTOM PAINTED DESIGNS



Additional fees apply for custom colors, custom painted designs, and custom printed graphics.

### ACCESSORIES

Accessory Parts	Part#
Suspension Strap Kit	UNL00211
OA Hinge Extension Stop Tree (5°, 10°, 15°, 30°)	OAH40029
OA Hinge Flexion Stop Tree (0°, 30°, 60°, 90°)	OAH40039
Wash Bag	UON60200

## REBOUND® CARTILAGE

Sleeves	Black Ossur One*	Black - Sleeve with Mesh**	Black - Ossur Knee Undersleeve***
XXS	OSO60021	NA	NA
XS	OSO60022	SLV60011	B-250150101
S	OSO60023	SLV60012	B-250150102
M	OSO60024	SLV60013	B-250150103
L	OSO60025	SLV60014	B-250150104
XL	OSO60026	SLV60015	B-250150105
XXL	OSO60027	SLV60016	B-250150106
3XL	OSO60028	NA	NA

\*Ossur One - 51% Tencel/30% Polyamide/19% Elastane

\*\*Sleeve with Mesh - Main fabric: 92% Cotton/8% Spandex - Mesh: 100% Polyester - Band: 80% Polyester/20% Spandex

\*\*\*Ossur Knee Undersleeve: 80% Nylon/20% Spandex

## REPLACEMENT PARTS

Service Kit*	Right medial/Left lateral	Right lateral/Left medial
Black, Service Kit	CRT416901	CRT415901

\*Includes Jersey knit thigh liner, Sensil calf liner, Popliteal pad standard, Hinge pad & Strap Kit Complete

Thigh Liners	Right medial/Left lateral	Right lateral/Left medial
Black, Jersey Knit	CRT526901	CRT525901

Calf Liners	Right medial/Left lateral	Right lateral/Left medial
Black, Sensil	CRT516952	CRT515952
Black, Jersey Knit	CRT516951	CRT515951

Strap Kit Complete*	Part #
Black, Strap Kit Complete	B-701290005

\*Includes 4 straps (1 Thigh, 1 Calf, 2 CPS), 5 Alligator tabs, 2 CPS Strap pads

CPS Strap Kit*	Part #
Black, CPS Kit	B-700290005

\*Includes 2 CPS Straps, 2 Alligator tabs, 2 CPS Strap pads

Buckle Clasp Kit	Right medial/Left lateral	Right lateral/Left medial
Black	B-708217001	B-708218001

Popliteal Pad	Part #
Black, Short	B-304010105
Black, Standard	B-304010106

Other Replacement Parts	Part #
Velcro Twist-Fit Attachment Kit	B-705290000
Hinge Pad	B-304120101
Alligator Kit-8 Pieces	B-704210001

## REBOUND® ACL

The Rebound ACL brace is an evolution in bracing, designed to comfortably apply an anatomically correct, dynamic load to the anterior cruciate ligament (ACL). Whether during conservative or surgical management, the Rebound ACL compliments protocols and supports an optimal biomechanical joint environment as a foundation for long-term successful patient outcomes. Innovative technology in the Dynamic Tension System® (DTS) allows individualized, adjustable settings to reduce strain on the healing ACL.

FUNCTIONAL HEALING – WHERE MOBILITY MEANS RECOVERY



## INDICATIONS FOR USE

### INDICATIONS

Conditions benefiting from an agonistic force that varies throughout flexion in proportion to ACL tension applied to the thigh and calf in the sagittal plane. These may include:

- Anterior cruciate ligament (ACL) injury
- Reconstruction
- Non-surgical treatment.

### CONTRAINDICATIONS

None known

### INTENDED USE

Post-injury, post-surgery, and conservative rehabilitation.

### TARGET GROUP


Patients requiring a dynamic knee orthosis who have suffered an ACL or combined instability, and are being treated post-injury with conservative or post-surgical management.

## FEATURES AND BENEFITS


- Dynamic technology for knee stability, with a Dynamic Tension System™ that provides an agonistic, dynamic force to the ACL that is flexion-angle dependent (LaPrade et al., 2017) and designed to reduce anterior-drawer, promoting knee stability.
- Specific load adaptation according to the patient's individual anatomy and rehabilitation requirements
- Supported by clinical & biomechanical studies, the Rebound ACL is shown to reduce strain on the ACL during activity and significantly reduce strain on the meniscus in ACL-deficient knees (Tomescu et al., 2017).
- Biocompatible, hi-tech materials including Blue is You™ patient touch points & ActiveGrip™ liners promote suspension & comfort
- Lightweight, aircraft aluminum frame
- Available in sizes SM-XL
- Flexion & extension range-of-motion lock-outs
- Posterior frame optimized for fit and support, with open anterior for ease of donning/doffing
- Clinically-tested comfort, validated by end-users and with load mapping data collected during rehabilitation activities, the Rebound® Dynamic Tension System (DTS) provides a comfortable load to the posterior thigh & anterior tibia.

# REBOUND® ACL

## REQUIRED MEASUREMENTS

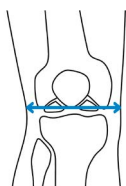

Measure knee caliber medio-laterally at joint width in standing, weight-bearing position.

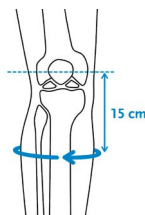

Measure calf circumference 15 cm (6") below mid-patella. Leg in extension.



Measure thigh circumference 15 cm (6") above mid-patella. Leg in extension.

## ORDERING INFORMATION

### STANDARD

Size	Left	Right	Caliper	Calf circumference 6" (15 cm) below mid patella	Thigh circumference 6" (15 cm) above mid patella
SM	B-124601191	B-124501191	3.5-4" (8.9-10.2 cm)	11.5-13.5" (29.2-34.3 cm)	16.5-18.5" (41.9-47.0 cm)
MD	B-124601192	B-124501192	4-4.5" (10.2-11.4 cm)	13.5-15.5" (34.3-39.4 cm)	18.5-20.5" (47.0-52.1 cm)
LG	B-124601193	B-124501193	4.5-5.0" (11.4-12.7 cm)	15.5-17.5" (39.4-44.5 cm)	20.5-22.5" (52.1-57.2 cm)
XL	B-124601194	B-124501194	5.0-5.5" (12.7-14.0 cm)	17.5-19.5" (44.5-49.5 cm)	22.5-24.5" (57.2-62.2 cm)

Recommended for patients 5'1" (1.56m) and taller

### REPLACEMENT PARTS

Description	Part#
Extension stop kit (0°, 5°, 10°, 20°, 30°, 40°)	B-124702035
Flexion stop kit (0°, 15°, 25°, 35°, 45°, 55°, 65°, 75°, 90°, 105°)	B-124702034
Shear Knob Kit	B-705130051
SM/MD Strap Kit	B-124702030
LG/XL Strap Kit	B-124702031
Condyle Pad Kit	B-701090009
Tibial Pad	B-124702032
Tibial Hex Pad	B-124702033

# REBOUND® ACL

## ACCESSORIES

### ANTI-MIGRATION WRAP

SIZE	RIGHT	LEFT
S/M	B-124501195	B-124601195
S/M HEX	B-124501196	B-124601196
L/XL	B-124501197	B-124601197
L/XL HEX	B-124501198	B-124601198

### UPPER DOESKIN LINER KIT

SIZE	RIGHT	LEFT
S	B-124702004	B-124702000
M	B-124702005	B-124702001
L	B-124702006	B-124702002
XL	B-124702007	B-124702003

### LOWER ACTIVEGRIP LINER KIT

SIZE	RIGHT	LEFT
S	B-124702014	B-124702010
M	B-124702015	B-124702011
L	B-124702016	B-124702012
XL	B-124702017	B-124702013

### LOWER DOESKIN LINER KIT

SIZE	RIGHT	LEFT
S	B-124702024	B-124702020
M	B-124702025	B-124702021
L	B-124702026	B-124702022
XL	B-124702027	B-124702023



## REBOUND® ACL CUSTOM

The Rebound ACL brace is an evolution in bracing, designed to comfortably apply an anatomically correct, dynamic load to the anterior cruciate ligament (ACL). Whether during conservative or surgical management, the Rebound ACL compliments protocols and supports an optimal biomechanical joint environment as a foundation for long-term successful patient outcomes. Innovative technology in the Rebound® Dynamic Tension System (DTS) allows individualized, adjustable settings to reduce strain on the healing ACL.



### FUNCTIONAL HEALING – WHERE MOBILITY MEANS RECOVERY

## INDICATIONS FOR USE

### INDICATIONS

Conditions benefiting from an agonistic force that varies throughout flexion in proportion to ACL tension applied to the thigh and calf in the sagittal plane. These may include:

- Anterior cruciate ligament (ACL) injury
- Reconstruction
- Non-surgical treatment

### CONTRAINDICATIONS

None known

### INTENDED USE

Post-injury, post-surgery, and conservative rehabilitation.

### TARGET GROUP

Patients requiring a dynamic knee orthosis who have suffered an ACL or combined instability, and are being treated post-injury with conservative or post-surgical management.

## FEATURES AND BENEFITS

- Dynamic technology for knee stability, with a Dynamic Tension System™ that provides an agonistic, dynamic force to the ACL that is flexion-angle dependent (LaPrade et al., 2017) and designed to reduce anterior-drawer, promoting knee stability.
- Specific load adaptation according to the patient's individual anatomy and rehabilitation requirements
- Supported by clinical & biomechanical studies, the Rebound ACL is shown to reduce strain on the ACL during activity and significantly reduce strain on the meniscus in ACL-deficient knees (Tomescu et al., 2017).
- Biocompatible, hi-tech materials including Blue is You™ patient touch points & ActiveGrip™ liners promote suspension & comfort
- Lightweight, aircraft aluminum frame
- Available in sizes SM-XL & Custom
- Flexion & extension range-of-motion lock-outs
- Posterior frame optimized for fit and support, with open anterior for ease of donning/doffing
- Clinically-tested comfort, validated by end-users and with load mapping data collected during rehabilitation activities, the Rebound® Dynamic Tension System (DTS) provides a comfortable load to the posterior thigh & anterior tibia.

# REBOUND® ACL CUSTOM



## SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilizes a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.

Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required.

## REQUIRED MEASUREMENTS

- Calf circumference 15 cm (6") below mid patella
- Calf circumference 20 cm (8") below mid patella
- Calf circumference 7.5 cm (3") below mid patella
- Knee caliper
- Thigh circumference 15 cm (6") above mid patella
- Thigh circumference 20 cm (8") above mid patella
- Thigh circumference 7.5 cm (3") above mid patella

## ORDERING INFORMATION

### CUSTOM

Part #	Type	Min / Max Caliper	Min / Max Circ. above	Min / Max Circ. below	Measuring
B-124701000	Right	3.5 – 5.5 in	16.5 – 24.5 in	11.5 – 19.5 in	SmartMeasure, Cast
B-124701001	Left	(8.9 – 14 cm)	(41.9 - 62.2 cm )	(29.2-49.5 cm)	

Caliper = M/L width at knee center

Circumference = 6" (15 cm) above / below mid-patella

Ideal for patients 5'1" (1.56m) and taller

### REPLACEMENT PARTS

Description	Part#
Extension stop kit (0°, 5°, 10°, 20°, 30°, 40°)	B-124702035
Flexion stop kit (0°, 15°, 25°, 35°, 45°, 55°, 65°, 75°, 90°, 105°)	B-124702034
Shear Knob Kit	B-705130051
SM/MD Strap Kit	B-124702030
LG/XL Strap Kit	B-124702031
Condyle Pad Kit	B-701090009
Tibial Pad	B-124702032
Tibial Hex Pad	B-124702033

# REBOUND® ACL CUSTOM

## ACCESSORIES

### ANTI-MIGRATION WRAP

Size	Right	Left
S/M	B-124501195	B-124601195
S/M HEX	B-124501196	B-124601196
L/XL	B-124501197	B-124601197
L/XL HEX	B-124501198	B-124601198

### UPPER DOESKIN LINER KIT

Size	Right	Left
S	B-124702004	B-124702000
M	B-124702005	B-124702001
L	B-124702006	B-124702002
XL	B-124702007	B-124702003

### LOWER ACTIVEGRIP LINER KIT

Size	Right	Left
S	B-124702014	B-124702010
M	B-124702015	B-124702011
L	B-124702016	B-124702012
XL	B-124702017	B-124702013

### LOWER DOESKIN LINER KIT

Size	Right	Left
S	B-124702024	B-124702020
M	B-124702025	B-124702021
L	B-124702026	B-124702022
XL	B-124702027	B-124702023

## REBOUND® PCL

The PCL, compared to the ACL, has the intrinsic ability to heal and regain continuity after an injury. The PCL experiences variable tensile forces during knee flexion. Insufficient healing after a PCL rupture leads to significant pathologies, such as, chronic anterior knee pain or knee OA. Healing of the PCL in an elongated position can lead to chronic instability and disability. Therefore, physiologically correct positioning of the PCL during conservative or post-surgical rehabilitation is important.

The custom made Rebound PCL is the world's first Dynamic Force PCL brace. It applies a physiologically correct dynamic force to the PCL during rehabilitation.

Our expertise in Injury Solutions has driven us to take the next step forward by developing indication-specific products which are designed to optimize healing while maintaining function and mobility.

### FUNCTIONAL HEALING – WHERE MOBILITY MEANS RECOVERY

#### INDICATIONS FOR USE

- Non-surgical treatment of PCL ruptures
- Post-surgical rehabilitation for:
- Partial PCL tears
- PCL reconstruction / augmentation
- PCL revision

#### FEATURES AND BENEFITS

- Dynamic Tension System (DTS) is designed to increase the load on the tibia as the knee goes through flexion, providing an anterior drawer force necessary to decrease load on the healing PCL to allow it to heal
- 3 options to customize the standardized, biochemically correct posterior tibial load for each patient by using shear knobs. And having the possibility to re-adjust the posterior tibial load, e.g. in case of swelling.
- Lightweight brace with proprietary Sensil® silicone calf liner and doeskin thigh liner reduce migration while providing comfort to the patient
- Custom molded posterior frame for optimized fit and support, with open anterior for ease of donning/doffing
- Available in 12 colors

#### REQUIRED MEASUREMENTS

- Custom-made to measure

#### STANDARD COLORS



# REBOUND® PCL

## ORDERING INFORMATION



### SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilizes a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.

Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required

### REBOUND PCL CUSTOM

Description	Part#
Rebound PCL Left	B-124501155
Rebound PCL Right	B-124501154

### REBOUND PCL PRE-FABRICATED

Size	Left	Right
S	B-124501161	B-124501158
M	B-124501162	B-124501159
L	B-124501163	B-124501160

# REBOUND® PCL

## ACCESSORIES

### STRAPS

Size	Part#
X-Small – Large	PCL00061
X-Large – XX-Large	PCL00062

### OTHER ACCESSORIES

Description	Part#
Extension Stop Kit	B-705130042
Flexion Stop Kit	B-705130043
Shear Knob Kit	B-465350087
Condyle Pad Kit	B-705130046

### REBOUND PCL STANDARD UPPER LINER KIT

Size	Left	Right
S	B-124501171	B-124501165
M	B-124501172	B-124501166
L	B-124501173	B-124501167

### REBOUND PCL STANDARD LOWER LINER KIT

Size	Left	Right
S	B-124601177	B-124501177
M	B-124601178	B-124501178
L	B-124601179	B-124501179

### REBOUND PCL DOESKIN LOWER LINER KIT

Size	Left	Right
S	B-124601183	B-124501183
M	B-124601184	B-124501184
L	B-124601185	B-124501185



## CTi® CUSTOM

Introduced in 1983, CTi remains the cornerstone of our ligament bracing product line today. While the original CTi has evolved over time, our basic philosophy remains unchanged – a static support system consisting of a rigid frame will hold the bones in proper alignment and provide high level of control to prevent displacement. The constant development and improvement of this classic design is why the CTi remains the top choice of athletes and leading physicians around the world.

The rigid, carbon composite CTi knee brace frame captures the tibia, providing the brace-to-bone contact needed for maximum stabilization of the knee joint. The non-elastic cruciate strapping combined with a rigid frame that truly captures the tibia, results in a static support system.

The CTi Custom is generally used for high levels of support where comfort and strength are not compromised with weight and bulkiness. It can also offer greater advantages for accurate fitting where leg shape abnormalities present a problem.



### INDICATIONS FOR USE

- For knee conditions that may benefit from increased AP and ML stability around the knee, such as: ACL, MCL, LCL, PCL, rotary and combined instabilities

### FEATURES AND BENEFITS

- Anatomically-correct Accutrac® hinges with extension stops
- Proprietary Sensil® silicone liners designed to reduce migration
- Total Support System™ for ACL, MCL, LCL, PCL, rotary and combined instabilities
- Pre-preg carbon fiber, rigid frame construction
- Non-corrosive materials, ideal for water sports
- Custom paint option available
- Standard model for moderate to high impact levels
- Pro Sport model for the highest impact levels
- Vapor model for low to moderate impact levels
- Available in 12 colors

### REQUIRED MEASUREMENTS

- Custom-made to measure

#### CUSTOM STANDARD COLORS



#### CUSTOM PAINTED DESIGNS



No charge for custom standard colors. Additional fees required for other custom colors, custom painted designs and custom printed graphics.

# CTI® CUSTOM

## ORDERING INFORMATION

### CTI CUSTOM

Description	Left	Right
Standard	B-116600010	B-116500010
Pro Sport	B-117600010	B-117500010
Vapor	B-118600010	B-118500010



### SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilizes a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.

Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required

## ACCESSORIES

### CALF LINERS

Description	Part#
CTi Custom liner	B-411500115

### STRAPS KITS

Size	Part#
Small, Medium, Large	B-701130104
X-Large, XX-Large, XXX-Large, XXXX-Large	B-701130108

### PCL KITS

Size	Part#
Small, Medium	B-705000132
Large, X-Large	B-705000235

# CTI® CUSTOM

## ACCESSORIES

### STRAP PADS\*

Size	Part#
Small	B-304010057
Medium, Large	B-304010058
X-Large, XX-Large	B-304010059

\*Must also order hook-and-loop B-309020017

### AMS (ANTI-MIGRATION SYSTEM) WRAPS

Description	Part#
AMS Wrap EVA	22095
AMS Wrap Neoprene SM-LG (up to 17" in circumference at 3" below knee)	22094
AMS Wrap Neoprene XL (above 17" in circumference at 3" below knee)	22152

### MX GEAR

Description	Part#
Patella Cup	B-711130011
Gear Guards	B-705000003
MX Kit S (patella cup, gear guards, undersleeve, stickers)	B-711140002
MX Kit M (patella cup, gear guards, undersleeve, stickers)	B-711140003
MX Kit L (patella cup, gear guards, undersleeve, stickers)	B-711140004
MX Kit XL (patella cup, gear guards, undersleeve, stickers)	B-711140005

### SLEEVES

Size	Circumference 15 cm above mid patella	Softsleeve	Neoprene Undersleeve	Neoprene Oversleeve	Sportsleeve	Padded Sports Oversleeve
Small	<15" (38 cm)	20643	20251	20051	20682	20448
Medium	15 – 18" (38 – 45 cm)	20644	20252	20052	20683	20448
Large	18 – 19.5" (45 – 49.5 cm)	20644	20253	20053	20684	20449
X-Large	19.5 – 21.5" (49.5 – 54.5 cm)	20645	20254	20054	20685	20449
XX-Large	21.5 – 24.5" (54.5 – 62 cm)	20645	20255	20055	N/A	
XXX-Large	24.5 – 28" (62 – 71 cm)	20646	20256	20056	N/A	
XXXX-Large	28 – 34" (71 – 86 cm)	20647	20257	20057	N/A	

### OTHER ACCESSORIES

Description	Part#
CTi D-Ring Kit Standard 2x 1" and 2x 1.5" D-rings	B-704132600
CTi D-Ring Kit 2" 2x 2" D-rings	B-704132602
CTi Condyle Pad Kit	B-704130150
Add ACL Repair kit	B-705130041
Extension Stop Kit	22130
Flexion Stop Kit (Factory Installed)	19254
Extension strap Kit (Factory Installed)	21144

## CTi®

The proven technology of the CTi knee brace is available with off-the-shelf convenience, providing adjustability and an excellent out-of-the-box fit while maintaining the level of support and protection that you expect from a CTi knee brace.

Outfitted with anatomically-correct Accutrac® hinges and flexible subshells and d-rings, the rock-solid CTi carbon frame provides a rigid exoskeleton that stabilizes the knee joint. Breathable liners coated with bio-inert Össur Sensil® Silicone ensure CTi OTS remains properly positioned on the leg, for optimal functional benefit.



## INDICATIONS FOR USE


- For knee conditions that may benefit from increased AP and ML stability around the knee, such as: ACL, MCL, LCL, PCL, rotary and combined instabilities

## FEATURES AND BENEFITS

- Anatomically-correct Accutrac hinges with extension stops to glide with the knee
- Flexible subshell and Adjustable Tibial Strap capture the tibia and allow adjustments in hinge depth for a more customized fit
- Breathable liners coated with bio-inert Össur Sensil Silicone to reduce migration and skin irritation
- Total Support System™ for ACL, MCL, LCL, PCL, rotary and combined instabilities
- Non-corrosive materials, ideal for water sports
- Available in 2 models: Standard model for moderate to high impact levels, Pro Sport model for the highest impact levels

## REQUIRED MEASUREMENTS

- Caliper measurement: Medio-laterally at joint width in a standard, weight-bearing position
- Circumference: Just below the distal border of the patella

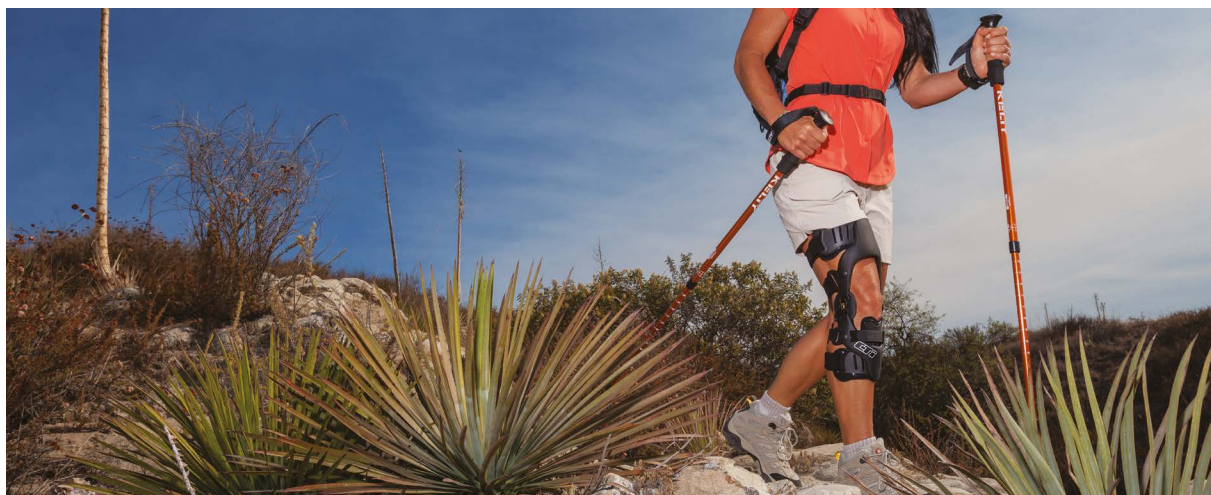

# CTI®

## ORDERING INFORMATION

### CTI

Size	Caliper	Circumference	Standard Left	Standard Right	Pro Sport Left	Pro Sport Right
Small	8.9 – 10.2 cm	28 – 34 cm	B-238600112	B-238500112	B-239600112	B-239500112
Medium	10.2 -11.4 cm	34 – 38 cm	B-238600113	B-238500113	B-239600113	B-239500113
Large	11.4 – 12.7 cm	38 – 41 cm	B-238600114	B-238500114	B-239600114	B-239500114
X-Large	12.7 – 14.0 cm	41 – 44 cm	B-238600115	B-238500115	B-239600115	B-239500115
XX-Large	14.0 – 15.2 cm	44 – 48 cm	B-238600116	B-238500116	B-239600116	B-239500116

Note: M/L measurements are the preferred and most accurate form of measurement for the CTI. Circumferential measurements are approximate.

### CTI PCL

Size	Caliper	Circumference	Standard Left	Standard Right	Pro Sport Left	Pro Sport Right
Small	8.9 – 10.2 cm	28 – 34 cm	B-238640112	B-238540112	B-239640112	B-239540112
Medium	10.2 -11.4 cm	34 – 38 cm	B-238600113	B-238540113	B-239640113	B-239540113
Large	11.4 – 12.7 cm	38 – 41 cm	B-238600114	B-238540114	B-239640114	B-239540114
X-Large	12.7 – 14.0 cm	41 – 44 cm	B-238600115	B-238540115	B-239640115	B-239540115
XX-Large	14.0 – 15.2 cm	44 – 48 cm	B-238600116	B-238540116	B-239640116	B-239540116

## ACCESSORIES

### LINERS

Size	Silicone tibia cuff	Silicone calf	Silicone thigh	Lycra thigh
Small, Medium	B-304030030	B-425580002	B-425571002	B-425570002
Large		B-425580004	B-425571004	B-425570004
X-Large, XX-Large		B-425580005	B-425571004	B-425570004

### SILICONE LINER KITS

Size	Description	Part#
Small, Medium	(tibia, calf, thigh and liners and strap pad)	B-703260012
Large	(tibia, calf, thigh and liners and strap pad)	B-703260014
X-Large, XX-Large, XXX-Large, XXXX-Large	(tibia, calf, thigh and liners and strap pad)	B-703260015

### STRAP KITS\*

Size	Part#
Small, Medium	B-701140103
Large, X-Large, XX-Large	B-701140104

\*Must also order hook-and-loop B-309020017

### PCL KITS

Size	Part#
Small, Medium	B-705000132
Large, X-Large	B-705000235

# CTI®

## ACCESSORIES

### SUBSHELLS

Size	Description	Part#
Small	(thigh and calf subshells, screws and washers)	B-713260002
Medium	(thigh and calf subshells, screws and washers)	B-713260003
Large, X-Large	(thigh and calf subshells, screws and washers)	B-713260004
XX-Large	(thigh and calf subshells, screws and washers)	B-713260006
Subshell Screw		B-301060096
Subshell Washer		FR16000

### AMS (ANTI-MIGRATION SYSTEM) WRAPS

Description	Part#
Silicone AMS Wrap Small, Medium	B-715260002
Silicone AMS Wrap Large, X-Large, XX-Large	B-715260004
Neoprene AMS Wrap Small, Medium	B-410000001
Neoprene AMS Wrap Large, X-Large	B-410000002

### MX GEAR

Description	Part#
Patella Cup	B-711130011
Gear Guards	B-705000003
MX Kit S (patella cup, gear guards, undersleeve, stickers)	B-711140002
MX Kit M (patella cup, gear guards, undersleeve, stickers)	B-711140003
MX Kit L (patella cup, gear guards, undersleeve, stickers)	B-711140004
MX Kit XL (patella cup, gear guards, undersleeve, stickers)	B-711140005

### SLEEVES

Circumference 6" (15 cm) above mid patella	Softsleeve	Neoprene Undersleeve	Neoprene Oversleeve	Sportsleeve	Padded Sports Oversleeve
<15" (38 cm)	20643	20251	20051	20682	20448
15 – 18" (38 – 45 cm)	20644	20252	20052	20683	20448
18 – 19.5" (45 – 49.5 cm)	20644	20253	20053	20684	20449
19.5 – 21.5" (49.5 – 54.5 cm)	20645	20254	20054	20685	20449
21.5 – 24.5" (54.5 – 62 cm)	20645	20255	20055	N/A	
24.5 – 28" (62 – 71 cm)	20646	20256	20056	N/A	
28 – 34" (71 – 86 cm)eee	20647	20257	20057	N/A	

### STRAP PADS

Size	Part#
Small	B-304010057
Medium, Large	B-304010058
X-Large, XX-Large	B-304010059

### OTHER ACCESSORIES

Description	Part#
CTi D-Ring Kit Standard 2x 1" and 2x 1.5" D-rings	B-704132600
CTi D-Ring Kit 2" 2x 2" D-rings	B-704132602
CTi Condyle Pad Kit	B-704130150
Extension Stop Kit	22130
Flexion Stop Kit	19254
Brace Bag	B-307020028



## CTi®3

CTi®3 is a comfortable, lightweight and durable ligament knee brace, which provides stabilization of the knee joint and an anatomical design suitable for both everyday activities and sports.



### INDICATIONS FOR USE

- For knee conditions that may benefit from increased AP and ML stability and protection, such as injuries to the ACL, MCL, LCL, PCL, rotary and combined instabilities

### FEATURES AND BENEFITS

- Enhanced construction - Lightweight, high strength carbon composite frames provide the optimal stiffness to weight ratio.
- Anatomical fit - Designed to fit a wider range of leg shapes and includes an Accutrak® hinge to closely follow the knee's anatomical movement.
- Elevated comfort - Unique liners enhance comfort with their special tread block pattern, which provides a balance of cushion and ventilation.
- Easy-to-use accessories and replacement parts, including MX kits, replacement strap sets, liner sets, and flexion stops.
- Available in 4 sizes, right and left.

### REQUIRED MEASUREMENTS

- M/L Width measurement: Medio-laterally at joint width in a standard, weight-bearing position.
- Thigh Circumference: Secondary measurement for patients in between sizes, measured 20cm (8") above mid-patella.
- Distal Patella Circumference: Just below the lower border of the knee cap.





# CTI® 3

## ORDERING INFORMATION

CTI3 - Includes Condyle Pad Set and Extension Stop Set, 0° Extension Stop and Standard Condyle Pad are pre-installed.

Size	M/L Width	Thigh Circumference	Distal Patella Circumference	Right	Left
S	8.9-10.2 cm (3.5"-4")	40-47 cm (15.7"-18.5")	28-34 cm (11"-13.4")	CTI-22002	CTI-23002
M	10-11.4 cm (3.9"-4.5")	45-55 cm (17.7"-21.7")	34-38 cm (13.4"-15")	CTI-22003	CTI-23003
L	11.2-12.7 cm (4.4"-5")	52-61 cm (20.5"-24")	38-41 cm (15"-16.1")	CTI-22004	CTI-23004
XL	12.5-14 cm (4.9"-5.5")	58-67 cm (22.8"-26.4")	41-44 cm (16.1"-17.3")	CTI-22005	CTI-23005

Note: M/L Width measurements are the preferred and most accurate measurement for CTI. Distal Patella Circumference can be used in place of M/L Width. Circumference measurements are approximate.

## ACCESSORIES & REPLACEMENT PARTS

CTI3 MX KIT - Includes Impact Guard, Sleeve with Mesh, CTi Decal Sheet, and CTi Product Bag.

Size	Right	Left
S	CTI-22132	CTI-23132
M	CTI-22133	CTI-23133
L	CTI-22134	CTI-23134
XL	CTI-22135	CTI-23135

### LINER SET

Size	Right	Left
S	CTI-22322	CTI-23322
M	CTI-22323	CTI-23323
L	CTI-22324	CTI-23324
XL	CTI-22325	CTI-23325

### HOOK SET

Size	Right	Left
S	CTI-22332	CTI-23332
M	CTI-22333	CTI-23333
L	CTI-22334	CTI-23334
XL	CTI-22335	CTI-23335

### STRAP SET

Size	Part #
S	CTI-24222
M	CTI-24223
L	CTI-24224
XL	CTI-24225

### MISCELLANEOUS

Description	Part #
Guard Elastic Cord	CTI-24130
Guard Screws	CTI-24140
Tibial Wedge Pad	22077
Condyle Pad Set (Thin, Standard, Thick)	CTI-24320
Extension Stop Set (0°, 5°, 10°, 20°, 30°)	CTI-24430
Flexion Stop Set (0°, 60°, 75°, 90°)	CTI-24440
Hinge Sticker Set	CTI-24450
D-Ring Set	CTI-24510
CTI3 PCL Strap Kit	CTI-24520

### SLEEVE WITH MESH

Size	Part #
S	SLV60012
M	SLV60013
L	SLV60014
XL	SLV60015

## REBOUND® DUAL

The Rebound® DUAL offers simplicity without compromise. With its malleable yet supportive frame, Rebound DUAL is the go-to for knee ligament support.

### INDICATIONS FOR USE

The brace is indicated for combined and non-combined instabilities of the:

- Anterior cruciate ligament (ACL)
- Medial collateral ligament (MCL)
- Lateral collateral ligament (LCL)
- Posterior cruciate ligament (PCL)
- Conditions requiring unicompartmental load control



### FEATURES AND BENEFITS

- Cold-malleable aluminum frame, trimmable straps, and field-serviceable D-rings accommodate various patient anatomies
- Acculign™ upper frame allows height and varus/valgus adjustment
- Tool-free polycentric hinge with ROM stop options for range-of-motion restriction
- Össur Hinge Alignment Guide, Blue is You™ touch points, and tactile numbered strap tabs guide the fitting process
- Optional SmartDosing® Kit accessory offers fine-tuned unicompartmental offloading (Available for OTS version only)
- Optional PCL kit accessory for multi-ligament support



# REBOUND® DUAL

## REQUIRED MEASUREMENTS: REBOUND DUAL CUSTOM

- Custom made to measure: Smartmeasure, 3D scan (.AOP, .STL, .OBJ, .CPV) or Patient Cast


## SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilizes a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.




Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000


## CUSTOM COLORS

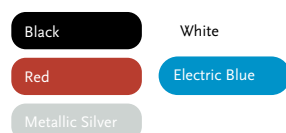

## REQUIRED MEASUREMENTS: REBOUND DUAL OTS

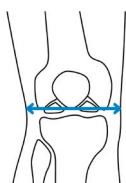

Measure knee caliper medio-laterally at joint width in standing, weight-bearing position.

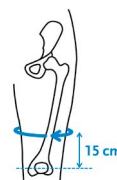

Measure thigh circumference 15 cm (6") above mid-patella. Leg in extension.

## ORDERING INFORMATION

### CUSTOM

Description	Right	Left
Black	B-155510001	B-155610001

### OTS

Size	Caliper	Circumference	Right	Left
XS	3-3.5" (7.6-8.9 cm)	15-16.5" (38.1-41.9 cm)	B-242510001	B-242610001
S	3.5-4" (8.9-10.2 cm)	16.5-18.5" (41.9-47 cm)	B-242510002	B-242610002
M	4-4.5" (10.2-11.4 cm)	18.5-20.5" (47-52.1 cm)	B-242510003	B-242610003
L	4.5-5" (11.4-12.7 cm)	20.5-22.5" (52.1-57.2 cm)	B-242510004	B-242610004
XL	5-5.5" (12.7-14 cm)	22.5-24.5" (57.2-62.2 cm)	B-242510005	B-242610005
XXL	5.5-6" (14-15.2 cm)	24.5-27.5" (62.2-69.9 cm)	B-242510006	B-242610006
3XL	6-6.5" (15.2 – 16.5 cm)	27.5-30" (69.9-76 cm)	B-242510007	B-242610007

# REBOUND® DUAL

## ACCESSORIES

### LINERS: CUSTOM

Size	Doeskin, Thigh, Black	Active Grip, Calf, Black
XS	B-701090150	B-701090160
S	B-701090151	B-701090161
M	B-701090152	B-701090162
L	B-701090153	B-701090163
XL	B-701090154	B-701090164
XXL	B-701090155	B-701090165
3XL	B-701090156	B-701090166

### LINER KITS

Size	Liner Kit, Black*
XS	B-701090110
S	B-701090111
M	B-701090112
L	B-701090113
XL	B-701090114
XXL	B-701090115
3XL	B-701090116

\*Includes Doeskin thigh liner and Sensil calf liner

### CALF LINERS: OTS, LEGACY BRACES

Size	Doeskin, Silver	Active Grip, Black	Active Grip, Silver
XS	B-701090028	B-701090160	B-701090017
S	B-701090029	B-701090161	B-701090018
M	B-701090030	B-701090162	B-701090019
L	B-701090031	B-701090163	B-701090020
XL	B-701090032	B-701090164	B-701090021
XXL	B-701090033	B-701090165	B-701090022
3XL	B-701090034	B-701090166	B-701090023

### THIGH LINERS: OTS, LEGACY BRACES

Size	Doeskin, Black	Doeskin, Silver	Active Grip, Silver
XS	B-701090150	B-701090010	B-701090041
S	B-701090151	B-701090011	B-701090042
M	B-701090152	B-701090012	B-701090043
L	B-701090153	B-701090013	B-701090044
XL	B-701090154	B-701090014	B-701090045
XXL	B-701090155	B-701090015	B-701090046
3XL	B-701090156	B-701090016	B-701090047

# REBOUND® DUAL

## ACCESSORIES

### STRAP KITS & PAD KITS

Size	Replacement Strap Kit	PCL Strap Kit
XS-L	B-701090105	B-701090057
XL-3XL	B-701090106	B-701090058

### STRAP KITS & PAD KITS

Size	Active Grip Strap Pad Kit	Strap Pad Kit
XS-M	B-701090039	B-701090037
L-XL	B-701090040	

### OTHER

Description	Part#
SmartDosing Kit RM/LL	B-701090060
SmartDosing Kit RL/LM	B-701090061
Flexion/Extension Stop Set - OTS Tool-Free Hinge	B-701090088
*Flexion Stop Set - Custom & OTS Legacy Hinge	B-701090004
**Extension Stop Set - Custom & OTS Legacy Hinge	B-701090008
D-Ring Kit	B-701090059
Tibial Wedge Kit	B-701090001
Condyle Pad Kit	B-701090009
Acculign Screw Kit (10pc)	B-701090050

\*Flexion Stops: 0,10,20,30,45,60,75,90

\*\*Extension Stops: 0,5,10,20,30,45

### SLEEVES

Size	Thigh Circumference	Sof sleeve	Neoprene Undersleeve	Neoprene Oversleeve	Padded Sports Oversleeve
S	< 15" (38 cm)	20643	20251	20051	20448
M	15 – 18" (38 – 46 cm)	20644	20252	20052	20448
L	18 – 19.5" (46 – 50 cm)	20644	20253	20053	20449
XL	19.5 – 21" (50 – 55 cm)	20645	20254	20054	20449
XXL	21.5 – 24.5" (55 – 62 cm)	20645	20255	20055	N/A
3XL	24.5 – 28" (62 – 71 cm)	20646	20256	20056	N/A
4XL	28 – 34" (71 – 86 cm)	20647	20257	20057	N/A

Size	Thigh Circumference	Sleeve with Mesh*
XS	<17" (43 cm)	SLV60011
S	17 – 18" (43 – 45 cm)	SLV60012
M	18 – 20" (45 – 51 cm)	SLV60013
L	20 – 21" (51 – 53.5 cm)	SLV60014
XL	21 – 22" (43.5 – 56 cm)	SLV60015
XXL	22 – 24" (56 – 61 cm)	SLV60016

Size	Thigh Circumference	Össur Knee Undersleeve**
XS	12 – 15" (30.5 – 38 cm)	B-250150101
S	15 – 18" (38 – 45.5 cm)	B-250150102
M	18 – 22" (45.5 – 56 cm)	B-250150103
L	22 – 26" (56 – 66 cm)	B-250150104
XL	26 – 29" (66 – 73.5 cm)	B-250150105
XXL	29 – 33" (73.5 – 84 cm)	B-250150106

\*Sleeve with Mesh: Main fabric: 92% Cotton/8% Spandex - Mesh: 100% Polyester - Band: 80% Polyester/20% Spandex

\*\*Össur Knee Undersleeve: 80% Nylon/20% Spandex

## REBOUND® DUAL ST

The Rebound® DUAL ST offers simplicity without compromise. With its malleable yet supportive frame, Rebound DUAL ST is the go-to for knee ligament support.

### INDICATIONS FOR USE

The brace is indicated for combined and non-combined instabilities of the:


- Anterior cruciate ligament (ACL)
- Medial collateral ligament (MCL)
- Lateral collateral ligament (LCL)
- Posterior cruciate ligament (PCL)

### FEATURES AND BENEFITS


- Cold-malleable aluminum frame, trimmable straps, and field-serviceable
- D-rings accommodate various patient anatomies
- Step-through design moves the rigid frame cuff behind the calf
- Tool-free polycentric hinge with ROM stop options for range-of-motion restriction
- Össur Hinge Alignment Guide, Blue is You™ touch points, and tactile numbered strap tabs guide the fitting process
- Optional PCL Kit accessory for multi-ligament support



### REQUIRED MEASUREMENTS

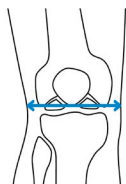

Measure knee caliper medio-laterally at joint width in standing, weight-bearing position.

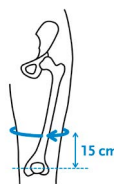

Measure thigh circumference 15 cm (6") above mid-patella. Leg in extension.

### SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilizes a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required



# REBOUND® DUAL ST

## ORDERING INFORMATION


### REBOUND DUAL ST

Size	Caliper	Circumference	Right	Left
XS	3-3.5" (7.6-8.9 cm)	15-16.5" (38.1-41.9 cm)	B-242520001	B-242620001
S	3.5-4" (8.9-10.2 cm)	16.5-18.5" (41.9-47 cm)	B-242520002	B-242620002
M	4-4.5" (10.2-11.4 cm)	18.5-20.5" (47-52.1 cm)	B-242520003	B-242620003
L	4.5-5" (11.4-12.7cm)	20.5"-22.5" (52.1-57.2 cm)	B-242520004	B-242620004
XL	5-5.5" (12.7-14cm)	22.5"-24.5" (57.2-62.2 cm)	B-242520005	B-242620005
XX	5.5-6" (14-15.2cm)	24.5"-27.5" (62.2-69.9 cm)	B-242520006	B-242620006
3XL	6-6.5" (15.2 – 16.5 cm)	27.5"-30" (69.9-76 cm)	B-242520007	B-242620007

### REBOUND DUAL ST CUSTOM

Description	Right	Left
Custom	B-155520001	B-155620002

### CUSTOM BRACE COLORS

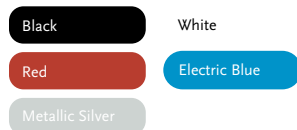

## ACCESSORIES

### OFF-THE-SHELF LINER KITS

Size	Liner Kit, Black*
XS	B-701090120
S	B-701090121
M	B-701090122
L	B-701090123
XL	B-701090124
XXL	B-701090125
3XL	B-701090126

### CUSTOM LINERS

Size	Calf, Active Grip, Black	Thigh, Doeskin, Black
XS	B-701090180	B-701090170
S	B-701090181	B-701090171
M	B-701090182	B-701090172
L	B-701090183	B-701090173
XL	B-701090184	B-701090174
XXL	B-701090185	B-701090175
3XL	B-701090186	B-701090176

\*Includes Doeskin thigh liner and Sensil calf liner

### LEGACY OFF-THE-SHELF LINERS

Size	Calf, Active Grip, Silver	Thigh, Doeskin, Silver
XS	B-701090074	B-701090067
S	B-701090075	B-701090068
M	B-701090076	B-701090069
L	B-701090077	B-701090070
XL	B-701090078	B-701090071
XXL	B-701090079	B-701090072
3XL	B-701090080	B-701090073

# REBOUND® DUAL ST

## STRAP KITS & PAD KITS

Size	Replacement Strap Kit	PCL Strap Kit
XS-L	B-701090107	B-701090065
XL-3XL	B-701090108	B-701090066

## ACTIVE GRIP STRAP PAD KIT

Size	Part#
XS-M	B-701090039
L-XXL	B-701090040

## SILICONE STRAP PAD KIT

Description	Part#
Pad Kit	B-701090037

## OTHER

Description	Part#
*Flexion/Extension Stop Set - Tool-Free Hinge	B-701090088
**Flexion Stop Set - Custom & Legacy Hinge	B-701090004
***Extension Stop Set - Custom & Legacy Hinge	B-701090008
D-Ring Kit	B-701090059
Condyle Pad Kit	B-701090009
Alligator Tab 1	RDP61106
Alligator Tab 2	RDP61102
Alligator Tab 3	RDP61103
Alligator Tab 4	RDP61104
Alligator Tab 5	RDP61105

\*Extension Stops: 0°,5°,10°,20°,30°,45° Flexion Stops: 0°,10°,20°,30°,45°,60°,75°,90°

\*\*Flexion Stops: 0°,30°,45°,60°,75°,90°

\*\*\* Extension Stops: 5°,20°,30°,40°

## SLEEVES

Size	Thigh Circumference	Sofsleeve	Neoprene Undersleeve	Neoprene Oversleeve	Padded Sports Oversleeve
S	< 15" (38 cm)	20643	20251	20051	20448
M	15 – 18" (38 – 46 cm)	20644	20252	20052	20448
L	18 – 19.5" (46 – 50 cm)	20644	20253	20053	20449
XL	19.5 – 21" (50 – 55 cm)	20645	20254	20054	20449
XXL	21.5 – 24.5" (55 – 62 cm)	20645	20255	20055	N/A
3XL	24.5 – 28" (62 – 71 cm)	20646	20256	20056	N/A
4XL	28 – 34" (71 – 86 cm)	20647	20257	20057	N/A

Size	Thigh Circumference	Sleeve with Mesh*
XS	<17" (43 cm)	SLV60011
S	17 – 18" (43 – 45 cm)	SLV60012
M	18 – 20" (45 – 51 cm)	SLV60013
L	20 – 21" (51 – 53.5 cm)	SLV60014
XL	21 – 22" (43.5 – 56 cm)	SLV60015
XXL	22 – 24" (56 – 61 cm)	SLV60016

Size	Thigh Circumference	Össur Knee Undersleeve**
XS	12 – 15" (30.5 – 38 cm)	B-250150101
S	15 – 18" (38 – 45.5 cm)	B-250150102
M	18 – 22" (45.5 – 56 cm)	B-250150103
L	22 – 26" (56 – 66 cm)	B-250150104
XL	26 – 29" (66 – 73.5 cm)	B-250150105
XXL	29 – 33" (73.5 – 84 cm)	B-250150106

\*Sleeve with Mesh: Main fabric: 92% Cotton/8% Spandex - Mesh: 100% Polyester - Band: 80% Polyester/20% Spandex

\*\*Össur Knee Undersleeve: 80% Nylon/20% Spandex

## REBOUND® DUAL ST CUSTOM

Evolving from the functional support and versatility of the Rebound® DUAL, Össur offers a knee ligament brace for everyone. The Rebound DUAL ST, a sleek step-through design built with the strength of aircraft aluminum. Leveraging a light-weight profile and patented hi-tech ActiveGrip™ liners, the Rebound DUAL ST is designed for exceptional comfort and fitting with ease.



### INDICATIONS FOR USE

For knee conditions that may benefit from increased AP and ML stability around the knee, such as: ACL, MCL, LCL, PCL, rotary and combined instabilities

### FEATURES AND BENEFITS

- Step-Through Design: advantage in comfort & fitting with ease
- Sleek, Strong Frame: offering the strength of aircraft aluminum with a light-weight design, the 14" (35 cm) frame length keeps a low-profile
- Anti-Gravity: Activegrip™ liners with the added advantage of Össur's anti-migration strap (AMS) system
- Easy for Everyone: simple strapping system and field-serviceable technology for adjustments on the go
- Versatile, comfortable, and reliable support for improved ligament stability
- Suitable for low-to-high impact activities as well as sliding sports
- Optional PCL kit accessory for support of the PCL
- Custom-made (also available in pre-fabricated sizes XS-3XL)

### 25 Color Options\*


Customize to match your team colors




\*Custom models only. Colors are approximate

# REBOUND® DUAL ST CUSTOM

## REQUIRED MEASUREMENTS

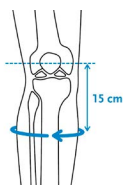


Measure calf circumference 15 cm (6") below mid-patella. Leg in extension.

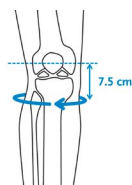


Measure calf circumference 7.5 cm (3") below mid-patella. Leg in extension.



Measure knee caliper medio-laterally at joint width in standing, weight-bearing position.

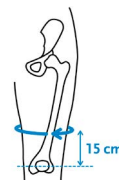

Measure thigh circumference 15 cm (6") above mid-patella. Leg in extension.

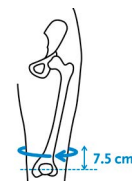

Measure thigh circumference 7.5 cm (3") above mid-patella. Leg in extension.

## ORDERING INFORMATION

### CUSTOM

Left	Right
B-155620002	B-155520001



### SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilizes a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.

Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required.

# REBOUND® DUAL ST CUSTOM

## ACCESSORIES

### PCL STRAP KIT

Size	Part#
XS-L	B-701090065
XL-3XL	B-701090066

### SPORT UNDERSLEEVE

Size	Part#
S	20682
M	20683
L	20684
XL	20685

### NEOPRENE UNDERSLEEVE

Size	Part#
S	20251
M	20252
L	20253
XL	20254
XXL	20255
3XL	20256
4XL	20257

### NEOPRENE OVERSLEEVE

Size	Part#
S	20051
M	20052
L	20053
XL	20054
XXL	20055
3XL	20056
4XL	20057

### PADDED SPORTS OVERSLEEVE

Size	Part#
S	20448
L	20449

### CALF LINER

Size	Part#
XS	B-701090074
S	B-701090075
M	B-701090076
L	B-701090077
XL	B-701090078
XXL	B-701090079
3XL	B-701090080

### THIGH LINER

Size	Part#
XS	B-701090067
S	B-701090068
M	B-701090069
L	B-701090070
XL	B-701090071
XXL	B-701090072
3XL	B-701090073

### STRAP KIT

Size	Part#
XS-L	B-701090005
XL-3XL	B-701090006

## OTHER ACCESSORIES

Description	Part#
Extension stop kit	B-701090008
Flexion stop kit	B-701090004
Condyle pad kit	B-701090009
Field servicable D-ring kit	B-701090059

# PARADIGM®

## INDICATIONS FOR USE

- ACL, MCL, LCL, rotary and combined instabilities
- Protection and stabilization of surgical grafts
- Recommended for low to medium contact / impact

## FEATURES AND BENEFITS

- Carbon fiber Triax® shells with flexible subshell allows for anatomical differences while providing firm control
- Hard-anodised, polycentric hinges
- Adjustable Tibial Strap captures the tibia and allows adjustments in hinge depth for a more customized fit
- Full ROM kit is available, enabling post surgical use
- Large, "open window" frame design for comfort and compliance



## REQUIRED MEASUREMENTS

- Circumference: 15 cm above mid-patella

## CUSTOM STANDARD COLORS



No charge for custom standard colors. Additional fees required for other custom colors.

## ORDERING INFORMATION

Size	Circumference	Left	Right
Custom	N/A	B-110600020	B-110500020

## ACCESSORIES

Size	Strap and Pad Kit	Liner Kit Left	Liner Kit Right
X-Small	B-701060001	B-703066001	B-703065001
Small	B-701060002	B-703066002	B-703065002
Medium	B-701060003	B-703066003	B-703065003
Large	B-701060004	B-703066004	B-703065004
X-Large	B-701060005	B-703066005	B-703065005
XX-Large	B-701060006	B-703066006	B-703065006

Description	Part#
Flexion / Extension Stop Kit	B-705000504



# EXTREME®

## INDICATIONS FOR USE

- Mild to severe ACL, MCL, LCL, PCL, rotary, combined instabilities, laxity or sprains
- Protection and stabilization of surgical grafts

## FEATURES AND BENEFITS

- The proven single-upright design provides dynamic stability, offering maximal support when the knee is under stress
- The unique Dynamic Force Strap® minimizes bulk and maximizes agility
- Custom fabricated brace
- Available in standard and in SmartDosing® version. Tool-free SmartDosing dial option enables fine-tuning and can be adjusted even when wearing the brace under clothing.
- SmartDosing version available with Sensil® silicone calf liner to reduce migration
- Available in matte and glossy paint finishing and in 12 standard colors. Also optional is to have the brace custom painted.



## REQUIRED MEASUREMENTS

- Custom-made to measure

## ORDERING INFORMATION

Description	Part#
Extreme Left with DFS strap	B-108600010
Extreme Right with DFS strap	B-108500010
Extreme with SmartDosing Left	B-108600020
Extreme with SmartDosing Right	B-108500020

## ACCESSORIES

Description	Part#
Patella cup	B-250110001
Hinge cover	B-251020000
AMS Wrap (custom made)	B-426000150
AMS Sleeve (custom made)	B-426000250

## SLEEVES

Size	Össur knee undersleeve	Sof sleeve	Neoprene undersleeve	Neoprene oversleeve
X-Small	B-250150101			
Small	B-250150102	20643	20251	20051
Medium	B-250150103	20644	20252	20052
Large	B-250150104	20644	20253	20053
X-Large	B-250150105	20645	20254	20054
XX-Large	B-250150106	20645	20255	20055
XXX-Large		20646	20256	20056
XXXX-Large		20647	20257	20057

# FLEX™

## INDICATIONS FOR USE

- ACL, MCL, LCL, PCL, rotary and functional support
- Long-term use for post-op, rehab and functional support
- Recommended for low to moderate activity

## FEATURES AND BENEFITS

- Accutrac® hinges with easy to use flexion/extension stops
- Heat-moldable, carbon composite frame accommodates edema and/or changes in leg shape during rehabilitation
- Gel Fit™ Condylar pad for improved suspension and comfort
- Total Support System™ for ACL, MCL, LCL, PCL, rotary and combined instabilities
- Adjustable Tibial Strap captured the tibia and allows adjustments in hinge depth for a customized fit
- Large, “open window” frame design accommodates post-op dressing and edema
- Non-corrosive materials, ideal for water sports



## REQUIRED MEASUREMENTS

- Caliper measurement: Medial-lateral at joint width in a standard, weight-bearing position
- Circumference: Just below the distal border of the patella

## ORDERING INFORMATION

			Standard		PCL	
Size	Caliper	Circumference	Left	Right	Left	Right
Small	3.5 – 4" (8.9 – 10.2 cm)	11 – 12.625" (27.9 – 32.1 cm)	FLEX-SL	FLEX-SR	FLEXPCL-SL	FLEXPCL-SR
Medium	4 – 4.625" (10.2 – 11.4 cm)	12.625 – 14.5" (32.1 – 36.2 cm)	FLEX-ML	FLEX-MR	FLEXPCL-ML	FLEXPCL-MR
Large	4.625 – 5.125" (11.4 – 12.4 cm)	14.5 – 16" (36.2 – 39.1 cm)	FLEX-LL	FLEX-LR	FLEXPCL-LL	FLEXPCL-LR
X-Large	5.125 – 5.625" (12.4 – 13.3 cm)	16 – 17.625" (39.1 – 42.2 cm)	FLEX-XLL	FLEX-XLR	FLEXPCL-XLL	FLEXPCL-XLR

## ACCESSORIES

### CALF & THIGH PADS

Size	Calf Pad Left	Calf Pad Right	Thigh Pad Left	Thigh Pad Right
Small	44048	44049	44046	44047
Medium	44040	44041	44042	44043
Large	44044	44045	44038	44039
X-Large	44052	44053	44050	44051

# FLEX™

## ACCESSORIES

### STRAPS KITS

Size	Part#
Small	44071
Medium	44069
Large	44070
X-Large	44068

### OTHER ACCESSORIES

Description	Part#
Flexion / Extension Stop Kit	44035
Hinge Cover Kit	44083
Condyle Pad Gel (1/4in)	22183
Condyle Pad Kit Neoprene (1/8in + 1/16in + cover)	44066
AMS (Anti-Migration System) Wrap S/M	23550
AMS (Anti-Migration System) Wrap L/XL	23551

# C180™ ROCKET

## INDICATIONS FOR USE

- ACL, MCL, LCL, PCL, rotary and combined instabilities
- Recommended for low to medium contact/impact
- Recommended for pediatric usage

## FEATURES AND BENEFITS

- Anatomically-correct Accutrak® hinges with easy-to-use extension stops
- Heat-Moldable frame for customization
- Adjustable Tibial Strap captures the tibia and allows adjustments in hinge depth for a more customized fit
- Available with an optional patella cup
- Total Support System™ for ACL, MCL, LCL, PCL, rotary and combined instabilities
- Semi-rigid frame consisting of woven carbon fiber infused with acrylic resin
- Lightweight, streamlined design for patient comfort and compliance
- Non-corrosive materials, ideal for water sports



## REQUIRED MEASUREMENTS

- Caliper measurement: Medial-lateral at joint width in a standard, weight-bearing position
- Circumference: Just below the distal border of the patella

## ORDERING INFORMATION

Size	Caliper	Circumference	Left	Right
Small	3 – 3.5" (7.6 – 8.9 cm)	9.5 – 11" (24.1 – 27.9 cm)	C180KSL	C180KSR
Large	3.3 – 3.75" (8.3 – 9.5 cm)	10 – 12" (26 – 29.8 cm)	C180KLL	C180KLR
X-Large	3.5 – 4" (8.9 – 10.2 cm)	11 – 12.5" (27.9 – 32.1 cm)	C1802XSL	C1802XSR

## ACCESSORIES

Description	Part#	
Motocross kit Small / Large	10833	
Motocross kit X-Large	B-705130072	
Patella Cup Small / Large	25029	
Patella Cup X-Large	B-711130000	
Calf Liner Small	25061 (L)	26062 (R)
Calf Liner Large	25057 (L)	26058 (R)
Calf Liner X-Large	25106 (L)	25107 (R)
Thigh Liner Small	25060 (L)	26063 (R)
Thigh Liner Large	25056 (L)	26059 (R)
Thigh Liner X-Large	25104 (L)	25105 (R)

## REBOUND® POST OP KNEE

The simple, versatile post-operative knee bracing solution that is easy for everyone. Whether adjusting ROM, setting the drop lock, or altering the brace length, you'll appreciate the simplicity of the Rebound Post-op knee brace.

### INDICATIONS FOR USE

- ACL, PCL, MCL, LCL repairs/injuries
- Meniscal repairs
- Tibial plateau fractures
- Patellar tendon repairs
- Osteochondral repairs
- Condylar fractures
- Sprains/strains of the knee
- HTOs (High Tibial Osteotomy)

### FEATURES AND BENEFITS

- Tool-free design and multi-purpose Smart-Fit™ packaging
- Available in universal, left or right contour and pediatric
- Telescoping frame from 18.5"-27" (Adult) and 15"-21" (Pediatric)
- OneTouch™ easy to adjust ROM hinge with lock-out capability
- Color-coding and numbered frame for patient education
- Quick-fit, gripping buckles for easy application

### REQUIRED MEASUREMENTS

Circumference: 6" (15 cm) above mid-patella

### ORDERING INFORMATION

#### REBOUND POST-OP KNEE

Description	Thigh circumference	Cool	Full Foam
Pediatric	12 – 25" (30.5 cm – 63.5 cm)	B-8381470	
Universal, Left contoured	16 – 32" (41 cm – 81 cm)	B-8381680	B-8382680
Universal, Right contoured	16 – 32" (41 cm – 81 cm)	B-8381580	B-8382580
Universal	16 – 32" (41 cm – 81 cm)	B-8381480	B-8382480
Universal, no box	16 – 32" (41 cm – 81 cm)	B-8385480	
Universal XL	16 – 38" (41 cm – 96.5 cm)	B-8381490	B-8382490

### ACCESSORIES

Description	Part #
ROM stop locks	B-8380005
Alligator hook and loop kit	B-8380006
Adhesive hook and loop kit pediatric	B-8380008
Adhesive hook and loop kit universal / universal XL	B-8380007
Anti-migration Strap kit pediatric	B-8381093
Anti-migration Strap kit Universal*	B-8381094
Anti-migration Strap kit XL	B-8381095
Standard buckle & pad kit	B-8381096
Incision pad kit 10pk	B-7031805

#### REPLACEMENT SOFTGOODS

Description	Cool	Full Foam
Pediatric	B-8381072	
Universal (*For universal, left contoured, right contoured)	B-8381082	B-8382082
Universal XL	B-8381092	B-8382092



## FORMFIT® POST-OP KNEE

Formfit Post-Op Knee provides range of motion control and immobilization following injury or surgery. Length adjustment is tool free and easy. The low profile ROM hinge can be adjusted with a simple pull, rotate and release motion. Wide range of flexion and extension settings allow you to choose the optimal range of motion.

### INDICATIONS FOR USE

- Post-op/post-injury ROM control or immobilization. Indicated for ligament and fracture-related surgeries and when controlled range of motion is necessary

### FEATURES AND BENEFITS

- Breakaway struts for easy, tool free length adjustment
- OneTouch™ easy to adjust ROM hinge with lock-out capability from -10° to 30° (10° incremental)
- Extension control from -10° to 90° (10° incremental)
- Flexion control from 0° to 120° (10° incremental)
- Available in 3 models, 3 lengths, and cool or full foam



### REQUIRED MEASUREMENTS

- Thigh circumference 15 cm (6") above mid patella

### ORDERING INFORMATION

Size	Thigh circumference	Cool	Full Foam
Knee PO 24in DL	UP TO 27" (69cm)	FPOK403100	FPOK403200
Knee PO XL 24in DL	27" – 35" (69-89cm)	FPOK403102	FPOK403202
Knee PO 28in DL	UP TO 27" (69cm)	FPOK403101	FPOK403201
Knee PO XL 28in DL	27" – 35" (69-89cm)	FPOK403103	FPOK403203
Knee PO 16in DL	UP TO 27" (69cm)	-	FPOK403300
Knee PO XL 16in DL	27" – 35" (69-89cm)	-	FPOK403301

### SOFT GOODS

Description	Cool	Full Foam
Replacement Softgood, Regular	403175	403275
Replacement Softgood, XL	403176	403277
Replacement Softgood, Regular Long	403276	403278

### REPLACEMENT SOFTGOODS

Size	
ROM stop locks (set of 20 lock clips)	B-8380005
Strap Reduction Kit Cool	403199
Strap Reduction Kit Full Foam	403299



# EXOFORM® KNEE IMMOBILIZER

## INDICATIONS FOR USE

- Post-injury
- Post-trauma

## FEATURES AND BENEFITS

- Slide-to-size adjustment enables you to provide the right fit for the largest variety of patient lengths and widths
- provides superior fit and limits migration
- Cool foam version can be worn directly against the skin for greater comfort on warm days
- Full foam version provides warmth and compression

## REQUIRED MEASUREMENTS

- Universal – fits leg inseams of 18 – 24" (46-61 cm)



## ORDERING INFORMATION

Description	Leg circumference	Part#
Cool	up to 28" (71.1 cm)	222100
Full foam	up to 28" (71.1 cm)	222000
Full foam XXXL	up to 38" (96.5 cm)	222050

## ACCESSORIES

Description	Part#
Replacement softgood, Cool	222101
Replacement softgood, Full Foam	222001
Replacement softgood, Full Foam, 3XL	222051

## REBOUND® KNEE IMMOBILIZER UNIVERSAL

Rebound® Knee Immobilizer Universal features a universal 3-panel design that can be adjusted to accommodate most leg circumferences. With an MR\* Conditional rating, patients can be safely scanned in an MR system, while maintaining immobilization.

\*See IFU for details.

### INDICATIONS FOR USE

- Emergency immobilization
- Pre and post-surgical knee immobilization
- Traumatic knee injuries

### FEATURES AND BENEFITS

- Adjustable medial/lateral stays and rigid posterior stays provide motion restriction of the knee
- Universal 3-piece design simplifies inventory needs
- MR\* Conditional: safely scan patients in an MR system while maintaining immobilization
- Min thigh circumference: 10.25" (26 cms)
- Max thigh circumference: 26.5" (67.31 cms)



### ORDERING INFORMATION

Length	Part#
12" (30 cm)	210012
14" (35 cm)	210014
16" (40 cm)	210016
18" (45 cm)	210018
20" (50 cm)	210020
22" (55 cm)	210022
24" (60 cm)	210024

### ACCESSORIES

Description	Part#
Hot/Cold Therapy Gel Pad	GB7

## FORMFIT® HINGED LATERAL J

### INDICATIONS FOR USE

- Lateral patellar subluxation and dislocations
- Lateral patellofemoral malalignment
- Patellofemoral maltracking and pain
- Chondromalacia due to lateral maltracking

### FEATURES AND BENEFITS

- Breathable material
- Removable hinges
- Easy pull-on
- Lateral J buttress
- Open popliteal



### REQUIRED MEASUREMENTS

- Thigh Circumference: 6" above mid-patella
- Calf Circumference: 6" below mid-patella

### ORDERING INFORMATION

Size	Thigh circumference	Knee circumference	Calf circumference	Right leg	Left leg
X-Small	13 - 15.5" (33 - 39.4cm)	12-13" (30.5-33cm)	10-12" (25.4-30.5cm)	709152	709162
Small	15.5-18.5" (39.4-47cm)	13-14" (33-35.6cm)	12-14" (30.5-35.6cm)	709154	709164
Medium	18.5-21" (47-53.3cm)	14-15" (35.6-38.1cm)	14-16" (35.6-40.6cm)	709155	709165
Large	21-23.5" (53.3-59.7cm)	15-17" (38.1-43.2cm)	16-18" (40.6-45.7cm)	709157	709167
X-Large	23.5-26.5" (59.7-67.3cm)	17-19" (43.2-48.3cm)	18-20" (45.7-50.8cm)	709158	709168
XX-Large	26.5-29.5" (67.3-74.9cm)	19-21" (48.3-53.3cm)	20-22" (50.8-55.9cm)	709159	709169

## FORMFIT® KNEE, HINGED

The Formfit® Knee brace is for patients suffering from mild to moderate strains, sprains or ligament tears (ACL/PCL/MCL/LCL). It provides excellent stability, protection and compression during injury recovery. Made from strong and breathable materials it offers comfort while remaining highly durable. The open popliteal area reduces material bulging and potential irritation, while the malleable rigid hinge arms provide an intimate fit.

The Formfit Knee brace comes in either a wrap or a sleeve version.

### INDICATIONS FOR USE

For knee conditions that may benefit from increased stability around the knee, such as:

- Mild medial/lateral knee instabilities

### FEATURES AND BENEFITS

- Breathable material
- Easy pull-on
- Malleable hinges
- Universal buttress (Sleeve only)
- Open popliteal

### REQUIRED MEASUREMENTS

- Measure thigh circumference 6" above mid-patella



### ORDERING INFORMATION

Size	Thigh Circumference	Sleeve	Wrap
XS	13" – 15.5" (33 – 39 cm)	504252	503252
S	15.5 – 18.5" (39 – 47 cm)	504253	503253
M	18.5 – 21" (47 – 53 cm)	504255	503255
L	21 – 23.5" (53 – 60 cm)	504257	503257
XL	23.5 – 26.5" (60 – 67 cm)	504258	503258
XXL	26.5 – 29.5" (67 – 75 cm)	504259	503259
3XL	29.5 – 32" (75 – 81 cm)		503260

### Accessories

Description	Part#
Formfit Knee Brace Buttress (Sleeve only)	503000

## FORMFIT® KNEE ROM

The Formfit® Knee brace is for patients suffering from mild to moderate strains, sprains or ligament tears (ACL/PCL/MCL/LCL). It provides excellent stability, protection and compression during injury recovery. Made from strong and breathable materials it offers comfort while remaining highly durable. The open popliteal area reduces material bulging and potential irritation, while the malleable rigid hinge arms provide an intimate fit.



The Formfit Knee brace comes in either a wrap or a sleeve version.

### INDICATIONS FOR USE

- Mild to moderate ACL and PCL tears
- MCL and LCL tears
- Combined mild knee sprains and strains
- Instabilities of the knee

### FEATURES AND BENEFITS

- Breathable material
- Easy pull-on
- Malleable hinges
- Open popliteal
- Flexion stops: 0°, 45°, 75°, 90°
- Extension stops: 0°, 20°, 30°, 40°, lockout at full extension

### REQUIRED MEASUREMENTS

- Measure thigh circumference 6" above mid-patella

### ORDERING INFORMATION

Size	Knee Circumference	Sleeve		Wrap	
		Long ROM Sleeve	Short ROM Sleeve	Long ROM Wrap	Short ROM Wrap
XSmall	13" – 15.5" (33 – 39 cm)	508352	508252	507352	507252
Small	15.5 – 18.5" (39 – 47 cm)	508353	508253	507353	507253
Medium	18.5 – 21" (47 – 53 cm)	508355	508255	507355	507255
Large	21 – 23.5" (53 – 60 cm)	508357	508257	507357	507257
XLarge	23.5 – 26.5" (60 – 67 cm)	508358	508258	507358	507258
XXLarge	26.5 – 29.5" (67 – 75 cm)	508359	508259	507359	507259
XXXLarge	29.5 – 32" (75 – 81 cm)			507360	507260

### ACCESSORIES

Description	Part#
Universal Butress	503000

## FORMFIT® TRACKER

The Formfit® Tracker with micro-adjustable ReflexWing™ buttress realigns the patella, providing dynamic lateral support for common patella conditions. Its FlyWeight™ fabric delivers lightweight breathability and a low-profile design that feels great against the skin and minimizes migration. PowerLock™ straps work in tandem with heat moldable CustomFit™ hinges to provide a customizable fit for your patient.

### INDICATIONS FOR USE

For knee conditions that may benefit from realignment of the patella and/or patients that would benefit from an added lateral support of the patella, such as:


- lateral patellar subluxation or dislocation
- lateral maltracking

### FEATURES AND BENEFITS


- CustomFit™ hinges for customized fit and support
- Anti-Migration Tech™ minimizes migration
- PowerLock™ straps for secure lateral support
- ReflexWing™ buttress for comfortable patella realignment
- FlyWeight™ fabric for soft feel and breathability
- Durable and machine washable



## REQUIRED MEASUREMENTS & ORDERING INFORMATION

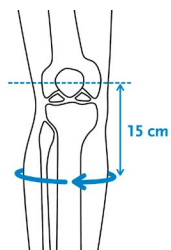

Measure calf circumference  
15 cm (6") below mid-patella.  
Leg in extension.

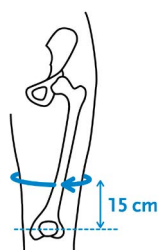

Measure thigh circumference  
15 cm (6") above mid-patella.  
Leg in extension.

Size	Calf circumference 15 cm (6") below mid patella		Thigh circumference 15 cm (6") above mid patella		Left	Right
	cm	inch	cm	inch		
XS	25 - 30	10 - 12	33 - 39	13 - 15½	719162	719152
S	30 - 35	12 - 14	39 - 47	15½ - 18½	719163	719153
M	35 - 40	14 - 16	47 - 53	18½ - 21	719164	719154
L	40 - 46	16 - 18	53 - 60	21 - 23½	719165	719155
XL	46 - 51	18 - 20	60 - 67	23½ - 26½	719166	719156
XXL	51 - 56	20 - 22	67 - 75	26½ - 29½	719167	719157
3XL	56 - 61	22 - 24	75 - 81	29½ - 32	719168	719158

## ACCESSORIES

Description	Part#
ReflexWing™ Buttress	719100



## REBOUND® KNEE BRACE

With six variations to choose from, Össur's Rebound hinged knee brace product line is a comprehensive portfolio of high-quality braces designed to serve the needs of a broad range of patients.

### INDICATIONS FOR USE

- ROM Hinge Models: Mild to moderate ACL and/or PCL, MCL and LCL instabilities, sprains and strains
- Non-ROM Hinge Models: Mild medial/lateral knee instabilities

### FEATURES AND BENEFITS

- Highly breathable aersopacer material reduces heat retention and improves comfort.
- Low profile hinges with extension and flexion stop kit options (ROM version) and malleable arms to provide a custom fit along with knee movement control.
- Contoured pre-tensed Cooltech™ fabric designed to reduce irritation and material bulging in the popliteal area.
- Strap pull-tab allows for consistent donning by patient.
- Universal buttresses are included with all sleeve versions for customizable patella support.



### REQUIRED MEASUREMENTS

- Thigh Circumference: 6" (15 cm) above mid-patella

### ORDERING INFORMATION

#### REBOUND ROM KNEE

Size	Circumference	Sleeve Long	Sleeve Short	Wrap Long	Wrap Short
XS	11.5 – 13.75" (29 – 35 cm)	708152	708052	707152	707052
S	13.75 – 16" (35 – 41 cm)	708153	708053	707153	707053
M	16 – 18" (41 – 46 cm)	708155	708055	707155	707055
L	18 – 20.5" (46 – 52 cm)	708157	708057	707157	707057
XL	20.5 – 22.5" (52 – 57 cm)	708158	708058	707158	707058
XXL	22.5 – 24.75" (57 – 63 cm)	708159	708059	707159	707059
3XL	24.75 – 28.5" (63 – 72 cm)	708160	708060	707160	707060
4XL	28.5 – 32" (72 – 81 cm)			707161	707061

#### REBOUND KNEE

Size	Circumference	Sleeve Short	Wrap Short
XS	11.5 – 13.75" (29 – 35 cm)	704052	703052
S	13.75 – 16" (35 – 41 cm)	704053	703053
M	16 – 18" (41 – 46 cm)	704055	703055
L	18 – 20.5" (46 – 52 cm)	704057	703057
XL	20.5 – 22.5" (52 – 57 cm)	704058	703058
XXL	22.5 – 24.75" (57 – 63 cm)	704059	703059
3XL	24.75 – 28.5" (63 – 72 cm)	704060	703060
4XL	28.5 – 32" (72 – 81 cm)		703061

## NEOPRENE HINGED KNEE SUPPORT

### INDICATIONS FOR USE

- Moderate ligament instabilities

### FEATURES AND BENEFITS

- Accordion molding in the popliteal area reduces bunching behind the knee
- Contoured fit for additional comfort and protection
- The hinges are molded into the brace to maintain a low profile

### REQUIRED MEASUREMENTS

- Knee circumference at mid-patella



### ORDERING INFORMATION

Size	Knee circumference	Part#
Small	11.5 – 13" (29 – 33 cm)	302153BLK
Medium	13 – 14.5" (33 – 37 cm)	302155BLK
Large	14.5 – 16" (37 – 41 cm)	302157BLK
X-Large	16 – 17.5" (41 – 44 cm)	302158BLK
XX-Large	17.5 – 19" (44 – 48 cm)	302159BLK

## NEOPRENE WRAPAROUND HINGED KNEE SUPPORT

### INDICATIONS FOR USE

- For patients with unstable or injured knees that may benefit from additional knee support, such as post-operation rehabilitation, tendonitis, bursitis, arthritis; and sprains and strains.

### FEATURES AND BENEFITS

- Dual action hinges incorporate our patented three dimensional moulding technology
- The hinges are molded into the brace to maintain a low profile

### REQUIRED MEASUREMENTS

- Knee circumference at mid-patella



### ORDERING INFORMATION

Size	Knee circumference	Part#
Small	13 – 14"	212003
Medium	14 – 15"	212005
Large	15 – 16"	212007
X-Large	16 – 17"	212008
XX-Large	17 – 18"	212009
3X-Large	18 – 23"	212010
4X-Large	19 – 25"	212011

# NEOPRENE KNEE SUPPORT WITH STABILIZED PATELLA

## INDICATIONS FOR USE

- For patients suffering from knee conditions that may benefit from additional knee and patella support such as patellar tendonitis, subluxation dislocation, post-operation rehabilitation, post arthroscopic surgery, condromalacia, bursitis, arthritis and tendonitis.

## FEATURES AND BENEFITS

- Molded hardware to maintain a low profile
- Accordion molding in the popliteal area eliminates bunching
- Soft cushioned liner to offer unequaled comfort
- Varied areas of thickness give additional protection
- Lycra® liner offers a breathable contact area for comfortable and extensive use

## REQUIRED MEASUREMENTS

- Knee circumference at mid-patella



## ORDERING INFORMATION

Size	Knee circumference	Part#
Small	11.5-13"	302563BLK
Medium	13-14.5"	302565BLK
Large	14.5-16"	302567BLK
X-Large	16-17.5"	302568BLK
XX-Large	17.5-19"	302569BLK

## FORMFIT® PRO KNEE

Össur Formfit® Pro Knee is a compressive knee sleeve that features a SmartPad™ buttress for targeted support and cushioning. Our patented design realigns the patella, suitable for conditions such as Patella Tendinitis (Jumper's Knee) and Chondromalacia Patella. The dynamic, 3D MotionTech™ knit is supportive, meets certified medical grade compression requirements and has been proven to be superior to the leading brands at moisture wicking and breathability, keeping your knee cool during wear. Lightweight, removable spiral stays ensure optimum shape and fit, while distinct patella graphics make donning intuitive.



MotionTech™



CoolVent™



StayBilizer™



SmartPad™



### INDICATIONS FOR USE

For knee conditions that may benefit from compression, such as:

- Patella tendinitis
- Chondromalacia Patella
- Mild sprains and strains
- Knee pain
- Feeling of instability


### CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

### FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit with superior moisture wicking and breathability, keeping your knee cool during wear.
- Certified medical grade compression (20-30 mmHg).
- Lightweight CoolVent™ knit in popliteal area for added breathability and comfort.
- Patented SmartPad™ for targeted support and cushioning.
- Patella graphics for intuitive placement.
- Patented FoldLock™ system for easier and more secure application.
- Lightweight, removable spiral steel stays.

### REQUIRED MEASUREMENTS

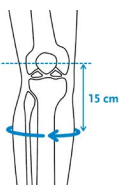

Measure calf circumference  
15 cm (6") below mid-patella.  
Leg in extension.



Measure thigh circumference  
15 cm (6") above mid-patella.  
Leg in extension.



Size	Calf circumference 15 cm (6") below mid patella		Thigh circumference 15 cm (6") above mid patella		Blue	Black
	cm	inch	cm	inch	Part#	Part#
XS	26 - 31	10¼ - 12¼	36 - 41	14¼ - 16¼	FP101001	FP100001
S	31 - 36	12¼ - 14¼	41 - 46	16¼ - 18	FP101002	FP100002
M	36 - 41	14¼ - 16¼	46 - 51	18 - 20	FP101003	FP100003
L	41 - 46	16¼ - 18	51 - 56	20 - 22	FP101004	FP100004
XL	46 - 50	18 - 19¾	56 - 60	22 - 23½	FP101005	FP100005

## FORMFIT® PRO KNEE FLITE

Össur Formfit® Pro Knee Flite is a performance 3D knitted sleeve that is micro-engineered from our custom MotionTech™ yarn to provide breathability, moisture wicking, and a skintight fit. Our unique FoldLock™ and StayBilizer™ technologies combine to ensure the brace stays in place for maximum benefit.



MotionTech™



CoolVent™



StayBilizer™



FoldLock™



### INDICATIONS FOR USE

For knee conditions that may benefit from compression, such as:

- Mild sprains and strains
- Knee pain
- Feeling of instability


### FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit
- CoolVent™ knit in popliteal area for added breathability
- Patented FoldLock™ system for easy and secure application
- Lightweight, removeable spiral steel stays

### CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

### REQUIRED MEASUREMENTS

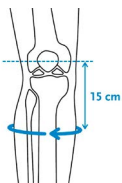

Measure calf circumference  
15 cm (6") below mid-patella.  
Leg in extension.



Measure thigh circumference  
15 cm (6") above mid-patella.  
Leg in extension.



Size	Calf circumference 15 cm (6") below mid patella		Thigh circumference 15 cm (6") above mid patella		Blue	Black
	cm	inch	cm	inch	Part#	Part#
XS	26 - 31	10¼ - 12¼	36 - 41	14¼ - 16¼	FP141001	FP140001
S	31 - 36	12¼ - 14¼	41 - 46	16¼ - 18	FP141002	FP140002
M	36 - 41	14¼ - 16¼	46 - 51	18 - 20	FP141003	FP140003
L	41 - 46	16¼ - 18	51 - 56	20 - 22	FP141004	FP140004
XL	46 - 50	18 - 19¾	56 - 60	22 - 23½	FP141005	FP140005

# NEOPRENE KNEE SLEEVE

## INDICATIONS FOR USE

- For patients suffering from weak and/or injured knees that may benefit from additional knee support.

## FEATURES AND BENEFITS

- Unique accordion design enhances comfort by eliminating bunching in the popliteal area
- Open or closed patella

## REQUIRED MEASUREMENTS

- Knee circumference at mid-patella



## ORDERING INFORMATION

		0.6 cm Neoprene		0.3 cm Neoprene	
Size	Circumference	Open Patella	Closed Patella	Open Patella	Closed Patella
Small	11.5 – 13" (29 – 33 cm)	402013BLK	412013BLK	302013BLK	312013BLK
Medium	13 – 14.5" (33 – 37 cm)	402015BLK	412015BLK	302015BLK	312015BLK
Large	14.5 – 16" (37 – 41 cm)	402017BLK	412017BLK	302017BLK	312017BLK
X-Large	16 – 17.5" (41 – 44 cm)	402018BLK	412018BLK	302018BLK	312018BLK
XX-Large	17.5 – 19" (44 – 48 cm)	402019BLK	412019BLK	302019BLK	312019BLK

## PREMIUM ELASTIC KNEE SUPPORT

### INDICATIONS FOR USE

- Knee conditions that may benefit from an elastic support, such as swelling, feeling of instability, and/or stability around the knee cap.

### FEATURES AND BENEFITS

- Cool, contoured, comfortable fit
- Made with high-quality, long-lasting, breathable materials
- Flexible support and compression
- Reduces swelling from minor injuries
- Adds stability to the knee cap

### REQUIRED MEASUREMENTS

- Knee circumference at mid-patella



### ORDERING INFORMATION

Size	Knee circumference	Part#
Small	13 – 15" (33 – 38 cm)	129603
Medium	15 – 18" (38 – 46 cm)	129595
Large	18 – 21" (46 – 53 cm)	129587
X-Large	21 – 24" (53 – 61 cm)	129578

## ELASTIC KNEE SUPPORT

### INDICATIONS FOR USE

- Knee conditions that may benefit from an elastic support, such as swelling and feeling of instability.

### FEATURES AND BENEFITS

- Cool, contoured, comfortable fit
- Made with high-quality, long-lasting, breathable materials
- Flexible support and compression
- Reduces swelling from minor injuries

### REQUIRED MEASUREMENTS

- Knee circumference at mid-patella



### ORDERING INFORMATION

Size	Knee circumference	Part#
Small	13 – 15" (33 – 38 cm)	13350
Medium	15 – 18" (38 – 46 cm)	13351
Large	18 – 21" (46 – 53 cm)	13352
X-Large	21 – 24" (53 – 61 cm)	13353





## FOOT AND ANKLE



## REBOUND® DIABETIC WALKER

Offloading has proven to be a key factor in foot ulcer treatment and entails reducing pressure of the plantar area in general and the ulcer site in particular. The Rebound Diabetic Walker is uniquely designed for effective treatment of plantar foot ulcers, with easy access and re-fitting during wound care, while improving the patient's mobility during the healing process.

The Rebound Diabetic Walker has major advantages over diabetic shoes and shoe inserts in its healing effectiveness and over total contact casting due to its ease of use, speed of application and low cost.



### INDICATIONS FOR USE

- Plantar ulcers needing offloading

### FEATURES AND BENEFITS

- Effective tri-laminate insole with removable shock-absorbing hexagons that promote ulcer offloading and a Plastazote, heat-moldable top layer that evenly distributes the pressure and minimizes friction
- Unique pressure relief valve for preventing over-inflation, while balancing compression with customized fit
- Positioning Portals allow the clinician to check the fit of the walker at the heel bottom, in order to ensure appropriate positioning of the foot in the walker
- Anatomically designed shell that immobilizes the foot and ankle at 90 degrees while providing protection
- Flex Edge™ technology for relief of pressure points
- Rocker Bottom outsole for a smooth roll over while allowing for individual natural toe out
- Soft and breathable, low friction, machine washable liner for hygiene promotion and prevention of infection
- Tamper evident strapping for compliance monitoring

### REQUIRED MEASUREMENTS

- Shoe size





# REBOUND® DIABETIC WALKER

## ORDERING INFORMATION

### REBOUND DIABETIC WALKER

Size	Men's Shoe Size US	Women's Shoe Size US	Shoe size UK	Shoe size EU	Part#
S	4.5-7	5.5-8.5	4 – 6.5	36 – 39	B-242500002
M	7-10.5	8.5-12	7 – 10	39.5 – 44	B-242500003
L	10.5-12.5	12-14	10.5 – 12	44.5 – 46.5	B-242500004
XL	12.5-15	14-16.5	12.5 – 14.5	47 – 50	B-242500005

## ACCESSORIES

### REPLACEMENT SOFTGOODS

Size	Liners	Insoles
S	B-418500102	B-418500202
M	B-418500103	B-418500203
L	B-418500104	B-418500204
XL	B-418500105	B-418500205

### WALKER COVERS

Size	Hygiene covers (2 pcs)	Durable weather cover	Disposable weather cover (20 pcs)
S	B-418500602	B-418500502	B-418500512
M	B-418500603	B-418500503	
L	B-418500604	B-418500504	B-418500514
XL	B-418500605	B-418500505	

### DIABETIC SOCKS (2 PCS)

Size	Part#
S/M	B-418500703
L/XL	B-418500704



HYGIENE COVER


DURABLE  
WEATHER COVER

DISPOSABLE  
WEATHER COVER


INSOLE

## REBOUND® AIR WALKER

The Rebound Air Walker features a host of details designed to provide added protection, superior compression, simplified application and usage, improved comfort and, ultimately, successful patient outcomes.



### INDICATIONS FOR USE

#### REBOUND AIR WALKER

Foot and ankle conditions that may benefit from motion restriction, such as:

- Grade 2 and 3 ankle sprains
- Stable fractures
- Tendon or ligamentous sprain
- During postoperative or rehab phase. If used with the Achilles Wedge Kit, Achilles ruptures can be treated as well.

#### REBOUND AIR WALKER LOW TOP

Foot and ankle conditions that may benefit from motion restriction, such as:

- Stable foot fractures
- Tendon or ligamentous sprain
- During postoperative or rehab phase.

### FEATURES AND BENEFITS

- Full circumferential shell with additional semi-soft toe bumper for frontal toe protection
- Soft Flex Edge™ reduces pressure points
- Full rocker bottom is especially engineered to be low and wide to help promote a natural gait
- Stretch-to-fit shell for optimal fit
- Shell can be trimmed for additional stretch in the calf
- Cushioning and breathable softgood provides comfort
- Linked medial/lateral bladders for easy inflation and equal compression
- Integrated pump and air release valve for easy access

### REQUIRED MEASUREMENTS

- Shoe size



# REBOUND® AIR WALKER

## ORDERING INFORMATION

### REBOUND AIR WALKER

Size	Men's Shoe Size US	Women's Shoe Size US	Standard	Low Top
X-Small	2 – 4	3.5 – 5.5	B-242900001	B-242900061
Small	4 – 7	5.5 – 8.5	B-242900002	B-242900062
Medium	7 – 10.5	8.5 – 12	B-242900003	B-242900063
Large	10.5 – 12.5	12 – 14	B-242900004	B-242900064
X-Large	12.5 – 15	14 – 16.5	B-242900005	B-242900065

## ACCESSORIES

### REPLACEMENT SOFTGOODS

Size	Slim Liner / High Top	Slim Liner / Low Top
X-Small	B-418030101	B-418030161
Small	B-418030102	B-418030162
Medium	B-418030103	B-418030163
Large	B-418030104	B-418030164
X-Large	B-418030105	B-418030165

### REBOUND ACHILLES WEDGE KIT

Size	Part#
Small	AWK0001
Medium	AWK0005
Large	AWK0009
Universal*	AWK00015

\* Only fits sizes M to XL

### WALKER COVERS

Size	Hygiene covers (2 pcs)	Durable weather cover	Disposable weather cover (20 pcs)
Small	B-418500602	B-418500502	B-418500512
Medium	B-418500603	B-418500503	
Large	B-418500604	B-418500504	B-418500514
X-Large	B-418500605	B-418500505	

### OTHER ACCESSORIES

Description	Part#
Adhesive Insole for Wedge	WHW-W0107
Strap Extension	B-502100101
Rebound Hightop Socks SM/MD	B-418500703
Rebound Hightop Sock LG/XL	B-418500704

## AFO DYNAMIC

The AFO Dynamic provides a strong, lightweight solution for patients with varying degrees of drop foot secondary to a variety of neurological conditions. Össur has taken the carbon fiber technology from its Flex-Foot® product and incorporated it into the new AFO Dynamic to provide increased support over traditional ankle foot orthoses. The energy storing properties of carbon fiber make it an ideal material for this dynamic ankle foot orthosis because it provides real energy return. Best for those people leading busy lives, the AFO Dynamic will ensure a more efficient walk for the user.



### INDICATIONS FOR USE

Drop foot secondary to:

- Cerebro Vascular Accident (CVA)
- Mild knee instability
- Other neurological conditions
- Charcot Marie Tooth disease (HMSN)
- Multiple Sclerosis

Contraindications:


- Severe ankle – foot deformities
- Fluctuating edema
- Severe spasticity

### FEATURES AND BENEFITS

- Carbon fiber design provides excellent strength-to-weight ratio and dynamic motion
- The tapered carbon heel stores energy and manages forces at heel strike
- Full-length toe lever provides full support while foot is loaded, as well as reduced shock to the contralateral limb
- Creates medial-lateral stability at mid-stance
- Reduces energy consumption by reducing hip hike
- Medial strut makes it less visible in shoe
- Open anterior tibia shell provides superior anatomical fit, and reduces heat and perspiration
- Weight limit: 120kg (265lbs)

### REQUIRED MEASUREMENTS

- Shoe size
- Calf height at thickest part of the calf

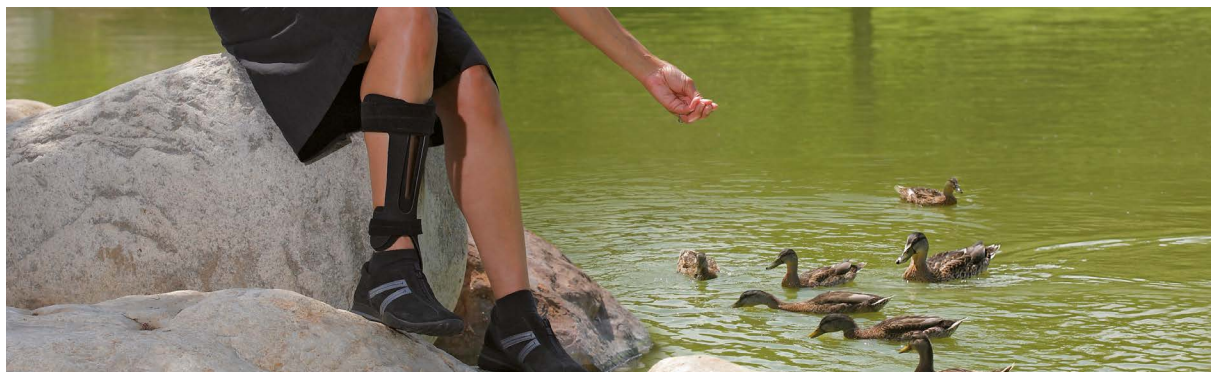

## AFO DYNAMIC

### ORDERING INFORMATION

#### AFO DYNAMIC

Size	Calf Height	Shoe Size US		Shoe Size UK		Shoe Size EU cm		Left	Right
		Male	Female	Male	Female	Male	Female		
X-Small	30 – 34 cm	3 – 5	4 – 6.5	2.5 – 4.5	< 4	34 – 37	< 37	O-102021	O-102011
Small	32 – 36 cm	5.5 – 7.5	7 – 9	5 – 7	4.5 – 6.5	37.5 – 40	37.5 – 40	O-102022	O-102012
Medium	34 – 38 cm	8 – 10.5	9.5 – 12	7.5 – 10	7 – 9.5	40.5 – 44.5	40.5 – 44	O-102024	O-102014
Large	36 – 40 cm	11 – 14		10.5 – 13.5		45 – 48.5		O-102026	O-102016

### ACCESSORIES

#### ACCESSORY KITS

Size	Part#
X-Small	O-102101
Small	O-102102
Medium	O-102104
Large	O-102106

Note: Accessory kits come with foam padding, upper and lower Velcro straps, and Velcro dots.

## AFO LEAF SPRING

AFO Leaf Spring is a pre-fabricated polypropylene ankle-foot orthosis designed to support flaccid drop foot. It provides a semirigid section for toe clearance and support, yet has a thin, flexible foot plate for easy trimming. The absence of a heel section makes the Leaf Spring more comfortable to wear and provides a better fit in shoes.

#### INDICATIONS FOR USE

- Drop foot
- Mild drop foot secondary to other neurological pathologies

#### FEATURES AND BENEFITS

- Lightweight injection-molded polypropylene
- Thin, flexible foot part which is easy to trim with scissors
- Absence of heel section for improved comfort



#### REQUIRED MEASUREMENTS

- Shoe size

### ORDERING INFORMATION

Size	Shoe Size US	Shoe Size UK	Shoe Size EU	Height	Left	Right	Foot Plate Length
Small – Medium	7.5 – 10.5	3.5 – 6.5	36 – 39	10.75" (27 cm)	O-110227	O-110127	24 cm
Large	11 – 14	7 – 10.5	39.5 – 45	12" (30 cm)	O-110230	O-110130	28 cm



## AFO LIGHT

AFO Light provides support only where it is needed, eliminating unnecessary bulk and providing a cosmetically-appealing look. Using the best of Flex-Foot® technology and the energy storing properties of carbon fiber, AFO Light offers a strong and lightweight solution for people with mild to moderate Drop Foot.



### INDICATIONS FOR USE

- Mild to moderate drop foot
- Severe ankle – foot deformities
- Medial-lateral instability
- Severe spasticity
- Extreme activity
- Fluctuating edema

### FEATURES AND BENEFITS

- Energy return and propulsion assistance of the limb at toe off
- Supports the foot for ground clearance during swing phase
- Carbon fiber design provides excellent strength-to-weight ratio and dynamic motion
- Gradual plantar flexion at initial contact
- Stiffness gradient at heel and the toe correspond to ground reaction forces as they are applied to the foot
- Easy to don and doff
- Weight limit: 100kg (220lbs)

### REQUIRED MEASUREMENTS

- Shoe size
- Calf height at thickest part of the calf

### ORDERING INFORMATION

#### AFO LIGHT

Size	Calf height	Shoe Size US		Shoe Size UK		Shoe Size EU		Left	Right
		Male	Female	Male	Female	Male	Female		
Small	32 – 36 cm	Up to 7.5	7 – 9	< 7	4.5 – 6.5	< 40	37.5 – 40	O-121022	O-121012
Medium	34 – 38 cm	8 – 10.5	9.5 – 12	7.5 – 10	7 – 9.5	40.5 – 44.5	40.5 – 44	O-121024	O-121014
Large	36 – 40 cm	11 – 14		10.5 – 13.5		45 – 48.5		O-121026	O-121016
X-Large	38 – 42 cm	12 – 15		11.5 – 14.5		46 – 50		O-121028	O-121018

### ACCESSORIES

Description	Part#
Replacement Kit	O-122101

## FOOT-UP®

Foot-Up is a lightweight ankle orthosis that offers dynamic support for Drop Foot or similar complaints. The revolutionary orthosis provides visible improvement in gait by supporting the foot the moment it is raised.



### INDICATIONS FOR USE

- Drop foot
- Support of dorsiflexion

### FEATURES AND BENEFITS

- Streamlined
- Padded
- Lightweight
- Easy to fit
- Invisible when worn
- Separate inlays available
- Comfortable to wear
- Available in black and beige
- Breathable material

### REQUIRED MEASUREMENTS

- Basic version: Circumference of ankle, measured around the malleoli
- Shoeless version: Circumference of foot immediately behind the ball of the foot

### ORDERING INFORMATION

#### FOOT-UP

Size	Ankle Circumference	Beige	Black
Medium	7 – 8.5" (18 - 21 cm)	07811 -1	07810-1
Large	8.5 – 10" (22 - 26 cm)	07811 -2	07810-2
X-Large	10 – 13" (27 - 33 cm)	07811 -3	07810-3

#### SHOELESS VERSION\*

Size	Foot Circumference	Beige	Black
Small	< 8" (20 cm)	27811-1	27810-1
Medium	8 – 9.5" (20 - 24 cm)	27811-2	27810-2
Large	9.5 – 11" (24 - 28 cm)	27811-3	27810-3
X-Large	> 11" (28 cm)	27811-4	27810-4

\*Shoeless version item comprises of foot wrap only and must be ordered with the Foot-Up unless the patient already has one.

### ACCESSORIES

Description	Beige	Black
Foot-Up plastic insert	17811	17810

## REBOUND FOOT-UP®

Rebound Foot-Up is a lightweight ankle/foot orthosis (AFO) that offers dynamic support for drop-foot or related disorders that require dorsiflexion support. The Rebound Foot-Up supports the ankle dorsiflexion during swing phase. Dynamic support is provided by an elastic strap connected to the shoe by either a transparent plastic inlay placed between shoelaces, or by a nut and bolt fastened through the eyelets. The positioning of the fastening allows for individualizing the support by providing the option of positioning the dorsiflexion support on the medial or lateral side. When using the plastic inlay, the dorsiflexion support becomes neutral. The Rebound Foot-Up Ankle Cuff has a smart clamp that holds the cuff in place and facilitates one-handed donning and doffing.



### INDICATIONS FOR USE

- Drop foot
- Disorders that require dorsiflexion support

### FEATURES AND BENEFITS

- Dynamic and discreet support for drop-foot or related disorders that require dorsiflexion support
- Improved function, ease of use, comfort, and quality
- Allows for voluntary plantar flexion
- The elastic strap is adjustable and easily replaceable
- The Rebound Foot-Up Ankle Cuff and Rebound Foot-Up Foot Wrap are made from breathable material that can be worn comfortably for long periods of time
- The Rebound Foot-Up is hand-washable
- Positioned medial or lateral support
- Contoured to the shape of leg
- Adjustable strap length
- Smart clamp facilitates one-handed donning / doffing
- Optional Rebound Foot-Up Foot Wrap can be used without shoes or in open shoes or sandals

### REQUIRED MEASUREMENTS

- Circumference of ankle, measured around the malleoli

### ORDERING INFORMATION

#### ANKLE CUFF

Size	Ankle Circumference	Part #
Small/Medium	7 – 8.5" (18 - 21 cm)	RFU50001
Large/X-Large	10 – 13" (26 - 33 cm)	RFU50002

#### FOOT WRAP

Size	Foot Circumference	Part #
One Size	8 – 11" (20 - 28 cm)	RFU50101

\*Shoeless version item comprises of foot wrap only and must be ordered with the Foot-Up unless the patient already has one.

### ACCESSORIES

Description	Part #
Rebound Foot-Up Attachment Kit (includes plastic inlay, elastic strap, and 1 x nut and screw)	RFU50201
Rebound Foot-Up Replacement Nuts And Bolts (5 nuts included: 2 short and 3 long screws)	RFU50202

# FORMFIT® WALKER, AIR

## INDICATIONS FOR USE

- Soft tissue injuries
- Grade 2 and 3 sprains
- Stable fractures
- Post-operative, trauma or rehab use

## FEATURES AND BENEFITS

- High and low top versions available
- Integrated pneumatic system
- Low, wide rocker bottom promotes a stable, natural gait
- Shock absorbing sole for reduced impact at heel strike
- Breathable, seamless liner



## REQUIRED MEASUREMENTS

- Shoe size

## ORDERING INFORMATION

Size	Men's Shoe Size US	Women's Shoe Size US	Shoe Size UK	Shoe Size EU	Standard	Low Top
X-Small	2 – 4	3.5 – 5.5	1.5 – 3.5	33 – 35.5	A-W0200BLK	A-W0300BLK
Small	4.5 – 7	6 – 8	4 – 6.5	36 – 39	A-W0400BLK	A-W0500BLK
Medium	7.5 – 10.5	8.5 – 11.5	7 – 10	39.5 – 44	A-W0600BLK	A-W0700BLK
Large	10.5 – 12.5	11.5 – 13.5	10.5 – 12	44.5 – 46.5	A-W0800BLK	A-W0900BLK
X-Large	12.5 +	13.5 +	12.5 – 14.5	47 – 50	A-W1000BLK	A-W1100BLK

## ACCESSORIES

### REPLACEMENT SOFTGOODS

Size	Standard	Low Top
X-Small	12349	12356
Small	10675	12357
Medium	10678	12358
Large	10681	12359
X-Large	12347	12360

# FORMFIT® WALKER

## INDICATIONS FOR USE

- Soft tissue injuries
- Grade 2 and 3 sprains
- Stable fractures
- Post-operative, trauma or rehab use

## FEATURES AND BENEFITS

- High and low top versions available
- Shock absorbing sole reduces impact during heel strike
- Low, wide rocker bottom promotes a stable, natural gate
- Breathable, seamless liner

## REQUIRED MEASUREMENTS

- Shoe size



## ORDERING INFORMATION

Size	Men's Shoe Size US	Women's Shoe Size US	Shoe Size UK	Shoe Size EU	Standard	Low Top
X-Small	2 – 4	3.5 – 5.5	1.5 – 3.5	33 – 35.5	W0200BLK	W0300BLK
Small	4.5 – 7	6 – 8	4 – 6.5	36 – 39	W0400BLK	W0500BLK
Medium	7.5 – 10.5	8.5 – 11.5	7 – 10	39.5 – 44	W0600BLK	W0700BLK
Large	10.5 – 12.5	11.5 – 13.5	10.5 – 12	44.5 – 46.5	W0800BLK	W0900BLK
X-Large	12.5 +	13.5 +	12.5 – 14.5	47 – 50	W1000BLK	W1100BLK

## ACCESSORIES

### REPLACEMENT SOFTGOODS

Size	Standard	Low Top
X-Small	WKSG200BLK	WKSG0300BLK
Small	WKSG400BLK	WKSG500BLK
Medium	WKSG600BLK	WKSG700BLK
Large	WKSG800BLK	WKSG900BLK
X-Large	WKSG1000BLK	WKSG1100BLK

### OTHER ACCESSORIES

Description	Part#
10 cm softgood width extension	WKSG-1BLK
Tibia and calf plates for use when additional splinting is required	WKSP100
Hot/Cold therapy insert for safe and reusable cryotherapy	WKGEL100
Heel wedge, Small	AWK0001
Heel wedge, Medium	AWK0005
Heel wedge, Large	AWK0009
Adhesive insole for heel wedge	WHW-W0107
Walker Strap Extender	10882

## ROM WALKER

### INDICATIONS FOR USE


- Foot and ankle conditions that may benefit from motion restriction or controlled range of motion, such as:
- Grade 2 and 3 ankle sprains
- Stable fractures
- Achilles ruptures
- During postoperative or rehab phase

### FEATURES AND BENEFITS

- Lightweight and low profile aluminum side struts
- Rocker bottom design assists gait
- Decreased heel-strike force on calcaneus
- Simple to apply and adjust by circumferential straps
- Dial lock hinges with adjustable ROM limitation
- ROM control from 0-45° of dorsi/plantar flexion (7.5° interval)
- Available in Standard and Air version



## ORDERING INFORMATION

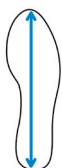

Shoe size.

Size	Shoe size				Standard	Air
	Men's Shoe Size US	Women's Shoe Size US	Shoe Size UK	Shoe Size EU	Part#	Part#
XS	< 4	< 4.5	up to 3.5	up to 35.5	RY-WK002-XS	RY-WK003-XS
S	4.5 - 7	5 - 8.5	4 - 6.5	36 - 39	RY-WK002-SM	RY-WK003-SM
M	7.5 - 10	9 - 11	7 - 9.5	40 - 44	RY-WK002-MD	RY-WK003-MD
L	10.5 - 12.5	11.5 - 13.5	10 - 12	44 - 46.5	RY-WK002-LG	RY-WK003-LG
XL	12.5 +	13.5 +	12 +	46.5 +	RY-WK002-XL	RY-WK003-XL

## DH OFFLOADING WALKER®

### INDICATIONS FOR USE

- Treatment of plantar ulcers

### FEATURES AND BENEFITS

- Shear force reduction by stabilizing foot and ankle at 90°
- Wide low profile rocker bottom provides a smooth roll-through and increased stability for altered gaits
- The softgood's patented inner bootie prevents foot from migrating on the insole
- Designed around Össur's unique Active Offloading Insole
- Provides a safe environment for the simultaneous use of wound care material
- Easy to apply and modify

### REQUIRED MEASUREMENTS

- Shoe size



### ORDERING INFORMATION

#### DH OFFLOADING WALKER

Size	Shoe Size US (Men)	Shoe Size US (Women)	Shoe Size EU	Part#
X-Small	2 – 4	3.5 – 5.5	33 – 35.5	DH0200BLK
Small	4.5 – 7	6 – 8	36 – 39	DH0400BLK
Medium	7.5 – 10.5	8.5 – 11.5	39.5 – 44	DH0600BLK
Large	10.5 – 12.5	11.5 – 13.5	44.5 – 46.5	DH0800BLK
X-Large	12.5+	13.5+	47 – 50	DH1000BLK

### ACCESSORIES

#### REPLACEMENT PARTS

Size	Softgoods	Insoles
X-Small	10958	11742C
Small	10958	PP04C
Medium	10959	PP06C
Large	10960	11740C
X-Large	10960	11741C

## DH OFFLOADING POST-OP STYLE SHOE

### INDICATIONS FOR USE

- Plantar wounds

### FEATURES AND BENEFITS

- Designed to accommodate either the left or the right foot
- Designed around Össur's patented Active Offloading Insole
- Designed to accommodate bulky dressings
- Super-soft nylon upper with reinforced heel counter provides extra comfort and stability
- Patented removable toe piece aides in wound accessibility for daily wound site inspection

### REQUIRED MEASUREMENTS

- Shoe size



### ORDERING INFORMATION

#### DH POST-OP SHOE

Size	Shoe Size US (Men)	Shoe Size US (Women)	Shoe Size EU	Part#
Small	4 – 6	6.5 – 8.5	36 – 39	10340
Medium	6.5 – 8.5	9 – 11	39.5 – 44	10341
Large	9 – 11	12+	44.5 – 46.5	10342
X-Large	11.5 – 14	13+	47 – 50	10343

### ACCESSORIES

#### REPLACEMENT PARTS

Size	Insoles
Small	12365
Medium	12366
Large	12367
X-Large	12368



## PEDIATRIC WALKER

### INDICATIONS FOR USE

Foot and ankle conditions that may benefit from motion restriction, such as:

- Grade 2 and 3 ankle sprains
- Stable fractures
- During postoperative or rehab phase.

### FEATURES AND BENEFITS

- High quality rigid outer skeleton for protection
- Dense foam lining within the sole to provide comfort and support
- Shock absorbing sole to improve comfort and protects the user from slips, providing additional security in each step
- Uprights give additional stability by restricting motion of the leg
- Lightweight design helps in keeping kids active

### REQUIRED MEASUREMENTS

- Shoe size



## ORDERING INFORMATION

### PEDIATRIC WALKER

Size	Shoe Size US	Part#
Small	Child 6.5 – 9.5	PEW0400
Medium	Child 10 – Youth 1	PEW0600
Large	Youth 1.5 – 2.5	PEW0800

### REPLACEMENT LINERS

Size	Shoe Size US	Part#
Small	Child 6.5 – 9.5	PEW0001
Medium	Child 10 – Youth	PEW0002
Large	Youth 1.5 – 2.5	PEW0003

# METAL UPRIGHT AIR WALKER

## INDICATIONS FOR USE

Foot and ankle conditions that may benefit from motion restriction, such as:

- Grade 2 and 3 ankle sprains
- Stable fractures
- During postoperative or rehab phase

## FEATURES AND BENEFITS

- Malleable struts allow you to customized fit
- Rocker bottom sole promotes a more natural gait
- Pneumatic pump allows air compression to be adjusted as needed for a more customized fit.



## REQUIRED MEASUREMENTS

- Shoe size

## ORDERING INFORMATION

Size	Men's shoe size US	Women's shoe size US	Part#
X-Small	up to 4	up to 5.5	MAW0200
Small	4.5 – 7	6 – 8	MAW0400
Medium	7.5 – 10.5	8.5 – 11.5	MAW0600
Large	10.5 – 12.5	11.5 – 13.5	MAW0800
X-Large	12.5 & up	13.5 & up	MAW1000

## FORMFIT® PRO ANKLE

Össur Formfit® Pro Ankle is a compressive ankle sleeve with 2 integrated malleoli pads, designed to be worn after mild ankle injuries. The dynamic 3D MotionTech™ knit is supportive and meets certified medical grade compression requirements. Proven superior to leading brands for moisture wicking and breathability, your ankle stays cool during wear. CoolVent™ technology in the area in front of the ankle further enhances breathability and comfort.



MotionTech™



CoolVent™



SmartPad™



FoldLock™



### INDICATIONS FOR USE

For Ankle conditions that may benefit from compression, such as:

- Ankle injury rehabilitation
- Mild sprains and strains
- Mild instability


### CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated

### FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit with superior moisture wicking and breathability, keeping your ankle cool during wear
- Certified medical grade compression (20-30 mmHg)
- CoolVent™ technology in the area in front of the ankle for added breathability and comfort
- 2 integrated malleoli SmartPads™ for targeted support and cushioning
- Patented FoldLock™ system for easier and more secure application

### REQUIRED MEASUREMENTS

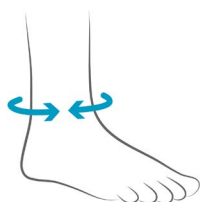

Measure ankle circumference.



#### BLUE

Size	Ankle circumference		Blue		Black	
	cm	inch	Left	Right	Left	Right
XS	18 - 20	7 - 7¾	FP201201	FP201101	FP200201	FP200101
S	20 - 23	7¾ - 9	FP201202	FP201102	FP200202	FP200102
M	23 - 26	9 - 10¼	FP201203	FP201103	FP200203	FP200103
L	26 - 29	10¼ - 11½	FP201204	FP201104	FP200204	FP200104
XL	29 - 32	11½ - 12½	FP201205	FP201105	FP200205	FP200105

# REBOUND® ANKLE BRACE

## INDICATIONS FOR USE

- Grade 1, 2, 3 and high ankle sprains
- Transition out of a walker and/or cast after fracture or surgery
- Chronic ankle instability
- Prophylactic use

## FEATURES AND BENEFITS

- Adjustable calf band accommodates a broad range of leg circumferences and creates a more custom fit
- Open front, crossover strap design allows simple and secure application
- Version with stability strap offers numerous strapping configurations for a range of stability levels. Can be stepped down or removed as the patient progresses through rehabilitation



## REQUIRED MEASUREMENTS

- Shoe size

## ORDERING INFORMATION

Size	Men's Shoe Size US	Women's Shoe size US	Shoe Size UK	Shoe Size EU	Left	Right
Small	6 – 7.5	7.5 – 9	2 – 7.5	35 – 39.5	B-231602512	B-231502512
Medium	8 – 12	9.5 – 13.5	8 – 10.5	39.5 – 42	B-231602513	B-231502513
Large	12.5 – 16	N/A	11+	42+	B-231602514	B-231502514


## ACCESSORIES

Description	Part#
Width Extender Kit	B-231900000
Stability Strap Kit	B-701160080

## FORMFIT® ANKLE BRACE

The Formfit® Ankle brace provides stability and protection for the ankle – whether used prophylactically, for chronic instability or following an injury. A hybrid between a rigid ankle stirrup and a soft ankle support, the Formfit Ankle ensures a close fit which limits inversion, eversion, flexion and extension movements. The low profile design fits easily into shoes and the quick lace up feature makes it easy to don and doff. Breathable material ensures it can be comfortably worn all day.

The Formfit Ankle with Figure 8 strap version, offers an even greater level of support, by further limiting the range of motion of the ankle.

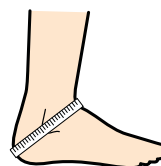

### INDICATIONS FOR USE

- Grade I-II sprains and strains
- Chronic instability
- Prophylactic use

### FEATURES AND BENEFITS

- Breathable material
- Quick lace Up
- Inversion/Eversion Control
- Available with Figure 8 strap

### REQUIRED MEASUREMENTS

- Measure ankle circumference diagonally over the base of the heel and across the ankle bone.

### ORDERING INFORMATION

Size	Ankle Circ.*	Formfit Ankle Brace	Formfit Ankle with Figure 8
X-Small	11 – 12" (28 – 30.5 cm)	B-212010001	B-212000001
Small	12 – 13" (30.5 – 33 cm)	B-212010002	B-212000002
Medium	13 – 14" (33 – 35.5 cm)	B-212010003	B-212000003
Large	14 – 15" (35.5 – 38 cm)	B-212010004	B-212000004
X-Large	15 – 16" (38 – 40.5 cm)	B-212010005	B-212000005

## FORMFIT® ANKLE WITH SPEEDLACE

The Formfit® Ankle Brace with Speedlace provides stability and protection for the ankle – whether used prophylactically, for chronic instability or following an injury. Made from durable materials, its low profile, quick-lace design is comfortable and fits easily into shoes. Removable semi-rigid stays provide additional support and the Figure 8 straps are designed for inversion/eversion control.

### INDICATIONS FOR USE

- Grade I-II sprains and strains
- Chronic instability
- Prophylactic use

### FEATURES AND BENEFITS

- Durable materials
- Quick-Pull closure straps
- Figure 8 straps
- Removable stays
- Circumferential cuff

### REQUIRED MEASUREMENTS

- Measure ankle circumference diagonally over the base of the heel and across the ankle bone.



## ORDERING INFORMATION

Size	Ankle Circ.*	Part#
XXSmall	9 – 10" (23 cm – 25.5 cm)	W-10621
XSmall	10 – 11" (25.5 cm – 28 cm)	W-10622
Small	11 – 12" (28 cm – 30.5 cm)	W-10623
Medium	12 – 13" (30.5 cm – 33 cm)	W-10625
Large	13 – 14" (33 cm – 35.5 cm)	W-10627
XLarge	14 – 15" (35.5 cm – 38 cm)	W-10628
XXLarge	15 – 16" (38 cm – 40.5 cm)	W-10629

\*Measure ankle circumference diagonally over the base of the heel and across the ankle bone.

## FORMFIT® ANKLE STIRRUPS

Formfit Ankle Stirrups are lightweight and sturdy. Whether worn to accommodate an acute injury, chronic instability, or post cast healing, Formfit Ankle Stirrups stabilize injured ankles by restricting the inversion/eversion motion. The universal design has been carefully engineered to accommodate the malleoli on both sides and ensures a great, comfortable fit. The slip resistant heel strap can be adjusted to accommodate most ankles. Our signature Flex-Edge™ overmold surrounds the stirrup's outer shell protecting patient's skin from any hard edges, further adding to a comfortable fit. The universal stirrup design is available with three different liner options: inflatable air, gel, or foam, offering each patient a solution that best fits their needs.



### INDICATIONS FOR USE

Situations requiring inversion/eversion restriction. These may include:

- Mild to severe ankle sprains
- Chronic ankle instability
- Post-cast and post walker support
- Ankle injury rehabilitation and/or post-surgical use

### FEATURES AND BENEFITS

- Universal shell, carefully crafted to fit both right and left ankles
- Soft Flex-Edge overmold for increased comfort
- Adjustable heel strap to optimize patient fit and comfort
- Slip-resistant silicone dots to reduce slipping and stabilize the foot inside of shoe
- 2 bidirectional hook and loop straps ensure a secure and stable fit
- Choice of inflatable, foam, and gel liners to suit your needs
- Available in Adult and Youth sizes

### REQUIRED MEASUREMENTS

- Patient's height

### ORDERING INFORMATION



Size	Patient's Height	Formfit Air Ankle Stirrup	Formfit Gel Ankle Stirrup	Formfit Foam Ankle Stirrup
Adult	Above 5'3" (160 cm)	70350	70351	70352
Youth	Below 5'3" (160 cm)	71350	71351	71352



## AIRFORM® UNIVERSAL INFLATABLE ANKLE STIRRUP

### INDICATIONS FOR USE

- Acute Ankle Injuries
- Post-cast healing and post-walker support
- Chronic Instability

### FEATURES AND BENEFITS

- Universal design fits both left and right ankles, reducing inventory needs
- Soft Flex-Edge molding provides a comfort barrier between the shell and ankle
- Inflatable air bladders provides adjustable pneumatic compression
- Adjustable heel strap allow for easy width adjustments



### ORDERING INFORMATION

Size	White Single	White 6 pack	Black Single
Adult	80250	802506	80350
Youth	80251	802516	

## AIRFORM® PRE-INFLATED ANKLE STIRRUP

### INDICATIONS FOR USE

- Ankle injury rehabilitation/ immobilization for grades I & II strains and sprains
- Chronic Instability
- Post Cast healing

### FEATURES AND BENEFITS

- Pre-inflated, pulsating air bladder with regulated chamber reduces bottoming out and decreases edema
- No straws or over inflation
- Adjustable pivoting heel strap increases comfort and allows for full range of motion
- Side specific right or left sizing



### ORDERING INFORMATION

Size	White Left	White Right
Adult	80100	80101
Youth	70100	70101

## FORMFIT® HONEYCOMB ANKLE STIRRUP

### INDICATIONS FOR USE

- Ankle injury rehabilitation/ immobilization for grades I & II strains and sprains
- Chronic Instability
- Post Cast healing

### FEATURES AND BENEFITS

- Patented Honeycomb pads
- Flex-Edges and adjustable pivoting heel straps



### ORDERING INFORMATION

Size	Left	Right
Adult	30100	30101
Youth	40100	40101

## EXOFORM<sup>®</sup> DORSAL NIGHT SPLINT

### INDICATIONS FOR USE

- Plantar fasciitis
- Post-static pain
- General heel pain
- Drop foot
- Achilles tendonitis
- Lower extremity overuse injuries

### FEATURES AND BENEFITS

- Adjustable aluminum stays can be positioned to maintain flexion between 85° – 90°
- Soft, Flex edge over molding improves patient comfort
- Full calf and heel strapping prevent brace from rotating
- Slip resistant padding



### REQUIRED MEASUREMENTS

- Shoe size

### ORDERING INFORMATION

Size	Men's Shoe Size US	Women's Shoe Size US	Shoe Size UK	Shoe Size EU	Part#
Medium	5 – 9	6 -10	4.5 – 8	37.5 – 42.5	W-50085
Large	9.5 – 14	10.5 – 14	8.5 – 13.5	43 – 48.5	W-50087

# FORMFIT® NIGHT SPLINT

## INDICATIONS FOR USE

- Plantar fasciitis
- Lower extremity overuse injuries
- Achilles tendinopathy

## FEATURES AND BENEFITS

- Straps adjust from 0° – 10° of dorsiflexion
- Contoured for a comfortable fit
- Soft foam interior is washable and breathable
- quick release buckles for easy application
- Slip resistant tread



## REQUIRED MEASUREMENTS

- Shoe size

## ORDERING INFORMATION

	Shoe Size US		Shoe Size UK		Shoe Size EU		Part#
Size	Male	Female	Male	Female	Male	Female	Standard
Small	Up to 7	Up to 7.5	Up to 6.5	Up to 5	Up to 39	Up to 38	50023
Medium	7.5-10	8-10.5	7 – 9.5	5.5 – 8	39.5 – 44	38.5 – 43	50025
Large	10.5 and up	11 and up	10 and up	8.5 and up	44.5 and up	43.5 and up	50027

## ACCESSORIES

Description	Size	Part#
Foam Wedge		10700
Replacement Liner	Small	11760
Replacement Liner	Medium	11761
Replacement Liner	Large	11762

## AIRFORM® NIGHT SPLINT

### INDICATIONS FOR USE

- Plantar Fasciitis
- Achilles Tendonitis
- Drop foot
- Post-static pain

### FEATURES AND BENEFITS

- Light weight, low profile design
- Easy to adjust
- Available in singles or 6 packs

### REQUIRED MEASUREMENTS

- Shoe size



### ORDERING INFORMATION

	Shoe Size US		Shoe Size UK		Shoe Size EU		Part#	
Size	Male	Female	Male	Female	Male	Female	Standard	6 Pack
Medium	5 – 9	6 – 10	4.5 – 8.5	3.5 – 7.5	37 – 43	36 – 42	50045	500456
Large	9.5 – 14	10.5 – 15	9 – 13.5	8 – 12	43.5 – 48.5	42.5 – 48	50047	500476

## SQUARE TOE POST-OP SHOE

### FEATURES AND BENEFITS

- Square Toe design to increase protection as square acts as bumper.
- Rocker bottom sole reduces pressure on plantar surface resulting in increased comfort and compliance.
- Strapless closure eliminates buckle pressure on the forefoot and can expand to accommodate dressing up to 17" (43cm) in circumference.



Shoe size.



### ORDERING INFORMATION

	Shoe Size US		Part#	
Size	Male	Female	Male	Female
Small	6 - 8	4 - 6	18113	18103
Medium	8.5 - 10	6.5 - 8	18115	18105
Large	10.5 - 12	8.5 - 10	18117	18107
X-Large	12.5 - 14	10.5 - 15	18118	-

## SOFT TOP POST-OP SHOE

### INDICATIONS FOR USE

- For recovery after soft tissue procedures and post trauma

### FEATURES AND BENEFITS

- The cotton-poly blend upper aids in breathability and comfort
- The contact closure system is designed for ease of use.

### REQUIRED MEASUREMENTS

- Shoe size



### ORDERING INFORMATION

	Shoe Size US		Shoe Size UK		Shoe Size EU		Part#	
Size	Male	Female	Male	Female	Male	Female	Male	Female
Pediatric	Preschool 10 - Youth 1						18002	
Small	4 - 6	5 - 6	3.5 - 5.5	2.5 - 3.5	35.5 - 38	35 - 36	18013	18003
Medium	6.5 - 8.5	7 - 8	6 - 8	4 - 5.5	38.5 - 42	36.5 - 38.5	18015	18005
Large	9 - 11	9 +	8.5 - 10.5	6 +	42.5 - 45	39 +	18017	18007
X-Large	11.5 - 14		11 - 13.5		45.5 - 48.5		18018	

## MESH TOP POST-OP SHOE

### INDICATIONS FOR USE

- For recovery after soft tissue procedures and post trauma

### FEATURES AND BENEFITS

- Breathable mesh upper with reinforced heel counter and padded heel collar, more stability overall to the rear of the foot while reducing slippage
- Semi-rigid sole provides superior protection and control after osteotomies
- Extended strap and dual side closure system accommodates extra bulky dressings

### REQUIRED MEASUREMENTS

- Shoe size



### ORDERING INFORMATION

	Shoe Size US		Shoe Size UK		Shoe Size EU			
Size	Male	Female	Male	Female	Male	Female	Male	Female
Small	4 – 6	5 – 6	3.5 – 5.5	2.5 – 3.5	35.5 – 38	35 – 36	17013	17003
Medium	6.5 – 8.5	7 – 8	6 – 8	4 – 5.5	38.5 – 42	36.5 – 38.5	17015	17005
Large	9 – 11	9 +	8.5 – 10.5	6 +	42.5 – 45	39 +	17017	17007
X-Large	11.5 – 14		11 – 13.5		45.5 – 48.5		17018	

## CANVAS ROCKER BOTTOM CAST SHOE

### INDICATIONS FOR USE

- For use with a short leg cast

### FEATURES AND BENEFITS

- Cushioned rocker bottom cast sandal
- Contact closure, open toe

### REQUIRED MEASUREMENTS

- Insole length



### ORDERING INFORMATION

Size	Insole length	Blue	Black
X-Small	7" (18 cm)	308XS	
Small	7.5 – 9" (19 – 23 cm)	308SM	B308SM
S/Medium	9 – 9.75" (23 – 25 cm)	308SM/MED	B308S/M
Medium	10 – 10.5" (25.5 – 26.5 cm)	308MED	B308MED
M/Large	10.5 – 11" (26.5 – 28 cm)	308MLG	B308MLG
Large	11 – 12" (28 – 30.5 cm)	308LG	B308LG
X-Large	12 – 13.75" (30.5 – 35 cm)	308XLG	B308XLG





SPECIALITY CARE,  
CASTING & TRACTION



## COLD RUSH®

### INDICATIONS FOR USE

- For cold therapy pain relief, as needed

### FEATURES AND BENEFITS

- Consistent cooling and excellent compression, even at high elevation.
- Brushless motor produces a whisper quiet unit that allows patients to get the rest they need to properly rehabilitate.
- Highest ice-to-water ratio in the industry and a design that produces less heat transfer from the motor allows therapy pain relief for up to 6 continuous hours.
- Unique locking lid prevents spills
- Insulated walls improve cooling



## COLD RUSH® COMPACT

### INDICATIONS FOR USE

- For anything that may benefit from cold therapy

### FEATURES AND BENEFITS

- Convenient, quiet and easy-to-use
- Single flow setting
- Easy fill instructions



### ORDERING INFORMATION

Description	Cold Rush Part #	Cold Rush Compact Part #
Device Only	B-232000010	B-232100010
Device w/ Knee Pad	B-232000011	B-232100011
Device w/ Unv Pad	B-232000012	B-232100012
Device w/ Hip Pad LT	B-232000013	B-232100013
Device w/ Hip Pad RT	B-232000014	B-232100014
Device w/ Shoulder Pad SM	B-232000015	B-232100015
Device w/ Shoulder Pad LG	B-232000016	B-232100016
Device w/ Ankle Pad	B-232000017	B-232100017
Device w/ Rectangular Pad SM	B-232000018	B-232100018
Device w/ Lumbar Pad	B-232000019	B-232100019


### ACCESSORY

Description	Part #
Replacement Power Cord	B-301090001


## COLD RUSH® PADS

Compatible with Cold Rush and Cold Rush Compact.


Description	Part #
Cold Rush knee pad	B-232002000
Cold Rush hip pad, left	B-232602100
Cold Rush hip pad, right	B-232502100
Cold Rush shoulder pad, small	B-232002202
Cold Rush shoulder pad, large	B-232002204
Cold Rush universal pad	B-232002300
Cold Rush rectangular pad, 5" x 10" (12.7 cm x 25.4 cm)	B-232002402
Cold Rush lumbar pad, large	B-232002404
Cold Rush foot & ankle pad	B-232002500

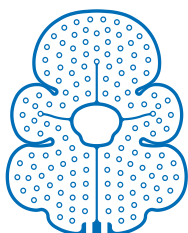


Knee Pad

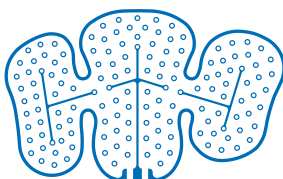


Hip Pad, Left

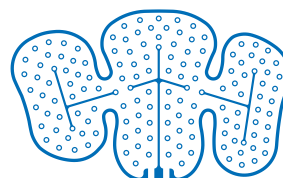


Hip Pad, Right

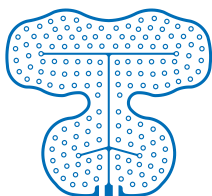


Shoulder Pad,  
Small

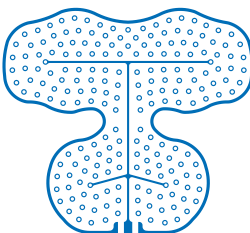


Shoulder Pad,  
Large

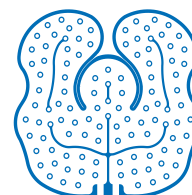


Universal Pad

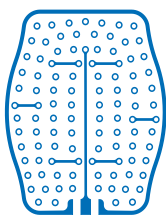

Lumbar Pad

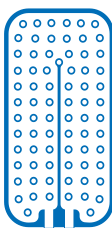

Rectangular Pad

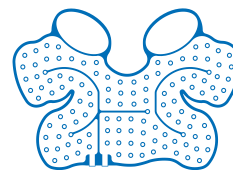

Foot & Ankle Pad

## CAST PADDING

### ORDERING INFORMATION

Width	U/M	Synthetic	Cotton
2"	12 rolls	V-101003	V-110002
3"	12 rolls	V-101003	V-110003
4"	12 rolls	V-101004	V-110004
6"	6 rolls	V-101006	V-110006

## STOCKINETTE

- 100% cotton or synthetic tubular stockinette
- Provides an additional layer between the skin and the cast

### ORDERING INFORMATION

Width	U/M	Synthetic	Cotton
1"	25 yard dispenser box	2101	2000A
2"	25 yard dispenser box	2102	2001
3"	25 yard dispenser box	2103	2002
4"	25 yard dispenser box	2104	2003
6"	25 yard dispenser box	2106	2005



# TECHFORM® PREMIUM

## INDICATIONS FOR USE

- Immobilization

## FEATURES AND BENEFITS

- Thinner material is lightweight and easy to cut
- Soft, conformable rolls allow for easy handling
- Excellent lamination ensures a strong and durable cast
- Smooth finished cast is non-abrasive and will not snag on clothing
- High radiotranslucence allows for clearer imaging through the cast
- Finger holes in the pouch allow for easy opening



## ORDERING INFORMATION

WIDTH	WH	BK	OR	RD	PK	HP	LB	DB	PR	GN	RB
1.5"	154WH										
2"	204WH	204BK	204OR	204RD	204PK	204HP	204LB	204DB	204PR	204GN	204RB
3"	304WH	304BK	304OR	304RD	304PK	304HP	304LB	304DB	304PR	304GN	304RB
4"	404WH	404BK	404OR	404RD	404PK	404HP	404LB	404DB	404PR	404GN	404RB
5"	504WH										

Note: Packed in boxes of 10

White (WH)

Black (BK)

Orange (OR)

Red (RD)

Pink (PK)

Hot Pink (HP)

Lt Blue (LB)

Dk Blue (DB)

Purple (PR)

Green (GN)

## TECHFORM® ORTHO-ROLL®

Developed in collaboration with orthopaedic professionals, Techform Ortho-Roll represents the first major breakthrough in orthopaedic splinting technology in decades.

In focus groups, Techform Ortho-Roll rated from "Good" to "Excellent" in every key category, including handling, conformability, padding, water retention, dispensing method, ease-of-use, overall appearance, and overall satisfaction. Techform Ortho-Roll simply performs better.

### FEATURES AND BENEFITS

- Quickdry is a patented, revolutionary orthopaedic padding material. Quickdry padding never really stays wet. Water simply passes straight through it. The result is less risk of skin maceration. Plus, it saves you application time.
- Instructions are right on the box
- Box opens easily along perforation
- Closure rod creates an air tight seal



### ORDERING INFORMATION

#### TECHFORM ORTHO-ROLL

Part#	Size	Part#	Size
TR0115	1" x 15ft.	TR0415	4" x 15ft.
TR0215	2" x 15ft.	TR0515	5" x 15ft.
TR0315	3" x 15ft.	TR0615	6" x 15ft.

#### ACCESSORIES

Part#	Size
13302	Positioning Strap (12 pack)
13332	Closure Rod
P-550606	Splinting Guide Poster

## TECHFORM® PRE-CUT SPLINTS

Techform Pre-Cut Splints have the same key features as our patented Ortho-Roll splinting material, including our patented Quickdry padding. Each splint comes pre-packaged in an individual foil pouch, perfect for those who prefer to eliminate waste of splint material.

### FEATURES AND BENEFITS

- Our selection includes 5 sizes, a full variety of lengths and widths to accommodate all your splinting needs.
- Pre-cut splints come in various lengths so you can choose the appropriate size to match your specific needs



### ORDERING INFORMATION

Part#	Size	Quantity
TC0312	3" x 12"	10/Box
TC0335	3" x 35"	5/Box
TC0415	4" x 15"	10/Box
TC0430	4" x 30"	5/Box
TC0530	5" x 30"	10/Box

## DUALPULSE™ COMPACT

Össur DualPulse Compact is an approachable solution for NMES and TENS. The user-friendly design is cordless and ultra-portable for use on the go. A simplified design over traditional NMES/TENS units allows for easily switching between modes and adjusting the intensity with the push of a button.



### INDICATIONS FOR USE

#### NMES:

- For the prevention or retardation of disuse atrophy and muscle re-education
- Relaxation of muscle spasms, muscle re-education, and maintaining or increasing range of motion
- Immediate post-surgical stimulation of calf muscles to prevent venous thrombosis

#### TENS:

- For symptomatic relief or management of chronic intractable pain
- Post-surgical and post-traumatic acute pain

### CONTRAINDICATIONS

#### NMES:

- Powered muscle stimulators should not be used on patients with cardiac demand pacemakers

#### TENS:

- Do not use within first 36 weeks of pregnancy except on the advice of your doctor or midwife
- Do not place electrodes on abdomen during pregnancy
- TENS devices can affect the operation of demand type cardiac pacemakers
- In patients with known myocardial disease, epilepsy or arrhythmias, TENS should be used only with consultation and evaluation by a physician
- Do not use TENS on the eyelids
- Do not place electrodes directly over the carotid sinus nerves or arteries and laryngeal or pharyngeal muscles
- Do not apply TENS for undiagnosed pain syndromes until etiology (actual cause) is established
- Do not place electrodes in any position that allows current to flow transcranially (through the head)
- Do not use TENS on broken skin or on areas where normal sensation is absent
- Do not use TENS on children under the age of 12 unless under medical supervision

### FEATURES AND BENEFITS

- Rechargeable and cordless for ultimate portability
- High-visibility screen displays intensity level
- 19 intensity levels
- User-replaceable electrode pads
- Includes DualPulse Compact, (1) electrode adhesive gel pad, and charger

### ORDERING INFORMATION

Description	Part#
DualPulse Compact	DPC
Replacement adhesive electrode gel pad	DPC-RP





[WWW.OSSUR.COM](http://WWW.OSSUR.COM)

USA (800) 233-6263  
[WWW.OSSUR.COM](http://WWW.OSSUR.COM)

CANADA (800) 663-5982  
[WWW.OSSUR.CA](http://WWW.OSSUR.CA)

