

Bracing & Support

2024 Catalogus



ÖSSUR BRACING & SUPPORT CATALOGUE

TABLE OF CONTENT

ORDERING INFORMATION	8
INTRODUCTION TO SOLUTIONS.....	10
KNEE	13
UNLOADER ONE® X	15
UNLOADER ONE®	21
UNLOADER® CUSTOM	24
MEASUREMENT FORM CAST, DIGITAL KNEEBRACE	26
REBOUND® DUAL	27
REBOUND® DUAL ST.....	31
REBOUND® DUAL BASIC AND REBOUND® DUAL BASIC ST.....	34
REBOUND® DUAL CUSTOM	36
OA EASE.....	38
FORMFIT® PRO KNEE OA	40
REBOUND® CARTILAGE	41
FORMFIT® TRACKER	44
FORMFIT® HINGED LATERAL J	45
CTi®3	46
18 CTi® CUSTOM	48
MEASUREMENT FORM CTi® CUSTOM BRACE	51
REBOUND® PCL	54
REBOUND® ACL	58
EXTREME®	62
18 REBOUND® POST-OP KNEE	63
FORMFIT POST-OP KNEE	64
REBOUND® KNEE IMMOBILIZER UNIVERSAL	65
FORMFIT® PRO KNEE	66
FORMFIT® PRO KNEE FLITE	68
FORMFIT® KNEE HINGED	69
FORMFIT® KNEE MCL	70
FORMFIT® KNEE ROM BRACE	71
18 C180™ ROCKET	72
SPINE	73
MIAMI J®	75
MIAMI J® SELECT	77
MIAMI JTO® THORACIC EXTENSION	78
18 OCCIAN® BACK	79
18 MIAMI Jr®	80
PHILADELPHIA® ADJUSTABLE TRACHEOTOMY COLLAR	81
18 PHILADELPHIA® TRACHEOTOMY COLLAR	82
NECLOC®	83
18 NECLOC® KIDS	84
18 PAPOOSE®	85

ÖSSUR BRACING & SUPPORT CATALOGUE

MIAMI LSO®	86
MIAMI TLSO 464	88
MIAMI TLSO 456	90
MIAMI LUMBAR® POSTEO	92
FORMFIT® PRO BACK	93
FORMFIT® BACK SUPPORT AIR	94
FORMFIT® BACK SUPPORT	95
HIP	97
REBOUND® HIP	99
UNLOADER® HIP	101
UPPER EXTREMITY	103
FORMFIT® SHOULDER BRACE	105
PREMIUM SHOULDER IMMOBILISER	106
BERGSCHULTZ UNIVERSAL ARM SLING	106
HIGH ARM SLING	107
18 PAEDIATRIC UNIVERSAL ARM SLING	107
FORMFIT® PRO ELBOW	108
REBOUND® POST-OP ELBOW	109
FORMFIT® TENNIS ELBOW	111
FORMFIT® PRO WRIST	112
FORMFIT® WRIST / FORMFIT® WRIST & FOREARM	113
18 FORMFIT® UNIVERSAL WRIST & THUMB	114
18 FORMFIT® THUMB SPICA	115
WRIST / THUMB BRACE	116
SPECTRA® WRIST BRACE	117
FOOT AND ANKLE.....	119
RANGE OF MOTION WALKER	121
REBOUND® DIABETIC WALKER	122
DH OFFLOADING WALKER®	124
REBOUND® AIR WALKER	125
FORMFIT® WALKER AIR	127
FORMFIT® WALKER	128
18 PAEDIATRIC WALKER	129
AFO DYNAMIC	130
AFO LIGHT	132
AFO LEAF SPRING	133
PROLITE AFO	134
EXTRA STRONG PROLITE	134
REBOUND FOOT-UP®	135
FOOT-UP®	136
FORMFIT® PRO ANKLE	137
FORMFIT® ANKLE BRACE	139
FORMFIT® ANKLE WITH SPEEDLACE	140
18 FORMFIT® ANKLE STIRRUPS	141

ÖSSUR BRACING & SUPPORT CATALOGUE

EXOFORM® DORSAL NIGHT SPLINT	142
FORMFIT® NIGHT SPLINT	143
AIRFORM® NIGHT SPLINT	144
DH OFFLOADING POST-OP STYLE SHOE	145
MESH TOP POST-OP SHOE	146
SOFT TOP POST-OP SHOE	146
CANVAS ROCKER BOTTOM CAST SHOE	147
NOTES	148



WELCOME FROM ÖSSUR!

As the background might suggest, our roots are not in the Netherlands, but in Iceland, a special and mystical country. Founded in 1971, we are active in the field of prosthetics and orthosis. We would like to take this opportunity to tell you a little bit more about who we are, what we do and what we would like to achieve.

The growth of a company can never be predicted, but great energy and enthusiasm can take you far. The foundation of our company is that we strive daily to improve people's mobility and help them live a 'life without limitations', with more than 3,700 employees around the world happy to contribute to this cause.

I hope you enjoy reading this and see you soon.

On behalf of the Benelux team,



Dries Coenen
Director Sales Benelux

OUR MISSION, OUR PASSION

Passion for innovation and quality has been driving successful patient and customer outcomes for decades. Our mission is to improve people's mobility so they can live a 'life without limitations'.

We have the vision to be a leading organization in the field of non-invasive orthopaedic devices. We focus on improving people's mobility by successfully making innovative technology available, within the field of prosthetics and orthotics. We also have extensive knowledge of compression therapy. Össur generates value for its customers, shareholders and employees through a strategy focused on innovation, efficiency and growth.

Innovation

Execute Ideas that add Value

We embrace innovation in all our actions by creating value for our customers. We will be at the forefront of indication-related innovation to ensure our consistently strong position in the market.

Efficiency

Do Business Efficiently

We strive to increase efficiency and drive continuous improvement. We run efficient operations in the most optimal locations, hire passionate employees and deliver strong profit and cash flow.

Growth

Sustainable Growth

We will achieve growth by successfully commercializing our innovation through our local go-to-market strategy and commitment to our customers' needs. We will further develop our business and leverage untapped market opportunities.

Our core values

HONESTY
Stay true

FRUGALITY
Make every step count

COURAGE
Aim higher

THE CUSTOMER FIRST

We can only be successful if our customers are too. Therefore, we believe that it is important that every contact with our clients adds value and the interests of the user are always paramount. We want to be a partner for our customers in the care issues that arise every day, so that they can focus completely on their patient's recovery and ensuring a positive experience.

The collection and dispatch of orders takes place from our logistics center in Eindhoven. Our Customer Service department and After Sales department are also located here, and they are available every working day to take care of orders and to follow up on practical questions.

In addition, we have a team of specialists in sales and clinical support: our Area Managers. They are on the road every day for customer visits, to provide support with clinical issues and work with various departments.

At Össur, we only go for long-term customer relationships and do everything we can to exceed customer expectations. In doing so, we adhere to our values: honesty, frugality and courage. We strive to do better every day and are happy to discuss this with our customers.



ORDERING INFORMATION

- Please quote your account number when placing an order
- Please quote model numbers and any other specification required when ordering
- Please quote your purchase order number
- Orders placed by telephone and then emailed should be clearly marked with 'CONFIRMATION ORDER' to avoid risk of duplication

RESPONSE TO ORDERS

We are proud of our service to our customers and maintain large stocks of products to expedite delivery. Orders received before 14.00 will be processed and dispatched same day.

METHOD OF DISPATCH

Your order will be dispatched by the most economical means available, with regard to weight / volume and the degree of urgency. Unless otherwise instructed, orders will be sent by standard 2 - 3 day delivery. Timed deliveries are available on request. Please contact Customer Care for pricing.

CUSTOM PRODUCTS

Össur®, through our own manufacturing facility, have the expertise and resources to create one-off custom requirements for products such as the Unloader Custom, Rebound PCL Custom, Rebound ACL Custom, CTi Custom. Where this is the case, please allow 10 working days for delivery.

INCORRECT DELIVERY

On receipt of the goods the customer is required to check the consignment. In the unlikely event that the type or quantity of goods is incorrect then you are requested to notify our Customer Care Department within 7 working days of the date of delivery. After 7 working days Össur® cannot accept responsibility for an alleged incorrect delivery.

RETURN OF GOODS POLICY

You can return your unused products to us free of charge for up to 30 days after you purchased them. Authorization must be issued by Össur® Europe B.V. prior to goods being returned, please contact Customer Care for a return reference number and ensure sections 1-5 of Return Advice Note (on the back of the dispatch note) are fully completed. Soiled, custom manufactured, modified or obsolete goods may not be returned. Goods accepted for return must be in a resalable condition, guaranteed infection free and in the original packaging. Össur® reserves the right to charge delivery costs and / or re-stocking fee of up to 20% on returned goods.

30-DAY MONEY BACK GUARANTEE ON THE UNLOADER ONE AND UNLOADER ONE X

A product that is risk-free; if the product doesn't work for a patient, you get a full refund.

ÖSSUR NO QUIBBLES GUARANTEE.

At Össur we know that accidents can happen. To put your mind at ease and to avoid unexpected costs, we are pleased to announce that the Unloader One is covered by our Össur Care, No Quibbles Guarantee.

This coverage runs concurrent with the original 2 year warranty as applicable and covers accidental damage during the normal working process of the brace and in accordance with the safe usage guidelines provided in the product manuals.

TERMS AND CONDITIONS OF SALE

Our full Terms and Conditions of Sale can be provided upon request.

The product range and availability may change during the period that the current catalogue runs.

The returns policy does not apply to custom made or modified goods. Statutory rights are not affected. Please check with Customer Care at time of sale if in doubt.

Telephone Orders: 00800 3539 3668

E-Mail: orders.benelux@ossur.com

SINGLE PATIENT USE

It is a condition of Sale that any orthopaedic product purchased from Össur® is restricted to 'Single use Only' by the originally-fitted patient in order to protect the care provider and the patient against potentially adverse consequences of infectious disease transmission or material instability resulting from previous use by the original wearer. Any expressed or implied warranties are voided if the orthosis is re-used for another patient. Any orthosis purchased from Össur® must be prescribed and applied by a qualified clinician to ensure that it meets the needs of the particular patient and accomplishes the desired results.



ÖSSUR OA SOLUTIONS

Enhance quality of life and mobility for people living with osteoarthritis.

These product solutions have been built on decades of experience and are designed to enhance the quality of life of people living with osteoarthritis by reducing pain and improving mobility. Through innovation leadership, OA expertise and partnerships with medical professionals, we offer comprehensive, proven and effective treatment options.

UNLOADER®

The Össur Unloader® brand name has become synonymous with osteoarthritis bracing, representing OA Solutions' technology as the industry standard, with more than 15 clinical studies demonstrating its efficacy.

Össur strives to maintain its position as the innovation leader for OA Solutions by continuing to develop simple, yet effective clinically proven treatment solutions that increase people's quality of life.



ÖSSUR INJURY SOLUTIONS

For people recovering from fractures, ligament injuries or in need of post-operative treatment solutions.

These product solutions have been built on decades of experience and are designed to enhance the healing process of bone and soft tissue injuries. Through innovation leadership, proven biomechanical approaches, and partnerships with leading medical professionals, we strive to offer comprehensive and effective treatment options.

Össur Injury Solutions are globally recognised, primarily under the brand names CTi®, Rebound® and Miami®, all of which are developed to assist healing and increase people's quality of life.

Our expertise in Injury Solutions has driven us to take the next step forward by developing indication-specific products which are designed to optimise healing while maintaining function and mobility. These Functional Healing products incorporate Össur proprietary technology and are designed to enhance clinically relevant outcomes.



FUNCTIONAL HEALING –
WHERE MOBILITY MEANS RECOVERY.

KNEE

KNEE
SOLUTIONS



TABLE OF CONTENTS

KNEE	13
UNLOADER ONE® X	15
UNLOADER ONE®	21
UNLOADER® CUSTOM	24
MEASUREMENT FORM CAST, DIGITAL KNEEBRACE	26
REBOUND® DUAL	27
REBOUND® DUAL ST	31
REBOUND® DUAL BASIC AND REBOUND® DUAL BASIC ST	34
REBOUND® DUAL CUSTOM	36
OA EASE	38
FORMFIT® PRO KNEE OA	40
REBOUND® CARTILAGE	41
FORMFIT® TRACKER	44
FORMFIT® HINGED LATERAL J	45
CTi®3	46
18 CTi® CUSTOM	48
MEASUREMENT FORM CTi® CUSTOM BRACE	51
REBOUND® PCL	54
REBOUND® ACL	58
EXTREME®	62
18 REBOUND® POST-OP KNEE	63
FORMFIT POST-OP KNEE	64
REBOUND® KNEE IMMOBILIZER UNIVERSAL	65
FORMFIT® PRO KNEE	66
FORMFIT® PRO KNEE FLITE	68
FORMFIT® KNEE HINGED	69
FORMFIT® KNEE MCL	70
FORMFIT® KNEE ROM BRACE	71
18 C180™ ROCKET	72

UNLOADER ONE® X

Unloader One X improves mobility by providing pain relief from knee osteoarthritis, degenerative meniscal tears, and other unicompartmental knee conditions that may benefit from unloading.

It features Össur's unique, proprietary, and clinically proven 3-point Leverage System that generates unicompartmental unloading of the knee joint.

Available in custom and OTS versions to help you fit your patient's different leg anatomies.



INDICATIONS FOR USE

- Moderate to severe unicompartmental knee osteoarthritis
- Degenerative meniscal tears
- Other knee conditions such as:
 - Articular cartilage defect repair
 - Meniscal repair
 - Avascular necrosis
 - Condylar bone marrow lesions
 - Tibial plateau fractures

FEATURES AND BENEFITS

- Relieves Pain - 3 Point Leverage System with Dynamic Force System® straps, rigid shells, and a single upright hinge biomechanically unloads the affected knee compartment.
- Control - Tool-free SmartDosing® dials enable the patient to fine-tune the strap tension to control pain.
- Tool-Free - Tool-free polycentric hinge with optional ROM stops for range-of-motion restriction.
- Comfortable - Skin-friendly liners feel great against the skin and reduce pressure points. Anti-migration strap technology design captures the widest part of the calf to reduce brace migration.
- Easy-to-Use - Alignment guide on hinge facilitates correct positioning of the brace on the leg and color coded touch points aid in easy donning and doffing.
- Easy-to-Wear - Slender frame with soft edges easily contours to individual anatomies and remains discreet whether worn under or over clothing.
- Easy Care - Entire brace is machine washable (washbag included).
- Field serviceable.


UNLOADER ONE® X

REQUIRED MEASUREMENTS: UNLOADER ONE X OTS



Measure calf circumference 15 cm below mid-patella. Leg in extension.

Measure leg length from floor to groin.

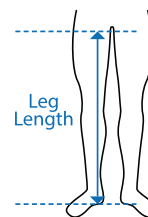

Size	Calf circumference 15 cm below mid patella
XS - M	25 - 43
L - XL	43 - 62

Model	Leg Length
Standard	> 85 cm (33.5")
Short	≤ 85 cm (33.5")

COLORS

OTS COLOR



Black Brace /
Black Components

Actual colors may vary.

ORDERING INFORMATION

Hinge is placed on the side of the affected compartment.
Short version recommended for patients with a leg length of ≤ 85 cm.

OTS

Standard	Right Medial	Right Lateral	Left Medial	Left Lateral
XS - M	UOX0518713	UOX0528713	UOX0618713	UOX0628713
L - XL	UOX0518715	UOX0528715	UOX0618715	UOX0628715

Short	Right Medial	Right Lateral	Left Medial	Left Lateral
XS - M	UOX1518713	UOX1528713	UOX1618713	UOX1628713
L - XL	UOX1518715	UOX1528715	UOX1618715	UOX1628715


UNLOADER ONE® X

OTS - NO BOX

Standard	Right Medial	Right Lateral	Left Medial	Left Lateral
XS - M	UOX0518718	UOX0528718	UOX0618718	UOX0628718
L - XL	UOX0518719	UOX0528719	UOX0618719	UOX0628719


Short	Right Medial	Right Lateral	Left Medial	Left Lateral
XS - M	UOX1518718	UOX1528718	UOX1618718	UOX1628718
L - XL	UOX1518719	UOX1528719	UOX1618719	UOX1628719

REPLACEMENT PARTS

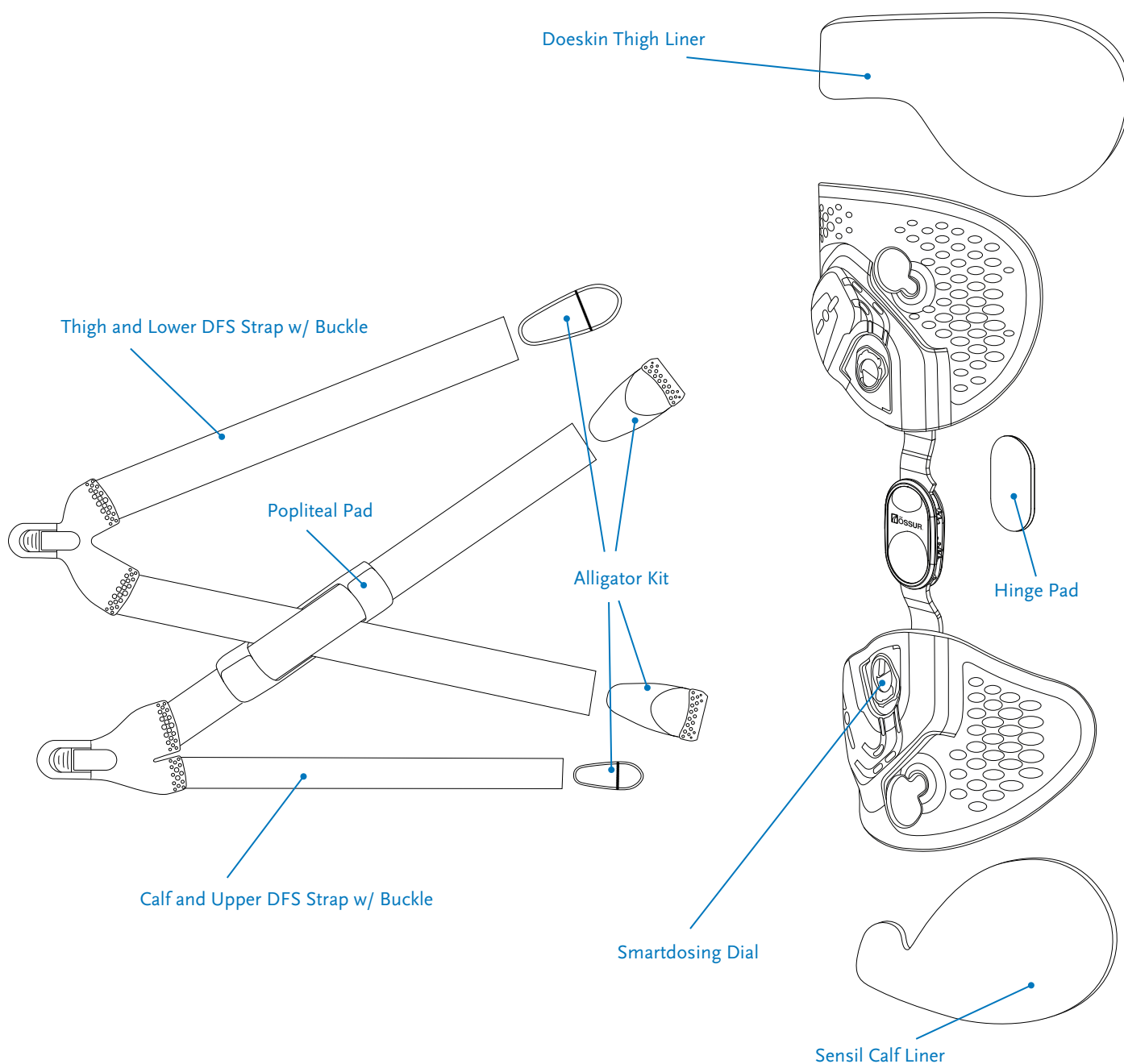

UNLOADER ONE® X

ACCESSORIES

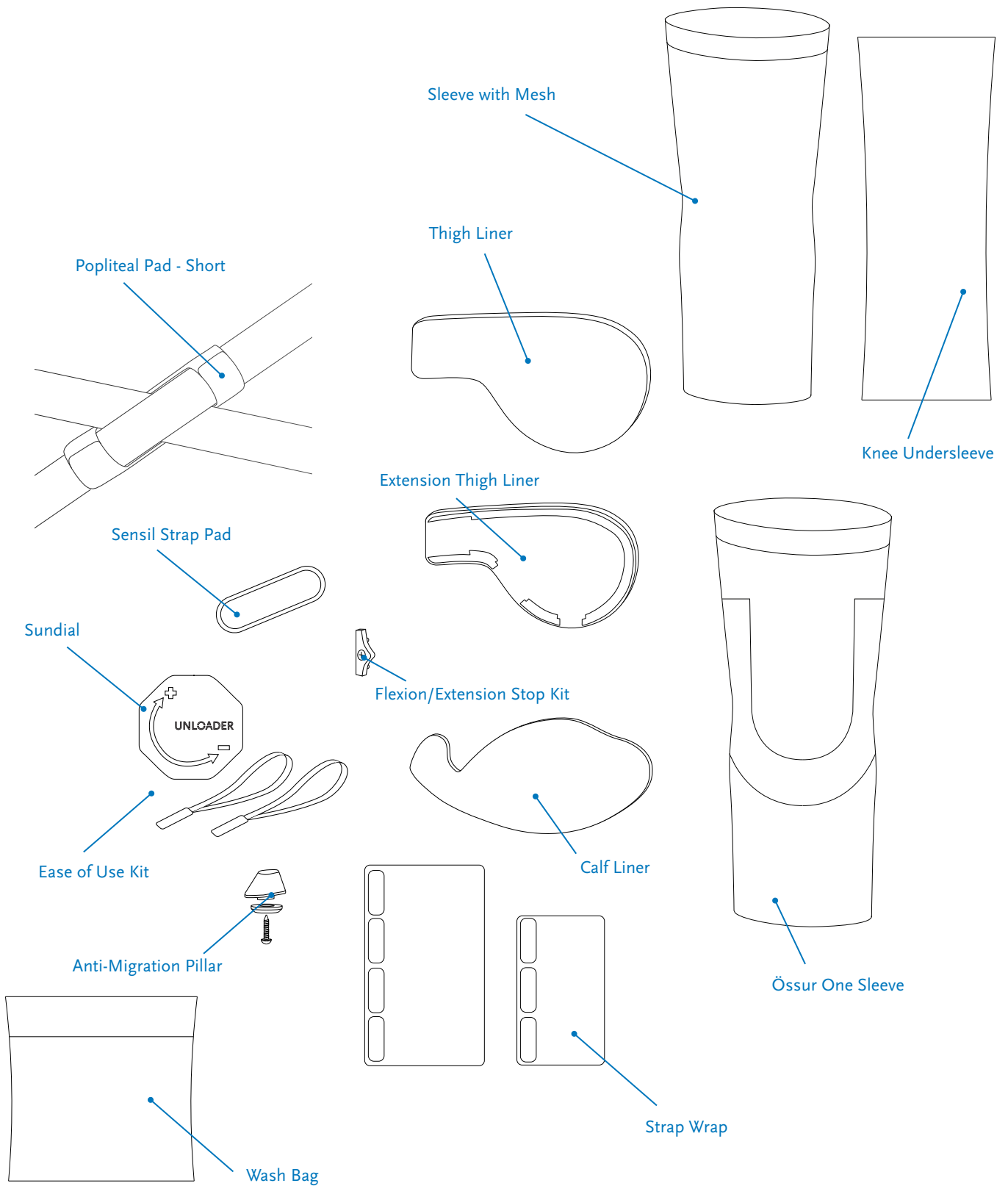

UNLOADER ONE® X

THIGH LINERS	THIGH, Right Medial, Left Lateral	THIGH, Right Lateral, Left Medial	THIGH EXTENSION, Right Medial, Left Lateral	THIGH EXTENSION, Right Lateral, Left Medial
Black, Sensil	UOX526801	UOX525801	NA	NA
Black, Doeskin	UOX526901	UOX525901	UOX546901	UOX545901

CALF LINERS	Right Medial, Left Lateral	Right Lateral, Left Medial
Black, Sensil XS-M	UOX516952	UOX515952
Black, Sensil L-XL	UOX516962	UOX515962
Black, Doeskin XS-M	UOX516951	UOX515951
Black, Doeskin L-XL	UOX516961	UOX515961

THIGH AND LOWER DFS STRAP*	Right Medial, Left Lateral	Right Lateral, Left Medial
Black	UOX406902	UOX405902

*Includes Thigh and DFS Strap with buckle

CALF AND UPPER DFS STRAP*	Right Medial, Left Lateral	Right Lateral, Left Medial
Black	UOX406901	UOX405901

*Includes Calf and DFS Strap with buckle

UOX SERVICE KIT*	Right Medial, Left Lateral	Right Lateral, Left Medial
Black, XS-M	UOX416901	UOX415901
Black, L-XL	UOX416911	UOX415911

* Includes Doeskin Thigh Liner, Sensil Calf Liner, Thigh and Lower DFS Strap with Buckle, Calf and Upper DFS Strap with Buckle, Hinge Pad, Popliteal Pad, and Alligator Kit

ANTI-MIGRATION PILLAR	Right Medial, Left Lateral	Right Lateral, Left Medial
Black	UOX700015	UOX700035

UNLOADER ONE® X

OTHER REPLACEMENT PARTS AND ACCESSORIES

Description	Black	Grey
Alligator Kit*	UOX900001	UOX900002
Hinge Pad	UOX700091	UOX700011
Popliteal Pad	UOX700012	UOX700092
Popliteal Pad - Short	UOX700022	UOX700082
Sensil Strap Pad	UON0181	UON0182
Sensil Strap Pad - 10 Pieces	UOX700113	UOX700193
Smartdosing Dial	UOX1000001	UOX1000002
Strap Wrap Set	UOX700014	UOX700094

* Includes 4 Alligator Clips: 2 DFS, 1 Thigh and 1 Calf

Description	Part #
Ease of Use Kit*	UOX800001
OA Hinge Extension Stop Tree (5°, 10°, 15°, 30°)	OAH40029
OA Hinge Flexion Stop Tree (0°, 30°, 60°, 90°)	OAH40039
Sundial	UOX1200001
Wash Bag	UON60200

* Includes Sundial and 2 buckle finger-loops

ÖSSUR ONE SLEEVES*

Size	Calf circumference 15 cm below mid patella	Thigh circumference 15 cm above mid patella	Grey	Black
2XS	25 - 29cm	31 - 36cm	OSO60011	OSO60021
XS	29 - 33cm	36 - 42cm	OSO60012	OSO60022
S	33 - 37cm	42 - 48cm	OSO60013	OSO60023
M	37 - 42cm	48 - 55cm	OSO60014	OSO60024
L	42 - 47cm	55 - 62cm	OSO60015	OSO60025
XL	47 - 52cm	62 - 70cm	OSO60016	OSO60026
XXL	52 - 57cm	70 - 78cm	OSO60017	OSO60027
3XL	57 - 62cm	78 - 86cm	OSO60018	OSO60028



* Össur One: 51% Tencel™/30% Polyamide/19% Elastane

SLEEVE WITH MESH*

Size	Measurements	Part #
XS	< 43cm	SLV60011
S	43 - 46cm	SLV60012
M	46 - 51cm	SLV60013
L	51 - 53cm	SLV60014
XL	53 - 56cm	SLV60015
XXL	56 - 61cm	SLV60016



* Sleeve with Mesh: Main fabric: 92% Cotton/8% Spandex - Mesh: 100% Polyester - Band: 80% Polyester/20% Spandex

ÖSSUR KNEE UNDERSLEEVE*

Size	Measurements	Grey	Black
XS	30.5 - 38cm	B-250150201	B-250150101
S	38.1 - 45cm	B-250150202	B-250150102
M	45.1 - 49.5cm	B-250150203	B-250150103
L	49.6 - 54.5cm	B-250150204	B-250150104
XL	54.6 - 62cm	B-250150205	B-250150105
XXL	62 - 71cm	B-250150206	B-250150106

Note: When using an undersleeve we advice to use the Silicone AMS (Anti-Migration System) to minimize migration.

* Össur Knee Undersleeve: 80% Nylon/20% Spandex

UNLOADER ONE®

Unloader One® is designed to relieve osteoarthritis knee pain and wearing it may help improve mobility. It features Össur's unique, patented 3-point leverage system, which combines a single hinge with dual Dynamic Force System (DFS) straps to provide unloading of the knee joint. Clinical studies demonstrate that this system may relieve osteoarthritis knee pain and improve function and quality of life while being worn. Colour-coded Quick Fit buckles make donning and doffing simple, while flexible shells and breathable liners are designed to improve comfort and reduce migration, thus helping to ensure compliance.



INDICATIONS FOR USE

- Moderate to severe unicompartmental knee osteoarthritis
- Degenerative meniscal tears
- Other knee conditions that may benefit from unicompartmental unloading, such as Articular cartilage defect repair, Avascular necrosis, Tibial plateau fracture, Bone marrow lesions (bone bruises)


FEATURES AND BENEFITS

- Dual Dynamic Force Straps® designed to reduce pressure on sensitive bony surfaces of the knee, resulting in reduction of pain
- Lightweight frame, perforated for flexibility and ventilation, with flexible, low profile upright that contours to patient anatomy and allows use underneath clothing
- Tool-free SmartDosing® dial enables fine-tuned pain management and can be adjusted even when wearing the brace under clothing
- Colour-coded quick fit buckles enable easy application and removal
- Proprietary Sensil® silicone liners designed to reduce migration
- Available in standard and short version, for a perfect fit on shorter legs, too

REQUIRED MEASUREMENTS

- Circumference measurement: 15cm below mid-patella

Size	Measurements
S	31 - 35.5cm
M	35.5 - 42.5cm
L	42.5 - 51cm
XL	51 - 61.5cm

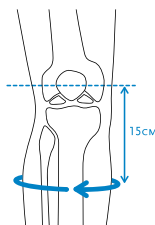

UNLOADER ONE®

ORDERING INFORMATION

UNLOADER ONE WITH SMARTDOSING

Size	Calf circumference	Left medial	Right medial	Left lateral	Right lateral
S	31 - 35.5cm	B-240618712	B-240518712	B-240628712	B-240528712
M	35.5 - 42.5cm	B-240618713	B-240518713	B-240628713	B-240528713
L	42.5 - 51cm	B-240618714	B-240518714	B-240628714	B-240528714
XL	51 - 61.5cm	B-240618715	B-240518715	B-240628715	B-240528715

UNLOADER ONE SHORT WITH SMARTDOSING

Size	Calf circumference	Left medial	Right medial	Left lateral	Right lateral
XS	25 - 31cm	B-241618711	B-241518711	B-241628711	B-241528711
S	31 - 35.5cm	B-241618712	B-241518712	B-241628712	B-241528712
M	35.5 - 42.5cm	B-241618713	B-241518713	B-241628713	B-241528713
L	42.5-51cm	B-241618714	B-241518714	B-241628714	B-241528714
XL	51 - 61.5cm	B-241618715	B-241518715	B-241628715	B-241528715

UNLOADER ONE WITH RATCHET

Size	Calf circumference	Left medial	Right medial	Left lateral	Right lateral
S	31 - 35.5cm	B-240619712	B-240519712	B-240629712	B-240529712
M	35.5 - 42.5cm	B-240619713	B-240519713	B-240629713	B-240529713
L	42.5 - 51cm	B-240619714	B-240519714	B-240629714	B-240529714
XL	51 - 61.5cm	B-240619715	B-240519715	B-240629715	B-240529715

UNLOADER ONE SHORT WITH RATCHET

Size	Calf circumference	Left medial	Right medial	Left lateral	Right lateral
XS	25 - 31cm	B-241619711	B-241519711	B-241629711	B-241529711
S	31 - 35.5cm	B-241619712	B-241519712	B-241629712	B-241529712
M	35.5 - 42.5cm	B-241619713	B-241519713	B-241629713	B-241529713
L	42.5 - 51cm	B-241619714	B-241519714	B-241629714	B-241529714
XL	51 - 61.5cm	B-241619715	B-241519715	B-241629715	B-241529715

Standard version is recommended for patients above 1.65m
Short version is recommended for patients under 1.65m tall.
Hinge is placed on the side of the affected compartment

ACCESSORIES

ACCESSORY KITS

Description	SMARTDOSING		RATCHET	
	RM/LL	RL/LM	RM/LL	RL/LM
Strap & Strap Pad & Buckle Clasp Kit	B-705277003	B-705278003	B-701217008	B-701218008
Strap & Strap Pad Kit	B-705270003		B-701210006	
Strap Kit	B-701270013		B-701210004	
Strap Kit Short	B-701240012			
Pad Kit	B-702210003		B-702210003	

REPLACEMENT PARTS AND ACCESSORIES

Description	Part#
Extension Stop Kit (5°, 10°, 15°, 30°)	OAH40029
Flexion Stop Kit (0°, 30°, 60°, 90°)	OAH40039

UNLOADER ONE®

ACCESSORIES

ACCESSORY KITS

Description	Right medial/Left lateral	Right lateral/Left medial
Buckle Clasp Kit	B-708217003	B-708218003
Alligator Clip Kit (included in all strap kits)	B-704210003	
Extension Stop Kit	B-301040046	
Twist-Fit Hook & Loop Attachment Kit	B-721240003	

UNDERSLEEVE

Size	Thigh Circumference	Part#
XS	25.4 - 30.9cm	B-250150201
S	30.9 - 35.6cm	B-250150202
M	35.6 - 42.5cm	B-250150203
L	42.5 - 50.8cm	B-250150204
XL	50.8 - 61cm	B-250150205
XXL	61 - 81.3cm	B-250150206

When using an undersleeve we advice to use the suspension strap kit to minimise migration



Note: This picture doesn't show the latest product

SENSIL SILICONE LINERS

Size	Tibia		Femur	
	RM/LL*	RL/LM**	RM/LL*	RL/LM**
XS	B-419700311	B-419800311	B-409700321	B-409800321
S	B-419700312	B-419800312	B-409700322	B-409800322
M	B-419700313	B-419800313	B-409700323	B-409800323
L	B-419700314	B-419800314	B-409700324	B-409800324
XL	B-419700315	B-419800315	B-409700325	B-409800325

* Right medial/Left lateral / ** Right lateral/Left medial



Note: This picture doesn't show the latest product

DOESKIN LINERS

Size	Tibia		Femur	
	RM/LL*	RL/LM**	RM/LL*	RL/LM**
XS	B-424703721	B-424803721	B-412701721	B-412801721
S	B-424703722	B-424803722	B-412701722	B-412801722
M	B-424703723	B-424803723	B-412701723	B-412801723
L	B-424703724	B-424803724	B-412701724	B-412801724
XL	B-424703725	B-424803725	B-412701725	B-412801725

When using a doeskin tibia liner we advice to use a suspension strap kit to minimise migration / * Right medial/Left lateral / ** Right lateral/Left medial



Note: This picture doesn't show the latest product

SENSIL AMS (ANTI-MIGRATION SYSTEM) WRAPS

Size	Right medial/Left lateral	Right lateral/Left medial
XS	B-715217031	B-715218031
S	B-715217032	B-715218032
M	B-715217033	B-715218033
L	B-715217034	B-715218034
XL	B-715217035	B-715218035



Note: This picture doesn't show the latest product

SUSPENSION STRAP KIT

Description	Part#
Suspension Strap Kit	UNL00212

UNLOADER® CUSTOM

With a semi-rigid frame and an adjustable hinge, Unloader Custom accommodates a wide variety of leg shapes and sizes. That adjustable dynamic hinge, with a diagonal Dynamic Force Strap, allows for some of the largest unloading forces with the greatest amount of patient comfort. The clinically proven Unloader Custom braces have numerous options and can be further customized with a variety of colors, designs and accessories.



INDICATIONS FOR USE

- Mild to severe unicompartmental knee osteoarthritis
- Other knee conditions that may benefit from unicompartmental unloading

FEATURES AND BENEFITS

- Custom moulded semi-flexible shells are durable, adaptable to leg shape changes, yet rigid enough to contain tissue while force is applied at the knee.
- Patented Adjustable Dynamic Joint™ (ADJ) high-carbon steel hinge is strong enough to unload the pressure from the compartment of the knee.
- Dynamic Force Strap® designed to reduce pressure on sensitive bony surfaces of the knee, resulting in reduction of pain.
- Optional tool-free SmartDosing® dial option enables fine-tuned pain management and can be adjusted even when wearing the brace under clothing.

REQUIRED MEASUREMENTS

- Custom made to measure (3D-Scan or patient cast)



UNLOADER® CUSTOM

ORDERING INFORMATION

UNLOADER CUSTOM

Description	Part#
Unloader Custom Right	B-102500010
Unloader Custom Left	B-102600010

UNLOADER CUSTOM WITH SMARTDOSING

Description	Part#
Unloader Custom with SmartDosing Right	B-122500020
Unloader Custom with SmartDosing Left	B-122600020

To order an Unloader XT a special order form needs to be filled out and a negative cast needs to be taken.

ACCESSORIES

Description	Part#
Patella cup	B-250110001
Hinge cover	B-251020000
AMS Wrap (custom made)	B-426000150
AMS Sleeve (custom made)	B-426000250

SLEEVES

Size	Össur knee undersleeve	Softsleeve	Neoprene undersleeve	Neoprene oversleeve
XS	B-250150101			
S	B-250150102	20643	20251	20051
M	B-250150103	20644	20252	20052
L	B-250150104	20644	20253	20053
XL	B-250150105	20645	20254	20054
XXL	B-250150106	20645	20255	20055
XXXL		20646	20256	20056
XXXXL		20647	20257	20057

CUSTOM STANDARD COLORS




MEASUREMENT FORM CAST, DIGITAL KNEEBRACE

Customer

Customer name: _____
 Customer number: _____
 CPO: _____
 Phone: _____
 Medical specialist: _____
 Diagnosis: _____
 Measuring Method: Plaster cast
 3D scan
 (e.g. .AOP .STL & Smartmeasure®)

Patient information

Reference number: _____
 (For traceability of the order)
 Sex(m/f): M F
 Knee: Left Right ilateral
 Height: _____ cm Weight: _____ kg

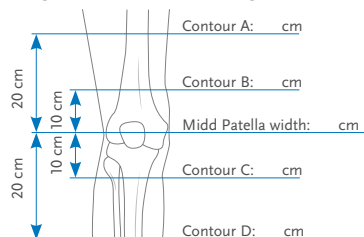

STEP I - choice of orthosis

LIGAMENT STABILIZATION

Product: EXTREME™
 EXTREME™ SHORT
 (Patient height < 1,65m)
 Hinge: Poli-Axial® free
 Poli-Axial® with lock 170° 180°
 KAFO partly
 (Upper leg socket + hinge + straps)

UNICOMPARTMENTAL UNLOADING

Product: UNLOADER® XT
 UNLOADER® XT SHORT
 (Patient height < 1,65m)
 Compartment: Medial (relief of the internal compartment)
 Varum correction: _____ degrees
 Lateral (relief of the outer compartment)
 Valgum correction: _____ degrees

STEP II - flexion / extension stops

Flexion stop: 50° 60° 70°
 Extension stop: 0° 10° 15° 20° 30°
 Built-in*: 5° 10° 15°
 *(this restriction is not removable)

Flexion stop: 50° 60° 70°
 Extension stop: 0° 5° 10° 15°

STEP III - liners and straps

Liners: Standard - Doeskin thigh / Doeskin calf
 Better grip - Doeskin thigh / Sensil silicones calf
 Specific grip - Sensil silicones thigh / Sensil silicones calf (Recommended only for voluminous patient)
 Straps: Dynamic Force Strap (DFS) Standard
 Dual Dynamic Force straps (DFS) standard (recommended only for extreme rotation of the knee)
 Special elastic calf strap (Wide strap for comfort with excess soft tissue)

STEP IV - colors and accessories

Colors: Yellow Pink Charcoal Ocean Blue Red Champagne
 Black Navy Blue White Silver Orange Lime Green
 Custom paint*: (See other measurement form for text input): _____ (Contact Ossur for additional cost)
 Select finish: MATTE GLOSS
 Accessories: Protection: Motorcross patella cup (knee protection)
 Tools: Robertson screwdriver
 Sleeve: Sofsleeve undersleeve Sleeve with Mesh Neoprene oversleeve
 Extra liner set Calf Doeskin (standard) Silicone
 Thigh Doeskin (standard) Silicone

STEP V - Revision

Complete revision: Revision of hinge, liners & straps

REBOUND® DUAL

The Rebound® DUAL offers simplicity without compromise. With its malleable yet supportive frame, Rebound DUAL is the go-to for knee ligament support.



INDICATIONS FOR USE

The brace is indicated for combined and non-combined instabilities of the:

- Anterior cruciate ligament (ACL)
- Medial collateral ligament (MCL)
- Lateral collateral ligament (LCL)
- Posterior cruciate ligament (PCL)
- Acculign version only: Conditions requiring unicompartmental load control.

FEATURES AND BENEFITS

- Cold-malleable aluminum frame, trimmable straps, and field-serviceable D-rings accommodate various patient anatomies
- Acculign™ upper frame allows height and varus/valgus adjustment
- Tool-free polycentric hinge with ROM stop options for range-of-motion restriction
- Össur Hinge Alignment Guide, Blue is You™ touch points, and tactile numbered strap tabs guide the fitting process
- Optional SmartDosing® Kit accessory offers fine-tuned unicompartmental offloading
- Optional PCL kit accessory for multi-ligament support

REQUIRED MEASUREMENTS

- Caliper measurement: Medio-laterally at joint width in a standard, weight-bearing position
- Circumference: 15 cm above mid-patella

REBOUND® DUAL

ORDERING INFORMATION

REBOUND DUAL

Size	Caliper	Circumference	Right	Left
XSmall	7.6 - 8.9 cm	38.1 - 41.9 cm	B-242510001	B-242610001
Small	8.9 - 10.2 cm	41.9 - 47 cm	B-242510002	B-242610002
Medium	10.2 - 11.4 cm	47 - 52.1 cm	B-242510003	B-242610003
Large	11.4 - 12.7 cm	52.1 - 57.2 cm	B-242510004	B-242610004
X-Large	12.7 - 14 cm	57.2 - 62.2 cm	B-242510005	B-242610005
XX-Large	14 - 15.2 cm	62.2 - 69.9 cm	B-242510006	B-242610006
XXX-Large	15.2 - 16.5 cm	69.9 - 76 cm	B-242510007	B-242610007

M/L at joint width in a standard, weight-bearing position 2 Thigh Circumference: 6" above mid-patella

REBOUND DUAL CUSTOM

Measuring System	Right	Left
SmartMeasure, DMS, Coordinate Cast System, Computer Aided Design	B-155510001	B-155610001

SMARTMEASURE®


A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilizes a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required

CUSTOM BRACE COLORS

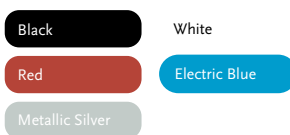

REBOUND® DUAL

ACCESSORIES

SERVICE KITS & LINER KITS

Size	Softgoods Kit, Black*	Liner Kit, Black**
XS	B-701090091	B-701090110
S	B-701090092	B-701090111
M	B-701090093	B-701090112
L	B-701090094	B-701090113
XL	B-701090095	B-701090114
XXL	B-701090096	B-701090115
3XL	B-701090097	B-701090116

*Includes Doeskin thigh liner, Active Grip calf liner, straps, gastroc pad, tibial pad, numbered alligator tabs, hook strip

**Includes Doeskin thigh liner and Sensil calf liner

CALF LINERS

Size	Doeskin, Silver	Active Grip, Black	Active Grip, Silver
XS	B-701090028	B-701090160	B-701090017
S	B-701090029	B-701090161	B-701090018
M	B-701090030	B-701090162	B-701090019
L	B-701090031	B-701090163	B-701090020
XL	B-701090032	B-701090164	B-701090021
XXL	B-701090033	B-701090165	B-701090022
3XL	B-701090034	B-701090166	B-701090023

THIGH LINERS

Size	Doeskin, Black	Doeskin, Silver	Active Grip, Silver
XS	B-701090150	B-701090010	B-701090041
S	B-701090151	B-701090011	B-701090042
M	B-701090152	B-701090012	B-701090043
L	B-701090153	B-701090013	B-701090044
XL	B-701090154	B-701090014	B-701090045
XXL	B-701090155	B-701090015	B-701090046
3XL	B-701090156	B-701090016	B-701090047

STRAP KITS & PAD KITS

Size	Replacement Strap Kit	PCL Strap Kit	Active Grip Strap Pad Kit
XS-L	B-701090105	B-701090057	B-701090039
XL-3XL	B-701090106	B-701090058	B-701090040

REBOUND® DUAL

ACCESSORIES

OTHER

Description	Part#
SmartDosing Kit RM/LL	B-701090060
SmartDosing Kit RL/LM	B-701090061
Flexion/Extension Stop Set - Tool-Free Hinge	B-701090088
*Flexion Stop Set - Custom & Legacy	B-701090004
**Extension Stop Set - Custom & Legacy	B-701090008
D-Ring Kit	B-701090059
Tibial Wedge Kit	B-701090001
Condyle Pad Kit	B-701090009

*Flexion Stops: 0, 10, 20, 30, 45, 60, 75, 90

**Extension Stops: 0, 5, 10, 20, 30, 45

SLEEVES

Size	Sleeve with Mesh, Black*	Össur One Sleeve, Black**	Neoprene Undersleeve, Black	Neoprene Oversleeve, Black
XXS		OSO60021		
XS	SLV60011	OSO60022		
S	SLV60012	OSO60023	20251	20051
M	SLV60013	OSO60024	20252	20052
L	SLV60014	OSO60025	20253	20053
XL	SLV60015	OSO60026	20254	20054
XXL	SLV60016	OSO60027	20255	20055
3XL		OSO60028	20256	20056
4XL			20257	20057

*Sleeve with Mesh: Main fabric: 92% Cotton/8% Spandex - Mesh: 100% Polyester - Band: 80% Polyester/20% Spandex

** Össur One: 51% Tencel™/30% Polyamide/19% Elastane

REBOUND® DUAL ST

The Rebound® DUAL ST offers simplicity without compromise. With its malleable yet supportive frame, Rebound DUAL ST is the go-to for knee ligament support.



INDICATIONS FOR USE

The brace is indicated for combined and non-combined instabilities of the:

- Anterior cruciate ligament (ACL)
- Medial collateral ligament (MCL)
- Lateral collateral ligament (LCL)
- Posterior cruciate ligament (PCL)

FEATURES AND BENEFITS

- Cold-malleable aluminum frame, trimmable straps, and field-serviceable D-rings accommodate various patient anatomies
- Step-through design moves the rigid frame cuff behind the calf
- Tool-free polycentric hinge with ROM stop options for range-of-motion restriction
- Össur Hinge Alignment Guide, Blue is You™ touch points, and tactile numbered strap tabs guide the fitting process
- Optional PCL Kit accessory for multi-ligament support

REQUIRED MEASUREMENTS



Measure knee caliper medio-laterally at joint width in standing, weight-bearing position.



Measure thigh circumference 15 cm (6") above mid-patella. Leg in extension.

Size	Caliper	Circumference	Right	Left
XSmall	7.6 - 8.9 cm	38.1 - 41.9 cm	B-242520001	B-242620001
Small	8.9 - 10.2 cm	41.9 - 47 cm	B-242520002	B-242620002
Medium	10.2 - 11.4 cm	47 - 52.1 cm	B-242520003	B-242620003
Large	11.4 - 12.7 cm	52.1 - 57.2 cm	B-242520004	B-242620004
X-Large	12.7 - 14 cm	57.2 - 62.2 cm	B-242520005	B-242620005
XX-Large	14 - 15.2 cm	62.2 - 69.9 cm	B-242520006	B-242620006
XXX-Large	15.2 - 16.5 cm	69.9 - 76 cm	B-242520007	B-242620007

REBOUND® DUAL ST

ORDERING INFORMATION

REBOUND DUAL CUSTOM

Measuring System	Right	Left
SmartMeasure, DMS, Coordinate Cast System, Computer Aided Design	B-155520001	B-155620002

SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilizes a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required

ACCESSORIES

LINER KITS

Size	Liner Kit, Black*
XS	B-701090120
S	B-701090121
M	B-701090122
L	B-701090123
XL	B-701090124
XXL	B-701090125
3XL	B-701090126

THIGH LINERS

Size	Doeskin, Black	Doeskin, Silver
XS	B-701090170	B-701090067
S	B-701090171	B-701090068
M	B-701090172	B-701090069
L	B-701090173	B-701090070
XL	B-701090174	B-701090071
XXL	B-701090175	B-701090072
3XL	B-701090176	B-701090073

*Includes Doeskin thigh liner and Sensil calf liner

CALF LINERS

Size	Active Grip, Black	Active Grip, Silver
XS	B-701090180	B-701090074
S	B-701090181	B-701090075
M	B-701090182	B-701090076
L	B-701090183	B-701090077
XL	B-701090184	B-701090078
XXL	B-701090185	B-701090079
3XL	B-701090186	B-701090080

REBOUND® DUAL ST

STRAP KITS & PAD KITS

Size	Replacement Strap Kit	PCL Strap Kit	Active Grip Strap Pad Kit
XS-L	B-701090107	B-701090065	B-701090039
XL-3XL	B-701090108	B-701090066	B-701090040

OTHER

Description	Part#
Flexion/Extension Stop Set - Tool-Free Hinge	B-701090088
*Flexion Stop Set - Custom & Legacy	B-701090004
**Extension Stop Set - Custom & Legacy	B-701090008
D-Ring Kit	B-701090059
Condyle Pad Kit	B-701090009

*Flexion Stops: 0, 10, 20, 30, 45, 60, 75, 90

**Extension Stops: 0, 5, 10, 20, 30, 45

SLEEVES

Size	Sleeve with Mesh, Black*	Össur One Sleeve, Black**	Neoprene Undersleeve, Black	Neoprene Oversleeve, Black
XXS		OSO60021		
XS	SLV60011	OSO60022		
S	SLV60012	OSO60023	20251	20051
M	SLV60013	OSO60024	20252	20052
L	SLV60014	OSO60025	20253	20053
XL	SLV60015	OSO60026	20254	20054
XXL	SLV60016	OSO60027	20255	20055
3XL		OSO60028	20256	20056
4XL			20257	20057

*Sleeve with Mesh: Main fabric: 92% Cotton/8% Spandex - Mesh: 100% Polyester - Band: 80% Polyester/20% Spandex

** Össur One: 51% Tencel™/30% Polyamide/19% Elastane

REBOUND® DUAL BASIC AND REBOUND® DUAL BASIC ST

The Rebound DUAL Basic and Rebound DUAL Basic ST offer the value and simplicity you need to navigate the complex coverage landscape. Within the versatility of the Rebound DUAL collection, Össur offers high quality solutions from a single source, to help users transition to a successful recovery without compromise.



Rebound DUAL Basic

Rebound DUAL Basic ST
(Step-Through)

INDICATIONS FOR USE

The Rebound DUAL Basic is indicated for combined and non-combined instabilities of the:

- Anterior cruciate ligament (ACL)
- Medial collateral ligament (MCL)
- Lateral collateral ligament (LCL)
- Posterior cruciate ligament (PCL)
- (Only Rebound DUAL Basic) Acculign version only: Conditions requiring unicompartmental load control

FEATURES AND BENEFITS

- (Only Rebound DUAL Basic ST) Step-Through Design: advantage in comfort & fitting with ease
- (Only Rebound DUAL Basic) Multi-purpose length-adjustable upper frame accommodates the user's height, helping to ensure the perfect fit and enabling adjustment for varus/valgus alignment
- Össur Hinge alignment guide provides easy & accurate re-fitting for users, ensuring correct placement of the brace relative to the knee
- Lightweight aircraft aluminium frame keeps a low profile
- Doeskin liners for comfort & no-slip Activegrip™ strap pad
- Trimmable straps with tactile strap tabs
- Low-profile polycentric hinge with stop options for stability & range-of-motion restriction
- Step-through or Anterior frame design
- Suitable for low-high impact and sliding sports
- Optional PCL kit accessory for multi-ligament support
- Available in pre-fabricated sizes XS-3XL

REQUIRED MEASUREMENTS

- Knee caliper
- Thigh circumference 15 cm above mid patella

REBOUND® DUAL BASIC AND REBOUND® DUAL BASIC ST

ORDERING INFORMATION

X = 5 (right)
6 (left)
Y = 1 (Rebound DUAL Basic)
2 (Rebound DUAL Basic ST)

Description	Size	Caliper	Thigh Circumference	Part #
Rebound DUAL Basic or Rebound DUAL Basic ST*	XS	7.6 - 8.9cm	38.1 - 41.9cm	B-242XY0001B
	S	8.9 - 10.2cm	41.9 - 47cm	B-242XY0002B
	M	10.2 - 11.4cm	47 - 52.1cm	B-242XY0003B
	L	11.4 - 12.7cm	52.1 - 57.2cm	B-242XY0004B
	XL	12.7 - 14cm	57.2 - 62.2cm	B-242XY0005B
	XXL	14 - 15.2cm	62.2 - 69.9cm	B-242XY0006B
	3XL	15.2 - 16.5cm	69.9 - 76cm	B-242XY0007B

REPLACEMENT LINER KITS

Measuring	Basic Liner Part #	Basic ST Liner Part #
XS	B-701090130	B-701090140
S	B-701090131	B-701090141
M	B-701090132	B-701090142
L	B-701090133	B-701090143
XL	B-701090134	B-701090144
XXL	B-701090135	B-701090145
3XL	B-701090136	B-701090146

REPLACEMENT PARTS

Size	Part #
RDUAL Basic Strap Kit XS-L	B-701090137
RDUAL Basic Strap Kit XL-3XL	B-701090138
RDUAL Basic ST Strap Kit XS-L	B-701090147
RDUAL Basic ST Strap Kit XL-3XL	B-701090148
Condyle Pad Kit	B-701090009

ACCESSORIES

Size	Part #
RDUAL Basic ROM Stop Kit	B-701090109
RDUAL Basic PCL Strap Kit XS-L	B-701090057
RDUAL Basic PCL Strap Kit XL-3XL	B-701090058
RDUAL Basic ST PCL Strap Kit XS-L	B-701090065
RDUAL Basic ST PCL Strap Kit XL-3XL	B-701090066

* Rebound DUAL Basic ST is 35.5cm

REBOUND® DUAL CUSTOM

The Rebound® DUAL family is a series of double upright aluminum frame knee ligament braces. Available in step-through, anterior calf frame, or a custom upper/lower frame combination, Rebound DUAL offers a solution for you.



INDICATIONS FOR USE

The brace is indicated for combined and non-combined instabilities of the:

- Anterior cruciate ligament (ACL)
- Medial collateral ligament (MCL)
- Lateral collateral ligament (LCL)
- Posterior cruciate ligament (PCL)
- Acculign version only: Conditions requiring unicompartmental load control

FEATURES AND BENEFITS

- Cold-malleable aluminum frame, trimmable straps, and field-serviceable D-rings accommodate various patient anatomies
- Acculign™ upper frame allows height and varus/valgus adjustment
- Polycentric hinge with ROM stop options for range-of-motion restriction
- Össur Hinge Alignment Guide, Blue is You™ touch points, and tactile numbered strap tabs guide the fitting process
- Custom-made for your patient
- Optional SmartDosing® Kit accessory offers fine-tuned unicompartmental offloading
- Optional PCL Kit accessory for multi-ligament support

REQUIRED MEASUREMENTS

- Thigh circumference 20 cm above mid patella
- Thigh circumference 15 cm above mid patella
- Thigh circumference 7.5 cm above mid patella
- Knee caliper
- Knee circumference (at) mid-patella
- Calf circumference 7.5 cm below mid patella
- Calf circumference 15 cm below mid patella
- Calf circumference 20 cm below mid patella

ORDERING INFORMATION

STANDARD

Type	Part#
Left	B-155610001
Right	B-155510001

M/L at joint width in a standard, weight-bearing position 2 Thigh Circumference: 6" above mid-patella

REBOUND® DUAL CUSTOM

ACCESSORIES

SLEEVES

Size	Neoprene undersleeve	Neoprene oversleeve	Sport undersleeve
S	20251	20051	20682
M	20252	20052	20683
L	20253	20053	20684
XL	20254	20054	20685
XXL	20255	20055	
3XL	20256	20056	
4XL	20257	20057	

PADDED SPORTS OVERSLEEVE

Size	Part#
S-M	20448
L-XL	20449

SMARTDOSING® KITS

Description	Part#
Right Medial, Left Lateral	B-701090060
Right Lateral, Left Medial	B-701090061

EXTENSION STOP KIT

Colour	Part#
Black	B-701090008

Includes: 5°, 10°, 20°, 30°, 40° extension stops. (0° and 10° are included with brace).

FLEXION STOP KIT

Size	Colour	Part#
Universal	Black	B-701090004

Includes: 45°, 60°, 75°, 90° flexion stops

PCL STRAP KIT

Size	Part#
XS-XL	B-701090057
XL-3XL	B-701090058

STRAP KIT

Size	Part#
XS-L	B-701090105
XL-3XL	B-701090106

TIBIAL WEDGE KIT

Size	Part#
Universal	B-701090001

LINER KIT

Part#
B-701090050

CONDYLE PAD KIT

Part#
B-701090009

FIELD SERVICABLE D-RING KIT

Part#
B-701090059

LINERS

Size	Doeskin calf liner	ActiveGrip thigh liner
XS	B-701090028	B-701090041
S	B-701090029	B-701090042
M	B-701090030	B-701090043
L	B-701090031	B-701090044
XL	B-701090032	B-701090045
XXL	B-701090033	B-701090046
3XL	B-701090034	B-701090047

THERMOFORMED LINER KITS (STANDARD)

Size	Part#
XS	B-701090110
S	B-701090111
M	B-701090112
L	B-701090113
XL	B-701090114
XXL	B-701090115
3XL	B-701090116

OA EASE

The Formfit OA Ease is a single upright brace with a Dynamic Force System™ strap designed to relieve pain from mild-to-moderate unicompartmental knee osteoarthritis (OA). Its wraparound design is easy to apply and remove using simple rubber pull-tabs, colour-coded guide dots, and strap tension indicator lines. This low-profile knee brace is comfortable, easy to use, and adapts to a variety of leg anatomies. The Formfit OA Ease can also be machine washed for effortless care.



INDICATIONS FOR USE


Take control of mild-to-moderate OA pain with the Formfit OA Ease, an easy-to-wear knee brace with a simple wraparound design that can be applied and removed with ease.

- Mild-to-moderate unicompartmental knee osteoarthritis and/or degenerative meniscal tears
- Other conditions that may benefit from unicompartmental unloading

FEATURES AND BENEFITS

- Easy Wraparound Design - Simple rubber pull-tabs, colour-coded guide dots and a convenient hinge alignment guide make application and removal simple
- Relieves OA Pain - 3-Point Leverage System with Dynamic Force System™ strap and shells unload the affected knee compartment
- Machine Washable - Lightweight and breathable fabric feels great against the skin and can be machine washed for effortless care
- Tool-Free - Tool-free polycentric hinge with available ROM stop options for range-of-motion restriction
- Control – Fine-tune strap tension for individual pain relief, with marked dosing indicator lines
- Extension kits and liners are available to further enhance fit and comfort

REQUIRED MEASUREMENTS

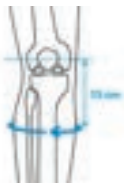

Measure calf circumference 15cm below mid-patella. Leg in extension.



Measure thigh circumference 15cm above mid-patella. Leg in extension.

OA EASE

ORDERING INFORMATION

Size	Calf circ. 15cm below mid patella	Thigh circ. 15cm above mid patella	Right Medial	Left Medial	Right Lateral	Left Lateral
XS	28 - 32cm	36 - 41cm	FOA0518711	FOA0618711	FOA0528711	FOA0628711
S*	32 - 36cm	41 - 46cm	FOA0518712	FOA0618712	FOA0528712	FOA0628712
M	36 - 41cm	46 - 53cm	FOA0518713	FOA0618713	FOA0528713	FOA0628713
L*	41 - 46cm	53 - 60cm	FOA0518714	FOA0618714	FOA0528714	FOA0628714
XL	46 - 51cm	60 - 67cm	FOA0518715	FOA0618715	FOA0528715	FOA0628715
XXL*	51 - 56cm	67 - 75cm	FOA0518716	FOA0618716	FOA0528716	FOA0628716

*All sizes are available but for consolidated stock option this size can be achieved by combining the smaller size with accessories below.

ACCESSORIES

Description	Part#
ActiveGrip Calf Liner Kit 2MM	FOA000101
ActiveGrip Calf Liner Kit 6MM	FOA000102
Doeskin Thigh Liner Kit 2MM	FOA000201
Doeskin Thigh Liner Kit 6MM	FOA000202
Small Extension Kit 2 Pieces	FOA000301
Small Extension Kit 10 Pieces	FOA000302
Large Extension Kit XS-SM (Right Medial/Left Lateral)	FOA000401
Large Extension Kit MD-LG (Right Medial/Left Lateral)	FOA000402
Large Extension Kit XL-XXXL (Right Medial/Left Lateral)	FOA000403
Large Extension Kit XS-SM (Right Lateral/Left Medial)	FOA000404
Large Extension Kit MD-LG (Right Lateral/Left Medial)	FOA000405
Large Extension Kit XL-XXXL (Right Lateral/Left Medial)	FOA000406
Wash Bag	WB000001
Suspension Strap	OAW00007
OA Hinge Extension Stop Kit (5°, 10°, 15°, 30°)	OAH40029
OA Hinge Flexion Stop Kit (0°, 30°, 60°, 90°)	OAH40039

REPLACEMENT PARTS

Description	Part#
Alligator Clip (Right Medial/Left Lateral)	FOW00501
Alligator Clip (Right Lateral/Left Medial)	FOW00502
Hinge Pad	B-304120101

FORMFIT® PRO KNEE OA

Össur Formfit® Pro Knee OA is a unique, compressive knee sleeve with a Dynamic Force Strap system, designed to provide pain relief from the early stages of osteoarthritis (OA). The dynamic, 3D MotionTech™ knit is supportive and meets certified medical-grade compression requirements, and has been proven to be superior to the leading brands at moisture wicking and breathability, keeping your knee cool during wear. CoolVent™ technology in the popliteal area enhances breathability and comfort, while the distinct patella graphics help with intuitive placing of the sleeve. The integrated StayBilizer™ hinge, constructed from lightweight, flexible EVA material, is suitable for sports, such as football.



MotionTech™



CoolVent™



StayBilizer™



FoldLock™

INDICATIONS FOR USE

For osteoarthritis and other knee conditions which may benefit from unloading and/or compression, such as;

- Medial or lateral knee pain
- Early or mild unicompartmental osteoarthritis of the knee
- Symptomatic, unicompartmental meniscus degeneration


CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.


FEATURES AND BENEFITS

- Patented DFS system.
- Certified medical grade compression (20-30 mmHg).
- Lightweight CoolVent™ knit in popliteal area for added breathability and comfort.
- Patented FoldLock™ system for easier and more secure application.
- Sports-compatible StayBilizer™ hinge made from light, flexible EVA material for optimum shape and fit. Patella graphics for intuitive placement.
- Available in 5 standard sizes and 3 comfort sizes.

REQUIRED MEASUREMENTS

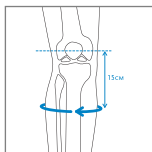

Measure calf circumference
15cm below mid-patella.
Leg in extension.

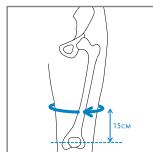

Measure thigh circumference
15cm above mid-patella.
Leg in extension.

Size	Calf circumference 15 cm below mid patella	Thigh circumference 15 cm above mid patella	Right medial, left lateral	Right lateral, left medial
XS	26 - 31cm	36 - 41cm	FP110401	FP110301
S	31 - 36cm	41 - 46cm	FP110402	FP110302
M	36 - 41cm	46 - 51cm	FP110403	FP110303
M Comfort	36 - 41cm	51 - 56cm	FP110413	FP110313
L	41 - 46cm	51 - 56cm	FP110404	FP110304
L Comfort	41 - 46cm	56 - 60cm	FP110414	FP110314
XL	46 - 50cm	56 - 60cm	FP110405	FP110305
XL Comfort	46 - 50cm	60 - 64cm	FP110415	FP110315

REBOUND® CARTILAGE

The success of an articular cartilage repair is dependent on patient compliance to the rehabilitation process. The protocol is based on restricted cartilage load combined with gradual increase in activity. Many rehabilitation protocols suggest crutches for partial weight-bearing and it is the standard of care for successful outcomes.

However, patient compliance to partial weight-bearing instructions with crutches has been shown to be inconsistent. The Rebound Cartilage is a protective functional solution designed to support regeneration of the knee cartilage after repair procedures such as Microfracture, OATS and (M)ACI. Based on the clinically-proven 3-Point Leverage System and proprietary Cartilage Protection Straps, joint unloading is maintained in flexion for protection of the cartilage during the healing process.



INDICATIONS FOR USE

For knee conditions that may benefit from unicompartmental load reduction or range of movement restriction, such as:

- Articular cartilage defect repair
- Meniscal repair
- Avascular necrosis
- Condylar bone marrow lesions (bone bruises)
- Tibial plateau fractures
- Mild to severe unicompartmental knee osteoarthritis

FEATURES AND BENEFITS

- Easy-to-Stock - Just 4 configurations to fit most of your patients
- Comfortable - Skin-friendly liner feels great against the skin and reduces pressure points.
- Anti-Migration Technology - Strap design captures the thickest part of the calf to reduce brace migration during activity.
- Control - SmartDosing® dials put you in control of your pain by allowing you to fine-tune strap tension, while Cartilage Protection Straps maintain joint unloading in flexion.
- Tool-Free - Tool-free polycentric hinge with optional ROM stops for range-of-motion restriction.

REQUIRED MEASUREMENTS

- Circumference measurement: 15 cm below mid-patella

REBOUND® CARTILAGE

ORDERING INFORMATION

Rebound Cartilage	Left medial	Right medial	Left lateral	Right lateral
Universal 31 - 61.5 cm	B-254610006	B-254510006	B-254620006	B-254520006

Rebound Cartilage - No Box	Left medial	Right medial	Left lateral	Right lateral
Universal 31 - 61.5 cm	B-254610007	B-254510007	B-254620007	B-254520007

Rebound Cartilage - Custom	Left	Right
Calf circumference 25 - 61.5 cm	B-154600001	B-154500001

SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilizes a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.




Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required

CUSTOM STANDARD COLORS



CUSTOM PAINTED DESIGNS

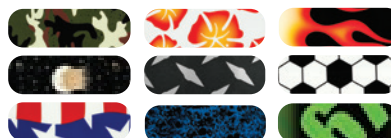

ACCESSORIES

Accessory Parts	Part#
Suspension Strap Kit	UNL00211
OA Hinge Extension Stop Tree (5°, 10°, 15°, 30°)	OAH40029
OA Hinge Flexion Stop Tree (0°, 30°, 60°, 90°)	OAH40039
Wash Bag	UON60200

REBOUND® CARTILAGE

Sleeves	Black Ossur One*	Black - Sleeve with Mesh**	Black - Ossur Knee Undersleeve***
XXS	OSO60021	NA	NA
XS	OSO60022	SLV60011	B-250150101
S	OSO60023	SLV60012	B-250150102
M	OSO60024	SLV60013	B-250150103
L	OSO60025	SLV60014	B-250150104
XL	OSO60026	SLV60015	B-250150105
XXL	OSO60027	SLV60016	B-250150106
3XL	OSO60028	NA	NA

*Ossur One - 51% Tencel/30% Polyamide/19% Elastane

**Sleeve with Mesh - Main fabric: 92% Cotton/8% Spandex - Mesh: 100% Polyester - Band: 80% Polyester/20% Spandex

***Ossur Knee Undersleeve: 80% Nylon/20% Spandex

REPLACEMENT PARTS

Service Kit*	Right medial/Left lateral	Right lateral/Left medial
Black, Service Kit	CRT416901	CRT415901

*Includes Jersey knit thigh liner, Sensil calf liner, Popliteal pad standard, Hinge pad & Strap Kit Complete

Thigh Liners	Right medial/Left lateral	Right lateral/Left medial
Black, Jersey Knit	CRT526901	CRT525901

Calf Liners	Right medial/Left lateral	Right lateral/Left medial
Black, Sensil	CRT516952	CRT515952
Black, Jersey Knit	CRT516951	CRT515951

Strap Kit Complete*	Part #
Black, Strap Kit Complete	B-701290005

*Includes 4 straps (1 Thigh, 1 Calf, 2 CPS), 5 Alligator tabs, 2 CPS Strap pads

CPS Strap Kit*	Part #
Black, CPS Kit	B-700290005

*Includes 2 CPS Straps, 2 Alligator tabs, 2 CPS Strap pads

Buckle Clasp Kit	Right medial/Left lateral	Right lateral/Left medial
Black	B-708217001	B-708218001

Popliteal Pad	Part #
Black, Short	B-304010105
Black, Standard	B-304010106

Other Replacement Parts	Part #
Velcro Twist-Fit Attachment Kit	B-705290000
Hinge Pad	B-304120101
Alligator Kit-8 Pieces	B-704210001

FORMFIT® TRACKER

The Formfit® Tracker with micro-adjustable ReflexWing™ buttress realigns the knee cap, providing dynamic lateral support for common knee cap conditions. Its FlyWeight™ fabric delivers lightweight breathability and a low-profile design that feels great against the skin and minimises migration. PowerLock™ straps work in tandem with heat mouldable CustomFit™ hinges to provide a customisable fit for your patient.



INDICATIONS FOR USE

For knee conditions that may benefit from realignment of the knee cap and/or patients that would benefit from an added lateral support of the knee cap, such as:

- lateral knee cap subluxation or dislocation
- lateral maltracking

FEATURES AND BENEFITS

- CustomFit™ hinges for customised fit and support
- Anti-Migration Tech™ minimises migration
- PowerLock™ straps for secure lateral support
- ReflexWing™ buttress for comfortable knee cap realignment
- FlyWeight™ fabric for soft feel and breathability
- Durable and machine washable

REQUIRED MEASUREMENTS & ORDERING INFORMATION



Calf circumference 15cm below mid patella



Thigh circumference 15cm above mid patella

Size	Calf circumference 15cm below mid patella	Thigh circumference 15cm above mid patella	Left	Right
XS	25 - 30cm	33 - 39cm	719162	719152
S	30 - 35cm	39 - 47cm	719163	719153
M	35 - 40cm	47 - 53cm	719164	719154
L	40 - 46cm	53 - 60cm	719165	719155
XL	46 - 51cm	60 - 67cm	719166	719156
XXL	51 - 56cm	67 - 75cm	719167	719157
3XL	56 - 61cm	75 - 81cm	719168	719158

ACCESSORIES

Description	Part#
ReflexWing™ Buttress	719100

FORMFIT® HINGED LATERAL J

Formfit Knee Hinged Lateral J is a sleeve design knee brace with a breathable Lycra-material and open popliteal area with two removable hinges and J-shaped buttress providing support to the patella. The knee brace is designed for patients that would benefit from realignment of the patella, and/or patients that would benefit from an added lateral support of the patella.

INDICATIONS FOR USE

Patients that would benefit from realignment of the patella, and/or patients that would benefit from an added lateral support of the patella.

- Lateral patellar subluxations and dislocations.
- Lateral patellofemoral malalignment.

FEATURES AND BENEFITS

- Breathable material
- Removable hinges
- Easy pull-on
- Lateral J buttress
- Open popliteal area

REQUIRED MEASUREMENTS

- Thigh Circumference: 6"/15cm above mid-patella
- Calf Circumference: 6"/15cm below mid-patella



ORDERING INFORMATION

Size	Thigh circumference	Knee circumference	Calf circumference	Right leg	Left leg
XS	33 - 39.4cm	30.5 - 33cm	25.4 - 30.5cm	709152	709162
S	39.4-47cm	33 - 35.6cm	30.5 - 35.6cm	709154	709164
M	47-53.3cm	35.6 - 38.1cm	35.6 - 40.6cm	709155	709165
L	53.3-59.7cm	38.1 - 43.2cm	40.6 - 45.7cm	709157	709167
XL	59.7-67.3cm	43.2 - 48.3cm	45.7 - 50.8cm	709158	709168
XXL	67.3-74.9cm	48.3 - 53.3cm	50.8 - 55.9cm	709159	709169

CTi®3

CTi®3 brings a new vision to double upright knee braces. Innovative engineering makes CTi3 an excellent option for most of your patient population. Suitable for both everyday activities and sport, CTi3 offers great quality with its enhanced construction and anatomical fit for elevated comfort.



INDICATIONS

- For knee conditions that may benefit from increased AP and ML stability and protection, such as injuries to the ACL, MCL, LCL, PCL, rotary and combined instabilities.

FEATURES AND BENEFITS

- Enhanced construction – Lightweight, high strength carbon composite frames provide the optimal stiffness to weight ratio
- Anatomical fit – Designed to fit a wider range of leg shapes and includes an Accutrac® hinge to closely follow the knee's anatomical movement
- Elevated comfort – Unique liners enhance comfort with their special tread block pattern, which provides a balance of cushion and ventilation
- Easy-to-use accessories and replacement parts, including MX kits, replacement strap sets, liner sets and flexion stops

REQUIRED MEASUREMENTS

- Knee caliper
- Thigh circumference 20 cm above mid patella

ORDERING INFORMATION

CTi3

Size	Caliper	Circumference	Left	Right
S	8 - 10 cm	40 - 47 cm	CTI-23002	CTI-22002
M	10 - 11 cm	45 - 55 cm	CTI-23003	CTI-22003
L	11 - 13 cm	52 - 61 cm	CTI-23004	CTI-22004
XL	13 - 14 cm	58 - 67 cm	CTI-23005	CTI-22005

Includes: Condyle Pad Set and Extension Stop Set, 0° Extension Stop and Standard Condyle Pad are pre-installed.

Note: M/L measurements are the preferred and most accurate form of measurement for CTi. Circumferential measurements are approximate.

CTi[®]3

ACCESSORIES & REPLACEMENT PARTS

CTi3 MX KIT

Size	Left	Right
S	CTI-23132	CTI-22132
M	CTI-23133	CTI-22133
L	CTI-23134	CTI-22134
XL	CTI-23135	CTI-22135

Includes: Impact Guard, Sleeve with Mesh, CTi Decal Sheet and CTi Product Bag.

LINER SET

Size	Left	Right
S	CTI-23322	CTI-22322
M	CTI-23323	CTI-22323
L	CTI-23324	CTI-22324
XL	CTI-23325	CTI-22325

HOOK SET

Size	Left	Right
S	CTI-23332	CTI-22332
M	CTI-23333	CTI-22333
L	CTI-23334	CTI-22334
XL	CTI-23335	CTI-22335

STRAP SET

Size	Part #
S	CTI-24222
M	CTI-24223
L	CTI-24224
XL	CTI-24225

MISCELLANEOUS

Description	Part#
Guard Elastic Cord	CTI-24130
Guard Screws	CTI-24140
Tibial Wedge Pad	22077
Condyle Pad Set (Thin, Standard, Thick)	CTI-24320
Extension Stop Set (0°, 5°, 10°, 20°, 30°)	CTI-24430
Flexion Stop Set (0°, 60°, 75°, 90°)	CTI-24440
Hinge Sticker Set	CTI-24450
D-Ring Set	CTI-24510
CTi3 PCL Strap Kit	CTI-24520

SLEEVE WITH MESH

Size	Part#
S	SLV60012
M	SLV60013
L	SLV60014
XL	SLV60015

CTi® CUSTOM

≤18

CTi braces provide the ultimate combination of stabilisation and protection of the knee joint. This truly custom made brace, with rock-solid CTi carbon frame, uses Accutrac® hinges with extension stops to glide with the knee and breathable liners coated with bio-inert Sensil® silicone to ensure the CTi remains properly positioned on the leg, for optimal functional benefit. CTi custom braces are available in different models and can be further customised with a variety of options, colours, designs and accessories.



INDICATIONS FOR USE

For knee conditions that may benefit from increased AP and ML stability around the knee, such as: ACL, MCL, LCL, PCL, rotary and combined instabilities

FEATURES AND BENEFITS

- Reversible field-serviceable d-rings and trimmable straps facilitate fast adjustments and customised fit (optional)
- Corrosion resistant materials, ideal for water sports
- Anatomically-correct Accutrac® hinges with extension stops
- Proprietary Sensil® silicone liners designed to reduce migration
- Available in 3 different models: Standard model for moderate to high impact levels, Pro Sport model for the impact levels and Vapor model for low to moderate impact levels


REQUIRED MEASUREMENTS

- Custom made to measure (CMS, Smartmeasure, 3D-Scan or patient cast)


CUSTOM STANDARD COLORS



CUSTOM PAINTED DESIGNS

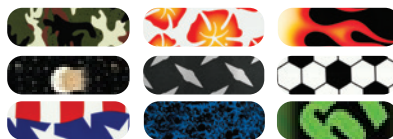

No charge for custom standard colors. Additional fees required for other custom colors, custom painted designs and custom printed graphics.

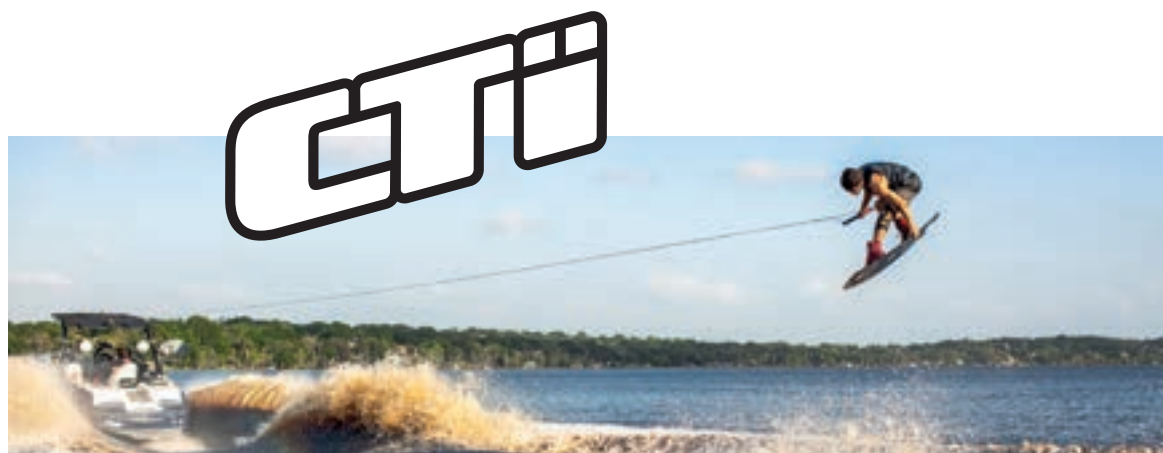

CTI® CUSTOM

ORDERING INFORMATION

CTI CUSTOM

Description	Left	Right
Vapor	B-118600010	B-118500010
Standard	B-116600010	B-116500010
Pro Sport	B-117600010	B-117500010

CTI OA

Description	Left	Right
Vapor	B-118630010	B-118530010
Standard	B-116630010	B-116530010
Pro Sport	B-117630010	B-117530010

Note: CTi OA can be constructed with up to 7° of varus or valgus correction.



SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace, including the CTi. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.

ACCESSORIES

CALF LINERS

Description	Part#
CTi Custom liner	B-411500115

STRAPS KITS

Size	Part#
S-M	B-701130104
L-XL-XXL	B-701130108

PCL KITS

Size	Part#
S-M	B-705000132
L-XL-XXL	B-705000235



CTI® CUSTOM

ACCESSORIES

STRAP PADS

Size	Part#
S	B-304010057
M-L	B-304010058
XL-XXL	B-304010059



Strap pads

AMS (ANTI-MIGRATION SYSTEM) WRAPS

Description	Part#
AMS Wrap EVA	22095
AMS Wrap Neo	22094
AMS Wrap Neo XL	22152



AMS Wrap

MX GEAR

Description	Part#
Patella Cup	B-711130011
Gear Guards	B-705000003
MX Kit S (patella cup, gear guards, undersleeve, stickers)	B-711140002
MX Kit M (patella cup, gear guards, undersleeve, stickers)	B-711140003
MX Kit L (patella cup, gear guards, undersleeve, stickers)	B-711140004
MX Kit XL (patella cup, gear guards, undersleeve, stickers)	B-711140005
MX Kit XXL (patella cup, gear guards, Sleeve with Mesh, stickers)	B-711140006



MX Kit

Gear guards

Patella cup

Unloader One X Sleeve

Stickers

Neoprene under- or oversleeve

Softsleeve

Sleeve with Mesh

SLEEVES

Size	Circumference 15cm above mid patella	Softsleeve	Undersleeve	Oversleeve
S	< 38cm	20643	20251	20051
M	38 - 45cm	20644	20252	20052
L	45 - 49.5cm	20644	20253	20053
XL	49.5 - 54.5cm	20645	20254	20054
XXL	54.5 - 62cm	20645	20255	20055
XXXL	62 - 71cm	20646	20256	20056
XXXXL	71 - 86cm	20647	20257	20057

Size	Circumference 15cm above mid patella	Sleeve with Mesh
XS	< 43 cm	SLV60011
S	43 - 46 cm	SLV60012
M	46 - 51 cm	SLV60013
L	51 - 54 cm	SLV60014
XL	54 - 56 cm	SLV60015
XXL	56 - 61 cm	SLV60016

OTHER ACCESSORIES

Description	Part#
CTi D-Ring Kit Standard 2x 1" and 2x 1.5" d-rings	B-704132600
CTi D-Ring Kit 2" 2x 2" d-rings	B-704132602
CTi Condyle Pad Kit	B-704130150
Extension Stop Kit	22130
Flexion Stop Kit	19254

MEASUREMENT FORM CTi[®] CUSTOM BRACE

CUSTOM MEASUREMENT SYSTEM

Date: _____ Customer #: _____ Purchase order #: _____

Required patient information

Patient ID: _____ Diagnosis: _____

Height: _____ cm. Weight: _____ kg. Date of Birth: _____ Sex(m/f): _____ Left leg Right leg

Bilats - l/r

Customer:

Contact: _____ Prescribing consultant: _____

Phone: _____ Clinic: _____

Company: _____

Please fill out all mandatory fields (marked in bold)

- VAPOR**
(For low to medium impact levels)
- STANDARD***
(For medium to high impact levels)
- PRO SPORT**
(For highest impact levels)

OPTIONS

D-ring attachment

- Screwed Riveted*
- Internal External*

Thigh Liner

- None* Sensil
- Evazote/Foam

Calf Liner

- Sensil* Evazote/Foam

Extension Stops

- 0° 10°* 20° 30° 40°

SPECIAL MODIFICATIONS

(Additional charges may apply)

- Flexion Stop Kit
_____° (0-90° installed)
- PCL System
(Helps reduce posterior drawer)
- ACL Cable Kit (Additional stability in ext)
- Hyperextension Straps (5th strap added)
- Ski Boot Attachment (Brace-to-boot)

Other: _____

SPECIAL NOTES:

- Please contact me about this order

*This configuration is delivered by default, if nothing is filled in.

SPECIAL CONDITIONS:

- OA (Osteoarthritis unloading)
 - Medial
 - Lateral _____°
(Choose 1° to 7° of unloading, 4°*)
- Prominent VMO
- Prominent VLO
- Hockey Quads
- Prominent Tibialis Anterior
- Prominent Fibular Head
- RTM
(Rounded Tibial Member)
- Osgood-Schlatters Disease
- Super Short
(For 1.60m and under)
- Shorten Frame Component
Tibial 1.3cm 2.6cm 3.9cm
Femoral 1.3cm 2.6cm 3.9cm

COLORS:

- Select finish:
- MATTE*** **GLOSS**
- Select color:
- Black* Yellow
 - Ocean Blue Orange
 - White Lime Green
 - Silver Pink
 - Red Charcoal
 - Champagne Navy Blue

ACCESSORIES:

(**Bolded** are free with initial order only)

- AMS WRAP**
 - Neoprene*** **Evazote**
- Sofsleeve**
 - Neoprene Undersleeve
 - Neoprene Oversleeve
 - Padded Sports Oversleeve
 - Sleeve with Mesh
 - MX Patella Cup
 - MX Gear Guards
 - MX Complete Kit
(Patella Cup, Gear Guards, Sleeve with Mesh)
- Other: _____

CUSTOM PAINT:

(Additional fee plus extra delivery time)

- Select custom finish:
- MATTE*** **GLOSS**
- Select custom color:
- Hibiscus Sports Equipment
 - Flag Camouflage
 - Flames Single Color
 - Metal 2-color Fade
 - Sponge 2-color Half & Half
 - Celestial Sports Fan
- Special
(Provide design file with order)

MEASUREMENT FORM CTi® CUSTOM BRACE

CUSTOM MEASUREMENT SYSTEM

CUSTOM MEASURING SYSTEM

Left leg Right leg

1: _____ Hinge Width
2: _____ Hinge Depth

LATERAL MEASUREMENTS

3: _____ Contour
4: _____ A-P
5: _____ Contour
6: _____ A-P
7: _____ Contour
8: _____ A-P
9: _____ Tibia
10: _____ Tibia
11: _____ Contour
12: _____ A-P
13: _____ Tibia

MEDIAL CONTOURS

14: _____ Contour
15: _____ Contour
16: _____ Contour
17: _____ Contour

CIRCUMFERENCES (in centimeters)

18. Level A _____
19. Level B _____
20. Level C _____
21. Level E _____
22. Distal Border of Patella _____

JOINT SPACE

23.
Caliper measurement at joint space (1.6 - 2.0 cm less than #1) Take measurement in a weight bearing position at full extension

SHORTEN TIBIA COMPONENT/ SKI BOOT MEASUREMENT

_____ cm from the distal border of the patella to the top of the boot. Ski boot attachment is not available if the measurement is less than 18cm.

CMS INSTRUCTIONS

Patient should not be wearing shoes for the measurement procedure.

There should be no space between the block and the back of the knee. The system will not be accurate if the knee is measured in a flexed position.

1. Slide the lateral pad to the knee. Tighten to patient's individual comfort level. This measurement will be the actual width of the patient's knee brace.

2. Pre-set hinge depth at approximately 3.5cm, adjust if needed.

3. – 13. Patient should remain still with foot dorsiflexed throughout the lateral measurements and recording process.

9., 10. and 13. Measure using the horizontal bar of the tool measure the position of the highest point of the tibialis anterior.

- 9. As reference take the #1 hinge width / 2 - 1 = ...
(for example 13cm / 2 - 1 = 5.5cm)
- 10. should be 0.5cm less than 9.
- 13. should be 1cm less than 9.


14. – 17. The patient may now relax throughout measuring the medial contours from the contracted position. Please note that with larger legs this measurement may be a negative value. Please read the numbers carefully and record negative values if necessary, such as -0.5 or -1.5

18. – 22. Take the measurements with a soft measuring tape ensuring that the tape is snug but not to loose/tight.


MEASUREMENT FORM CTi® CUSTOM BRACE

CUSTOM MEASUREMENT SYSTEM


SPECIAL CONDITIONS REFERENCES

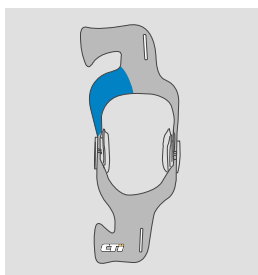


Prominent VLO

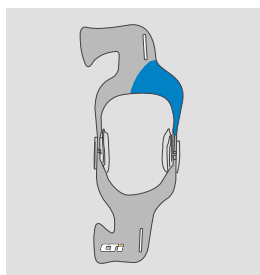


Prominent VMO

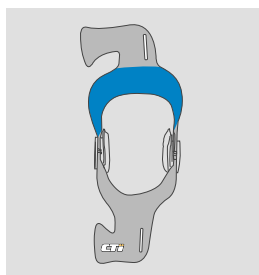


Hockey Quads

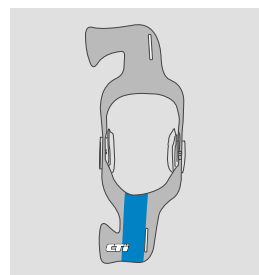


Prominent Tibialis Anterior

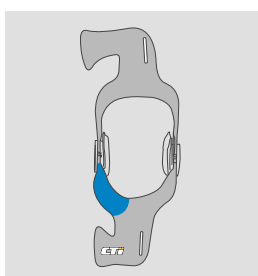


Prominent Fibular Head

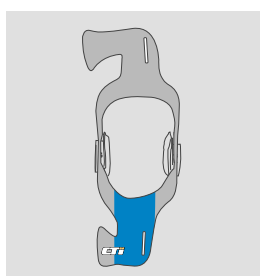


RTM
(Rounded Tibial Member)

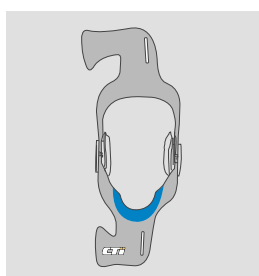


Osgood-Schlatters Disease

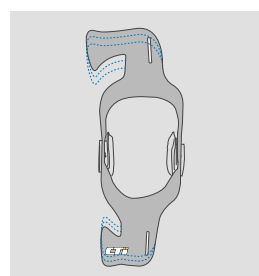

Shorten Femoral Component
Shorten Tibial Component

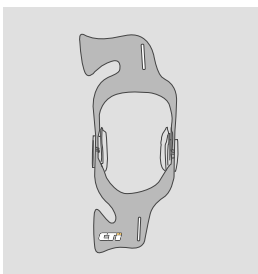

Super Short

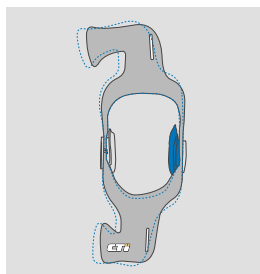

Osteoarthritis Unloading

REBOUND® PCL

Unlike the anterior cruciate ligament (ACL), the PCL has the intrinsic ability to heal and regain functionality after injury. Physiologically correct positioning of the PCL during conservative or post-surgical rehabilitation is important, as the PCL experiences variable tensile forces during knee flexion, and healing of the PCL in an elongated position can lead to chronic instability and disability. Insufficient healing after a PCL rupture leads to significant pathologies, such as chronic anterior knee pain or knee osteoarthritis.

The Rebound PCL applies a physiologically correct dynamic force to the PCL during rehabilitation. The dynamic load on the tibia is generated by applying an anterior directed dynamic force on the calf area, and opposing counterforces on the anterior aspect of the leg.


FUNCTIONAL HEALING - WHERE MOBILITY MEANS RECOVERY

INDICATIONS FOR USE

- Non-surgical treatment of PCL ruptures

Post-surgical rehabilitation for:

- Partial PCL tears
- PCL reconstruction/augmentation
- PCL revision

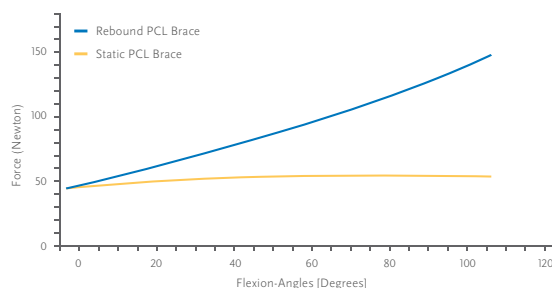

LaPrade RF, Smith SD, Wilson KJ, Wijdicks CA. Quantification of functional brace forces for posterior cruciate ligament injuries on the knee joint: an in vivo investigation. *Knee Surg Sports Traumatol Arthrosc* [Internet]. 2014 Aug 22 [cited 2014 Dec 11]; Available from: <http://www.ncbi.nlm.nih.gov/pubmed/25145947>

FEATURES AND BENEFITS

- The Dynamic Tension System™ (DTS) is designed to facilitate healing by increasing the load on the tibia as the knee goes through flexion, and providing an anterior drawer force necessary to decrease load on the healing posterior cruciate ligament (PCL).
- Three shear knob options permit customisation of the biochemical correct posterior tibial load for each patient, and allow for adjustment, e.g. in case of swelling.
- A polycentric hinge with extension and flexion stops allows for range of motion adjustment. Lightweight brace with proprietary Sensil® silicone calf liner and doeskin thigh liner to reduce migration, while providing comfort to the user.
- Posterior frame for optimised support, with open anterior for ease of donning/doffing.

REQUIRED MEASUREMENTS

- Calf circumference 15cm below mid patella
- Thigh circumference 15cm above mid patella

REBOUND® PCL

ORDERING INFORMATION

REBOUND PCL

Size	Caliper M/L*	Calf Circ.**	Thigh Circ.***	Left	Right
S	8.9 - 10.2cm	29 - 34.5cm	42 - 45cm	B-124501161	B-124501158
M	10.2 - 11.4cm	34.5 - 38cm	45 - 51cm	B-124501162	B-124501159
L	11.4 - 12.7cm	38 - 42cm	51 - 58.5cm	B-124501163	B-124501160
XL	12.7 - 14.0 cm	42 - 45.7 cm	58.5 - 66.7cm	B-124501155XL	B-124501154XL

* M/L measurement: In a standard, weight bearing position - Using a Caliper: M/L at joint width, Using a Constant Force Caliper: M/L at knee axis line

** Calf circumference measurement: 15cm below knee centre

*** Thigh circumference measurement: 15cm above knee centre

DOESKIN THIGH LINER

Size	Left	Right
XS	PCL60031	PCL60011
S	PCL60032	PCL60012
M	PCL60033	PCL60013
L	PCL60034	PCL60014
XL	PCL60035	PCL60015
XXL	PCL60036	PCL60016

SENSIL CALF LINER

Size	Left	Right
XS	PCL60171	PCL60071
S	PCL60172	PCL60072
M	PCL60173	PCL60073
L	PCL60174	PCL60074
XL	PCL60175	PCL60075
XXL	PCL60176	PCL60077

DOESKIN CALF LINER

Size	Left	Right
XS	PCL60121	PCL60021
S	PCL60122	PCL60022
M	PCL60123	PCL60023
L	PCL60124	PCL60024
XL	PCL60125	PCL60025
XXL	PCL60126	PCL60026

STANDARD THIGH LINER KIT

Size	Left	Right
XS	B-124501170	B-124501164
S	B-124501171	B-124501165
M	B-124501172	B-124501166
L	B-124501173	B-124501167
XL	B-124501174	B-124501168
XXL	B-124501175	B-124501169

Includes: Doeskin thigh liner, doeskin femoral cuff liner (softgood on lower thigh strap), doeskin pre-femoral liner (softgood on upper thigh strap), hook and loop (4x round coins, 8x rec. tabs)

REBOUND® PCL

ORDERING INFORMATION

STANDARD CALF LINER KIT

Size	Left	Right
XS	B-124601176	B-124501176
S	B-124601177	B-124501177
M	B-124601178	B-124501178
L	B-124601179	B-124501179
XL	B-124601180	B-124501180
XXL	B-124601181	B-124501181

Includes: Silicone calf liner/AMS, doeskin tibia cuff liner (softgood on lower calf strap), hook and loop (2x round coins, 4x rec. tabs)

DOESKIN CALF LINER KIT

Size	Left	Right
XS	B-124601182	B-124501182
S	B-124601183	B-124501183
M	B-124601184	B-124501184
L	B-124601185	B-124501185
XL	B-124601186	B-124501186
XXL	B-124601187	B-124501187

Includes: Doeskin calf liner/AMS, doeskin tibia cuff liner (softgood on lower calf strap), tibia strap pad, hook and loop (2x round coins, 4x rec. tabs)



SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.

REBOUND PCL CUSTOM

Description	Part#
Rebound PCL Custom Left	B-124501155
Rebound PCL Custom Right	B-124501154

*Please allow 10 working days for delivery

CUSTOM STANDARD COLORS



REBOUND® PCL

ACCESSORIES

STRAPS

Size	Part#
S-L	PCL00061
XL-XXL	PCL00062

OTHER ACCESSORIES

Description	Part#
Extension Stop Kit	B-705130042
Flexion Stop Kit	B-705130043
Shear Knob Kit	B-465350087
Condyle Pad Kit	B-705130046



REBOUND® ACL

The Rebound ACL brace is an evolution in bracing, designed to comfortably apply an anatomically correct, dynamic load to the anterior cruciate ligament (ACL). As an element in either conservative or surgical management, the Rebound ACL brace complements clinical protocols and is conducive to an ideal biomechanical joint environment as a foundation for long-term successful patient outcomes. The innovative technology in the Rebound® Dynamic Tension System (DTS) offers individualised adjustment options to reduce strain on the healing ACL.




FUNCTIONAL HEALING – WHERE MOBILITY MEANS RECOVERY

INDICATIONS FOR USE

Conditions benefiting from an agonistic force that varies throughout flexion in proportion to ACL tension applied to the thigh and calf in the sagittal plane. These may include:

- Anterior cruciate ligament (ACL) injury
- Reconstruction
- Non-surgical treatment

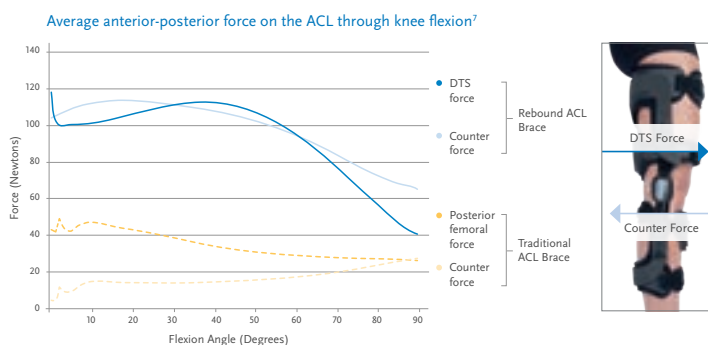

FEATURES AND BENEFITS

- Dynamic technology for knee stability, with a Dynamic Tension System™ that applies an agonistic, dynamic force to the ACL that is flexion-angle dependent (LaPrade et al., 2017) and designed to reduce ACL deficiency (positive anterior-drawer test), promoting knee stability.
- Specific load adaptation according to the patient's individual anatomy and rehabilitation requirements. Clinical and biomechanical studies indicate that the Rebound ACL brace reduces strain on the ACL during activity and significantly reduces strain on the meniscus in ACL-deficient knees (Tomescu et al., 2017). Biocompatible, hi-tech materials, including Blue is You™ patient touchpoints and ActiveGrip™ liners, promote suspension and comfort.
- Lightweight, aircraft-grade aluminium frame
- Available in sizes S-XL
- Flexion & extension range-of-motion lock-outs.
- Posterior frame optimised for fit and support, with open anterior for ease of donning/doffing Clinically-tested comfort, validated by wearers and with load mapping data collected during rehabilitation activities, the Rebound® Dynamic Tension System (DTS) provides a comfortable load to the posterior thigh and anterior tibia.

REBOUND® ACL

REQUIRED MEASUREMENTS



Measure knee caliper medio-laterally at joint width in standing, weight-bearing position.



Measure calf circumference 15cm below mid-patella. Leg in extension.



Measure thigh circumference 15cm above mid-patella. Leg in extension.

ORDERING INFORMATION

STANDARD

Size	Left	Right	Caliper	Calf circumference 15cm below mid patella	Thigh circumference 15cm above mid patella
SM	B-124601191	B-124501191	8.9 - 10.2cm	29.2 - 34.3cm	41.9 - 47.0cm
MD	B-124601192	B-124501192	10.2 - 11.4cm	34.3 - 39.4cm	47.0 - 52.1cm
LG	B-124601193	B-124501193	11.4 - 12.7cm	39.4 - 44.5cm	52.1 - 57.2cm
XL	B-124601194	B-124501194	12.7 - 14.0cm	44.5 - 49.5cm	57.2 - 62.2cm

Recommended for patients 1.56m and taller

REPLACEMENT PARTS

Description	Part#
Extension stop kit (0°, 5°, 10°, 20°, 30°, 40°)	B-124702035
Flexion stop kit (0°, 15°, 25°, 35°, 45°, 55°, 65°, 75°, 90°, 105°)	B-124702034
Shear Knob Kit	B-705130051
SM/MD Strap Kit	B-124702030
LG/XL Strap Kit	B-124702031
Condyle Pad Kit	B-701090009
Tibial Pad	B-124702032
Tibial Hex Pad	B-124702033

REBOUND® ACL

ACCESSORIES

ANTI-MIGRATION WRAP

SIZE	RIGHT	LEFT
S-M	B-124501195	B-124601195
S-M HEX	B-124501196	B-124601196
L-XL	B-124501197	B-124601197
L-XL HEX	B-124501198	B-124601198

UPPER DOESKIN LINER KIT

SIZE	RIGHT	LEFT
S	B-124702004	B-124702000
M	B-124702005	B-124702001
L	B-124702006	B-124702002
XL	B-124702007	B-124702003

LOWER ACTIVEGRIP LINER KIT

SIZE	RIGHT	LEFT
S	B-124702014	B-124702010
M	B-124702015	B-124702011
L	B-124702016	B-124702012
XL	B-124702017	B-124702013

LOWER DOESKIN LINER KIT

SIZE	RIGHT	LEFT
S	B-124702024	B-124702020
M	B-124702025	B-124702021
L	B-124702026	B-124702022
XL	B-124702027	B-124702023

REBOUND® ACL

ORDERING INFORMATION FOR REBOUND ACL CUSTOM

REBOUND ACL CUSTOM

Part#	Type	Min / Max Caliper	Min / Max Circ. above	Min / Max Circ. below	Measuring
B-124701000	Right	8.9 - 14cm	41.9 - 62.2cm	29.2 - 49.5cm	SmartMeasure, Cast
B-124701001	Left				

Caliper = M/L width at knee center
 Circumference = 15cm above / below mid-patella
 Ideal for patients 1.56m and taller

SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required.

REQUIRED MEASUREMENTS

- Custom made to measure (Smartmeasure, 3D-Scan or patient cast)

EXTREME®

For bilateral bracing applications, saddle sports, and other situations where a single, lateral hinge is advantageous, consider Extreme. The custom made Extreme is durable enough for rigorous athletic challenges, yet its Triax® shells are malleable, offering adaptability to changes in leg size and varus or valgus alignment. A carbon steel Poli-axial® hinge and Dynamic Force Strap® combine to provide excellent rotational control and proprioception.

INDICATIONS FOR USE

- For knee conditions that may benefit from increased AP and ML stability around the knee, such as: ACL, MCL, LCL, PCL, rotary and combined instabilities

FEATURES AND BENEFITS

- The proven single-upright design provides dynamic stability, offering maximal support when the knee is under stress
- Dynamic Force Strap® minimises bulk and maximises agility.
- Optional tool-free SmartDosing® dial option enables fine-tuned pain management and can be adjusted even when wearing the brace under clothing.

REQUIRED MEASUREMENTS

- Custom made to measure (Smartmeasure, 3D-Scan or patient cast)



ORDERING INFORMATION

Description	Part#
Extreme Left with DFS strap	B-108600010
Extreme Right with DFS strap	B-108500010
Extreme with SmartDosing Left	B-108600020
Extreme with SmartDosing Right	B-108500020

ACCESSORIES

Description	Part#
Patella cup	B-250110001
Hinge cover	B-251020000
AMS Wrap (custom made)	B-426000150
AMS Sleeve (custom made)	B-426000250

GEL PAD FOR HOT/COLD THERAPY

Size	Össur knee undersleeve	Softsleeve	Neoprene undersleeve	Neoprene oversleeve
XS	B-250150101			
S	B-250150102	20643	20251	20051
M	B-250150103	20644	20252	20052
L	B-250150104	20644	20253	20053
XL	B-250150105	20645	20254	20054
XXL	B-250150106	20645	20255	20055
XXXL		20646	20256	20056
XXXXL		20647	20257	20057

REBOUND® POST-OP KNEE

≤18

The simple, versatile post-operative knee bracing solution that is easy for everyone. Whether adjusting ROM, setting the drop lock, or altering the brace length, you'll appreciate the simplicity of the Rebound Post-Op knee brace.



INDICATIONS FOR USE

- ACL, PCL, MCL, LCL repairs/injuries
- Meniscal repairs
- Tibial plateau fractures
- Patellar tendon repairs
- Osteochondral repairs
- Condylar fractures
- Sprains/strains of the knee
- HTOs (High Tibial Osteotomy)

FEATURES AND BENEFITS

- Tool-free design and multi-purpose Smart-Fit™ packaging
- Available in universal, left or right contour and paediatric
- Telescoping frame from 47cm-68cm (Adult) and 39.5cm - 54cm (paediatric)
- OneTouch™ easy to adjust ROM hinge with lock-out capability
- Colour-coding and numbered frame for patient education
- Quick-fit, gripping buckles for easy application

REQUIRED MEASUREMENTS

- Circumference: 15cm above mid-patella

ORDERING INFORMATION

REBOUND POST-OP

Description	Thigh circumference	Cool	Full foam
Paediatric	30.5 - 63.5cm	B-8381470	
Universal, Left contoured	41 - 81cm	B-8381680	B-8382680
Universal, Right contoured	41 - 81cm	B-8381580	B-8382580
Universal	41 - 81cm	B-8381480	B-8382480
Universal XL	41 - 96,5cm	B-8381490	B-8382490

ACCESSORIES

Description	Part#	Description	Part#
ROM stop locks	B-8380005	Standard buckle & pad kit	B-8381096
Alligator hook & loop kit	B-8380006	AMS Kit Paediatric	B-8381093
Adhesive Hook & Loop kit Paediatric	B-8380008	AMS Kit Regular	B-8381094
Adhesive Hook & Loop kit universal / universal XL	B-8380007	AMS Kit XL	B-8381095

REPLACEMENT SOFTGOODS

Description	Cool	Full foam
Paediatric	B-8381072	
Universal*	B-8381082	B-8382082
Universal XL	B-8381092	B-8382092

*For universal, left contoured, right contoured

FORMFIT POST-OP KNEE

INDICATIONS FOR USE

- Post-op/post-injury ROM control or immobilisation
- Indicated for ligament and fracture-related surgeries and when controlled range of motion is necessary

FEATURES AND BENEFITS

- Breakaway struts for easy, tool free length adjustment
- OneTouch™ easy to adjust ROM hinge with lock-out capability from -10° to 30° (10° incremental)
- Extension control from -10° to 90° (10° incremental)
- Flexion control from 0° to 120° (10° incremental)
- Available in cool or foam models, 3 lengths, and two sizes (Regular and XL)



REQUIRED MEASUREMENTS



Measure thigh circumference
15cm above mid-patella.
Leg in extension.

Size	Thigh circumference 15cm above mid patella
XL	69 - 89cm
Regular	0 - 69cm

ORDERING INFORMATION

Size	Thigh circumference	Cool	Full Foam
Knee PO 61cm	UP TO 69cm	FPOK403100	FPOK403200
Knee PO XL 61cm	69 - 89cm	FPOK403102	FPOK403202
Knee PO 71cm	UP TO 69cm	FPOK403101	FPOK403201
Knee PO XL 71cm	69 - 89cm	FPOK403103	FPOK403203
Knee PO 41cm	69cm	-	FPOK403300
Knee PO XL 41cm	69 - 89cm	-	FPOK403301

SOFT GOODS

Description	Cool	Full Foam
Replacement Softgood, Regular	403175	403275
Replacement Softgood, XL	403176	403277
Replacement Softgood, Regular Long	403276	403278

REPLACEMENT SOFTGOODS

Size	
ROM stop locks (set of 20 lock clips)	B-8380005
Strap Reduction Kit Cool	403199
Strap Reduction Kit Full Foam	403299

REBOUND® KNEE IMMOBILIZER UNIVERSAL

Rebound® Knee Immobilizer Universal features a universal 3-panel design that can be adjusted to accommodate most leg circumferences. With an MR* Conditional rating, patients can be safely scanned in an MR system, while maintaining immobilisation.

*See IFU for details.



INDICATIONS FOR USE

- Emergency immobilisation
- Pre- and post-surgical knee immobilisation
- Traumatic knee injuries

FEATURES AND BENEFITS

- Adjustable medial/lateral stays and rigid posterior stays provide motion restriction of the knee
- Universal 3-piece design simplifies inventory needs
- MR* Conditional: safely scan patients in an MR system while maintaining immobilisation
- Min thigh circumference: 26cm
- Max thigh circumference: 67.31cm

ORDERING INFORMATION

Length	Part#
30 cm	210012
35 cm	210014
40 cm	210016
45 cm	210018
50 cm	210020
55 cm	210022
60 cm	210024

ACCESSORIES

Description	Part#
Hot/Cold Therapy Gel Pad	GB7
PCL Pad	B-705000036



PCL Pad

FORMFIT® PRO KNEE

Össur Formfit® Pro Knee is a compressive knee sleeve that features a SmartPad™ buttress for targeted support and cushioning. Our patented design realigns the patella, suitable for conditions such as Patella Tendonitis (Jumper's Knee) and Chondromalacia Patella. The dynamic, 3D MotionTech™ knit is supportive, meets certified medical grade compression requirements and has been proven to be superior to the leading brands at moisture wicking and breathability, keeping your knee cool during wear. Lightweight, removable spiral stays ensure optimum shape and fit, while distinct patella graphics make donning intuitive.



MotionTech™



CoolVent™



StayBilizer™



SmartPad™

INDICATIONS FOR USE

For knee conditions that may benefit from compression, such as;

- Patella tendonitis
- Chondromalacia Patella
- Mild sprains and strains
- Knee pain
- Feeling of instability


CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.


FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit with superior moisture wicking and breathability, keeping your knee cool during wear
- Certified medical grade compression (20-30 mmHg)
- Lightweight CoolVent™ knit in popliteal area for added breathability and comfort
- Patented SmartPad™ for targeted support and cushioning
- Patella graphics for intuitive placement
- Patented FoldLock™ system for easier and more secure application
- Lightweight, removable spiral steel stays

REQUIRED MEASUREMENTS

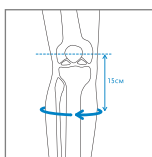

Measure calf circumference
15cm below mid-patella.
Leg in extension.

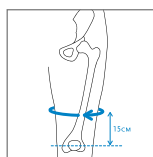

Measure thigh circumference
15cm above mid-patella.
Leg in extension.

Size	Calf circumference 15cm below mid patella	Thigh circumference 15cm above mid patella
XS	26 - 31cm	36 - 41cm
S	31 - 36cm	41 - 46cm
M	36 - 41cm	46 - 51cm
L	41 - 46cm	51 - 56cm
XL	46 - 50cm	56 - 60cm

FORMFIT® PRO KNEE

ORDERING INFORMATION

BLACK

BLUE

Size	Part#	Size	Part#
XS	FP100001	XS	FP101001
S	FP100002	S	FP101002
M	FP100003	M	FP101003
L	FP100004	L	FP101004
XL	FP100005	XL	FP101005



FORMFIT® PRO KNEE FLITE

Össur Formfit® Pro Knee Flite is a performance 3D knitted sleeve that is micro-engineered from our custom MotionTech™ yarn to provide breathability, moisture wicking, and a skintight fit. Our unique FoldLock™ and StayBilizer™ technologies combine to ensure the brace stays in place for maximum benefit.



MotionTech™



CoolVent™



StayBilizer™



FoldLock™

INDICATIONS FOR USE

For knee conditions that may benefit from compression, such as:

- Mild sprains and strains
- Knee pain
- Feeling of instability


FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit
- CoolVent™ knit in popliteal area for added breathability
- Patented FoldLock™ system for easy and secure application
- Lightweight, removeable spiral steel stays

CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

REQUIRED MEASUREMENTS

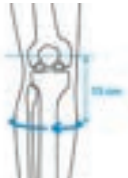

Measure calf circumference 15 cm below mid-patella. Leg in extension.



Measure thigh circumference 15 cm above mid-patella. Leg in extension.



Size	Calf circumference 15 cm below mid patella	Thigh circumference 15 cm above mid patella	Blue	Black
XS	26 - 31 cm	36 - 41 cm	FP141001	FP140001
S	31 - 36 cm	41 - 46 cm	FP141002	FP140002
M	36 - 41 cm	46 - 51 cm	FP141003	FP140003
L	41 - 46 cm	51 - 56 cm	FP141004	FP140004
XL	46 - 50 cm	56 - 60 cm	FP141005	FP140005

FORMFIT® KNEE HINGED

The Formfit Knee brace is for patients suffering from mild to moderate strains, sprains or ligament tears (ACL/PCL/MCL/LCL). It provides excellent stability, protection and compression during injury recovery. Made from strong and breathable materials it offers comfort while remaining highly durable. The open popliteal area reduces material bulging and potential irritation, while the malleable rigid hinge arms provide an intimate fit.

The Formfit Knee brace comes in either a wrap or a sleeve version.

INDICATIONS FOR USE

- Mild to moderate ACL and PCL tears
- MCL and LCL tears
- Combined mild knee sprains and strains
- Instabilities of the knee

FEATURES AND BENEFITS

- Breathable material
- Easy pull-on
- Malleable hinges
- Universal buttress
- Open popliteal area

REQUIRED MEASUREMENTS

- Knee circumference at mid-patella



ORDERING INFORMATION

Size	Knee Circumference	Sleeve	Wrap
XS	33 - 39cm	504252	
S	39 - 47cm	504253	503253
M	47 - 53cm	504255	503255
L	53 - 60cm	504257	503257
XL	60 - 67cm	504258	503258
XXL	67 - 75cm	504259	503259
XXXL	75 - 81cm		503260

FORMFIT® KNEE MCL

The Formfit Knee MCL provides external stability and protection for low to medium contact/impact activities and while recovering from injury. A unique, single hinged, hybrid wrap/sleeve design sets this brace apart from the competition. The lateral, range of motion hinge keeps the medial side free of obstruction. The Dynamic Force Strap aids in the resistance of valgus forces during knee extension, while the adjustable tibial strap improves suspension by minimising slippage.

INDICATIONS FOR USE

- Mild to Moderate Medial Collateral Ligament (MCL) sprains (Grade I and Grade II)

FEATURES AND BENEFITS

- Single, steel ROM lateral hinge (0° or 20° extension stop, 70° flexion stop)
- Dynamic Force Strap resist valgus forces
- Hybrid wrap/sleeve for easy application
- Adjustable tibial strap for improved suspension

REQUIRED MEASUREMENTS

- Knee circumference at mid-patella



ORDERING INFORMATION

Size	Thigh Circumference	Formfit Knee MCL Left	Formfit Knee MCL Right
XS	33 - 39cm	B-237600001	B-237500001
S	39 - 47cm	B-237600002	B-237500002
M	47 - 53cm	B-237600003	B-237500003
L	53 - 60cm	B-237600004	B-237500004
XL	60 - 67cm	B-237600005	B-237500005

FORMFIT® KNEE ROM BRACE

INDICATIONS FOR USE

- Mild to moderate ACL and PCL tears
- MCL and LCL tears
- Combined mild knee sprains and strains
- Instabilities of the knee

FEATURES AND BENEFITS

- With 6 options to choose from, you can confidently meet the unique needs of any patient
- When you need to limit range-of-motion, our ROM hinge provides the control you need. When you don't, you can opt for a more streamlined, polycentric hinge
- ROM braces available in 12" or 16" Length
- A removable, universal buttress is included with every sleeve version of the Formfit Knee Brace for additional, customisable patella support, if necessary



REQUIRED MEASUREMENTS

- Knee circumference at mid-patella

ORDERING INFORMATION

Size	Knee Circumference	Sleeve Short	Wrap Short	Sleeve Long	Wrap Long
XS	33 - 39cm	508252	507252	508352	507352
S	39 - 47cm	508253	507253	508353	507353
M	47 - 53cm	508255	507255	508355	507355
L	53 - 60cm	508257	507257	508357	507357
XL	60 - 67cm	508258	507258	508358	507358
XXL	67 - 75cm	508259	507259	508359	507359
XXXL	75 - 81cm		507260		507360

C180™ ROCKET

≤18

With a condensed size and Accutrac™ hinges that are modified to more accurately track young knees, the lightweight C180 Rocket provides support for children with ligament instabilities.

INDICATIONS FOR USE

- For knee conditions that may benefit from increased AP and ML stability around the knee, such as: ACL, MCL, LCL, PCL, rotary and combined instabilities

FEATURES AND BENEFITS

- Anatomically-correct Accutrac® hinges with easy-to-use extension stops
- Semi-rigid frame consisting of woven carbon fibre infused with acrylic resin, is heat-mouldable for customisation

REQUIRED MEASUREMENTS

- Caliper measurement: Medio-laterally at joint width in a standard, weight-bearing position
- Circumference: Just below the distal border of the patella



ORDERING INFORMATION

Size	Caliper	Circumference	Left	Right
S	7.6 - 8.9cm	24.1 - 27.9cm	C180KSL	C180KSR
L	8.3 - 9.5cm	26 - 29.8cm	C180KLL	C180KLR
XL	8.9 - 10.2cm	27.9 - 32.1cm	C1802XSL	C1802XSR

ACCESSORIES

Description	Part#
Motocross kit S / L	10833
Motocross kit XL	B-705130072
Patella Cup S / L	25029
Patella Cup XL	B-711130000



TABLE OF CONTENT

SPINE	73
MIAMI J®	75
MIAMI J® SELECT	77
MIAMI JTO® THORACIC EXTENSION	78
18 OCCIAN® BACK	79
18 MIAMI Jr®	80
PHILADELPHIA® ADJUSTABLE TRACHEOTOMY COLLAR	81
18 PHILADELPHIA® TRACHEOTOMY COLLAR	82
NECLOC®	83
18 NECLOC® KIDS	84
18 PAPOOSE®	85
MIAMI LSO®	86
MIAMI TLSO 464	88
MIAMI TLSO 456	90
MIAMI LUMBAR® POSTEO	92
FORMFIT® PRO BACK	93
FORMFIT® BACK SUPPORT AIR	94
FORMFIT® BACK SUPPORT	95

MIAMI J®

Known as the superior c-spine immobiliser through multiple independent studies, the Miami J's patented design meets the specialised needs of individuals with cervical injuries. Bioengineered to minimise pressure points in key known areas: Chin, occiput, trapezius, and clavicle. Miami J offers features that enhance compliance, while providing immobilisation, inhibition of skin deterioration, and patient comfort.



INDICATIONS FOR USE

Situations requiring gross immobilisation of the cervical spine, these may include:

- Pre and post c-spine surgery
- C-spine precaution for trauma patients


FEATURES AND BENEFITS

- Sorbatex padding is antibacterial and clinically proven to inhibit microbial growth, which enhances comfort and helps assure patient compliance
- Comfort adjustment buttons for occiput and sternal pad is removable when supine, lying and swallowing
- MRI, CT and X-ray lucent, MR compatible

REQUIRED MEASUREMENTS

- Sizing based on 6 phenotypes to ensure correct fit and optimum immobilisation

Phenotype-driven sizing system makes size selection simple while drastically reducing margin of error in fitting a patient. Each collar has a bioengineered front with a specific back designed to work together for superior immobilisation. For paediatric sizing, please see Miami Jr in this catalogue.

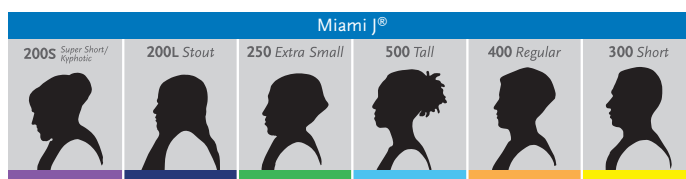

MIAMI J®

ORDERING INFORMATION

MIAMI J

Size	Collar set*	Collar	Collar Back	Collar Front
Super Short	MJR-200S	MJ-200S	MJ-400BK	MJ-200SFRT
Stout	MJR-200L	MJ-200L	MJ-200LBK	MJ-200LFRT
XS	MJR-250	MJ-250	MJ-250BK	MJ-250FRT
Short	MJR-300	MJ-300	MJ-300BK	MJ-300FRT
Regular	MJR-400	MJ-400	MJ-400BK	MJ-400FRT
Tall	MJR-500	MJ-500	MJ-500BK	MJ-500FRT
Bariatric	MJR-200B	MJ-200B		

*Collar Sets come with one additional set of replacement pads.

Miami J Front and back panels may be interchanged to accommodate patient anatomy, e.g. a Short front may be used with a Tall back. Choose collar size based on patient neck height, not patient body size. Super Short collars are packaged with regular back panels

ACCESSORIES

REPLACEMENT PADS

Size	Part#
Super Short	MJP-100
Stout	MJP-200L
XS	MJP-250
Short	MJP-100
Regular	MJP-100
Tall	MJP-100
Bariatric	MJP-200B

MIAMI J® SELECT

The Miami J Select cervical collar by Össur offers the range of height adjustability in the market with a single, universal design that replicates the comfort, immobilisation and quality you expect from a Miami J product. Featuring a patented, first-to-market locking mechanism to encourage patient compliance and anti-microbial Sorbatex padding, Miami J Select is designed to support successful patient outcomes.



SPINE SOLUTIONS

INDICATIONS FOR USE

- Situations requiring gross immobilisation of the cervical spine, these may include:
- Pre and post c-spine surgery
- C-spine precaution for trauma patients

FEATURES AND BENEFITS

- Easy height adjustability with a tamper-proof locking mechanism – ‘Left to Lock’
- Anti-microbial & biocompatible Sorbatex padding prevents bacterial build-up
- Universal design that doesn’t compromise immobilisation or comfort
- 7 precise fitting sizes with opportunity to fine-tune
- Miami J® inspired chin tray
- One-piece, anatomically inspired collar back with Flex Edge® designed to reduce occipital pressure
- Easy to don and doff with patented Reproducible Fit option
- Intuitive blue Patient Touch-Points to send patients home with confidence – Blue is You™
- Simplified comfort adjustments with integrated and patented Sternal-Relief Dial
- Large tracheal opening – improved function for ICU patients
- Easy to clean and replace multi-coloured padding
- X-ray and CT lucent, MR-safe
- Compatible with Occian® Back & Miami JTO® accessories

ORDERING INFORMATION

COLLAR SET

Description	Part#
Collar set	MJSR-101

MIAMI J SELECT COLLAR

Description	Part#
Miami J Select Collar	MJS-101

OTHER ACCESSORIES

Description	Part#
Miami J Select Replacement Pads	MJSP-101
Miami J Select Tabs Reproducible Fit Tabs (10 pack)	MJST-101

MIAMI JTO[®] THORACIC EXTENSION

The Miami JTO Thoracic Extension is designed in conjunction with leading spine surgeons, orthotists and nurses. The orthosis is used exclusively with all Miami J[®] collars when even greater stability at the cervical junctions is indicated.

Miami JTO Thoracic Extension is a versatile, tool-free device with a patented chin strut, providing more control than a collar alone can provide, designed to treat cervical and high-thoracic injuries.



INDICATIONS FOR USE

- Low c-spine and high t-spine precaution for trauma patients
- Immobilisation for pre and post low c-spine and high t-spine surgery
- Other conditions requiring gross immobilisation of the low-cervical and/or high-thoracic-spine
- Conditions requiring gross immobilisation of the low cervical and/or high-thoracic-spine

FEATURES AND BENEFITS

- Patented design with 'floating back' permits maximum immobilisation without the risk of compensatory motion
- Snap on and off without tools
- Sorbatex II padding is antibacterial and clinically proven to inhibit microbial growth, which enhances comfort and helps assure patient compliance
- MRI, CT and X-ray lucent

REQUIRED MEASUREMENTS

- Universal, one size

ORDERING INFORMATION

MIAMI JTO THORACIC EXTENSION

Description	Part#
Miami JTO	MT-900
Miami JTO with replacement softgoods	MTR-900

Compatible with all sizes of the Miami J[®], Miami J[®] Select and the P3 Miami Jr[®]. Miami J collar is not included with Miami JTO Thoracic Extension. Miami J collar needs to be purchased also

ACCESSORIES

Description	Part#
Replacement softgoods*	MTP-950

*Contains 2 x 4" Shoulder Strap Pads / 4 x 8" Waist Strap Pads / 4 x Snap Rivet Sets

OCCIAN® BACK

≤18

The Occian Back is designed exclusively for the specialised needs of multi-system trauma patient in the ICU. Universally sized, specifically engineered and designed with advanced, Intuitect™ pressure-relief memory foam, the Occian Back helps caregivers manage c-sp patients who are at high risk of occipital breakdown.



SPINE SOLUTIONS

INDICATIONS FOR USE

- Patients not awake and alert within 24 - 48 hours after admission
- Patients with pre-existing skin injuries and/or lacerations to the head/neck
- Burn patients who must wear a collar
- Elective surgery patients desiring night time post-operative comfort

FEATURES AND BENEFITS

- Intuitect pressure relieving memory foam
- Maintains c-spine stability
- Provides unique pressure-relieving comfort by replacing the back section of a Miami J Collar.
- Easy to clean
- MRI, CT and X-ray lucent

REQUIRED MEASUREMENTS

- Universal, one size

ORDERING INFORMATION

Description	Part#
Occian Back	ACB-100
Occian Back Junior	ACB-200

Compatible with all sizes of the Miami J®, Miami J® Advanced, Miami J® Select and the Miami Jr®. Collar front panel not included.

MIAMI Jr®

≤18

Miami Jr is engineered to fit the unique anatomies of patients aged 12 and under. The Miami Jr line, which spans the growth cycle, is sized according to age. Össur is the only company that has a paediatric line of collars based on the anatomical differences between adults and children. We realise that children are not just “little adults” - they require innovative design.



INDICATIONS FOR USE

Situations requiring gross immobilisation of the cervical spine, these may include:

- pre and post c-spine surgery
- c-spine precaution for trauma patients

FEATURES AND BENEFITS

- Designed exclusively for children 12 years and under
- Sorbatex II pads make the collar “skin friendly” and protect skin integrity during extended wear
- MRI, CT and X-ray lucent, MR compatible

REQUIRED MEASUREMENTS

- Age

ORDERING INFORMATION

MIAMI JR

Size	Collar set*	Collar
0 - 6 months	MJR-P0	MJ-P0
6 - 24 months	MJR-P1	MJ-P1
2 - 6 years	MJR-P2	MJ-P2
6 - 12 years	MJR-P3	MJ-P3

*Collar sets come with one additional set of replacement pads.

ACCESSORIES

REPLACEMENT PADS

Size	Part#
0 - 6 months	MJP-P0
6 - 24 months	MJP-P1
2 - 6 years	MJP-P2
6 - 12 years	MJP-P3

PHILADELPHIA® ADJUSTABLE TRACHEOTOMY COLLAR

Philadelphia tracheotomy collars have been an industry standard for decades. Philadelphia Adjustable is constructed from the same high-quality materials as the original, and combines convenient size adjustability with immobilisation and comfort.

INDICATIONS FOR USE

Situations requiring gross immobilisation of the cervical spine, these may include:

- Pre and post c-spine surgery
- C-spine precaution for trauma patients

FEATURES AND BENEFITS

- Circumference adjustable and height adjustable with permanent lock
- Anatomically shaped Plastazote® foam leaving clearance for cervical veins
- Well ventilated and water-resistant for bathing and hydro-therapy
- MRI, CT and X-ray lucent, MR compatible

REQUIRED MEASUREMENTS

- Universal, one size



SPINE SOLUTIONS

ORDERING INFORMATION

Description	Part#
Philadelphia Adjustable Tracheotomy Collar	PHP-AA

PHILADELPHIA® TRACHEOTOMY COLLAR

≤18

The Philadelphia Tracheotomy Collar is a hospital and rehabilitation collar that is designed to maintain neutral alignment of the cervical spine after injury to the neck. The collar contains a large tracheal opening, allows carotid pulse monitoring and emergency tracheotomies.

INDICATIONS FOR USE

- C-spine precaution for trauma patients
- Immobilisation for pre c-spine surgery
- Other conditions requiring gross immobilisation of the mid-cervical spine

FEATURES AND BENEFITS

- Anterior and posterior rigid plastic reinforcement supports limit movement
- Total cervical arch support maintains neutral alignment of the cervical spine
- Water-resistant for bathing and hydro-therapy
- Easily customised for individual requirements
- Does not contain natural rubber latex, non-toxic and hypoallergenic Plastazote® foam reduces skin irritation

REQUIRED MEASUREMENTS

- Neck circumference and height from tip of chin to sternal notch



ORDERING INFORMATION

Size	Neck circumference	6cm height	8cm height	11cm height	13cm height
XS	< 25cm	PHP-T2XS			
S	25 - 33cm	PHP-T2S	PHP-T3S	PHP-T4S	PHP-T5S
M	33 - 41cm	PHP-T2M	PHP-T3M	PHP-T4M	PHP-T5M
L	41 - 48cm	PHP-T2L	PHP-T3L	PHP-T4L	PHP-T5L
XL	> 48cm	PHP-T2XL	PHP-T3XL	PHP-T4XL	PHP-T5XL

Size	Neck circumference	Part#
Infant	15 - 20cm	PHP-TI
Paediatric	20 - 28cm	PHP-TP

ACCESSORIES

PHILADELPHIA COOLMAX COTTON LINER

Size	Neck circumference	Part#
Child	20 - 28cm	PHP-LCP
S	25 - 33cm	PHP-LCS
M	33 - 41cm	PHP-LCM
L	41 - 48cm	PHP-LCL

PHILADELPHIA TRACHEOTOMY ATTACHMENTS

Size	Stabiliser
Adult	PHP-OS

NECLOC®

INDICATIONS FOR USE

Situations requiring gross immobilisation of the cervical spine, these may include:

- Pre and post c-spine surgery
- C-spine precaution for trauma patients

FEATURES AND BENEFITS

- Ergonomic two-piece clamshell design and retaining strap secure flexion during application
- Five unique phenotype-driven sizes make it easy to select the correct size
- Minimal manipulation of patient during application
- X-ray and CT lucent. MR safe




REQUIRED MEASUREMENTS

- Sizing based on 5 phenotypes to ensure correct fit and optimum immobilisation

SIZING

- Phenotype-driven sizing system makes size selection simple while drastically reducing margin of error in fitting a patient. Each collar has bioengineered size specific front and back sections designed to work together for superior immobilisation. For paediatric sizing, please see NecLoc Kids.

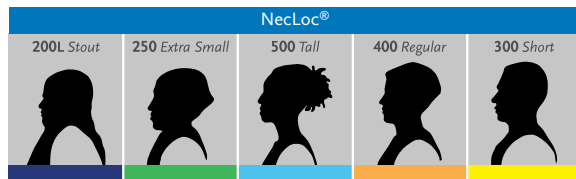

ORDERING INFORMATION

Size	Part#
Stout	NL-200EL
XS	NL-250E
S	NL-300E
M	NL-400E
L	NL-500E

NECLOC® KIDS

≤18

INDICATIONS FOR USE

Situations requiring gross immobilisation of the cervical spine, these may include:

- Pre and post c-spine surgery
- C-spine precaution for trauma patients

FEATURES AND BENEFITS

- Ergonomic two-piece clamshell design and retaining strap secure flexion during application
- Three paediatric sizes ensure optimal airway and cervical spine immobilisation
- Minimal manipulation of patient during application to children
- Meets the unique developing anatomical requirements of children
- X-ray and CT lucent. MR Safe

REQUIRED MEASUREMENTS

- Age



ORDERING INFORMATION

Age	Part#
0 - 2 years	NL-P1
2 - 6 years	NL-P2
6 - 12 years	NL-P3

PAPOOSE®

≤18

Papoose is an innovative design for newborns up to 3 months (Broselow Gray). This true cervical spine immobiliser cradles the baby's body, securely while gently immobilising the head and spine in the proper anatomical airway and spinal alignment to help achieve optimal patient outcome while allowing for continued intimacy between caregiver and child.



INDICATIONS FOR USE

- C-Spine precaution for infants
- Immobilisation for pre and post c-spine surgery
- Other conditions requiring gross immobilisation of the infant cervical spine

FEATURES AND BENEFITS

- X-ray & CT Lucent. MRI Safe

REQUIRED MEASUREMENTS

- Universal, one size

ORDERING INFORMATION

PAPOOSE

Description	Part#
Papoose without front collar	PO-100
Papoose without front with replacement pads	POR-100
Papoose with front collar	PA-100
Papoose with front with replacement pads	PAR-100

ACCESSORIES

REPLACEMENT PADS

Description	Part#
Papoose replacement pads (with front)	PAP-100
Papoose replacement pads (without front)	POP-100

MIAMI LSO®

Miami LSO is a spinal orthosis designed for pain relief, protection of injured ligaments or muscles and post-surgical immobilisation. It provides low-profile, anatomical immobilisation, easy fit and wear, options for modular support and an advanced pulley system with a length-adjustable pulley cord.



INDICATIONS FOR USE

Conditions requiring gross immobilisation of the trunk in the lumbar region such as:

- Stable non-displaced spinal fractures (L1-L5)
- Spinal Stenosis
- Herniated Disks
- Degenerative Spinal Pathologies
- Spondylolysis
- Spondylolisthesis

FEATURES AND BENEFITS

- Slim, breathable material
- Proven Miami double-pulley system with length-adjustable pulley cord
- Intuitive fitting without trimming with fold-over belt arms & Precise-Size™ landmarks
- Optional fit tool for perfect fitting directly on the patient
- Universal belt sizing accommodates waists up to 178cm
- New Universal Posterior panel to accommodate more patients with less inventory
- Contoured posterior panels sized S-XL for a precise, anatomical fit
- Quick-Fit pocket enclosure and blue highlights for patient ease of use

REQUIRED MEASUREMENTS

- Waist circumference

MIAMI LSO®

ORDERING INFORMATION

MIAMI LSO BELT

Size	Waist Circumference	Part#
6" Universal (Slim)	66 - 127cm	MLSO-1E
8" Universal (Standard)	66 - 127cm	MLSO-2E



OPTIONAL BELT EXTENDERS (2 PCS)

Size	Waist Circumference	Part#
6" (Slim) Belt Extension	127 - 177cm	MLSO-1EXT
8" (Standard) Belt Extension	127 - 177cm	MLSO-2EXT

STANDARD POSTERIOR PANEL

Size	Part#
Standard Posterior Panel	MLSO-P56



POSTERIOR PANEL WITH LATERAL SUPPORTS

Size	Waist Circumference	Part#
S	66 - 86cm	MLSO-P12
M	86 - 96cm	MLSO-P22
L	96 - 127cm	MLSO-P32
XL	127 - 178 cm	MLSO-P42



*Non-lateral and non-lordotic panels available on request.

ANTERIOR PANEL

Size	Part#
6" Panel (Slim)	MLSO-PA6
8" Panel (Standard)	MLSO-PA8



QUICK FIT TOOL

Size	Part#
Quick fit tool	B-040100000



MIAMI TLSO 464

The Miami TLSO from Össur directly addresses the needs of patients and practitioners, simplifying preparation without compromising on fit or performance. This easy-to-use pre-assembled Miami TLSO brace offers three configurations in one.

Compliance: Easily applied in over the shoulder or underarm configuration, donning and doffing is further simplified with the conveniently located quick release buckles.

Adjustability: The universal belt designs allows for setting the belt length on the patient for a more accurate sizing experience.

Comfort: Truly shaped to fit the anatomy, the anatomically contoured design better immobilises the spine while maximizing patient comfort.

Ease of Use: The Miami TLSO brace requires minimal preparation and is designed to provide optimal patient comfort.



INDICATIONS FOR USE

Conditions requiring gross immobilisation of the thoracic and lumbar spine. Such conditions may include:

- Post-surgical immobilisation
- Spinal support TLSO attachment is up to T7
- Spinal stenosis
- Herniated disc
- Degenerative spinal fractures
- Spondylolithesis
- Spondylolysis

CONTRAINDICATIONS

- Unstable, displaced fractures

FEATURES AND BENEFITS

- Significantly superior comfort and ease of donning/doffing
- Fully assembled for fitting with ease
- Accommodates heights 155-193cm
- Posterior thoracic extension option with ergonomic axillary strapping
- Step-down to an LSO as one progresses through rehabilitation
- One-touch telescoping anterior-thoracic extension option with single or double-hinge
- The convenience of a one-size Universal or individualise with a uniquely-sized anatomical design
- Improve comfort and immobilisation with multiple strapping configurations and removable anchors

REQUIRED MEASUREMENTS

- Waist circumference

MIAMI TLSO 464

ORDERING INFORMATION

ATE/PTE, LORDOTIC

Size	Part#	Waist circumference
Universal	MTLSO-2351	66 - 177cm
S	MTLSO-2111	66 - 86cm
M	MTLSO-2121	86 - 107cm
L	MTLSO-2131	107 - 127cm
XL	MTLSO-2141	127 - 177cm

*Non-lateral and non-lordotic available on request

*Belt extensions are required for waist circumferences larger than 127cm

ATE/PTE, LORDOTIC, SLIM

Size	Part#	Waist circumference
Universal	MTLSO-1351	66 - 177cm
S	MTLSO-1111	66 - 86cm
M	MTLSO-1121	86 - 107cm
L	MTLSO-1131	107 - 127cm
XL	MTLSO-1141	127 - 177cm

*Belt extensions are required for waist circumferences larger than 127cm

OTHER ACCESSORIES

Description	Part#
5MM Hex Long Wrench	MTLSO-0004
Axillary strap accessory kit	MTLSO-0003
Quick fit tool	B-040100000
Belt extension, standard (Recommended for waist circumferences 127-167cm. Fits size XL of Miami LSO.)	MLSO-2EXT
Belt extension, slim (Recommended for waist circumferences 127-167cm. Fits size XL of Miami LSO.)	MLSO-1EXT

MIAMI TLSO 456

The Miami TLSO 456 from Össur directly addresses the needs of patients and practitioners, simplifying preparation without compromising on fit or performance. This easy-to-use pre-assembled Miami TLSO brace offers the convenience of a one-size Universal or a uniquely-sized anatomical design.

Compliance: Blue is You™ patient touch-points help send patients home with confidence.

Adjustability: The universal belt design allows for setting the belt length on the patient for a more accurate sizing experience. Heat moldable posterior panels allow a customised fit, while the modular design allows step-down support to an LSO.

Comfort: Featuring the Miami LSO™ Advanced Compression pulley system with lumbar compression you can feel, and an anatomically contoured design that better immobilises the spine while maximising patient comfort.

Ease of Use: Easily applied with over-the-shoulder straps, the Miami TLSO brace requires minimal preparation and is designed to provide optimal patient comfort.



INDICATIONS FOR USE

Conditions requiring gross immobilisation of the thoracic and lumbar spine. Such conditions may include:

- Post-surgical immobilisation
- Spinal support TLSO attachment is up to T7
- Spinal stenosis
- Herniated disc
- Degenerative spinal fractures
- Spondylolithesis
- Spondylolysis

CONTRAINDICATIONS

- Unstable, displaced fractures

FEATURES AND BENEFITS

- Significantly superior comfort and ease of donning/doffing (when compared to Miami Lumbar TLSO)
- Fully assembled for fitting with ease
- Accommodates heights 155-193cm
- Step-down to an LSO as one progresses through rehabilitation
- The convenience of a one-size Universal or individualise with a uniquely-sized anatomical design

REQUIRED MEASUREMENTS

- Waist circumference

MIAMI TLSO 456

ORDERING INFORMATION

PTE LORDOTIC

Size	Part#	Waist circumference
Universal	MTLSO-2352	66 - 177cm
S	MTLSO-2112	66 - 86cm
M	MTLSO-2122	86 - 107cm
L	MTLSO-2132	107 - 127cm
XL	MTLSO-2142	127 - 177cm

*Belt extensions are required for waist circumferences larger than 127cm

PTE LORDOTIC, SLIM

Size	Part#	Waist circumference
Universal	MTLSO-1352	66 - 177cm
S	MTLSO-1112	66 - 86cm
M	MTLSO-1122	86 - 107m
L	MTLSO-1132	107 - 127cm
XL	MTLSO-1142	127 - 177cm

*Non-lateral and non-lordotic available on request.

*Belt extensions are required for waist circumferences larger than 127cm

OTHER ACCESSORIES

Description	Part#
Quick fit tool	B-040100000
Belt extension, standard (Recommended for waist circumferences 127-167cm. Fits size XL of Miami LSO)	MLSO-2EXT
Belt extension, slim (Recommended for waist circumferences 127-167cm. Fits size XL of Miami LSO)	MLSO-1EXT

MIAMI LUMBAR® POSTEO

Designed with patient comfort and compliance in mind, the Miami Lumbar Posteo provides functional relief of vertebral fracture pain, and facilitates return to mobility. The lumbar compression system offers both pain relief via active vertebral offloading and better grip of patient's hips for greater ease of use and compliance. The Miami Lumbar Posteo includes an innovative approach to ease of use and consistent donning.



INDICATIONS FOR USE

- Back pain caused by stable vertebral compression fractures (VCF)
- Post-operative immobilisation

FEATURES AND BENEFITS

- Active offloading via thoracic extension and lumbar compression
- Integrated pulley system for customised fit
- Trimmable axillary straps with adjustable tension for optimal stabilisation
- Cold-moldable aluminum PTE to fit different anatomies
- Extra cushioning around the aluminum strut for minimised skin pressure

REQUIRED MEASUREMENTS

- Back length measured from the top of the sacrum up to C7 vertebrae, following the spinal curvature

SIZING

Please note that the Miami Lumbar Posteo includes a universal lumbar belt which can be trimmed down to the appropriate size.

ORDERING INFORMATION

MIAMI LUMBAR POSTEO

Size	Back Length	Part#
X-Small	37 - 44cm	LMB-POSTEO-XS
Small	44 - 51cm	LMB-POSTEO-SM
Medium	49 - 56cm	LMB-POSTEO-MD
Large	55 - 61cm	LMB-POSTEO-LG

FORMFIT® PRO BACK

Össur Formfit® Pro Back provides relief for patients suffering from pains and strains in the lower back. Manufactured with our dynamic, 3D MotionTech™ knitting technology, our belts feature superior moisture wicking capabilities and are highly breathable. The patented back panel design utilises our StayBilizer™ technology to provide optimum shape and fit, while conveniently placed finger loops make donning easy. The SmartPad™ provides a targeted support and cushioning and can be removed if desired.



MotionTech™



StayBilizer™



SmartPad™



CoolVent™

INDICATIONS FOR USE

- For lower back conditions that may benefit from compression and gross motion restriction, such as:
- Lower back pain (Lumbago)
- Lumbosacral sprain/strain
- Degenerative joint/disc disease


CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit
- Removable lumbar SmartPad™ for targeted support and cushioning
- Lightweight, malleable aluminum stays are shaped to the anatomy of the lower back

REQUIRED MEASUREMENTS

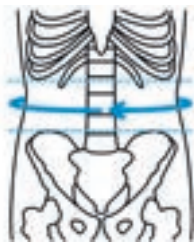

Measure circumference of the trunk at a level midway between the lowest ribs and the upper iliac crest. Standing.



ORDERING INFORMATION

Size	Waist circumference	Part#
XS	70 - 80cm	FP500001
S	80 - 90cm	FP500002
M	90 - 100cm	FP500003
L	100 - 115cm	FP500004
XL	115 - 130cm	FP500005

FORMFIT® BACK SUPPORT AIR

The Form Fit Back Support with Air provides compression and support for those suffering from lower back pain. The integrated pneumatic system is intuitive and conveniently located so that adjustments can be made as demands on the body change throughout the day. The double wrap design improves compression, while durable finger loops make applying the brace easy. Made from breathable materials, the Form Fit Back Support provides compression and support for those suffering from lower back pain. The double wrap design improves easy compression, while the durable finger loops make applying the brace easy.



INDICATIONS FOR USE

Lower back conditions that may benefit from gross motion restriction, such as;

- Muscular low back pain
- Lumbosacral strains
- Chronic low back pain
- Sciatica

FEATURES AND BENEFITS

- Self-contained pneumatic system is low-profile and easy to use
- Mesh panels and ventilation holes increase breathability and enhance comfort
- Overlapping double pull strap design provides additional support to the lumbar and abdominal areas
- Flexible stays provide optimum support in the lumbar and abdominal areas
- Gripper strips prevent migration
- Adjustable air bladder provides a custom fit and offers focused compression in the lumbar area

REQUIRED MEASUREMENTS

- Waist circumference

ORDERING INFORMATION

Size	Waist circumference	Standard
Small	63 - 76cm	209133
Medium	76 - 86cm	209135
Large	86 - 97cm	209137
X-Large	97 - 107cm	209138
XX-Large	107 - 142cm	209139

ACCESSORIES

Description	Part#
Gel pad	GB28

FORMFIT® BACK SUPPORT

Made from breathable materials, the Formfit Back Support provides compression and support for those suffering from lower back pain. The double wrap design improves compression, while the durable finger loops make applying the brace easy

INDICATIONS FOR USE

- Muscular low back pain
- Lumbo-sacral strains
- Chronic low back pain
- Sciatica

FEATURES AND BENEFITS

- Anti-migration system reduces slippage
- Double wrap design improves compression
- Finger loops for easy application
- Hot/Cold Gel pad available separately

REQUIRED MEASUREMENTS

- Waist circumference



SPINE SOLUTIONS

ORDERING INFORMATION

Size	Waist Circumference	Standard
S	25-30"	209143
M	30-34"	209145
L	34-38"	209147
XL	38-42"	209148
XXL	42-56"	209149
XXXL	56-60"	209151

ACCESSORIES

Description	Part#
Gel pad	GB28

HIP

HIP
SOLUTIONS



TABLE OF CONTENTS

HIP	97
REBOUND® HIP	99
UNLOADER® HIP	101

REBOUND® HIP

Rebound Hip delivers gentle controlling forces and effective functional ROM restriction in extension / flexion and abduction / adduction to promote optimal post-surgical outcomes. The low profile, lightweight design supports patient comfort and compliance. The universal frame and softgood are easy to stock, assemble and quickly fit.



INDICATIONS FOR USE

Hip conditions that could benefit from motion restriction such as:

- Postoperative after (total) hip arthroplasty
- Postoperative after revision of (total) hip arthroplasty
- Postoperative hip arthroscopy
- Post-op following FAI arthroscopic surgery
- Post-op following gluteus medius repair surgery
- Post-op following hamstring repair surgery
- Inoperable hip abnormalities
- Preventively when there is risk of hip dislocation
- Conventional treatment after dislocation/reduction

FEATURES AND BENEFITS

- Low-profile, lightweight design that supports patient comfort and compliance
- Quick-release pegs easily adjust the desired range-of-motion needed for the patient's optimal rehab
- Quick-fit with sliding mechanism on frame to adjust for patient's height
- Optional, adjustable ROM hinge enables adduction/abduction from 0° to 10°, 20° or 30°
- OR kit offers patient protection of ankle and perineum skin during intraoperative traction on a surgical table

REQUIRED MEASUREMENTS

- Circumference at iliac crest height
- Length from greater trochanter to mid patella

ORDERING INFORMATION

REBOUND HIP

Description	Part#
Rebound Hip	98013XRB
Rebound Hip with adjustable hinge	98012XRB

X = 0 (Preset to left) or 5 (Preset to right)

*Frame: Universal; Softgood: Universal fitting to 48" (122cm) hip circumference

REBOUND® HIP

ACCESSORIES

Description	Part#
Rebound Hip alligator kit	980501
Rebound Hip v2 softgood kit	980601
Rebound Hip v2 belt only kit	980701
OR Kit	980301
Belt Extender (Fitting 122 - 170cm hip circumference)	980121RB
Rotation Strap	RHK-100024
Fitting Tool	B-040100000

UNLOADER® HIP

Unloader Hip is designed to improve activities of daily living by optimizing load dispersion for patients suffering from mild and moderate osteoarthritis of the hip, thus contributing to hip stability and improved mobility.

Unloader Hip delivers proprioceptive support during the gait cycle by providing compression with a proven pulley system design. If required it can decrease the internal rotation and adduction moment with its unique and clinically proven Dynamic Rotation Straps™.



HIP SOLUTIONS

INDICATIONS FOR USE

Conditions benefitting from hip repositioning and/or compression around the greater trochanters, this may include:

- Hip osteoarthritis (OA)
- Femoroacetabular impingement (FAI)
- Labrum tears

FEATURES AND BENEFITS

- The Dynamic Rotation Straps with SmartDosing dial allows individual adjustment of the external rotation force, to achieve optimum pain relief for each patient
- Easy donning and doffing thanks to pocket wrap closure system and zipper
- Pulley system for improved stability
- Premium quality Lycra® for breathability and a snug fit


REQUIRED MEASUREMENTS

- Measure the hip circumference at the anterior superior iliac-spine.

ORDERING INFORMATION

UNLOADER HIP

Size	Hip Circumference	Woman	Men
S	80 – 88cm	B-465350030	B-465350031
M	88 – 96cm	B-465350032	B-465350033
L	96 – 104cm	B-465350034	B-465350035
XL	104 – 114cm	B-465350036	B-465350037
XXL	114 – 124cm	B-465350038	B-465350039

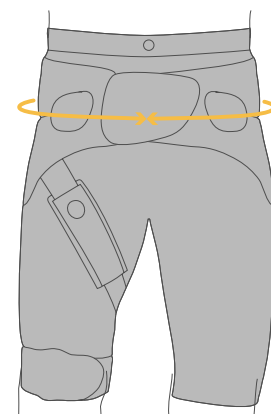

UNLOADER® HIP

ACCESSORIES/ OPTIONS

REPLACEMENT TROUSERS

Size	Woman	Men
S	B-465350077	B-465350078
M	B-465350079	B-465350080
L	B-465350081	B-465350082
XL	B-465350083	B-465350084
XXL	B-465350085	B-465350086

OTHER ACCESSORIES

Description	Part#
Rotation strap	B-465350064
Washbag Kit	WB000001
Trochanter pad	B-465350066
Thigh strap S	B-465350067
Thigh strap M	B-465350068
Thigh strap L	B-465350069
Thigh strap XL	B-465350070
Thigh strap XXL	B-465350071
Bilateral kit* S	B-465350072
Bilateral kit* M	B-465350073
Bilateral kit* L	B-465350074
Bilateral kit* XL	B-465350075
Bilateral kit* XXL	B-465350076

*The bilateral kit consists of a rotation control strap, a thigh strap and a trochanter pad

UPPER EXTREMITY



TABLE OF CONTENTS

UPPER EXTREMITY	103
FORMFIT® SHOULDER BRACE	105
PREMIUM SHOULDER IMMOBILISER	106
BERGSCHULTZ UNIVERSAL ARM SLING	106
HIGH ARM SLING	107
18 PAEDIATRIC UNIVERSAL ARM SLING	107
FORMFIT® PRO ELBOW	108
REBOUND® POST-OP ELBOW	109
FORMFIT® TENNIS ELBOW	111
FORMFIT® PRO WRIST	112
FORMFIT® WRIST / FORMFIT® WRIST & FOREARM	113
18 FORMFIT® UNIVERSAL WRIST & THUMB	114
18 FORMFIT® THUMB SPICA	115
WRIST / THUMB BRACE	116
SPECTRA® WRIST BRACE	117

FORMFIT® SHOULDER BRACE

The Formfit Shoulder Brace is designed to provide immobilisation following shoulder injury/surgery. With its streamlined fit, sporty look and breathability it offers excellent comfort and ease of use. Same as its predecessor, the SmartSling®, the Formfit Shoulder Brace also provides abduction of the shoulder (when the abduction pillow is attached), or external rotation of the shoulder (when the external rotation pillow is attached).

Optional axilla pillow helps distribute weight over a larger surface area.

INDICATIONS FOR USE

- Immobilisation for rotator cuff repairs
- Bankart repairs
- Capsular shifts
- Glenohumeral dislocations/subluxations
- Soft tissue repairs/strains
- Shoulder instabilities/injuries
- Post-surgical rehabilitation

FEATURES AND BENEFITS

- Breathable material
- Quick-release buckles
- Offloading strap
- Extra padding
- Universal thumb loop

REQUIRED MEASUREMENTS

- Olecranon to end of closed fist



Formfit Shoulder Brace

Axilla Pillow

ORDERING INFORMATION

FORMFIT SHOULDER BRACE

Size	Length	Part
S	up to 29cm	SSS-SO
M	29 - 40cm	SSR-SO
L	40 - 44.5cm	SSL-SO

FORMFIT SHOULDER BRACE WITH ABDUCTION

Size	Length	Part
S	up to 29cm	SSS
M	29 - 40cm	SSR
L	40 - 44.5cm	SSL

FORMFIT SHOULDER BRACE WITH EXTERNAL ROTATION

Size	Length	Part
S	up to 29cm	SSS-ER
M	29 - 40cm	SSR-ER
L	40 - 44.5cm	SSL-ER

Description	Part
Axilla Pillow	SSAXP

PREMIUM SHOULDER IMMOBILISER


INDICATIONS FOR USE

For shoulder conditions that may benefit from immobilizing the shoulder, such as:

- Pre or post-operative
- Soft tissue repairs/strains
- Shoulder instabilities/injuries

FEATURES AND BENEFITS

- Effectively controls abduction and rotation
- Humerus cuff, wrist cuff and shoulder strap may be placed in desired position for optimum fit/comfort
- One size for reduced stockholding
- Fits sizes S-XL

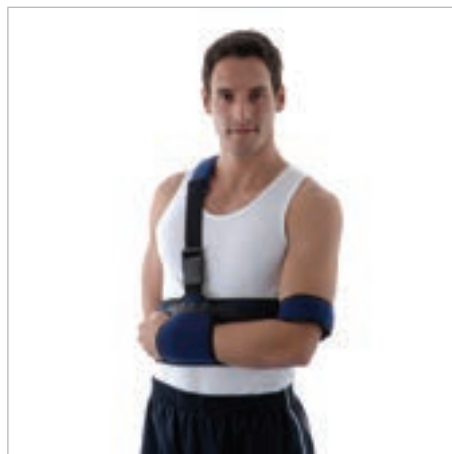

REQUIRED MEASUREMENTS

- Universal, one size (for upper waist diameter up to 106cm)

ORDERING INFORMATION

Description	Part#
Premium Shoulder Immobiliser	0380
Premium Shoulder Immobiliser XL	0380XL

BERGSCHULTZ UNIVERSAL ARM SLING

INDICATIONS FOR USE

- Post-trauma
- Post-operative

FEATURES AND BENEFITS

- Designed by Jim Bergschultz. R.O.T.
- Blue heavy denim material with contact closure

REQUIRED MEASUREMENTS

- Universal, one size



ORDERING INFORMATION

Description	Part#
Bergschulz Universal Arm Sling	204070

HIGH ARM SLING

INDICATIONS FOR USE

- Post upper limb trauma
- Post upper limb surgery
- Hand surgery

FEATURES AND BENEFITS

- Maintains upper arm in raised position
- Breathable nylon/foam laminate construction
- Easy to apply with touch and close fastening
- Waist strap included



ORDERING INFORMATION

Description	Part#
High Arm Sling	0860

PAEDIATRIC UNIVERSAL ARM SLING

≤18

INDICATIONS FOR USE

- Post upper limb trauma
- Post upper limb surgery

FEATURES AND BENEFITS

- Prong buckle closure correct fit without slippage

REQUIRED MEASUREMENTS

- Universal, one size



ORDERING INFORMATION

Description	Part#
Paediatric Universal Arm Sling Packed in boxes of 6	204020

FORMFIT® PRO ELBOW

Formfit® Pro Elbow provides relief from elbow pain. This 3D compressive elbow sleeve is manufactured using our MotionTech™ technology and features two integrated epicondyle SmartPads™ for more targeted support and cushioning. The RadialStrap™ provides additional compression and support.



MotionTech™



CoolVent™



SmartPad™



FoldLock™

INDICATIONS FOR USE

- For elbow conditions that may benefit from compression, such as:
- Medial and lateral epicondylitis
- Elbow pain and swelling

CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

FEATURES AND BENEFITS

- 3D MotionTech™ for compression and dynamic fit keep
- Epicondyle SmartPads™ for targeted support and cushioning
- Patented FoldLock™ system for easier and more secure application
- RadialStrap™ for increased compression and support

REQUIRED MEASUREMENTS



Measure circumference
12cm below elbow.

ORDERING INFORMATION

Size	Elbow circumference	Part#
XS	17 - 20cm	FP300001
S	20 - 23cm	FP300002
M	23 - 26cm	FP300003
L	26 - 29cm	FP300004
XL	29 - 32cm	FP300005



REBOUND® POST-OP ELBOW

The Rebound® Post-Op Elbow brace by Össur combines intuitive features for fitters and users with a design that aims to improve the post-op experience. The industry's only elbow immobiliser that combines a drop-lock hinge and four telescoping paddles, individual post-op protocols are easily catered for. The Rebound Post-Op Elbow, lightest of its kind, promotes user comfort with straps that stay in place, anatomical left and right fit with mouldable stability cuffs and blue patient touch-points for easy education.



INDICATIONS FOR USE

Indications requiring elbow immobilisation and/or range of movement restriction. This may include:

- Stable fractures of the elbow, distal humerus, proximal radius or ulna
- Tendon and ligament injuries or repairs
- Collateral ligament reconstructions
- Elbow hyperextension
- Triceps tendon repair
- Chronic elbow injuries
- Tennis elbow release

FEATURES AND BENEFITS

- Designed to be the most lightweight brace on the market
- Telescoping paddles relocate to individualise length and keep incision sites clear
- Easy education for users with intuitive patient touchpoints, including Blue is You™ buckles and numbered strapping system
- Drop-Lock hinge enables secure range of movement with bump-resistant pull adjustment stops and optional ROM lock-outs
- Mouldable stability cuffs designed to be individually customised for improved fit and support
- Strap guides maintain strap location for added security and comfort
- Universal fit for both left and right elbows

REQUIRED MEASUREMENTS

- Universal, one size

REBOUND® POST-OP ELBOW

ORDERING INFORMATION

Side	Part#
Left	B-8391680
Right	B-8391580

ACCESSORIES

Description	Part#
Elbow Wrist Control Kit	B-8390003
Elbow ROM Locks Kit	B-8390005

REPLACEMENT PARTS

Description	Part#
Elbow Liner Kit	B-8390002
Elbow Sling Kit	B-8390004
Strap Tab Kit	B-8390006
Adhesive Hook Sticker Kit	B-8390007
Anti-Migration Pad Kit	B-8390008
Elbow Strap Kit	B-8390009

FORMFIT® TENNIS ELBOW

Formfit Tennis Elbow is made with a patented, ergonomically-shaped plastic. There are two blue cushions with ribbed surface on each side that reduce traction on the tendons. The durable design and materials enable everyday use for both work and athletic activities.

INDICATIONS FOR USE

- Tendonitis of the elbow
- Prevention during work, daily life and sport
- Epicondylitis

FEATURES AND BENEFITS

- Adjustable strap
- Patented ergonomically-shaped plastic eliminates tourniquet effect associated with traditional designs.
- Twin blue cushions provide counterforce, reducing traction on the tendons and thus inflammation
- Ribbed surfaces reduce vibration to the tendon, increasing comfort
- Durable design and materials enable everyday use for both work and athletic activities

REQUIRED MEASUREMENTS

- Circumference of the elbow



ORDERING INFORMATION

Size	Circumference	Part#
S	22 - 26cm	B-247000002
M	27 - 32cm	B-247000003
L	33 - 38cm	B-247000004

FORMFIT® PRO WRIST

Formfit Pro Wrist is constructed using our superior signature 3D MotionTech knit, and ideal for injuries that may benefit from compression. Malleable aluminum stays can be adjusted to conform to an individual's anatomy. CoolVent™ technology in the basal thumb area enhances breathability and comfort.



MotionTech™



CoolVent™



StayBilizer™



FoldLock™

INDICATIONS FOR USE

- For wrist conditions that may benefit from compression, such as:
- Rheumatoid Arthritis (RA) and other chronic pain conditions
- Wrist sprain and strains
- Overuse and pain conditions
- Wrist tendonitis


CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit has been proven to be superior to the leading brands at moisture wicking and breathability, keeping patients cool during wear
- Malleable aluminum stay for a customised fit
- CoolVent™ technology in thumb base area for extra breathability and comfort

REQUIRED MEASUREMENTS

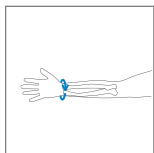

Measure wrist circumference at the level of the radial styloid with the hand outstretched. Wrist in 0 degrees of flexion.

ORDERING INFORMATION

Size	Wrist circumference	Left	Right
XS	14 - 15cm	FP400201	FP400101
S	15 - 16cm	FP400202	FP400102
M	16 - 17cm	FP400203	FP400103
L	17 - 18cm	FP400204	FP400104
XL	18 - 19cm	FP400205	FP400105
XXL	19 - 20cm	FP400206	FP400106



FORMFIT® WRIST / FORMFIT® WRIST & FOREARM

Formfit Wrist, and Formfit Wrist & Forearm, provides immobilisation and protection for patients suffering from tendonitis, select soft tissue injuries – including mild to moderate sprains and strains, as well as for post-cast healing. The lightweight, breathable material and contour fit make this comfortable product an excellent choice.



INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Tendonitis
- Post-cast healing
- Other selected soft tissue injuries

FEATURES AND BENEFITS

- Breathable material
- Contoured fit
- Contact closure straps
- Customisable stays

REQUIRED MEASUREMENTS

- Wrist circumference

ORDERING INFORMATION

FORMFIT WRIST & FOREARM

Size	Wrist Circumference	8"		10"	
		Left	Right	Left	Right
XS	13 - 16cm	317082	317072	617082	617072
S	16 - 18.5cm	317083	317073	617083	617073
M	18.5 - 20.5cm	317085	317075	617085	617075
L	20.5 - 23.5cm	317087	317077	617087	617077
XL	23.5 - 25.5cm	317088	317078	617088	617078

ACCESSORIES

GEL PAD FOR HOT/COLD THERAPY

Description	Left	Right
Form Fit Wrist (8" version)*	GB20L	GB20R
Form Fit Wrist & Forearm (10" version)	GB22L	GB22R



* Fits XS/S. For M-XL, please use Form Forearm 10" part numbers



FORMFIT® UNIVERSAL WRIST & THUMB

≤18

Formfit Universal Wrist and Thumb products are designed to restrict motion of the wrist and the thumb. Providing immobilisation and protection from select soft tissue injuries, mild to moderate strains and sprains, including gamekeeper's thumb and post cast healing. The lightweight, universal design also reduces inventory needs.

INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Tendonitis
- Post-cast healing
- Other selected soft tissue injuries

FEATURES AND BENEFITS

- Breathable material
- Quick-Pull close strap
- Positional dorsal stay
- Customisable thumb stay
- Malleable palmar stay

REQUIRED MEASUREMENTS

- Wrist circumference



ORDERING INFORMATION

FORMFIT WRIST

Size	Wrist Circumference	Left	Right
Paediatric	3.5 - 5"	B-252603301	B-252603300
8" Universal	4.5 - 9.5"	B-252603109	B-252503109
8" XL	7.5 - 11"	B-252603106	B-252603105

FORMFIT WRIST & FOREARM UNIVERSAL

Size	Wrist Circumference	Left	Right
10" Universal	4.5 - 9.5"	B-252603210	B-252603209

FORMFIT THUMB SPICA UNIVERSAL

Size	Wrist Circumference	Left	Right
Paediatric	3.5 - 5"	B-253603301	B-253603300
8" Universal	4.5 - 9.5"	B-253603109	B-253503109
8" XL	7.5 - 11"	B-253603106	B-253603105

FORMFIT® THUMB SPICA

≤18

The Formfit Thumb Spica is designed to restrict wrist and thumb motion. It provides immobilisation and protection for patients suffering from tendonitis (e.g. DeQuervain's Tendonitis), select soft tissue injuries – including mild to moderate sprains and strains and Gamekeeper's Thumb –, as well as for post-cast healing. The lightweight, breathable material and contour fit make this comfortable product an excellent choice.



INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Tendonitis
 - e.g., DeQuervain's Tendonitis
- Post-cast healing
- Other selected soft tissue injuries
 - e.g., Gamekeeper's Thumb

FEATURES AND BENEFITS

- Breathable material
- Contoured fit
- Contact closure straps
- Customisable stays

REQUIRED MEASUREMENTS

- Wrist circumference

ORDERING INFORMATION

Size	Wrist Circumference	Left	Right
XS	13 - 16cm	3020	3120
S	16 - 18.5cm	3030	3130
M	18.5 - 20.5cm	3050	3150
L	20.5 - 23.5cm	3070	3170
XL	23.5 - 25.5cm	3080	3180

ACCESSORIES

GEL PAD FOR HOT/COLD THERAPY

Description	Left	Right
Form Fit Wrist (8" version)*	GB20L	GB20R
Form Fit Wrist & Forearm (10" version)	GB22L	GB22R



* Fits XS/S. For M-XL, please use Form Forearm 10" part numbers

WRIST / THUMB BRACE

A simple yet effective wrist brace offering superior comfort.

INDICATIONS FOR USE

- Stabilising hyper-or hypotonic wrist/thumb flexors
- Wrist / Thumb over use injury
- Wrist / Thumb trauma

FEATURES AND BENEFITS

- Simple touch-and-close fastenings
- Full length malleable aluminium thumb stay

REQUIRED MEASUREMENTS

- Wrist circumference



ORDERING INFORMATION

Size	Wrist circumference	Elastic		Neoprene	
		Left	Right	Left	Right
XS	11 - 13cm	150/ET	155/ET	160/NT	165/NT
S	13 - 15cm	151/ET	156/ET	161/NT	166/NT
M	15 - 17cm	152/ET	157/ET	162/NT	167/NT
L	17 - 19cm	153/ET	158/ET	163/NT	168/NT
XL	19 - 21cm	154/ET	159/ET	164/NT	169/NT

SPECTRA® WRIST BRACE

The Spectra wrist brace is designed to immobilise the wrist at a neutral angle. Its single-pull lacing closure makes it easier to apply than a conventional wrist brace, meeting the needs of patients with arthritis and other hand dexterity issues.

INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Tendonitis
- Carpal tunnel and other repetitive stress injuries
- Post cast healing

FEATURES AND BENEFITS

- Wicking liner moves moisture away from the skin for increased comfort
- Single-pull lace closure enables easy application
- Adjustable webspace strap allows size adjustment for increased comfort
- Removable/adjustable palmar stay provides adjustability of immobilisation angle
- Suede soft good material is durable and breathable

REQUIRED MEASUREMENTS

- Wrist circumference



ORDERING INFORMATION

Size	Wrist circumference	Left	Right
XS	12 - 14cm	6602210	6602315
S	14 - 16cm	6602422	6602535
M	16 - 18cm	6602624	6602734
L	18 - 20cm	6602755	6602793
XL	20 - 22cm	6602826	6602932

FOOT AND ANKLE



FOOT & ANKLE
SOLUTIONS

TABLE OF CONTENTS

FOOT AND ANKLE.....	119
RANGE OF MOTION WALKER	121
REBOUND® DIABETIC WALKER	122
DH OFFLOADING WALKER®	124
REBOUND® AIR WALKER	125
FORMFIT® WALKER AIR	127
FORMFIT® WALKER	128
18 PAEDIATRIC WALKER	129
AFO DYNAMIC	130
AFO LIGHT	132
AFO LEAF SPRING	133
PROLITE AFO	134
EXTRA STRONG PROLITE	134
REBOUND FOOT-UP®	135
FOOT-UP®	136
FORMFIT® PRO ANKLE	137
FORMFIT® ANKLE BRACE	139
FORMFIT® ANKLE WITH SPEEDLACE	140
18 FORMFIT® ANKLE STIRRUPS	141
EXOFORM® DORSAL NIGHT SPLINT	142
FORMFIT® NIGHT SPLINT	143
AIRFORM® NIGHT SPLINT	144
DH OFFLOADING POST-OP STYLE SHOE	145
MESH TOP POST-OP SHOE	146
SOFT TOP POST-OP SHOE	146
CANVAS ROCKER BOTTOM CAST SHOE	147

RANGE OF MOTION WALKER

INDICATIONS FOR USE

- Soft tissue injuries
- Grade 2 and 3 sprains
- Stable fractures
- Post-operative ankle stabilisation
- Trauma and rehab use

FEATURES AND BENEFITS

- Malleable metal struts for adjustable fit
- Integrated ROM setting adjustability from 0-40° of plantar flexion
- The range of motion joint adjusts in 10° increments in both dorsi and plantar flexion, and can either be completely locked out at any desired interval, or left with desired range



REQUIRED MEASUREMENTS

- Shoe size

ORDERING INFORMATION

Size	Shoe Size UK	Shoe Size EU	Part#
XS	<3.5	<35.5	ROM1200
S	4 - 6.5	36 - 39	ROM1400
M	7 - 9	39.5 - 43	ROM1600
L	9.5 - 10.5	43.5 - 45.5	ROM1800
XL	11+	46+	ROM2000

REBOUND® DIABETIC WALKER

Offloading has proven to be a key factor in foot ulcer treatment and entails reducing pressure of the plantar area in general and the ulcer site in particular.

The Rebound Diabetic Walker is uniquely designed for effective treatment of plantar foot ulcers, with easy access and re-fitting during wound care, while improving the patient's mobility during the healing process.

The Rebound Diabetic Walker has major advantages over diabetic shoes and shoe inserts in its healing effectiveness and over total contact casting due to its ease of use, speed of application and low cost.

FUNCTIONAL HEALING - WHERE MOBILITY MEANS RECOVERY



INDICATIONS FOR USE

- Plantar foot ulcers and wound prevention

FEATURES AND BENEFITS

- Effective tri-laminate insole with removable shock-absorbing hexagons that promote ulcer offloading and a Plastazote, heat-moldable top layer that evenly distributes the pressure and minimises friction
- Unique pressure relief valve for preventing over-inflation, while balancing compression with customised fit
- Positioning Portals allow the clinician to check the fit of the walker at the heel bottom, in order to ensure appropriate positioning of the foot in the walker
- Anatomically designed shell that minimises shear forces by immobilising the foot and ankle at 90 degrees while providing protection
- Flex Edge® technology for relief of pressure points
- Rocker Bottom outsole for a smooth roll over while allowing for individual natural toe out
- Soft and breathable, low friction, machine washable liner for hygiene promotion and prevention of infection
- Tamper evident strapping for compliance monitoring

REQUIRED MEASUREMENTS

- Shoe size



REBOUND® DIABETIC WALKER

ORDERING INFORMATION

REBOUND DIABETIC WALKER

Size	Shoe size UK	Shoe size EU	Part#
S	4 - 6.5	36 - 39	B-242500002
M	7 - 10	39.5 - 44	B-242500003
L	10.5 - 12	44.5 - 46.5	B-242500004
XL	12.5 - 14.5	47 - 50	B-242500005

ACCESSORIES

REPLACEMENT SOFTGOODS

Size	Liners	Insoles
S	B-418500102	B-418500202
M	B-418500103	B-418500203
L	B-418500104	B-418500204
XL	B-418500105	B-418500205



WALKER COVERS

Size	1. Hygiene covers (2 pcs)	2. Durable weather cover	3. Disposable weather cover (20 pcs)
S	B-418500602	B-418500502	B-418500512
M	B-418500603	B-418500503	
L	B-418500604	B-418500504	B-418500514
XL	B-418500605	B-418500505	



DIABETIC SOCKS (2 PCS)

Size	Part#
S-M	B-418500703
L-XL	B-418500704

COMPLIANCE STRAPS (5 PCS)

Size	Part#
Universal	B-418500300

EVENUP

Size	Part#
S	10350
M	10351
L	10352



DH OFFLOADING WALKER®

Assists in the healing of plantar ulcers with a patented pressure relief insole

INDICATIONS FOR USE

- Treatment of plantar ulcers

FEATURES AND BENEFITS

- Shear force reduction by stabilising foot and ankle at 90°
- Wide low profile rocker bottom provides a smooth roll-through and increased stability for altered gaits
- Designed around Össur's unique Active Offloading Insole
- The softgood's patented inner bootie prevents the foot from migrating on the insole
- Provides a safe environment for the simultaneous use of wound care material
- Create an Instant TCC using a single roll of cast tape for patient compliance



REQUIRED MEASUREMENTS

- Shoe size

ORDERING INFORMATION

Size	Shoe Size UK	Shoe Size EU	Part#
XS	1.5 - 3.5	33 - 35.5	DH0200BLK
S	4 - 6.5	36 - 39	DH0400BLK
M	7 - 10	39.5 - 44	DH0600BLK
L	10.5 - 12	44.5 - 46.5	DH0800BLK
XL	12.5 - 14.5	47 - 50	DH1000BLK

ACCESSORIES

REPLACEMENT PARTS

Size	Softgoods	Insoles
XS	10958	11742C
S	10958	PP04C
M	10959	PP06C
L	10960	11740C
XL	10960	11741C

REBOUND® AIR WALKER

Rebound Air Walker is a circumferential walker with fully integrated air pump and release valve that provides easy, tool-free inflation and deflation for fine-tuned, consistent compression. It features a host of details designed to provide added protection, simplified application and usage.



INDICATIONS FOR USE

Foot and ankle conditions that may benefit from motion restriction, such as:

- Grade 2 and 3 ankle sprains
- Stable fractures
- Tendon or ligamentous sprain
- During postoperative or rehab phase. If used with the Achilles Wedge Kit, Achilles ruptures can be treated as well.
- Rebound Air Walker Low Top
- Foot and ankle conditions that may benefit from motion restriction, such as:
- Stable foot fractures
- Tendon or ligamentous sprain
- During postoperative or rehab phase.

FEATURES AND BENEFITS

- Linked medial and lateral air bladders with a single inflation point provide balanced compression and are shaped to cradle the Achilles, holding the heel in place
- Rocker bottom promotes a natural, stable gait, while a trimmable, semi-soft toe bumper provides added protection
- Moisture-wicking softgood helps keep skin dry and comfortable
- Straps fasten in front for easy application, adjustment and removal
- EVA material provides greater shock absorption in the heel
- Fully-integrated air pump and release valve provides easy, tool-free inflation and deflation for fine-tuned, consistent compression
- Flex Edge™ overmolding conforms to each patient's unique anatomy to help limit the formation of pressure points
- Ventilated panels wick moisture to prevent microbial buildup and improve patient comfort

REQUIRED MEASUREMENTS

- Shoe size



REBOUND® AIR WALKER

ORDERING INFORMATION

REBOUND AIR WALKER

Size	Shoe Size UK	Shoe Size EU	Standard Liner / Standard	Standard Liner / Low Top	Slim Liner / Standard	Slim Liner / Low Top
XS	1.5 - 3.5	33 - 35.5	B-242900001E	B-242900061E	B-242900001SE	B-242900061SE
S	4 - 6.5	36 - 39	B-242900002E	B-242900062E	B-242900002SE	B-242900062SE
M	7 - 10	39.5 - 44	B-242900003E	B-242900063E	B-242900003SE	B-242900063SE
L	10.5 - 12	44.5 - 46.5	B-242900004E	B-242900064E	B-242900004SE	B-242900064SE
XL	12.5 - 14.5	47 - 50	B-242900005E	B-242900065E	B-242900005SE	B-242900065SE

ACCESSORIES

REPLACEMENT SOFTGOODS

Size	Standard Liner / Standard	Standard Liner / Low Top	Slim Liner / Standard	Slim Liner / Low Top
XS	B-418000101	B-418000161	B-418030101	B-418030161
S	B-418000102	B-418000162	B-418030102	B-418030162
M	B-418000103	B-418000163	B-418030103	B-418030163
L	B-418000104	B-418000164	B-418030104	B-418030164
XL	B-418000105	B-418000165	B-418030105	B-418030165

ACHILLES HEEL WEDGE KIT

Size	Description	Part#
S	Wedge kit for walkers XS-S	AWK0001
M	Wedge Kit for walkers M	AWK0005
L	Wedge kit for walkers L-XL	AWK0009
N/A	5 layer Wedge kit	AWK00015

EVENUP

Size	Part#
S	10350
M	10351
L	10352

WALKER COVERS

Size	Hygiene covers (2 pcs)	Durable weather cover	Disposable weather cover (20 pcs)
S	B-418500602	B-418500502	B-418500512
M	B-418500603	B-418500503	
L	B-418500604	B-418500504	B-418500514
XL	B-418500605	B-418500505	

OTHER ACCESSORIES

Description	Part#
Adhesive Insole for Wedge	WHW-W0107
Strap Extension	B-502100101
Rebound Hightop Sock SM/MD SM/MD	B-418500703
Rebound Hightop Sock LG/XL LG/XL	B-418500704
Rebound Lowtop Sock SM/MD SM/MD	B-418500903
Rebound Lowtop Sock LG/XL LG/XL	B-418500904

FORMFIT® WALKER AIR

The Formfit Walker Air provides immobilisation for patients suffering from soft tissue injury; grades 2 & 3 strains and sprains, and stable fractures. The patented pneumatic system is designed to increase stability while decreasing pain and swelling. The intuitive pump and clearly marked decompression valve make air/pressure adjustments easy.

INDICATIONS FOR USE

For foot and ankle conditions that may benefit from motion restriction, such as:

- Grade 2 and 3 ankle sprains
- Stable fractures
- Soft tissue injuries
- During postoperative or rehab phase

FEATURES AND BENEFITS

- Available in low top or standard height
- Integrated pneumatic system
- Low, wide rocker bottom promotes a stable, natural gait
- Shock absorbing sole for reduced impact at heel strike
- Breathable, seamless liner

REQUIRED MEASUREMENTS

- Shoe size



ORDERING INFORMATION

Size	Shoe Size UK	Shoe Size EU	Standard	Low Top
XS	1.5 - 3.5	33 - 35.5	A-W0200BLK	A-W0300BLK
S	4 - 6.5	36 - 39	A-W0400BLK	A-W0500BLK
M	7 - 10	39.5 - 44	A-W0600BLK	A-W0700BLK
L	10.5 - 12	44.5 - 46.5	A-W0800BLK	A-W0900BLK
XL	12.5 - 14.5	47 - 50	A-W1000BLK	A-W1100BLK

ACCESSORIES

REPLACEMENT SOFTGOODS

Size	Standard	Low Top
XS	12349	12356
S	10675	12357
M	10678	12358
L	10681	12359
XL	12347	12360

OTHER ACCESSORIES

Description	Part#
10cm softgood width extension	WKSG-1BLK
Tibia and calf plates for use when additional splinting is required	WKSP100
Hot/Cold therapy insert for safe and reusable cryotherapy	WKGEL100
5 layer Wedge kit	AWK00015
Adhesive insole for heel wedge	WHW-W0107

FORMFIT® WALKER

The Formfit Walker provides immobilisation for patients suffering from soft tissue injury; grades 2 & 3 strains and sprains; and stable fractures.

INDICATIONS FOR USE

For foot and ankle conditions that may benefit from motion restriction, such as:

- Grade 2 and 3 ankle sprains
- Stable fractures
- Soft tissue injuries
- During postoperative or rehab phase

FEATURES AND BENEFITS

- Available in low top or standard height
- Low, wide rocker bottom promotes a stable, natural gait
- Shock absorbing sole for reduced impact at heel strike
- Breathable, seamless liner

REQUIRED MEASUREMENTS

- Shoe size



ORDERING INFORMATION

Size	Shoe Size UK	Shoe Size EU	Standard	Low Top
XS	1.5 - 3.5	33 - 35.5	W0200BLK	W0300BLK
S	4 - 6.5	36 - 39	W0400BLK	W0500BLK
M	7 - 10	40 - 44	W0600BLK	W0700BLK
L	10.5 - 12	44.5 - 46.5	W0800BLK	W0900BLK
XL	12.5 - 14.5	47 - 50	W1000BLK	W1100BLK

ACCESSORIES

REPLACEMENT SOFTGOODS

Size	Standard	Low Top
XS	WKSG200BLK	WKSG0300BLK
S	WKSG400BLK	WKSG500BLK
M	WKSG600BLK	WKSG700BLK
L	WKSG800BLK	WKSG900BLK
XL	WKSG1000BLK	WKSG1100BLK

OTHER ACCESSORIES

Description	Part#
10cm softgood width extension	WKSG-1BLK
Tibia and calf plates for use when additional splinting is required	WKSP100
Hot/Cold therapy insert for safe and reusable cryotherapy	WKGEL100
5 layer Wedge kit	AWK00015
Adhesive insole for heel wedge	WHW-W0107

PAEDIATRIC WALKER

≤18

Paediatric Walkers are designed to stabilise and protect the lower leg, ankle and foot. They are easy to operate and comfortable to wear, making them ideal for an active child who requires stability following a sprain, stress fracture, or surgery.

INDICATIONS FOR USE

- Soft tissue injuries
- Grade 2 and 3 sprains
- Stable fractures
- Post-operative ankle stabilisation
- Trauma and rehab use

FEATURES AND BENEFITS

- High quality rigid outer skeleton for protection
- Dense foam lining within the sole to provide comfort and support
- Shock absorbing sole to improve comfort and protects the user from slips, providing additional security in each step
- Uprights give additional stability by restricting motion of the leg
- Lightweight design perfectly designed for kids to keep active

REQUIRED MEASUREMENTS

- Shoe size



ORDERING INFORMATION

PAEDIATRIC WALKER

Size	Shoe Size	Part#
S	Child 6.5 - 9.5	PEW0400
M	Child 10 - Youth 1	PEW0600
L	Adult 1.5 - 2.5	PEW0800

REPLACEMENT LINERS

Size	Shoe Size	Part#
S	Child 6.5 - 9.5	PEW0001
M	Child 10 - Youth	PEW0002
L	Adult 1.5 - 2.5	PEW0003

AFO DYNAMIC

The AFO Dynamic provides a strong, lightweight solution for patients with varying degrees of drop foot secondary to a variety of neurological conditions. Össur has taken the carbon fibre technology from its Flex-Foot® product and incorporated it into the new AFO Dynamic to provide increased support over traditional ankle foot orthoses. The energy storing properties of carbon fibre make it an ideal material for this dynamic ankle foot orthosis because it provides real energy return. For those people leading busy lives, the AFO Dynamic will ensure a more efficient walk for the user.



INDICATIONS FOR USE

Drop foot secondary to:

- Cerebro Vascular Accident (CVA)
- Other neurological conditions
- Multiple Sclerosis
- Mild knee instability
- Charcot Marie Tooth disease (HMSN)

Contraindications:

- Severe ankle - foot deformities
- Fluctuating oedema
- Severe spasticity
- Neuropathy

FEATURES AND BENEFITS

- Carbon fibre design provides excellent strength-to-weight ratio and dynamic motion
- The tapered carbon heel stores energy and manages forces at heel strike
- Full-length toe lever provides full support while foot is loaded, as well as reduced shock to the contralateral limb
- Creates medio-lateral stability at mid-stance
- Reduces energy consumption by reducing hip hike
- Medial strut makes it less visible in shoe
- Open anterior tibia shell provides superior anatomical fit, and reduces heat and perspiration
- Weight limit: 120 kg

REQUIRED MEASUREMENTS

- Shoe size
- Calf height at widest part of the calf



AFO DYNAMIC

ORDERING INFORMATION

AFO DYNAMIC

Size	Calf Height	Shoe Size UK		Shoe Size EU		Left	Right
		Male	Female	Male	Female		
XS	30 - 34cm	2.5 - 4.5	< 4	34 - 37	< 37	O-102021	O-102011
S	32 - 36cm	5 - 7	4.5 - 6.5	37.5 - 40	37.5 - 40	O-102022	O-102012
M	34 - 38cm	7.5 - 10	7 - 9.5	40.5 - 44.5	40.5 - 44	O-102024	O-102014
L	36 - 40cm	10.5 - 13.5		45 - 48.5		O-102026	O-102016

ACCESSORIES

ACCESSORY KITS

Size	Part#
XS	O-102101
S	O-102102
M	O-102104
L	O-102106

Note: Accessory kits come with foam padding, upper and lower Hook & Loop straps, and Hook & Loop dots.

AFO LIGHT

AFO Light provides support only where it is needed, eliminating unnecessary bulk and providing a cosmetically-appealing look. Using the best of Flex-Foot® technology and the energy storing properties of carbon fibre, AFO Light offers a strong and lightweight solution for people with mild to moderate drop foot.



INDICATIONS FOR USE

- | | | |
|---|--|---|
| Drop foot secondary to: | <ul style="list-style-type: none"> • Charcot Marie Tooth disease (HMSN) • Lower extremity overuse injuries | Contraindications: |
| <ul style="list-style-type: none"> • Cerebro Vascular Accident (CVA) • Other neurological conditions • Multiple Sclerosis • Mild knee instability | | <ul style="list-style-type: none"> • Severe ankle - foot deformities • Fluctuating edema • Severe spasticity • Neuropathy |

FEATURES AND BENEFITS

- Energy return and propulsion assistance of the limb at toe off
- Supports the foot for ground clearance during swing phase
- Carbon fibre design provides excellent strength-to-weight ratio and dynamic motion
- Gradual plantar flexion at initial contact
- Stiffness gradient at heel and the toe correspond to ground reaction forces as they are applied to the foot
- Easy to don and doff
- Weight limit: 100kg

REQUIRED MEASUREMENTS

- Shoe size
- Calf height at widest part of the calf

ORDERING INFORMATION

AFO LIGHT

Size	Calf height	Shoe Size UK		Shoe Size EU		Left	Right
		Male	Female	Male	Female		
S	32 - 36cm	< 7	4.5 - 6.5	< 40	37.5 - 40	O-121022	O-121012
M	34 - 38cm	7.5 - 10	7 - 9.5	40.5 - 44.5	40.5 - 44	O-121024	O-121014
L	36 - 40cm	10.5 - 13.5		45 - 48.5		O-121026	O-121016
XL	38 - 42cm	11.5 - 14.5		46 - 50		O-121028	O-121018

ACCESSORIES

Description	Part#
Refurb Kit	O-122101

AFO LEAF SPRING

AFO Leaf Spring is a pre-fabricated polypropylene ankle-foot orthosis designed to support flaccid drop foot. It provides a semirigid section for toe clearance and support, yet has a thin, flexible foot plate for easy trimming. The absence of a heel section makes the Leaf Spring more comfortable to wear and provides a better fit in shoes.

INDICATIONS FOR USE

- Drop foot
- Mild drop foot secondary to other neurological pathologies

FEATURES AND BENEFITS

- Injection molded polypropylene is lightweight
- Variable thickness throughout the orthosis provides strength
- Provides good toe clearance and support
- Thin, flexible foot part is easy to trim with a pair of scissors
- Excellent fit for most types of shoes
- Absence of heel section for improved comfort

REQUIRED MEASUREMENTS

- Shoe size



ORDERING INFORMATION

Size	Shoe Size UK	Shoe Size EU	Height	Left	Right
S-M	3.5 - 6.5	36 - 39	27 cm	O-110227	O-110127
L	7 - 10.5	39.5 - 45	30 cm	O-110230	O-110130

PROLITE AFO

Comfortable, low profile prefabricated leaf spring design to correct flaccid foot drop, especially on post-stroke patients. Injection moulded polypropylene allows for different thicknesses throughout the orthosis — thicker on the vertical aspect for rigidity, thinner footplate for ease of trimming, comfort and flexibility.

INDICATIONS FOR USE

- Drop foot
- Mild drop foot secondary to other neurological pathologies

FEATURES AND BENEFITS

- Injection-moulded for a high quality finish
- Easily trimmed footplate
- May be heat modified for minor fitting adjustments

REQUIRED MEASUREMENTS

- Foot length



ORDERING INFORMATION

Size	Foot length	Brace height	Left	Right
S	24cm	26.5cm	3500OP	3505OP
M	28cm	28.5cm	3501OP	3506OP
L	28cm	33cm	3502OP	3507OP
XL	30cm	39cm	3503OP	3508OP

EXTRA STRONG PROLITE

INDICATIONS FOR USE

- Drop foot
- Mild drop foot secondary to other neurological pathologies

FEATURES AND BENEFITS

- Made from a specific acetal resin, polyoxymethylene, ensuring no moisture absorption, outstanding fatigue strength, impact resistance
- Increased rigidity in the sagittal plane
- May be heat modified for minor fitting adjustments

REQUIRED MEASUREMENTS

- Foot length



ORDERING INFORMATION

Size	Foot length	Brace height	Left	Right
S	24cm	26.5cm	3520BLU	3525BLU
M	28cm	28.5cm	3521BLU	3526BLU
L	28cm	33cm	3522BLU	3527BLU
XL	30cm	39cm	3523BLU	3528BLU

REBOUND FOOT-UP®

Rebound Foot-Up is a lightweight ankle/foot orthosis (AFO) that offers dynamic support for drop-foot or related disorders that require dorsiflexion support. The Rebound Foot-Up supports the ankle dorsiflexion during swing phase. Dynamic support is provided by an elastic strap connected to the shoe by either a transparent plastic inlay placed between shoelaces, or by a nut and bolt fastened through the eyelets. The positioning of the fastening allows for individualising the support by providing the option of positioning the dorsiflexion support on the medial or lateral side. When using the plastic inlay, the dorsiflexion support becomes neutral.

The Rebound Foot-Up Ankle Cuff has a smart clamp that holds the cuff in place and facilitates one-handed donning and doffing.

Optional: Rebound Foot-Up Foot Wrap can be used when barefoot or in open shoes or sandals. The adjustable elastic strap is attached to the Foot Wrap using a nut and bolt fastened through the eyelets.



INDICATIONS FOR USE

- Drop-foot
- Disorders that require dorsiflexion support

FEATURES AND BENEFITS

- Dynamic and discreet support for drop-foot or related disorders that require dorsiflexion support
- Improved function, ease of use, comfort, and quality
- Allows for voluntary plantar flexion
- The elastic strap is adjustable and easily replaceable
- The Rebound Foot-Up Ankle Cuff and Rebound Foot-Up Foot Wrap are made from breathable material that can be worn comfortably for long periods of time
- The Rebound Foot-Up is hand-washable
- Positioned medial or lateral support
- Contoured to the shape of leg
- Adjustable strap length
- Smart clamp facilitates one-handed donning / doffing
- Optional Rebound Foot-Up Foot Wrap can be used without shoes or in open shoes or sandals

REQUIRED MEASUREMENTS

- Circumference of ankle, measured around the malleoli



ORDERING INFORMATION

ANKLE CUFF

Description	Size	Ankle Circumference	Part #
Rebound Foot-Up Ankle Cuff	S-M	18 - 25cm	RFU50001
Rebound Foot-Up Ankle Cuff	L-LX	26 - 33cm	RFU50002

FOOT WRAP

Description	Size	Foot Circumference	Part #
Rebound Foot-Up Foot Wrap	One Size	20 - 28cm	RFU50101

ACCESSORIES

Description	Part #
Rebound Foot-Up Attachment Kit (includes plastic inlay, elastic strap, and 1 x nut and screw)	RFU50201
Rebound Foot-Up Replacement Nuts And Bolts (5 nuts included: 2 short and 3 long screws)	RFU50202

Description	Size	Foot Circumference	Part #
Rebound Foot-Up Bundle	-/M	18 - 25cm	RFU500011
Rebound Foot-Up Bundle	L-LX	26 - 33cm	RFU500021

FOOT-UP®

Foot-Up is a lightweight AFO that offers dynamic support for drop foot or similar complaints. The orthosis provides visible improvement in gait by supporting the foot the moment it is raised.



INDICATIONS FOR USE

- Drop foot
- Support of dorsiflexion

FEATURES AND BENEFITS

- Streamlined, lightweight and invisible when worn
- Breathable material, padded and comfortable to wear
- Easy to fit
- Available in black and beige
- Attached to the shoe, or available as shoeless model

REQUIRED MEASUREMENTS

- Basic version: Circumference of ankle, measured around the malleoli
- Shoeless version: Circumference of foot immediately behind the ball of the foot

ORDERING INFORMATION

FOOT-UP

Size	Ankle Circumference	Beige	Black
M	18 - 21cm	07811 -1	07810-1
L	22 - 26cm	07811 -2	07810-2
XL	27 - 33cm	07811 -3	07810-3

SHOELESS VERSION*

Size	Foot Circumference	Beige	Black
S	< 20cm	27811-1	27810-1
M	20 - 24cm	27811-2	27810-2
L	24 - 28cm	27811-3	27810-3
XL	> 28cm	27811-4	27810-4

*Shoeless version item comprises of foot wrap only and must be ordered with the Foot-Up unless the patient already has one.

ACCESSORIES

Description	Beige	Black
Foot-Up plastic insert	17811	17810

Description	Beige	Black
Foot-Up with Shoeless add-on Bundle	07811xy	07810xy

x = size of main foot-up part
 y = size of shoeless add-on
 e.g. 0781012 = size 1 Foot-up with a size 2 Shoeless add-on in Black

FORMFIT® PRO ANKLE

Össur Formfit® Pro Ankle is a compressive ankle sleeve with 2 integrated malleoli pads, designed to be worn after mild ankle injuries. The dynamic 3D MotionTech™ knit is supportive and meets certified medical grade compression requirements. Proven superior to leading brands for moisture wicking and breathability, your ankle stays cool during wear. CoolVent™ technology in the area in front of the ankle further enhances breathability and comfort.



MotionTech™



CoolVent™



SmartPad™



FoldLock™

INDICATIONS FOR USE

- For ankle conditions that may benefit from compression, such as:
- Ankle injury rehabilitation
- Mild sprains and strains
- Mild instability


CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit with superior moisture wicking and breathability, keeping your ankle cool during wear
- Certified medical grade compression (20-30 mmHg)
- CoolVent™ technology in the area in front of the ankle for added breathability and comfort
- 2 integrated malleoli SmartPads™ for targeted support and cushioning
- Patented FoldLock™ system for easier and more secure application

REQUIRED MEASUREMENTS

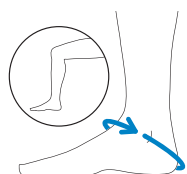

Measure ankle circumference diagonally over the base of the heel and across the ankle bone. Ankle in 90 degrees.

Size	Ankle circumference	Left	Right
XS	18 - 20cm	FP200201	FP200101
S	20 - 23cm	FP200202	FP200102
M	23 - 26cm	FP200203	FP200103
L	26 - 29cm	FP200204	FP200104
XL	29 - 32cm	FP200205	FP200105

FORMFIT® PRO ANKLE

ORDERING INFORMATION

Size	Left	Right
XS	FP200201	FP200101
S	FP200202	FP200102
M	FP200203	FP200103
L	FP200204	FP200104
XL	FP200205	FP200105



FORMFIT® ANKLE BRACE

The Formfit Ankle brace provides stability and protection for the ankle – whether used prophylactically, for chronic instability or following an injury. A hybrid between a rigid ankle stirrup and a soft ankle support, the Formfit Ankle ensures a close fit which limits inversion, eversion, flexion and extension movements. The low profile design fits easily into shoes and the quick lace up feature makes it easy to don and doff. Breathable material ensures it can be comfortably worn all day. The Formfit Ankle with Figure 8 strap version, offers an even greater level of support, by further limiting the range of motion of the ankle.



INDICATIONS FOR USE


- Ankle injury rehabilitation/immobilisation (Grade I/II sprains)
- Support chronic instability
- Prophylactic use

FEATURES AND BENEFITS

- Breathable material
- Quick lace Up
- Inversion/Eversion Control
- Figure 8 strap version

REQUIRED MEASUREMENTS

- Measure ankle circumference diagonally over the base of the heel and across the ankle bone.

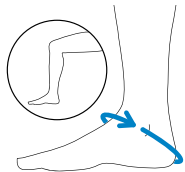

ORDERING INFORMATION

Size	Ankle Circ.	Form Fit Ankle Brace	Form Fit Ankle with Figure 8
XS	28 - 30.5cm	B-212010001	B-212000001
S	30.5 - 33cm	B-212010002	B-212000002
M	33 - 35.5cm	B-212010003	B-212000003
L	35.5 - 38cm	B-212010004	B-212000004
XL	38 - 40.5cm	B-212010005	B-212000005

FORMFIT® ANKLE WITH SPEEDLACE

INDICATIONS FOR USE

- Ankle injury rehabilitation/immobilization (Grade I/II sprains)
- Support chronic instability
- Prophylactic use


FEATURES AND BENEFITS

- Durable materials
- Quick-Pull closure straps
- Figure 8 straps
- Removable stays
- Circumferential cuff



REQUIRED MEASUREMENTS

- Measure ankle circumference diagonally over the base of the heel and across the ankle bone. Ankle in 90 degrees.

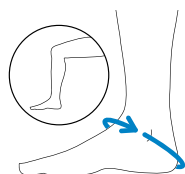

ORDERING INFORMATION

Size	Ankle Circ.	Part#
XXS	23 - 25.5cm	W-10621
XS	25.5 - 28cm	W-10622
S	28 - 30.5	W-10623
M	30.5 - 33cm	W-10625
L	33 - 35.5cm	W-10627
XL	35.5 - 38cm	W-10628
XXL	38 - 40.5cm	W-10629

FORMFIT® ANKLE STIRRUPS

≤18

Formfit Ankle Stirrups are lightweight and sturdy. Whether worn to accommodate an acute injury, chronic instability, or post-cast healing, Formfit Ankle Stirrups stabilise injured ankles by restricting the inversion/eversion motion. The universal design has been carefully engineered to accommodate the malleoli on both sides and ensures a great, comfortable fit. The slip resistant heel strap can be adjusted to accommodate most ankles. Our signature Flex-Edge™ overmould surrounds the stirrup's outer shell protecting patient's skin from any hard edges, further adding to a comfortable fit. The universal stirrup design is available with three different liner options: inflatable air, gel, or foam, offering each patient a solution that best fits their needs.



INDICATIONS FOR USE

Situations requiring inversion/eversion restriction. These may include:

- Mild to severe ankle sprains
- Chronic ankle instability
- Post-cast and post-walker support
- Ankle injury rehabilitation and/or post-surgical use

FEATURES AND BENEFITS

- Universal shell, carefully crafted to fit both right and left ankles
- Soft Flex-Edge overmould for increased comfort
- Adjustable heel strap to optimise patient fit and comfort
- Slip-resistant silicone dots to reduce slipping and stabilise the foot inside of shoe
- 2 bidirectional hook and loop straps ensure a secure and stable fit
- Choice of inflatable, foam, and gel liners to suit your needs
- Available in Adult and Youth sizes

REQUIRED MEASUREMENTS

- Patient's height

ORDERING INFORMATION



Size	Patient's Height	Formfit Air Ankle Stirrup	Formfit Gel Ankle Stirrup	Formfit Foam Ankle Stirrup
Adult	Above 160 cm	70350	70351	70352
Youth	Below 160 cm	71350	71351	71352

EXOFORM® DORSAL NIGHT SPLINT

INDICATIONS FOR USE

- Plantar fasciitis
- Post-static pain
- General heel pain
- Drop foot
- Achilles tendonitis
- Lower extremity overuse injuries

FEATURES AND BENEFITS

- Adjustable aluminum stay inside dorsal shell allows modification in the angle of dorsiflexion
- Soft, overmolded edges enhance comfort
- Hinged shell at the calf area prevents brace rotation
- Semi-rigid plantar pad reduces discomfort along the sides of the foot while providing an improved angle of dorsiflexion
- Non-slip surface on plantar pad prevents slipping during normal walking
- Circumferential heel and posterior strapping prevents migration while increasing overall comfort and effectiveness
- Strapping and shell perforations improve comfort and breathability



REQUIRED MEASUREMENTS

- Shoe size

ORDERING INFORMATION

Size	Shoe Size UK	Shoe Size EU	Part#
M	4.5 - 8	37.5 - 42.5	W-50085
L	8.5 - 13.5	43 - 48.5	W-50087

FORMFIT® NIGHT SPLINT

The Formfit Night Splint, designed to help relieve the pain associated with plantar fasciitis by providing a gentle overnight stretch. Straps can be adjusted from 0° up to 10° degrees as needed. The soft breathable liner and padded straps provide comfort and help keep patient cool. Straps adjust from 0°-10° dorsiflexion

INDICATIONS FOR USE

Conditions that may benefit from resting with the ankle in a neutral position, such as:

- Plantar fasciitis
- Achilles tendinopathy

FEATURES AND BENEFITS

- Straps adjust from 0°-10° of dorsiflexion
- Contoured for a comfortable fit
- Soft foam interior is washable and breathable quick release buckles for easy application
- Slip resistant tread

REQUIRED MEASUREMENTS

- Shoe size



ORDERING INFORMATION

Size	Shoe Size UK		Shoe Size EU		Standard with tread
	Male	Female	Male	Female	
S	Up to 6.5	Up to 5	Up to 39	Up to 38	50023
M	7 - 9.5	5.5 - 8	39.5 - 44	38.5 - 43	50025
L	10 and up	8.5 and up	44.5 and up	43.5 and up	50027

ACCESSORIES

Description	Part#
Foam Wedge	10700
Replacement Liner S	11760
Replacement Liner M	11761
Replacement Liner L	11762

AIRFORM® NIGHT SPLINT

Smaller and lighter than most, delivering both nighttime comfort and morning pain relief

INDICATIONS FOR USE

- Plantar fasciitis
- Drop foot
- Achilles tendonitis
- Post-static pain

FEATURES AND BENEFITS

- Light weight, low profile design
- Slips on and adjusts easily
- Available in singles or 6 packs for easy storage



REQUIRED MEASUREMENTS

- Shoe size

ORDERING INFORMATION

Size	Shoe Size UK		Shoe Size EU		Standard	6 Pack
	Male	Female	Male	Female		
M	4.5 - 8.5	3.5 - 7.5	37 - 43	36 - 42	50045	500456
L	9 - 13.5	8 - 12	43.5 - 48.5	42.5 - 48	50047	500476

DH OFFLOADING POST-OP STYLE SHOE

Designed for use in the treatment of plantar wounds, this shoe can be an excellent alternative to post-op style shoes with static pads (wool, felt, foam, etc.). It accommodates the ambulatory and compliant patient for short-term treatment (1 to 4 months).

INDICATIONS FOR USE

- Plantar wounds
- Post-operative use
- Plantar foot ulcers

FEATURES AND BENEFITS

- Designed to accommodate either left or right foot
- Designed around Össur's unique Active Offloading Insole
- Designed to accommodate bulky dressings
- Super soft nylon upper with reinforced heel counter provides extra comfort and stability
- Patented removable toe pieces aides in wound accessibility for daily wound site inspection



REQUIRED MEASUREMENTS

- Shoe size

ORDERING INFORMATION

Size	Shoe Size UK		Shoe Size EU		Part#
	Male	Female	Male	Female	
S	3.5 - 5.5	4 - 6	35.5 - 38	37 - 39	10340
M	6 - 8	6.5 - 8.5	38.5 - 42	39.5 - 43.5	10341
L	8.5 - 10.5	9 - 10	42.5 - 45	44 - 45	10342
XL	11 - 13.5	10.5+	45.5 - 48.5	45.5-48.5	10343

ACCESSORIES

REPLACEMENT PARTS

Size	Insoles
S	12365
M	12366
L	12367
XL	12368

MESH TOP POST-OP SHOE

INDICATIONS FOR USE

- For recovery after soft tissue procedures and post trauma

FEATURES AND BENEFITS

- Breathable mesh upper with reinforced heel counter and padded heel collar provides increased stability overall to the rear of the foot while reducing slippage
- Semi-rigid sole provides superior protection and control after osteotomies
- Extended strap and dual side closure system accommodates extra bulky dressings

REQUIRED MEASUREMENTS

- Shoe size



ORDERING INFORMATION

Size	Shoe Size UK		Shoe Size EU		Male	Female
	Male	Female	Male	Female		
S	3.5 - 5.5	2.5 - 3.5	35.5 - 38	35 - 36	17013	17003
M	6 - 8	4 - 5.5	38.5 - 42	36.5 - 38.5	17015	17005
L	8.5 - 10.5	6 +	42.5 - 45	39 +	17017	17007
XL	11 - 13.5		45.5 - 48.5		17018	

SOFT TOP POST-OP SHOE

INDICATIONS FOR USE

- Post-operative use or minor trauma wounds
- Geriatric or insensate foot

FEATURES AND BENEFITS

- The cotton-poly blend upper allows the patient's foot to breathe and provides added comfort
- Designed specifically for recovery after soft tissue procedures post trauma
- The contact closure system provides even torsion and is designed for ease of use

REQUIRED MEASUREMENTS

- Shoe size



ORDERING INFORMATION

Size	Shoe Size UK		Shoe Size EU		Male	Female
	Male	Female	Male	Female		
S	3.5 - 5.5	2.5 - 3.5	35.5 - 38	35 - 36	18013	18003
M	6 - 8	4 - 5.5	38.5 - 42	36.5 - 38.5	18015	18005
L	8.5 - 10.5	6 +	42.5 - 45	39 +	18017	18007
XL	11 - 13.5		45.5 - 48.5		18018	

CANVAS ROCKER BOTTOM CAST SHOE

INDICATIONS FOR USE

- For use with a short leg cast

FEATURES AND BENEFITS

- Cushioned rocker bottom cast sandal
- Contact closure, open toe

REQUIRED MEASUREMENTS

- Insole length



ORDERING INFORMATION

Size	Insole length	Blue	Black
XS	18cm	308XS	
S	19 - 23cm	308SM	B308SM
S-M	23 - 25cm	308SM/MED	B308S/M
M	25.5 - 26.5cm	308MED	B308MED
M-L	26.5 - 28cm	308MLG	B308MLG
L	28 - 30.5cm	308LG	B308LG
XL	30.5 - 35cm	308XLG	B308XLG

NOTES



WWW.OSSUR.NL

Össur Europe B.V.
Afleveradres: De Schakel 70
5651 GH Eindhoven

Postadres: Postbus 120
5690 AC Son en Breugel
The Netherlands

TEL NL 00800 3539 3668
TEL BE 0800 70927
orders.benelux@ossur.com

