


Bracing & Support

2025 Catalogue

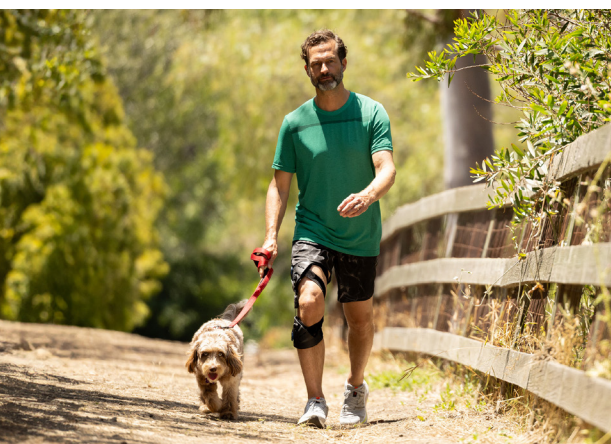

BRACING & SUPPORT SOLUTIONS CATALOGUE

TABLE OF CONTENTS

ÖSSUR OA SOLUTIONS.....	4
INTRODUCTION TO OA SOLUTIONS	5
UNLOADER® HIP	6
UNLOADER ONE® X.....	8
UNLOADER® CUSTOM	14
CTI® OA.....	16
FORMFIT® PRO KNEE OA	18
OA EASE.....	19
SPINE.....	23
PHILADELPHIA® ADJUSTABLE TRACHEOTOMY COLLAR	24
PHILADELPHIA® TRACHEOTOMY COLLAR.....	25
PATRIOT® COLLAR	26
MIAMI J® SELECT	27
MIAMI J®	28
OCCIAN® BACK & OCCIAN® BACK JR.....	30
MIAMI JTO® THORACIC EXTENSION	31
MIAMI TLSO™ 464	32
MIAMI TLSO 456	34
MIAMI LUMBAR® POSTEO.....	36
MIAMI LSO™	37
OAM RIGID LUMBAR	39
FORMFIT® BACK VS	40
FORMFIT® PRO BACK	41
FORMFIT® BACK SUPPORT, AIR	42
FORMFIT® BACK SUPPORT.....	43
MIAMI Jr®	44
PAPOOSE®	45
HIP	46
REBOUND® HIP.....	47
KNEE	48
CTI® 3.....	49
CTI® CUSTOM	51
C180™ ROCKET	54
REBOUND® CARTILAGE	55
REBOUND® PCL	57
REBOUND® ACL	60
REBOUND® ACL CUSTOM	63
FORMFIT® MCL	66
REBOUND® DUAL BASIC	67
REBOUND® POST OP KNEE	69
FORMFIT® POST-OP KNEE	71
FORMFIT® KNEE ROM.....	72
REBOUND® KNEE IMMOBILISER UNIVERSAL.....	73
EXOFORM® KNEE IMMOBILISER.....	74
REBOUND® KNEE BRACE.....	75

ÖSSUR BRACING & SUPPORTS CATALOGUE

TABLE OF CONTENTS

FORMFIT® KNEE, HINGED	76
FORMFIT® TRACKER	77
NEOPRENE HINGED KNEE SUPPORT	79
NEOPRENE KNEE SUPPORT WITH STABILISED PATELLA	79
FORMFIT® PRO KNEE	80
FOOT AND ANKLE.....	81
REBOUND® AIR WALKER	82
FORMFIT® WALKER, AIR	84
FORMFIT® WALKER.....	85
ROM WALKER	86
REBOUND® DIABETIC WALKER	87
DH OFFLOADING WALKER®	89
DH OFFLOADING POST-OP STYLE SHOE	90
FORMFIT® ANKLE BRACE (SPEED BRACE)	91
FORMFIT® ANKLE WITH SPEEDLACE.....	92
FORMFIT® ANKLE STIRRUPS.....	93
FORMFIT® PRO ANKLE.....	94
FORMFIT® NIGHT SPLINT	95
EXOFORM® DORSAL NIGHT SPLINT	96
AIRFORM® NIGHT SPLINT	97
AFO DYNAMIC	98
AFO LIGHT	100
AFO LEAF SPRING	101
REBOUND FOOT-UP®.....	102
UPPER EXTREMITY	103
FORMFIT® SHOULDER BRACE	104
REBOUND® POST-OP ELBOW.....	105
FORMFIT® TENNIS ELBOW	107
AIRFORM® TENNIS ELBOW SUPPORT.....	107
FORMFIT® PRO ELBOW	108
FORMFIT® WRIST / FORMFIT® WRIST & FOREARM	109
FORMFIT® WRIST & THUMB, UNIVERSAL	110
SPECTRA WRIST BRACE.....	111
EXOFORM® CARPAL TUNNEL WRIST	112
FORMFIT® THUMB SPICA.....	113
UNIVERSAL THUMB SPLINT.....	114
FORMFIT® PRO WRIST	115
CASTING	116
TECHFORM® PREMIUM	117

ÖSSUR OA SOLUTIONS

Enhance quality of life and mobility for people living with osteoarthritis.

These product solutions have been built on decades of experience and are designed to enhance the quality of life of people living with osteoarthritis by reducing pain and improving mobility. Through innovation leadership, OA expertise and partnerships with medical professionals, we offer comprehensive, proven and effective treatment options.

UNLOADER®

The Össur Unloader® brand name has become synonymous with Osteoarthritis bracing, representing OA Solutions' technology as the industry standard, with more than 15 clinical studies demonstrating its efficiency.

Össur strives to maintain its position as the innovation leader for OA Solutions by continuing to develop simple, yet effective clinically proven treatment solutions that increase people's quality of life.





HIP OA



UNLOADER[®] HIP

PAGE 6

KNEE OA



UNLOADER ONE[®] X

PAGE 8

UNLOADER[®] HIP

Unloader Hip is designed to reduce pain by optimising load dispersion for patients suffering from mild and moderate osteoarthritis of the hip, thus contributing to hip stability and improved mobility.



INDICATIONS FOR USE


- Mild and moderate osteoarthritis of the hip
- Conditions benefitting from repositioning of the hip and/or compression around the greater trochanters

FEATURES AND BENEFITS

- The Dynamic Rotation Straps with SmartDosing dial allows individual adjustment of the external rotation force, to achieve optimum pain relief for each patient
- Easy donning and doffing thanks to pocket wrap closure system and zipper
- Pulley system for improved stability
- Premium quality lycra for breathability and a snug fit

REQUIRED MEASUREMENTS

- Measure the hip circumference at the anterior superior iliac spine

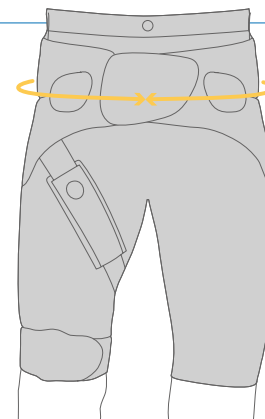

UNLOADER® HIP

ORDERING INFORMATION

UNLOADER HIP

Size	Hip circumference	Woman	Men
Small	80 – 88 cm	B-465350030	B-465350031
Medium	88 – 96 cm	B-465350032	B-465350033
Large	96 – 104 cm	B-465350034	B-465350035
X-Large	104 – 114 cm	B-465350036	B-465350037
XX-Large	114 – 124 cm	B-465350038	B-465350039

ACCESSORIES/ OPTIONS

REPLACEMENT TROUSERS

Size	Measurements	Woman	Men
Small	80 – 88 cm	B-465350077	B-465350078
Medium	88 – 96 cm	B-465350079	B-465350080
Large	96 – 104 cm	B-465350081	B-465350082
X-Large	104 – 114 cm	B-465350083	B-465350084
XX-Large	114 – 124 cm	B-465350085	B-465350086

OTHER ACCESSORIES

Description	Part#
Rotation strap	B-465350064
Washbag	B-465350065
Trochanter pad	B-465350066
Thigh strap S	B-465350067
Thigh strap M	B-465350068
Thigh strap L	B-465350069
Thigh strap XL	B-465350070
Thigh strap XXL	B-465350071
Bilateral kit* S	B-465350072
Bilateral kit* M	B-465350073
Bilateral kit* L	B-465350074
Bilateral kit* XL	B-465350075
Bilateral kit* XXL	B-465350076
Unloader Hip front closure	B-112600020

*The bilateral kit consists of a rotation control strap, a thigh strap and a trochanter pad

UNLOADER ONE® X

Unloader One X improves mobility by providing pain relief from knee osteoarthritis, degenerative meniscal tears, and other unicompartmental knee conditions that may benefit from unloading.

It features Össur's unique, proprietary, and clinically proven 3-point Leverage System that generates unicompartmental unloading of the knee joint.

Available in custom and OTS versions to help you fit your patient's different leg anatomies.



INDICATIONS FOR USE

For knee conditions that may benefit from unicompartmental load reduction, such as:

- Mild to severe unicompartmental knee osteoarthritis
- Degenerative meniscal tears
- Articular cartilage defect repair
- Meniscal repair
- Avascular necrosis
- Condylar bone marrow lesions (bone bruises)
- Tibial plateau fractures

FEATURES AND BENEFITS

- Relieves Pain - 3 Point Leverage System with Dynamic Force System® straps, rigid shells, and a single upright hinge biomechanically unloads the affected knee compartment
- Control - Tool-free SmartDosing® dials enable the patient to fine-tune the strap tension to control pain
- Tool-Free - Tool-free polycentric hinge with optional ROM stops for range-of-motion restriction
- Comfortable - Skin-friendly liners feel great against the skin and reduce pressure points. Anti-migration strap technology design captures the widest part of the calf to reduce brace migration
- Easy-to-Use - Alignment guide on hinge facilitates correct positioning of the brace on the leg and color coded touch points aid in easy donning and doffing
- Easy-to-Wear - Slender frame with soft edges easily contours to individual anatomies and remains discreet whether worn under or over clothing
- Easy Care - Entire brace is machine washable (washbag included)

REQUIRED MEASUREMENTS: UNLOADER ONE X CUSTOM

- Custom made to measure: Smartmeasure, 3D Scan (.AOP, .STL, .OBJ), .CPV) or Patient Cast

SMARTMEASURE®


A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000


UNLOADER ONE® X

REQUIRED MEASUREMENTS: UNLOADER ONE X OTS

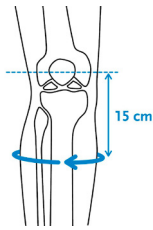

Measure calf circumference 15 cm below mid-patella. Leg in extension.

Measure leg length from floor to groin.

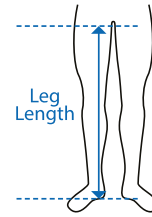

Size	Calf circumference 15 cm below mid patella
	cm
XS - M	25 - 43
L - XL	43 - 62

Model	Leg Length
Standard	> 85 cm
Short	≤ 85 cm

COLOURS

CUSTOM STANDARD COLOURS



Black Brace /
Black Components



Blue Brace /
Black Components



Charcoal Grey Brace /
Black Components



Red Brace /
Black Components



White Brace /
Black Components



Yellow Brace /
Black Components

OTS COLOUR



Black Brace /
Black Components

Actual colours may vary.

ORDERING INFORMATION

Hinge is placed on the side of the affected compartment.
Short version recommended for patients with a leg length of ≤ 85 cm (33.5").

CUSTOM

Description	Part#
Right	UOX0R
Left	UOX0L


OTS

Standard	Right Medial	Right Lateral	Left Medial	Left Lateral
XS - M	UOX0518713	UOX0528713	UOX0618713	UOX0628713
L - XL	UOX0518715	UOX0528715	UOX0618715	UOX0628715

Short	Right Medial	Right Lateral	Left Medial	Left Lateral
XS - M	UOX1518713	UOX1528713	UOX1618713	UOX1628713
L - XL	UOX1518715	UOX1528715	UOX1618715	UOX1628715


UNLOADER ONE® X

REPLACEMENT PARTS

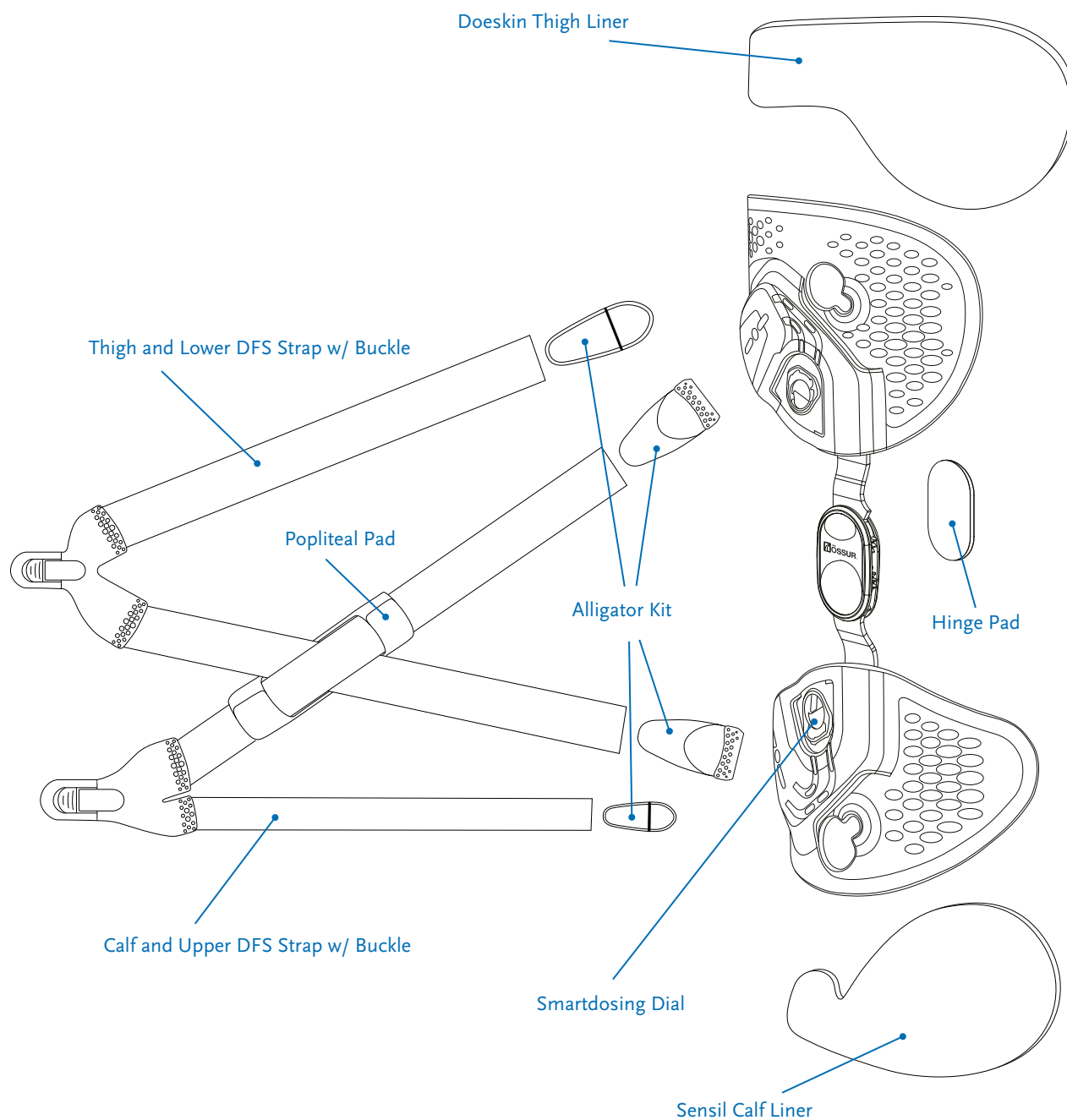

UNLOADER ONE® X

ACCESSORIES

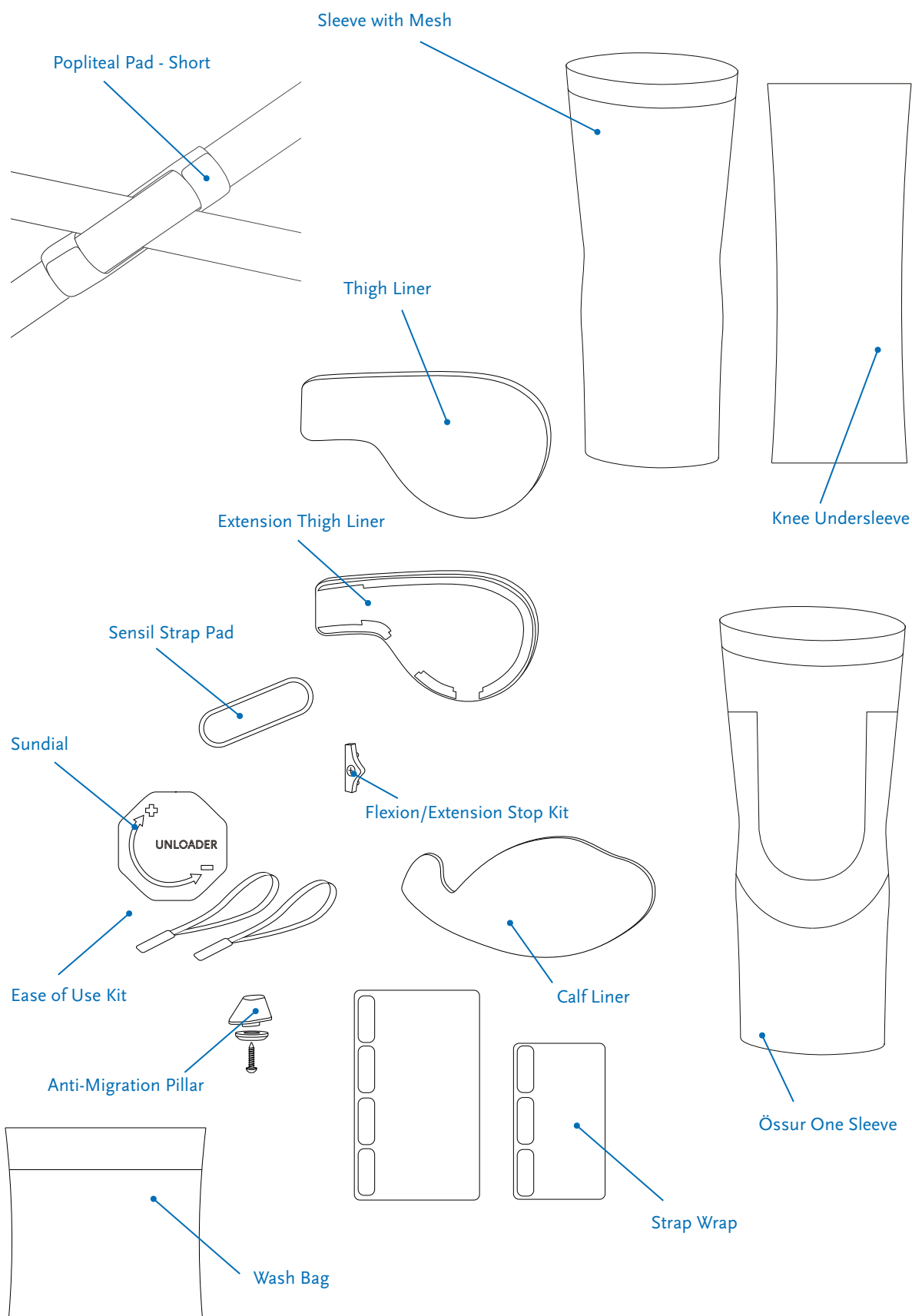

UNLOADER ONE® X

THIGH LINERS	THIGH, Right Medial, Left Lateral	THIGH, Right Lateral, Left Medial	THIGH EXTENSION, Right Medial, Left Lateral	THIGH EXTENSION, Right Lateral, Left Medial
Black, Sensil	UOX526801	UOX525801	NA	NA
Black, Doeskin	UOX526901	UOX525901	UOX546901	UOX545901

CALF LINERS	Right Medial, Left Lateral	Right Lateral, Left Medial
Black, Sensil XS-M	UOX516952	UOX515952
Black, Sensil L-XL	UOX516962	UOX515962
Black, Doeskin XS-M	UOX516951	UOX515951
Black, Doeskin L-XL	UOX516961	UOX515961

THIGH AND LOWER DFS STRAP*	Right Medial, Left Lateral	Right Lateral, Left Medial
Black	UOX406902	UOX405902

*Includes Thigh and DFS Strap with buckle

CALF AND UPPER DFS STRAP*	Right Medial, Left Lateral	Right Lateral, Left Medial
Black	UOX406901	UOX405901

*Includes Calf and DFS Strap with buckle

UOX SERVICE KIT*	Right Medial, Left Lateral	Right Lateral, Left Medial
Black, XS-M	UOX416901	UOX415901
Black, L-XL	UOX416911	UOX415911

* Includes Doeskin Thigh Liner, Sensil Calf Liner, Thigh and Lower DFS Strap with Buckle, Calf and Upper DFS Strap with Buckle, Hinge Pad, Popliteal Pad, and Alligator Kit

ANTI-MIGRATION PILLAR	Right Medial, Left Lateral	Right Lateral, Left Medial
Black	UOX700015	UOX700035

UNLOADER ONE® X

OTHER REPLACEMENT PARTS AND ACCESSORIES	Part#
Alligator Kit*	UOX900001
Hinge Pad	UOX700091
Popliteal Pad	UOX700012
Popliteal Pad - Short	UOX700022
Sensil Strap Pad	UON0181
Sensil Strap Pad - 10 Pieces	UOX700113
Smartdosing Dial	UOX1000001
Strap Wrap Set	UOX700014
Ease of Use Kit**	UOX800001
OA Hinge Extension Stop Tree (5°, 10°, 15°, 30°)	OAH40029
OA Hinge Flexion Stop Tree (0°, 30°, 60°, 90°)	OAH40039
Sundial	UOX1200001
Wash Bag	UON60200

* Includes 4 Alligator Clips: 2 DFS, 1 Thigh and 1 Calf

** Includes Sundial and 2 buckle finger-loops

SLEEVES	Black - Össur One*	Black - Sleeve with Mesh**	Black - Össur Knee Undersleeve***
XXS	OSO60021	NA	NA
XS	OSO60022	SLV60011	B-250150101
S	OSO60023	SLV60012	B-250150102
M	OSO60024	SLV60013	B-250150103
L	OSO60025	SLV60014	B-250150104
XL	OSO60026	SLV60015	B-250150105
XXL	OSO60027	SLV60016	B-250150106
3XL	OSO60028	NA	NA

* Össur One: 51% Tencel™/30% Polyamide/19% Elastane

**Sleeve with Mesh: Main fabric: 92% Cotton/8% Spandex - Mesh: 100% Polyester - Band: 80% Polyester/20% Spandex

***Össur Knee Undersleeve: 80% Nylon/20% Spandex

UNLOADER® CUSTOM

With a semi-rigid frame and an adjustable hinge, the Unloader Custom accommodates a wide variety of leg shapes and sizes. The Adjustable Dynamic Joint enables changes to the hinge and additional leverage for optimal pain management.



INDICATIONS FOR USE

- Mild to severe unicompartmental osteoarthritis

FEATURES AND BENEFITS

- Custom molded semi-flexible shells are durable, adaptable to leg shape changes, yet rigid enough to contain tissue while force is applied at the knee
- Patented Adjustable Dynamic Joint™ (ADJ) high-carbon steel hinge is strong enough to unload the pressure from the compartment of the knee. Adjustments are made easily
- The Dynamic Force Strap® designed to reduce pressure on sensitive bony surfaces of the knee, resulting in reduction of pain
- Available in standard and in SmartDosing version. Tool-free SmartDosing dial option enables fine-tuned pain management and can be adjusted even when wearing the brace under clothing
- SmartDosing version available with Sensil® silicone calf liner to reduce migration
- Available in 12 standard colours in matte or glossy finish or custom painted


REQUIRED MEASUREMENTS

- Custom-to-measurement

CUSTOM STANDARD COLOURS



CUSTOM PAINTED DESIGNS

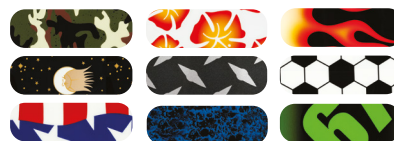

No charge for custom standard colours. Additional fees required for other custom colours, custom painted designs and custom printed graphics.



UNLOADER® CUSTOM

SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required

ORDERING INFORMATION

UNLOADER CUSTOM

Description	Part#
Right	B-122500020
Left	B-122600020

Description	Part#
Proximal Non-Elastic Thigh Strap	B-507010000
AMS Wrap (custom made)	B-426000150
AMS Sleeve (custom made)	B-426000250
ACL Band Kit	B-704000003

ACCESSORIES

Description	Part#
Knee Shield	B-250110000
Hinge cover	B-251030000
Water Ski Strap Lock	B-303010084
Distal Non-Elastic Thigh Strap	B-507010000

ELASTIC CALF SUSPENSION STRAP

Size	Part#	Size	Part#	Size	Part#
14"	B-504020114	18"	B-504020018	22"	B-504020022
15"	B-504020115	19"	B-504020019	23"	B-504020023
15 cm	B-504020116	20"	B-504020020	24"	B-504020024
17"	B-504020017	21"	B-504020021		

SLEEVES

Size	Thigh circumference	Softsleeve	Neoprene undersleeve	Neoprene oversleeve
Small	< 38 cm	20643	20251	20051
Medium	38 – 46 cm	20644	20252	20052
Large	46 – 50 cm	20644	20253	20053
X-Large	50 – 55 cm	20645	20254	20054
XX-Large	55 – 62 cm	20645	20255	20055
XXX-Large	62 – 71 cm	20646	20256	20056
XXXX-Large	71 – 86 cm	20647	20257	20057

Size	Thigh circumference	Össur knee undersleeve	Size	Thigh circumference	Össur knee undersleeve
X-Small	30.5 – 38 cm	B-250150101	Large	56 – 66 cm	B-250150104
Small	38 – 45.5 cm	B-250150102	X-Large	66 – 73.5 cm	B-250150105
Medium	45.5 – 56 cm	B-250150103	XX-Large	73.5 – 84 cm	B-250150106

CTI® OA

CTi® OA's composite shells can be constructed with up to 7° of varus or valgus correction, offering a high level of protection and stability for clinicians interested in treating osteoarthritis and ligament instability equally. Key features include a Total Support System, Accutrac® hinges that mimic the natural motion of the knee joint and breathable liners, coated with Össur® Sensil® Silicone to improve efficacy and virtually eliminate migration.



INDICATIONS FOR USE

- Mild to severe unicompartmental osteoarthritis
- ACL, MCL, LCL, PCL, rotary and combined instabilities
- Demanding activities of daily living

FEATURES AND BENEFITS

- Anatomically-correct Accutrac® hinges with extension stops
- Proprietary Sensil® silicone liners designed to reduce migration
- Total Support System™ for ACL, MCL, LCL, PCL, rotary and combined instabilities
- Carbon composite, rigid frame construction
- Custom-fabricated to each patient's unique measurements with up to 7° varus/valgus correction built into the frame
- Non-corrosive materials, ideal for water sports
- Standard model for medium to high impact levels. Vapor model for low to medium impact levels. Pro Sport model for highest impact levels

REQUIRED MEASUREMENTS

- Custom-made to measure



CTI® OA

ORDERING INFORMATION

CTI OA

Description	Left	Right
Standard	B-116630010	B-116530010
Pro Sport	B-117630010	B-117530010
Vapor	B-118630010	B-118530010



SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.

Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required

CUSTOM STANDARD COLOURS

Red	Pink	Silver
Orange	Black	White
Yellow	Navy Blue	Champagne
Lime Green	Ocean Blue	Charcoal

CUSTOM PAINTED DESIGNS



No charge for custom standard colours. Additional fees required for other custom colours, custom painted designs and custom printed graphics.

ACCESSORIES

Description	Part#
OA Condyle Pad Kit 1/4"	25021
OA Condyle Pad Kit 3/8"	25644

For all other Cti OA Accessories, see CTi Custom Accessories. Page 76.

FORMFIT® PRO KNEE OA

Össur Formfit® Pro Knee OA is a unique, compressive knee sleeve with a Dynamic Force Strap system, designed to provide pain relief from the early stages of osteoarthritis (OA). The dynamic, 3D MotionTech™ knit is supportive and meets certified medical grade compression requirements, and has been proven to be superior to the leading brands at moisture wicking and breathability, keeping your knee cool during wear. CoolVent™ technology in the popliteal area enhances breathability and comfort, while the distinct patella graphics help with intuitive placing of the sleeve. The integrated StayBilizer™ hinge, constructed from lightweight, flexible EVA material, is suitable for sports, such as football.



MotionTech™



CoolVent™



StayBilizer™



FoldLock™



INDICATIONS FOR USE

For osteoarthritis and other knee conditions which may benefit from unloading and/or compression, such as:

- Medial or lateral knee pain
- Early or mild unicompartmental osteoarthritis of the knee
- Symptomatic, unicompartmental meniscus degeneration

CONTRAINDICATIONS


- Must not be used by individuals for whom compression is contraindicated

FEATURES AND BENEFITS

- Patented DFS system
- Certified medical grade compression (20-30 mmHg)
- Lightweight CoolVent™ knit in popliteal area for added breathability and comfort
- Patented FoldLock™ system for easier and more secure application
- Sports-compatible StayBilizer™ hinge made from light, flexible EVA material for optimum shape and fit
- Patella graphics for intuitive placement
- Available in 5 standard sizes and 3 comfort sizes

REQUIRED MEASUREMENTS

Size	Calf circumference 15 cm below mid patella cm	Thigh circumference 15 cm above mid patella cm	Right Medial, Left Lateral	Right Lateral, Left Medial
XS	26 - 31	36 - 41	FP110401	FP110301
S	31 - 36	41 - 46	FP110402	FP110302
M	36 - 41	46 - 51	FP110403	FP110303
M Comfort	36 - 41	51 - 56	FP110413	FP110313
L	41 - 46	51 - 56	FP110404	FP110304
L Comfort	41 - 46	56 - 60	FP110414	FP110314
XL	46 - 50	56 - 60	FP110405	FP110305
XL Comfort	46 - 50	60 - 64	FP110415	FP110315

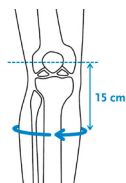

Measure calf circumference 15 cm below mid-patella. Leg in extension.



Measure thigh circumference 15 cm above mid-patella. Leg in extension.

OA EASE

The OA Ease is a single upright brace with a Dynamic Force System™ strap designed to relieve pain from mild-to-moderate unicompartmental knee osteoarthritis (OA). Its wraparound design is easy to apply and remove using simple rubber pull-tabs, color-coded guide dots, and strap tension indicator lines. This low-profile knee brace is comfortable, easy to use, and adapts to a variety of leg anatomies. The OA Ease can also be machine washed for effortless care.




INDICATIONS FOR USE

- Unicompartmental osteoarthritis of the knee
- Other conditions that may benefit from unicompartmental unloading, such as degenerative meniscal tears

FEATURES AND BENEFITS

- Easy Wraparound Design - Simple rubber pull-tabs, color-coded guide dots, and a convenient hinge alignment guide make application and removal simple
- Relieves OA Pain - 3-Point Leverage System with Dynamic Force System® strap and shells unload the affected knee compartment
- Machine Washable - Lightweight and breathable fabric feels great against the skin and can be machine washed for effortless care
- Tool-Free - Tool-free polycentric hinge with available ROM stop options for range-of-motion restriction
- Control - Fine-tune strap tension for individual pain relief, with marked dosing indicator lines
- Extension kits are available to further enhance fit and comfort

REQUIRED MEASUREMENTS

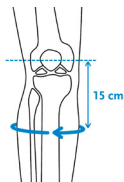

Measure calf circumference
15 cm below mid-patella. Leg in extension.



Measure thigh circumference
15 cm above mid-patella. Leg in extension.

OA EASE

ORDERING INFORMATION

Size	Calf circumference 15cm below mid patella	Thigh circumference 15cm above mid patella	Right Medial	Left Medial	Right Lateral	Left Lateral
XS	28 - 32	36 - 41	FOA0518711	FOA0618711	FOA0528711	FOA0628711
S	32 - 36	41 - 46	FOA0518712	FOA0618712	FOA0528712	FOA0628712
M	36 - 41	46 - 53	FOA0518713	FOA0618713	FOA0528713	FOA0628713
L	41 - 46	53 - 60	FOA0518714	FOA0618714	FOA0528714	FOA0628714
XL	46 - 51	60 - 67	FOA0518715	FOA0618715	FOA0528715	FOA0628715
XXL	51 - 56	67 - 75	FOA0518716	FOA0618716	FOA0528716	FOA0628716
3XL	56 - 61	75 - 81	FOA0518717	FOA0618717	FOA0528717	FOA0628717

Hinge is placed on the side of the affected compartment.

REPLACEMENT PARTS


Description	Part#
Alligator Clip (Right Medial/Left Lateral)	FOW00501
Alligator Clip (Right Lateral/Left Medial)	FOW00502
Hinge Pad	B-304120101

ACCESSORIES

Description	Part#
Small Extension Kit 2 Pieces	FOA000301
Small Extension Kit 10 Pieces	FOA000302
Large Extension Kit XS-S (Right Medial/Left Lateral)	FOA000401
Large Extension Kit M-L (Right Medial/Left Lateral)	FOA000402
Large Extension Kit XL-3XL (Right Medial/Left Lateral)	FOA000403
Large Extension Kit XS-S (Right Lateral/Left Medial)	FOA000404
Large Extension Kit M-L (Right Lateral/Left Medial)	FOA000405
Large Extension Kit XL-3XL (Right Lateral/Left Medial)	FOA000406
Wash Bag	WB000001
Suspension Strap Kit	OAW00007
OA Hinge Extension Stop Kit (5°, 10°, 15°, 30°)	OAH40029
OA Hinge Flexion Stop Kit (0°, 30°, 60°, 90°)	OAH40039

Small Extension Kit:

Increases calf and/or thigh circumference up to 4.5 cm

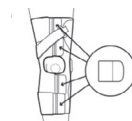


Small Extension Kit

Large Extension Kit:

XS-S increases thigh circumference up to 4 cm

M-L increases thigh circumference up to 5 cm

XL-3XL increases thigh circumference up to 6 cm

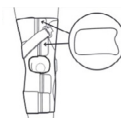

Large Extension Kit

ÖSSUR INJURY SOLUTIONS

For people recovering from fractures, ligament injuries or in need of post-operative treatment solutions.

These product solutions have been built on decades of experience and are designed to enhance the healing process of bone and soft tissue injuries. Through innovation leadership, proven biomechanical approaches, and partnerships with leading medical professionals, we strive to offer comprehensive and effective treatment options.

Össur Injury Solutions are globally recognized, primarily under the brand names CTi®, Rebound® and Miami®, all of which are developed to assist healing and increase people's quality of life.

Our expertise in Injury Solutions has driven us to take the next step forward by developing indication-specific products which are designed to optimise healing while maintaining function and mobility. These Functional Healing products incorporate Össur proprietary technology and are designed to enhance clinically relevant outcomes.



FUNCTIONAL HEALING –
WHERE MOBILITY
MEANS RECOVERY.



SPINE



MIAMI J®

PAGE 27



FUNCTIONAL HEALING



REBOUND®
CARTILAGE

PAGE 55



REBOUND®
PCL

PAGE 58

KNEES



CTi®

PAGE 49



REBOUND® DUAL
BASIC

PAGE 68

FEET



REBOUND® AIR WALKER

PAGE 83

SPINE



PHILADELPHIA® ADJUSTABLE TRACHEOTOMY COLLAR

INDICATIONS FOR USE

- Post-trauma
- Herniated cervical disc
- Cervical spondylitis
- Post-operative
- Cervical spondylosis
- RA and OA cervical spine

FEATURES AND BENEFITS

- Circumference and height adjustable
- Permanent lock on height adjustment
- Anatomically shaped Plastazote® foam (Not made with natural rubber latex, non-toxic and hypoallergenic to reduce skin irritation), leaving clearance for cervical veins
- Total cervical arch support maintains neutral alignment of the cervical spine
- Water-resistant for bathing and hydro-therapy
- Well ventilated
- X-ray and CT lucent. MR conditional

REQUIRED MEASUREMENTS

- Neck height: or 25mm of neck length adjustability accommodates varying patient neck length.

ORDERING INFORMATION

PHILADELPHIA ADJUSTABLE TRACHEOTOMY COLLAR

Size	Neck circumference
Small (2 tears)	25.4 – 33 cm
Medium (1 tear)	33 – 40.6 cm
Large (no tear)	40.6 – 48.3 cm

NOTE: The table above represents common sizing only. Sizing must be verified by the healthcare provider for each patient individually.

Part#	Size / Description
PHP-AA	Conforms to sizes 2, 3 and 4 of the original Philadelphia Tracheotomy Collar



PHILADELPHIA® TRACHEOTOMY COLLAR

INDICATIONS FOR USE

- Post-trauma
- Herniated cervical disc
- Cervical spondylitis
- Post-operative
- Cervical spondylosis
- Motor Neurone Disease
- RA and OA cervical spine



FEATURES AND BENEFITS

- Not made with natural rubber latex, non-toxic and hypoallergenic
Plastazote® foam reduces skin irritation
- Total cervical arch support maintains neutral alignment of the cervical spine
- Water-resistant for bathing and hydro-therapy

REQUIRED MEASUREMENTS

- Neck circumference and height from tip of chin to sternal notch

ORDERING INFORMATION

Size	Neck circumference	6 cm height	8 cm height	11 cm height	13 cm height
X-Small	< 25 cm	PHP-T2XS			
Small	25 – 33 cm	PHP-T2S	PHP-T3S	PHP-T4S	PHP-T5S
Medium	33 – 41 cm	PHP-T2M	PHP-T3M	PHP-T4M	PHP-T5M
Large	41 – 48 cm	PHP-T2L	PHP-T3L	PHP-T4L	PHP-T5L
X-Large	48 cm	PHP-T2XL	PHP-T3XL	PHP-T4XL	PHP-T5XL

Size	Neck circumference	Height	Part#
Infant	15 – 20 cm	3 cm	PHP-TI
Pediatric	20 – 28 cm	4.5 cm	PHP-TP

ACCESSORIES

PHILADELPHIA COOLMAX COTTON LINER

Size	Neck circumference	Part#
Infant	15 – 20 cm	PHP-LCI
X-Small	< 25 cm	PHP-LCXS
Small	25 – 33 cm	PHP-LCS
Medium	33 – 41 cm	PHP-LCM
Large	41 – 48 cm	PHP-LCL
X-Large	> 48 cm	PHP-LCX

PHILADELPHIA TRACHEOTOMY ATTACHMENTS

Size	Stabiliser	Halo
Pediatric	PHP-OSP	PHP-OHP
Adult	PHP-OS	PHP-OH

PATRIOT® COLLAR

INDICATIONS FOR USE

- Post-trauma

FEATURES AND BENEFITS

- Multi-height adjustment – Adult (4 positions) and Pediatric (3 positions)
- Open rear ventilation panel for fluid drainage, airflow and cervical spine palpation
- Universal sizing optimises inventory requirements



REQUIRED MEASUREMENTS

- Neck circumference

ORDERING INFORMATION

Size	Neck circumference	Blue
Child	20 – 46 cm	PHP-PP
Adult	28 – 58 cm	PHP-PA

MIAMI J® SELECT

The Miami J Select cervical collar by Össur offers the widest range of height adjustability in the market with a single, universal design that replicates the comfort, immobilisation and quality you expect from a Miami J product. Featuring a proprietary, first-to-market locking mechanism to encourage patient compliance and anti-microbial Sorbatex padding, Miami J Select is designed to support successful patient outcomes.



INDICATIONS FOR USE

Situations requiring gross immobilisation of the cervical spine, these may include:

- Pre- and post c-spine surgery
- C-spine precaution for trauma patients

FEATURES AND BENEFITS

- Easy height adjustability with a tamper-proof locking mechanism – ‘Left to Lock’
- Anti-microbial & biocompatible Sorbatex™ padding prevents bacterial build-up
- Universal design that doesn’t compromise immobilisation or comfort
- 7 precise fitting sizes with opportunity to fine-tune
- Miami J® inspired chin tray
- One-piece, anatomically inspired collar back with Flex Edge® designed to reduce occipital pressure
- Easy to don and doff with proprietary Reproducible Fit option
- Intuitive blue Patient Touch-Points to send patients home with confidence – Blue is You™
- Simplified comfort adjustments with integrated and proprietary Sternal-Relief Dial
- Large tracheal opening – improved function for ICU patients
- Easy to clean and replace multi-coloured padding
- X-ray and CT lucent, MR-safe.
- Compatible with Occian™ Back & Miami JTO® accessories

ORDERING INFORMATION

Description	Size	Part#
Miami J Select Collar Set	One-Size adjustable (Universal)	MJSR-101
Miami J Select Collar	One-Size adjustable (Universal)	MJS-101
Miami J Select Replacement Pads	One-Size adjustable (Universal)	MJSP-101
Miami J Select Reproducible Fit Tabs (10 pack)	One-Size adjustable (Universal)	MJST-101

MIAMI J®

Known as the superior c-spine immobiliser through multiple independent studies, the Miami J's patented design meets the specialised needs of all those afflicted with cervical injuries. The Miami J offers features that enhance compliance while providing immobilisation, inhibition of skin breakdown and patient comfort.



INDICATIONS FOR USE

- Post-trauma
- Stable cervical fractures
- Cervical spondylitis
- Cervical spondylosis
- RA and OA cervical spine
- Herniated cervical disc
- Post-operative
- Motor Neurone Disease

FEATURES AND BENEFITS

- Sorbatex padding is antibacterial and clinically proven to inhibit microbial growth, which enhances comfort and helps assure patient compliance
- Bioengineered to minimise pressure points in key known areas: Chin, occiput, trapezius and clavicle
- Angulation buttons for occiput and sternal pad is removable when supine, lying and swallowing
- MRI, CT and X-ray lucent, MR safe







REQUIRED MEASUREMENTS

- Sizing based on 6 phenotypes to ensure correct fit and optimum immobilisation

Phenotype-driven sizing system makes size selection simple while drastically reducing margin of error in fitting a patient. Each collar has a bioengineered front with a specific back designed to work together for superior immobilisation. For pediatric sizing, please see Miami Jr in this Catalogue.

Specialty Sizes (40% of population)

Standard Sizes (60%)

Kyphotic (10%)	XSmall (5%)	Stout (15%)	Tall (10%)	Regular (30%)	Short (30%)
 <ul style="list-style-type: none"> • kyphotic • chin-on-chest • geriatric • osteoporitic and/or ankylosing spondylitic 	 <ul style="list-style-type: none"> • Very short, thin neck • Principally female • More prevalent in the Asian population 	 <ul style="list-style-type: none"> • Very large neck circumference • Obese • "No neck" • Massive shoulders 	 <ul style="list-style-type: none"> • Long, tall neck • "Swan neck" • Young Women • Adolescents 	 <ul style="list-style-type: none"> • Female • Standard size for mature women • Thin, mature male 	 <ul style="list-style-type: none"> • Standard size for mature men • Short-necked women

*For individuals with neck circumference > 25", the Bariatric collar is recommended.

MIAMI J®

ORDERING INFORMATION

MIAMI J

Size	Collar set*	Collar	Collar Back	Collar Front
Super Short	MJR-200S	MJ-200S	MJ-400BK	MJ-200SFRT
Stout	MJR-200L	MJ-200L	MJ-200LBK	MJ-200LFRT
XS	MJR-250	MJ-250	MJ-250BK	MJ-250FRT
Short	MJR-300	MJ-300	MJ-300BK	MJ-300FRT
Regular	MJR-400	MJ-400	MJ-400BK	MJ-400FRT
Tall	MJR-500	MJ-500	MJ-500BK	MJ-500FRT
Bariatric	MJR-200B	MJ-200B		

*Collar sets come with one additional set of replacement pads.

Miami J Front and Back Panels may be interchanged to accommodate patient anatomy, e.g. a Short Front may be used with a Tall Back. Choose collar size based on patient neck height, not patient body size. Super Short collars are packaged with Regular Back Panels

ACCESSORIES

REPLACEMENT PADS

Size	Part#
Super Short	MJP-100
Stout	MJP-200L
XS	MJP-250
Short	MJP-100
Regular	MJP-100
Tall	MJP-100
Bariatric	MJP-200B

MIAMI J PATIENT PACK

Description	Part#
Miami J Patient Pack	MJ-PTPK2

OCCIAN® BACK & OCCIAN® BACK JR

The Occian Back is designed exclusively for the specialised needs of the multi-system trauma patient in the ICU. Universally sized, specifically engineered and designed with advanced, Intuitech™ pressure-relieving memory foam, the Occian Back helps caregivers manage c-spine patients who are at high risk of occipital breakdown.



INDICATIONS FOR USE

- Patients not awake and alert within 24 – 48 hours after admission
- Patients with pre-existing skin injuries and/or lacerations to the head/neck
- Burn patients who must wear a collar
- Elective surgery patients desiring night time post-operative comfort
- Patients who remain in a supine position for long periods of time related to respiratory, neurological, or haemodynamic instability

FEATURES AND BENEFITS

- Intuitech pressure relieving memory foam virtually eliminates occipital breakdown
- Maintains C spine stability
- Provides unique pressure-relieving comfort by replacing the back section of a Miami J Collar
- Clinically proven to virtually eliminate occipital breakdown without sacrificing c-spine stability
- Easy to clean
- MRI, CT & X-ray lucent, MR safe

REQUIRED MEASUREMENTS

- Universal, one size

ORDERING INFORMATION

Description	Part#
Occian Back*	ACB-100
Occian Back Junior**	ACB-200

*Compatible with Miami Jr® size P3 and all sizes of Miami J® and Miami J® Select.

** Compatible with Miami Jr® size P2.

MIAMI JTO® THORACIC EXTENSION

The Miami JTO Thoracic Extension is designed in conjunction with leading spine surgeons, orthotists and nurses. The orthosis is used exclusively with all Miami J® collars when even greater stability at the cervical junctions is indicated.



INDICATIONS FOR USE

Lower C-Spine disruptions

- Level one instabilities
- Isolated compartment fracture
- Lateral mass fracture
- Spinous Process fracture

Upper C-Spine disruptions

- For increased stability over the collar alone
- Stable Jefferson or Hangman's Fracture
- Type I Odontoid Fracture

FEATURES AND BENEFITS

- Provides more control than a collar alone can provide
- Innovative device designed to treat cervical and high-thoracic injuries
- Patented design with 'floating back' permits maximum immobilisation without the risk of compensatory motion
- Snap on and off without tools
- Sorbatex II padding is antibacterial and clinically proven to inhibit microbial growth, which enhances comfort and helps assure patient compliance
- MRI, CT and X-ray lucent, MR compatible

REQUIRED MEASUREMENTS

- Universal, one size

ORDERING INFORMATION

MIAMI JTO THORACIC EXTENSION

Description	Part#
Miami JTO	MT-900
Miami JTO with replacement softgoods	MTR-900

Compatible with all sizes of the Miami J®, level 1 through 5 of the Miami J® Select, and the P3 Miami Jr®. Any model of Miami J collar is not included with Miami JTO Thoracic Extension and is purchased separately.

ACCESSORIES

Description	Part#
Replacement softgoods	MTP-950

MIAMI TLSO™ 464

The Miami TLSO from Össur directly addresses the needs of patients and practitioners, simplifying preparation without compromising on fit or performance. This easy-to-use pre-assembled Miami TLSO brace offers three configurations in one. Compliance: Easily applied in over the shoulder or underarm configuration, donning and doffing is further simplified with the conveniently located quick release buckles. Adjustability: The universal belt designs allows for setting the belt length on the patient for a more accurate sizing experience. Comfort: Truly shaped to fit the anatomy, the anatomically contoured design better immobilises the spine while maximising patient comfort. Ease of Use: The Miami TLSO brace requires minimal preparation and is designed to provide optimal patient comfort.



INDICATIONS FOR USE

Conditions requiring gross immobilisation of the thoracic and lumbar spine. Such conditions may include:

- Post-surgical immobilisation
- Spinal Support TLSO attachment is up to T7
- Spinal stenosis
- Herniated disc
- Degenerative spinal fractures
- Spondylolithesis
- Spondylolysis

CONTRAINDICATIONS


- Unstable, displaced fractures

FEATURES AND BENEFITS

- Significantly superior comfort and ease of donning/doffing
- Fully assembled for fitting with ease
- Accommodates heights 155–193 cm
- Posterior thoracic extension option with ergonomic axillary strapping
- Step-down to an LSO as one progresses through rehabilitation
- One-touch telescoping anterior-thoracic extension option with single or double-hinge
- The convenience of a one-size Universal or individualise with a uniquely-sized anatomical design
- Improve comfort and immobilisation with multiple strapping configurations and removable anchors

REQUIRED MEASUREMENTS

Size	Waist circumference
	cm
Universal	66 - 177
S	66 - 86
M	86 - 107
L	107 - 127
XL	127 - 177

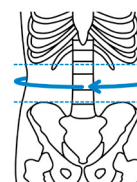

Measure circumference of the trunk at a level midway between the lowest ribs and the upper iliac crest. Standing.

MIAMI TLSO™ 464

ORDERING INFORMATION

ATE/PTE, LORDOTIC

Size	Part#
Universal	MTLSO-2351
S	MTLSO-2111
M	MTLSO-2121
L	MTLSO-2131
XL	MTLSO-2141

ATE/PTE, NON-LORDOTIC

Size	Part#
S	MTLSO-2211
M	MTLSO-2221
L	MTLSO-2231
XL	MTLSO-2241

ATE/PTE, LORDOTIC, SLIM

Size	Part#
Universal	MTLSO-1351
S	MTLSO-1111
M	MTLSO-1121
L	MTLSO-1131
XL	MTLSO-1141

ATE/PTE, NON-LORDOTIC, SLIM

Size	Part#
S	MTLSO-1211
M	MTLSO-1221
L	MTLSO-1231
XL	MTLSO-1241

OTHER ACCESSORIES

Description	Part#
5MM Hex Long Wrench	MTLSO-0004
Axillary strap accessory kit	MTLSO-0003
Quick fit tool	B-040100000
Belt extension, standard (Recommended for waist circumferences 127-177 cm. Fits size XL.)	MLSO-2EXT
Belt extension, slim (Recommended for waist circumferences 127-177 cm. Fits size XL.)	MLSO-1EXT

MIAMI TLSO 456

The Miami TLSO 456 from Össur directly addresses the needs of patients and practitioners, simplifying preparation without compromising on fit or performance. This easy-to-use pre-assembled Miami TLSO brace offers the convenience of a one-size Universal or a uniquely-sized anatomical design. Compliance: Blue is You™ patient touch-points help send patients home with confidence.

Adjustability: The universal belt design allows for setting the belt length on the patient for a more accurate sizing experience. Heat moldable posterior panels allow a customised fit, while the modular design allows step-down support to an LSO.

Comfort: Featuring the Miami LSO™ Advanced Compression pulley system with lumbar compression you can feel, and an anatomically contoured design that better immobilises the spine while maximising patient comfort.

Ease of Use: Easily applied with over-the-shoulder straps, the Miami TLSO brace requires minimal preparation and is designed to provide optimal patient comfort.



INDICATIONS FOR USE

Conditions requiring gross immobilisation of the thoracic and lumbar spine. Such conditions may include:

- Post-surgical immobilisation
- Spinal Support TLSO attachment is up to T7
- Spinal stenosis
- Herniated disc
- Degenerative spinal fractures
- Spondylolithesis
- Spondylolysis

FEATURES AND BENEFITS

- Significantly superior comfort and ease of donning/doffing (when compared to Miami Lumbar TLSO)
- Fully assembled for fitting with ease
- Accommodates heights 155–193 cm
- Step-down to an LSO as one progresses through rehabilitation
- The convenience of a one-size Universal or individualise with a uniquely-sized anatomical design


CONTRAINDICATIONS

- Unstable, displaced fractures

REQUIRED MEASUREMENTS

Size	Waist circumference*
	cm
Universal	66 - 177
S	66 - 86
M	86 - 107
L	107 - 127
XL	127 - 177

*Belt extensions are required for waist circumferences larger than 127 cm

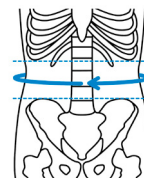

Measure circumference of the trunk at a level midway between the lowest ribs and the upper iliac crest. Standing.

MIAMI TLSO 456

ORDERING INFORMATION

PTE LORDOTIC

Size	Part#
Universal	MTLSO-2352
S	MTLSO-2112
M	MTLSO-2122
L	MTLSO-2132
XL	MTLSO-2142

PTE NON-LORDOTIC, SLIM

Size	Part#
S	MTLSO-1212
M	MTLSO-1222
L	MTLSO-1232
XL	MTLSO-1242

PTE NON-LORDOTIC

Size	Part#
S	MTLSO-2212
M	MTLSO-2222
L	MTLSO-2232
XL	MTLSO-2242

PTE LORDOTIC, SLIM

Size	Part#
Universal	MTLSO-1352
S	MTLSO-1112
M	MTLSO-1122
L	MTLSO-1132
XL	MTLSO-1142

OTHER ACCESSORIES

Description	Part#
Quick fit tool	B-040100000
Belt extension, standard (Recommended for waist circumferences 127-177 cm. Fits size XL.)	MLSO-2EXT
Belt extension, slim (Recommended for waist circumferences 127-177 cm. Fits size XL.)	MLSO-1EXT

MIAMI LUMBAR® POSTEO

Designed with patient comfort and compliance in mind, the Miami Lumbar Posteo provides functional relief of vertebral fracture pain, and facilitates return to mobility. The lumbar compression system offers both pain relief via active vertebral offloading and better grip of patient's hips for greater ease of use and compliance. The Miami Lumbar Posteo includes an innovative approach to ease of use and consistent donning.



INDICATIONS FOR USE

- Back pain caused by stable vertebral compression fractures (VCF)

FEATURES AND BENEFITS

- Active offloading via thoracic extension and lumbar compression
- Integrated pulley system for customised fit
- Trimmable axillary straps with adjustable tension for optimal stabilisation
- Cold-moldable aluminum PTE to fit different anatomies
- Extra cushioning around the aluminum strut for minimised skin pressure

REQUIRED MEASUREMENTS

- Back length measured from the top of the sacrum up to C7 vertebrae, following the spinal curvature

Please note that the Miami Lumbar Posteo includes a universal lumbar belt which can be trimmed down to the appropriate size. Calculation of the appropriate back length size, measured from the tip of the sacrum to the C7 vertebra, following the curvature of the spine.

ORDERING INFORMATION

MIAMI LUMBAR POSTEO

Size	Back Length	Part#
X-Small	37 – 44 cm	LMB-POSTEO-XS
Small	44 – 51 cm	LMB-POSTEO-SM
Medium	49 – 56 cm	LMB-POSTEO-MD
Large	55 – 61 cm	LMB-POSTEO-LG

ACCESSORIES

Description	Part#
Strap Pad	LMB-POSTEO ST PD

MIAMI LSO™

Miami LSO is a spinal orthosis designed for pain relief, protection of injured ligaments or muscles and post-surgical immobilisation. It provides low-profile, anatomical immobilisation, easy fit and wear, options for universal or modular support and an advanced pulley system with a length-adjustable pulley cord.



INDICATIONS FOR USE

Conditions requiring gross immobilisation of the trunk in the lumbar region. Such conditions may include:

- Stable, non-displaced spinal fractures (L1-L5)
- Spinal stenosis
- Herniated discs
- Degenerative spinal pathologies
- Spondylolisthesis
- Spondylolysis
- Contra-indication: Unstable, displaced spinal fractures

FEATURES AND BENEFITS

- Slim, breathable material
- Proven Miami double-pulley system with length-adjustable pulley cord
- Intuitive fitting without trimming with fold-over belt arms
- Optional fit tool for perfect fitting directly on the patient
- Universal belt sizing accommodates waists up to 178 cm
- New Universal Posterior panel to accommodate more patients with less inventory
- Contoured posterior panels sized S-XL for a precise, anatomical fit
- Quick-Fit pocket enclosure and blue highlights for patient ease of use
- Immobilisation from L1 – L5 with coverage from S1 – T9


REQUIRED MEASUREMENTS


- Waist circumference





MIAMI LSO™


ORDERING INFORMATION


MIAMI LSO BELT		
		
Size	Waist circumference	Part#
6" Universal (Slim)	66-127 cm	MLSO-1
8" Universal (Standard)	66-127 cm	MLSO-2
6" (Slim) Belt Extension	66-177 cm	MLSO-1EXT
8" (Standard) Belt Extension	66-177 cm	MLSO-2EXT

CONTOURED POSTERIOR PANEL		
		
Size	Waist circumference	Part#
Small (S)	66-86 cm	MLSO-P11
Medium (M)	86-96 cm	MLSO-P21
Large (L)	96-127 cm	MLSO-P31
Extra Large (XL)	127-167 cm	MLSO-P41

CONTOURED POSTERIOR PANEL WITH LATERAL SUPPORTS		
		
Size	Waist circumference	Part#
Small (S)	66-86 cm	MLSO-P12
Medium (M)	86-96 cm	MLSO-P22
Large (L)	96-127 cm	MLSO-P32
Extra Large (XL)	127-167 cm	MLSO-P42

STANDARD POSTERIOR PANEL	
	
Size	Part#
Standard	MLSO-P56

UNIVERSAL POSTERIOR PANEL	
	
Type	Part#
with Lateral	MLSO-P55
without Lateral	MLSO-P57

ANTERIOR PANEL	
	
Size	Part#
6" Panel (Slim)	MLSO-PA6
8" Panel (Standard)	MLSO-PA8

OAM RIGID LUMBAR

The OAM Rigid Lumbar offers you a discrete look with endless options.

First, you choose the belt.

Slim or Standard, 10 pre-cut sizes or universal.

Then, you choose the panels.

Different panels for different control, modular to allow you to remove as the patient heals.

Or, you choose the kit.


We make it even simpler by providing our most popular sizes pre-configured for you.



TWO SIMPLE OPTIONS FOR ORDERING

OPTION 1: ORDER BRACES BY COMPONENTS


OAM RIGID LUMBAR BELT

			
Size	Waist circumference	Standard part #	Slim part #
2XL	116-127cm	LMBGEN-006	LMBSLGEN-006
6XL	157-167cm	LMBGEN-010	LMBSLGEN-010

ANTERIOR PANELS

		
Size	Waist circumference	Part #
Standard	<116cm<	LMBGEN-ST
Slim	<116cm<	LMBGEN-SLST

POSTERIOR PANELS

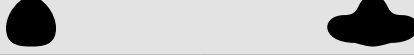			
Size	Waist circumference	Without lateral	With lateral
S	66-86cm	LMBGEN-NS	LMBGEN-LS
M	86-96cm	LMBGEN-NM	LMBGEN-LM
L	96-127cm	LMBGEN-NL	LMBGEN-LL
XL	127-167cm	LMBGEN-NX	LMBGEN-LX

FORMFIT® BACK VS

Formfit® Back VS is a lumbar support belt with versatile double strap. V-shaped metal stays in the back of the belt support the lumbosacral joint. The height of the second strap can be adjusted to apply the desired level of compression on the lower back.



INDICATIONS FOR USE

For lower back conditions that may benefit from gross motion restriction, such as:

- Muscular low back pain
- Lumbosacral strains
- Chronic low back pain
- Sciatica

CONTRAINDICATIONS

- Conditions requiring more rigid or higher immobilisation (> D12) or a surgical procedure, unless medically indicated.
- Pelvic floor dysfunction

FEATURES AND BENEFITS

- Removable, versatile double strap, adjustable in height
- Patented V-shaped stays
- Breathable materials
- Wide finger loops and hook & loop areas

REQUIRED MEASUREMENTS

- Waist circumference

ORDERING INFORMATION

FORMFIT BACK VS - 21 CM

Size	Waist circumference	Part#
XS	70 - 80 cm	209112
S	80 - 90 cm	209113
M	90 - 100 cm	209115
L	100 - 110 cm	209117
XL	110 - 125 cm	209118
XXL	125 - 140 cm	209119

FORMFIT BACK VS - 26 CM

Size	Waist circumference	Part#
XS	70 - 80 cm	209122
S	80 - 90 cm	209123
M	90 - 100 cm	209125
L	100 - 110 cm	209127
XL	110 - 125 cm	209128
XXL	125 - 140 cm	209129

ACCESSORIES

Description	Part#
Hot/Cold Therapy Gel Pad	GB28

FORMFIT® PRO BACK

Össur Formfit® Pro Back provides relief for patients suffering from pains and strains in the lower back. Manufactured with our dynamic, 3D-Motion Tech™ knitting technology, our belts feature superior moisture wicking capabilities and are highly breathable. The patented back panel design Utilises our StayBilizer™ technology to provide optimum shape and fit, while conveniently placed finger loops make donning easy. The SmartPad™ provides a targeted support and cushioning and can be removed if desired.



MotionTech™



StayBilizer™



SmartPad™



CoolVent™



INDICATIONS FOR USE

For lower back conditions that may benefit from compression and gross motion restriction, such as:

- Lower back pain (Lumbago)
- Lumbosacral sprain/strain
- Degenerative joint/disc disease

CONTRAINDICATIONS


- Must not be used by individuals for whom compression is contraindicated

FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit
- Removable lumbar SmartPad™ for targeted support and cushioning
- Lightweight, malleable aluminum stays are shaped to the anatomy of the lower back

REQUIRED MEASUREMENTS

Size	Waist circumference	Black
	cm	Part#
XS	70 - 80	FP500001
S	80 - 90	FP500002
M	90 - 100	FP500003
L	100 - 115	FP500004
XL	115 - 130	FP500005

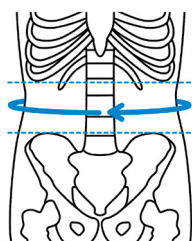

Measure circumference of the trunk at a level midway between the lowest ribs and the upper iliac crest. Standing.

FORMFIT® BACK SUPPORT, AIR

INDICATIONS FOR USE

- Muscular low back pain
- Lumbosacral strains
- Chronic low back pain
- Sciatica

FEATURES AND BENEFITS

- Intuitive integrated pneumatic system for more customised compression and support
- Anti-migration system reduces slippage
- Double wrap design for improved compression
- Finger loops for easy application
- Hot/Cold Gel Pad available separately



REQUIRED MEASUREMENTS

- Waist circumference

ORDERING INFORMATION

Size	Waist circumference	Standard
S	63 – 76 cm	209133
M	76 – 86 cm	209135
L	86 – 97 cm	209137
XL	97 – 107 cm	209138
XXL	107 – 142 cm	209139

ACCESSORIES

Description	Part#
Gel Pad	GB28

Description	Part#
Back Support Suspenders Clip On	10912

FORMFIT® BACK SUPPORT

INDICATIONS FOR USE

- Lumbar strains
- General posture support
- Lumbosacral strains
- Sciatica

FEATURES AND BENEFITS

- Muscular low back pain
- Lumbosacral strains
- Chronic low back pain
- Sciatica



REQUIRED MEASUREMENTS

- Waist circumference

ORDERING INFORMATION

Size	Waist Circumference	Standard
Small	63 – 76 cm	209143
Medium	76 – 86 cm	209145
Large	86 – 97 cm	209147
X-Large	97 – 107 cm	209148
XX-Large	107 – 142 cm	209149
XXX-Large	142 – 152 cm	209151

ACCESSORIES

Description	Part#
Back Support Suspenders Clip On	10912
Hot/Cold Therapy Gel Pad	GB19028

MIAMI Jr®

The Miami Jr is the breakthrough collar designed exclusively for children of 12 years and under. The Miami Jr line, which spans the growth cycle, is sized according to age. Össur is the only company that has a pediatric line of collars based on the anatomical differences between adults and children. We realise that children are not just “little adults” – they require innovative design.



INDICATIONS FOR USE

- Post-trauma
- Stable cervical fractures
- Cervical spondylitis
- Cervical spondylosis
- RA and OA cervical spine
- Herniated cervical disc
- Post-operative
- Motor Neurone Disease

FEATURES AND BENEFITS

- Innovative collar designed exclusively for children 12 years and under anatomy pediatric
- Sorbatex II pads make the collar “skin friendly” and protect skin integrity during extended wear
- MRI, CT and X-ray lucent, MR safe

REQUIRED MEASUREMENTS

- Age

ORDERING INFORMATION

MIAMI JR

Size	Collar set*	Collar
0 – 6 months	MJR-P0	MJ-P0
6 – 24 months	MJR-P1	MJ-P1
2 – 6 years	MJR-P2	MJ-P2
6 – 12 years	MJR-P3	MJ-P3

*Collar sets come with one additional set of replacement pads.

ACCESSORIES

REPLACEMENT PADS

Size	Part#
0 – 6 months	MJP-P0
6 – 24 months	MJP-P1
2 – 6 years	MJP-P2
6 – 12 years	MJP-P3

PAPOOSE®

The Papoose is an innovative design for the youngest and most challenging patients – newborn infants. The Papoose cradles the baby's body securely while gently immobilising the head and spine.



INDICATIONS FOR USE

- Immobilisation of the spine
- Spinal fractures

FEATURES AND BENEFITS

- True cervical spine immobiliser for infants up to 3 months
- Allows for continued intimacy between caregiver and child
- MRI, CT and X-ray lucent, MR compatible

REQUIRED MEASUREMENTS

- Universal, one size

ORDERING INFORMATION

PAPOOSE

Description	Part#
Papoose without front collar	PO-100
Papoose without front with replacement pads	POR-100
Papoose with front collar	PA-100
Papoose with front with replacement pads	PAR-100

ACCESSORIES

REPLACEMENT PADS

Description	Part#
Papoose replacement pads (with front)	PAP-100
Papoose replacement pads (without front)	POP-100

HIP



REBOUND® HIP

Össur's Rebound Hip brace has a low-profile, lightweight design that supports patient comfort and compliance. It delivers gentle controlling forces and effective functional ROM restriction in extension/flexion and abduction/adduction to promote optimal post-surgical outcomes. The universal frame and softgood are easy to stock, assemble and quickly fit on patients.



INDICATIONS FOR USE

Hip conditions that could benefit from motion restriction following surgery:

- Femoral Acetabular Impingement arthroscopic surgery
- Gluteus medius repair surgery
- Hamstring repair surgery

FEATURES AND BENEFITS

- Low-profile, lightweight design that supports patient comfort and compliance
- Quick-release pegs easily adjust the desired range-of-motion needed for the patient's optimal rehab
- Quick-fit with sliding mechanism on frame to adjust for patient's height
- Optional, adjustable ROM hinge from -10°, 0° 10°, or 20°

REQUIRED MEASUREMENTS

- Circumference at iliac crest height
- Length from greater trochanter to mid patella

ORDERING INFORMATION

REBOUND HIP

Description	Preset Left	Preset Right
Rebound Hip	980130RB	980135RB
Rebound Hip with adjustable hinge	980120RB	980125RB

ACCESSORIES

Description	Part#
OR Kit	980301
Belt Extender (Fitting 49-67" hip circumference)	980121RB
Fitting Tool	B-040100000
Rotation Strap	RHK-100024

KNEE



CTI®3

CTI®3 is a comfortable, lightweight and durable ligament knee brace, which provides stabilisation of the knee joint and an anatomical design suitable for both everyday activities and sports.



INDICATIONS FOR USE

- For knee conditions that may benefit from increased AP and ML stability and protection, such as injuries to the ACL, MCL, LCL, PCL, rotary and combined instabilities

FEATURES AND BENEFITS

- Enhanced construction - Lightweight, high strength carbon composite frames provide the optimal stiffness to weight ratio
- Anatomical fit - Designed to fit a wider range of leg shapes and includes an Accutrak® hinge to closely follow the knee's anatomical movement
- Elevated comfort - Unique liners enhance comfort with their special tread block pattern, which provides a balance of cushion and ventilation
- Easy-to-use accessories and replacement parts, including MX kits, replacement strap sets, liner sets, and flexion stops
- Available in 4 sizes, right and left

REQUIRED MEASUREMENTS

- M/L Width measurement: Medio-laterally at joint width in a standard, weight-bearing position
- Thigh circumference: Secondary measurement for patients in between sizes, measured 20cm above mid-patella
- Distal Patella circumference: Just below the lower border of the knee cap



CTI®3

ORDERING INFORMATION

CTi3 - Includes Condyle Pad Set and Extension Stop Set, 0° Extension Stop and Standard Condyle Pad are pre-installed.

Size	M/L Width	Thigh circumference	Distal Patella circumference	Right	Left
S	8.9-10.2 cm	40-47 cm	28-34 cm	CTI-22002	CTI-23002
M	10-11.4 cm	45-55 cm	34-38 cm	CTI-22003	CTI-23003
L	11.2-12.7 cm	52-61 cm	38-41 cm	CTI-22004	CTI-23004
XL	12.5-14 cm	58-67 cm	41-44 cm	CTI-22005	CTI-23005

Note: M/L Width measurements are the preferred and most accurate measurement for CTi. Distal Patella circumference can be used in place of M/L Width. Circumference measurements are approximate.

ACCESSORIES & REPLACEMENT PARTS

CTi3 mx kit - Includes Impact Guard, Sleeve with Mesh, CTi Decal Sheet, and CTi Product Bag.

Size	Right	Left
S	CTI-22132	CTI-23132
M	CTI-22133	CTI-23133
L	CTI-22134	CTI-23134
XL	CTI-22135	CTI-23135

LINER SET

Size	Right	Left
S	CTI-22322	CTI-23322
M	CTI-22323	CTI-23323
L	CTI-22324	CTI-23324
XL	CTI-22325	CTI-23325

HOOK SET

Size	Right	Left
S	CTI-22332	CTI-23332
M	CTI-22333	CTI-23333
L	CTI-22334	CTI-23334
XL	CTI-22335	CTI-23335

STRAP SET

Size	Part#
S	CTI-24222
M	CTI-24223
L	CTI-24224
XL	CTI-24225

MISCELLANEOUS

Description	Part#
Guard Elastic Cord	CTI-24130
Guard Screws	CTI-24140
Tibial Wedge Pad	22077
Condyle Pad Set (Thin, Standard, Thick)	CTI-24320
Extension Stop Set (0°, 5°, 10°, 20°, 30°)	CTI-24430
Flexion Stop Set (0°, 60°, 75°, 90°)	CTI-24440
Hinge Sticker Set	CTI-24450
D-Ring Set	CTI-24510
CTi3 PCL Strap Kit	CTI-24520

SLEEVE WITH MESH

Size	Part#
S	SLV60012
M	SLV60013
L	SLV60014
XL	SLV60015

CTi® CUSTOM

Introduced in 1983, CTi remains the cornerstone of our ligament bracing product line today. While the original CTi has evolved over time, our basic philosophy remains unchanged – a static support system consisting of a rigid frame will hold the bones in proper alignment and provide high level of control to prevent displacement. The constant development and improvement of this classic design is why the CTi remains the top choice of athletes and leading physicians around the world.

The rigid, carbon composite CTi knee brace frame captures the tibia, providing the brace-to-bone contact needed for maximum stabilisation of the knee joint. The non-elastic cruciate strapping combined with a rigid frame that truly captures the tibia, results in a static support system.

The CTi Custom is generally used for high levels of support where comfort and strength are not compromised with weight and bulkiness. It can also offer greater advantages for accurate fitting where leg shape abnormalities present a problem.



INDICATIONS FOR USE

- For knee conditions that may benefit from increased AP and ML stability around the knee, such as: ACL, MCL, LCL, PCL, rotary and combined instabilities

FEATURES AND BENEFITS

- Anatomically-correct Accutrac® hinges with extension stops
- Proprietary Sensil® silicone liners designed to reduce migration
- Total Support System™ for ACL, MCL, LCL, PCL, rotary and combined instabilities
- Pre-preg carbon fiber, rigid frame construction
- Non-corrosive materials, ideal for water sports
- Custom paint option available
- Standard model for moderate to high impact levels
- Pro Sport model for the highest impact levels
- Vapor model for low to moderate impact levels
- Available in 12 colours

REQUIRED MEASUREMENTS

- Custom-made to measure

CUSTOM STANDARD COLOURS



CUSTOM PAINTED DESIGNS



No charge for custom standard colours. Additional fees required for other custom colours, custom painted designs and custom printed graphics.

CTI® CUSTOM

ORDERING INFORMATION

CTI CUSTOM

Description	Left	Right
Standard	B-116600010	B-116500010
Pro Sport	B-117600010	B-117500010
Vapor	B-118600010	B-118500010

SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required

ACCESSORIES

CALF LINERS

Description	Part#
CTi Custom liner	B-411500115

STRAPS KITS

Size	Part#
Small, Medium, Large	B-701130104
X-Large, XX-Large, XXX-Large, XXXX-Large	B-701130108

PCL KITS

Size	Part#
Small, Medium	B-705000132
Large, X-Large	B-705000235

CTI® CUSTOM

ACCESSORIES

STRAP PADS*

Size	Part#
Small	B-304010057
Medium, Large	B-304010058
X-Large, XX-Large	B-304010059

*Must also order hook-and-loop B-309020017

AMS (ANTI-MIGRATION SYSTEM) WRAPS

Description	Part#
AMS Wrap EVA	22095
AMS Wrap Neoprene SM-LG (up to 17" in circumference at 3" below knee)	22094
AMS Wrap Neoprene XL (above 17" in circumference at 3" below knee)	22152

MX GEAR

Description	Part#
Patella Cup	B-711130011
Gear Guards	B-705000003
MX Kit S (patella cup, gear guards, undersleeve, stickers)	B-711140002
MX Kit M (patella cup, gear guards, undersleeve, stickers)	B-711140003
MX Kit L (patella cup, gear guards, undersleeve, stickers)	B-711140004
MX Kit XL (patella cup, gear guards, undersleeve, stickers)	B-711140005

SLEEVES

Size	Circumference 15 cm above mid patella	Softsleeve	Neoprene Undersleeve	Neoprene Oversleeve	Sportsleeve	Padded Sports Oversleeve
Small	38 cm	20643	20251	20051	20682	20448
Medium	38 – 45 cm	20644	20252	20052	20683	20448
Large	45 – 49.5 cm	20644	20253	20053	20684	20449
X-Large	49.5 – 54.5 cm	20645	20254	20054	20685	20449
XX-Large	54.5 – 62 cm	20645	20255	20055	N/A	
XXX-Large	62 – 71 cm	20646	20256	20056	N/A	
XXXX-Large	71 – 86 cm	20647	20257	20057	N/A	

OTHER ACCESSORIES

Description	Part#
CTi D-Ring Kit Standard 2x 1" and 2x 1.5" D-rings	B-704132600
CTi D-Ring Kit 2" 2x 2" D-rings	B-704132602
CTi Condyle Pad Kit	B-704130150
Add ACL Repair kit	B-705130041
Extension Stop Kit	22130
Flexion Stop Kit (Factory Installed)	19254
Extension strap Kit (Factory Installed)	21144

C180™ ROCKET

INDICATIONS FOR USE

- ACL, MCL, LCL, PCL, rotary and combined instabilities
- Recommended for low to medium contact/impact
- Recommended for pediatric usage

FEATURES AND BENEFITS

- Anatomically-correct Accutrac® hinges with easy-to-use extension stops
- Heat-Moldable frame for customization
- Adjustable Tibial Strap captures the tibia and allows adjustments in hinge depth for a more customised fit
- Available with an optional patella cup
- Total Support System™ for ACL, MCL, LCL, PCL, rotary and combined instabilities
- Semi-rigid frame consisting of woven carbon fiber infused with acrylic resin
- Lightweight, streamlined design for patient comfort and compliance
- Non-corrosive materials, ideal for water sports



REQUIRED MEASUREMENTS

- Caliper measurement: Medial-lateral at joint width in a standard, weight-bearing position
- Circumference: Just below the distal border of the patella

ORDERING INFORMATION

Size	Caliper	Circumference	Left	Right
Small	7.6 – 8.9 cm	24.1 – 27.9 cm	C180KSL	C180KSR
Large	8.3 – 9.5 cm	26 – 29.8 cm	C180KLL	C180KLR
X-Large	8.9 – 10.2 cm	27.9 – 32.1 cm	C1802XSL	C1802XSR

ACCESSORIES

Description	Part#	
Motocross kit Small / Large	10833	
Motocross kit X-Large	B-705130072	
Patella Cup Small / Large	25029	
Patella Cup X-Large	B-711130000	
Calf Liner Small	25061 (L)	26062 (R)
Calf Liner Large	25057 (L)	26058 (R)
Calf Liner X-Large	25106 (L)	25107 (R)
Thigh Liner Small	25060 (L)	26063 (R)
Thigh Liner Large	25056 (L)	26059 (R)
Thigh Liner X-Large	25104 (L)	25105 (R)

REBOUND® CARTILAGE

The success of an articular cartilage repair is dependent on patient compliance to the rehabilitation process. The protocol is based on restricted cartilage load combined with gradual increase in activity. Many rehabilitation protocols suggest crutches for partial weight-bearing and it is the standard of care for successful outcomes.

However, patient compliance to partial weight-bearing instructions with crutches has been shown to be inconsistent. The Rebound Cartilage is a protective functional solution designed to support regeneration of the knee cartilage after repair procedures such as Microfracture, OATS and (M)ACI. Based on the clinically-proven 3-Point Leverage System and proprietary Cartilage Protection Straps, joint unloading is maintained in flexion for protection of the cartilage during the healing process.



INDICATIONS FOR USE

For knee conditions that may benefit from unicompartmental load reduction or range of movement restriction, such as:

- Articular cartilage defect repair
- Meniscal repair
- Avascular necrosis
- Condylar bone marrow lesions (bone bruises)
- Tibial plateau fractures
- Mild to severe unicompartmental knee osteoarthritis

FEATURES AND BENEFITS

- Easy-to-Stock - Just 4 configurations to fit most of your patients
- Comfortable - Skin-friendly liner feels great against the skin and reduces pressure points
- Anti-Migration Technology - Strap design captures the thickest part of the calf to reduce brace migration during activity
- Control - SmartDosing® dials put you in control of your pain by allowing you to fine-tune strap tension, while Cartilage Protection Straps maintain joint unloading in flexion
- Tool-Free - Tool-free polycentric hinge with optional ROM stops for range-of-motion restriction

REQUIRED MEASUREMENTS

- Circumference measurement: 15 cm below mid-patella

REBOUND® CARTILAGE

ORDERING INFORMATION

REBOUND CARTILAGE

Size	Circumference	Left medial	Right medial	Left lateral	Right lateral
Small	31 - 35.5cm	B-254610002	B-254510002	B-254620002	B-254520002
Medium	35.5 - 42.5cm	B-254610003	B-254510003	B-254620003	B-254520003
Large	42.5 - 51cm	B-254610004	B-254510004	B-254620004	B-254520004
X-Large	51 - 61.5cm	B-254610005	B-254510005	B-254620005	B-254520005

ROM STOP KIT INCLUDED WITH BRACE:

Flexion stops: 0°/30°/60°/90°

Extension stops: 0°/5°/10°/15°/30°

REBOUND® PCL

The PCL, compared to the ACL, has the intrinsic ability to heal and regain continuity after an injury. The PCL experiences variable tensile forces during knee flexion. Insufficient healing after a PCL rupture leads to significant pathologies, such as, chronic anterior knee pain or knee OA. Healing of the PCL in an elongated position can lead to chronic instability and disability. Therefore, physiologically correct positioning of the PCL during conservative or post-surgical rehabilitation is important.

The custom made Rebound PCL is the world's first Dynamic Force PCL brace. It applies a physiologically correct dynamic force to the PCL during rehabilitation.

Our expertise in Injury Solutions has driven us to take the next step forward by developing indication-specific products which are designed to optimise healing while maintaining function and mobility.

FUNCTIONAL HEALING – WHERE MOBILITY MEANS RECOVERY

INDICATIONS FOR USE

- Non-surgical treatment of PCL ruptures
- Post-surgical rehabilitation for:
- Partial PCL tears
- PCL reconstruction / augmentation
- PCL revision

FEATURES AND BENEFITS

- Dynamic Tension System (DTS) is designed to increase the load on the tibia as the knee goes through flexion, providing an anterior drawer force necessary to decrease load on the healing PCL to allow it to heal
- 3 options to customize the standardized, biochemically correct posterior tibial load for each patient by using shear knobs And having the possibility to re-adjust the posterior tibial load, e.g. in case of swelling
- Lightweight brace with proprietary Sensil® silicone calf liner and doeskin thigh liner reduce migration while providing comfort to the patient
- Custom molded posterior frame for optimised fit and support, with open anterior for ease of donning/doffing
- Available in 12 colours

REQUIRED MEASUREMENTS

- Custom-made to measure

STANDARD COLOURS



REBOUND® PCL

ORDERING INFORMATION

REBOUND PCL PRE-FABRICATED

Size	Caliper M/L*	Calf Circ.**	Thigh Circ.***	Left	Right
Small	8.9 - 10.2cm	29 - 34.5cm	42 - 45cm	B-124501161	B-124501158
Medium	10.2 - 11.4cm	34.5 - 38cm	45 - 51cm	B-124501162	B-124501159
Large	11.4 - 12.7cm	38 - 42cm	51 - 58.5cm	B-124501163	B-124501160

* M/L measurement: In a standard, weight bearing position - Using a Caliper: M/L at joint width, Using a Constant Force Caliper: M/L at knee axis line

** Calf circumference measurement: 15cm below knee centre

*** Thigh circumference measurement: 15cm above knee centre



SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.

Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required

REBOUND PCL CUSTOM

Description	Part#
Rebound PCL Left	B-124501155
Rebound PCL Right	B-124501154

REBOUND® PCL

ACCESSORIES

STRAPS

Size	Part#
X-Small – Large	PCL00061
X-Large – XX-Large	PCL00062

OTHER ACCESSORIES

Description	Part#
Extension Stop Kit	B-705130042
Flexion Stop Kit	B-705130043
Shear Knob Kit	B-465350087
Condyle Pad Kit	B-705130046

ROM STOP KIT INCLUDES:

Flexion stops: 0°/15°/25°/35°/45°/55°/65°/75°/90° Extension stops: 0°/5°/10°/20°/30°/40°

REBOUND PCL STANDARD UPPER LINER KIT

Size	Left	Right
S	B-124501171	B-124501165
M	B-124501172	B-124501166
L	B-124501173	B-124501167

REBOUND PCL STANDARD LOWER LINER KIT

Size	Left	Right
S	B-124601177	B-124501177
M	B-124601178	B-124501178
L	B-124601179	B-124501179

REBOUND PCL DOESKIN LOWER LINER KIT

Size	Left	Right
S	B-124601183	B-124501183
M	B-124601184	B-124501184
L	B-124601185	B-124501185

REBOUND® ACL

The Rebound ACL brace is an evolution in bracing, designed to comfortably apply an anatomically correct, dynamic load to the anterior cruciate ligament (ACL). Whether during conservative or surgical management, the Rebound ACL compliments protocols and supports an optimal biomechanical joint environment as a foundation for long-term successful patient outcomes. Innovative technology in the Dynamic Tension System® (DTS) allows individualised, adjustable settings to reduce strain on the healing ACL.



FUNCTIONAL HEALING – WHERE MOBILITY MEANS RECOVERY

INDICATIONS FOR USE

INDICATIONS

Conditions benefiting from an agonistic force that varies throughout flexion in proportion to ACL tension applied to the thigh and calf in the sagittal plane. These may include:

- Anterior cruciate ligament (ACL) injury
- Reconstruction
- Non-surgical treatment

CONTRAINDICATIONS

None known

INTENDED USE

Post-injury, post-surgery, and conservative rehabilitation

TARGET GROUP


Patients requiring a dynamic knee orthosis who have suffered an ACL or combined instability, and are being treated post-injury with conservative or post-surgical management

FEATURES AND BENEFITS


- Dynamic technology for knee stability, with a Dynamic Tension System™ that provides an agonistic, dynamic force to the ACL that is flexion-angle dependent (LaPrade et al., 2017) and designed to reduce anterior-drawer, promoting knee stability
- Specific load adaptation according to the patient's individual anatomy and rehabilitation requirements
- Supported by clinical & biomechanical studies, the Rebound ACL is shown to reduce strain on the ACL during activity and significantly reduce strain on the meniscus in ACL-deficient knees (Tomescu et al., 2017)
- Biocompatible, hi-tech materials including Blue is You™ patient touch points & ActiveGrip™ liners promote suspension & comfort
- Lightweight, aircraft aluminum frame
- Available in sizes SM-XL
- Flexion & extension range-of-motion lock-outs
- Posterior frame optimised for fit and support, with open anterior for ease of donning/doffing
- Clinically-tested comfort, validated by end-users and with load mapping data collected during rehabilitation activities, the Rebound® Dynamic Tension System (DTS) provides a comfortable load to the posterior thigh & anterior tibia

REBOUND® ACL


REQUIRED MEASUREMENTS

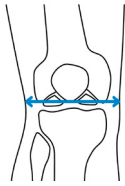

Measure knee caliper medio-laterally at joint width in standing, weight-bearing position.

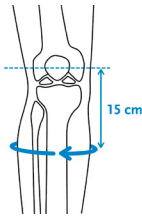

Measure calf circumference 15 cm below mid-patella. Leg in extension.

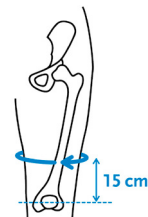

Measure thigh circumference 15 cm above mid-patella. Leg in extension.

ORDERING INFORMATION

STANDARD

Size	Left	Right	Caliper	Calf circumference 15 cm below mid patella	Thigh circumference 15 cm above mid patella
SM	B-124601191	B-124501191	8.9-10.2 cm	29.2-34.3 cm	41.9-47.0 cm
MD	B-124601192	B-124501192	10.2-11.4 cm	34.3-39.4 cm	47.0-52.1 cm
LG	B-124601193	B-124501193	11.4-12.7 cm	39.4-44.5 cm	52.1-57.2 cm
XL	B-124601194	B-124501194	12.7-14.0 cm	44.5-49.5 cm	57.2-62.2 cm

Recommended for patients 1.56m and taller

REPLACEMENT PARTS

Description	Part#
Extension stop kit (0°, 5°, 10°, 20°, 30°, 40°)	B-124702035
Flexion stop kit (0°, 15°, 25°, 35°, 45°, 55°, 65°, 75°, 90°, 105°)	B-124702034
Shear Knob Kit	B-705130051
SM/MD Strap Kit	B-124702030
LG/XL Strap Kit	B-124702031
Condyle Pad Kit	B-701090009
Tibial Pad	B-124702032
Tibial Hex Pad	B-124702033

REBOUND® ACL

ACCESSORIES

ANTI-MIGRATION WRAP

Size	Right	Left
S/M	B-124501195	B-124601195
S/M HEX	B-124501196	B-124601196
L/XL	B-124501197	B-124601197
L/XL HEX	B-124501198	B-124601198

UPPER DOESKIN LINER KIT

Size	Right	Left
S	B-124702004	B-124702000
M	B-124702005	B-124702001
L	B-124702006	B-124702002
XL	B-124702007	B-124702003

LOWER ACTIVEGRIP LINER KIT

Size	Right	Left
S	B-124702014	B-124702010
M	B-124702015	B-124702011
L	B-124702016	B-124702012
XL	B-124702017	B-124702013

LOWER DOESKIN LINER KIT

Size	Right	Left
S	B-124702024	B-124702020
M	B-124702025	B-124702021
L	B-124702026	B-124702022
XL	B-124702027	B-124702023

REBOUND® ACL CUSTOM

The Rebound ACL brace is an evolution in bracing, designed to comfortably apply an anatomically correct, dynamic load to the anterior cruciate ligament (ACL). Whether during conservative or surgical management, the Rebound ACL compliments protocols and supports an optimal biomechanical joint environment as a foundation for long-term successful patient outcomes. Innovative technology in the Rebound® Dynamic Tension System (DTS) allows individualised, adjustable settings to reduce strain on the healing ACL.

FUNCTIONAL HEALING – WHERE MOBILITY MEANS RECOVERY



INDICATIONS FOR USE

INDICATIONS

Conditions benefiting from an agonistic force that varies throughout flexion in proportion to ACL tension applied to the thigh and calf in the sagittal plane. These may include:

- Anterior cruciate ligament (ACL) injury
- Reconstruction
- Non-surgical treatment

CONTRAINDICATIONS

None known

INTENDED USE

Post-injury, post-surgery, and conservative rehabilitation

TARGET GROUP

Patients requiring a dynamic knee orthosis who have suffered an ACL or combined instability, and are being treated post-injury with conservative or post-surgical management

FEATURES AND BENEFITS

- Dynamic technology for knee stability, with a Dynamic Tension System™ that provides an agonistic, dynamic force to the ACL that is flexion-angle dependent (LaPrade et al., 2017) and designed to reduce anterior-drawer, promoting knee stability
- Specific load adaptation according to the patient's individual anatomy and rehabilitation requirements
- Supported by clinical & biomechanical studies, the Rebound ACL is shown to reduce strain on the ACL during activity and significantly reduce strain on the meniscus in ACL-deficient knees (Tomescu et al., 2017)
- Biocompatible, hi-tech materials including Blue is You™ patient touch points & ActiveGrip™ liners promote suspension & comfort
- Lightweight, aircraft aluminum frame
- Available in sizes SM-XL & Custom
- Flexion & extension range-of-motion lock-outs
- Posterior frame optimised for fit and support, with open anterior for ease of donning/doffing
- Clinically-tested comfort, validated by end-users and with load mapping data collected during rehabilitation activities, the Rebound® Dynamic Tension System (DTS) provides a comfortable load to the posterior thigh & anterior tibia

REBOUND® ACL CUSTOM

SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required.

REQUIRED MEASUREMENTS

- Calf circumference 15 cm below mid patella
- Calf circumference 20 cm below mid patella
- Calf circumference 7.5 cm below mid patella
- Knee caliper
- Thigh circumference 15 cm above mid patella
- Thigh circumference 20 cm above mid patella
- Thigh circumference 7.5 cm above mid patella

ORDERING INFORMATION

CUSTOM

Part #	Type	Min / Max Caliper	Min / Max Circ. above	Min / Max Circ. below	Measuring
B-124701000	Right	8.9 – 14 cm	41.9 - 62.2 cm	29.2 - 49.5 cm	SmartMeasure, Cast
B-124701001	Left				

Caliper = M/L width at knee center
Circumference = 15 cm above / below mid-patella
Ideal for patients 1.56m and taller

REPLACEMENT PARTS

Description	Part#
Extension stop kit (0°, 5°, 10°, 20°, 30°, 40°)	B-124702035
Flexion stop kit (0°, 15°, 25°, 35°, 45°, 55°, 65°, 75°, 90°, 105°)	B-124702034
Shear Knob Kit	B-705130051
SM/MD Strap Kit	B-124702030
LG/XL Strap Kit	B-124702031
Condyle Pad Kit	B-701090009
Tibial Pad	B-124702032
Tibial Hex Pad	B-124702033

REBOUND® ACL CUSTOM

ACCESSORIES

ANTI-MIGRATION WRAP

Size	Right	Left
S/M	B-124501195	B-124601195
S/M HEX	B-124501196	B-124601196
L/XL	B-124501197	B-124601197
L/XL HEX	B-124501198	B-124601198

UPPER DOESKIN LINER KIT

Size	Right	Left
S	B-124702004	B-124702000
M	B-124702005	B-124702001
L	B-124702006	B-124702002
XL	B-124702007	B-124702003

LOWER ACTIVEGRIP LINER KIT

Size	Right	Left
S	B-124702014	B-124702010
M	B-124702015	B-124702011
L	B-124702016	B-124702012
XL	B-124702017	B-124702013

LOWER DOESKIN LINER KIT

Size	Right	Left
S	B-124702024	B-124702020
M	B-124702025	B-124702021
L	B-124702026	B-124702022
XL	B-124702027	B-124702023

FORMFIT® MCL

INDICATIONS FOR USE

- Mild to moderate Medial Collateral Ligament (MCL) sprains (Grade I and Grade II)

FEATURES AND BENEFITS

- Single, steel ROM lateral hinge (0° or 20° extension stop, 70° flexion stop)
- Dynamic Force Strap resist valgus forces
- Hybrid wrap/sleeve for easy application
- Adjustable tibial strap for improved suspension

REQUIRED MEASUREMENTS

- Circumference: 15 cm above mid-patella



ORDERING INFORMATION

Size	Circumference	Left	Right
X-Small	34.3 – 39.4 cm	B-237600001	B-237500001
Small	39.4 – 44.5 cm	B-237600002	B-237500002
Medium	44.5 – 52.1 cm	B-237600003	B-237500003
Large	52.1 – 59.7 cm	B-237600004	B-237500004
X-Large	59.7 – 67.3 cm	B-237600005	B-237500005

REBOUND® DUAL BASIC

The Rebound DUAL Basic offers the value and simplicity you need to navigate the complex coverage landscape. Within the versatility of the Rebound DUAL collection, Össur offers high-quality solutions from a single source, to help users transition to a successful recovery without compromise.



INDICATIONS FOR USE

The Rebound DUAL Basic is indicated for combined and non-combined instabilities of the: Anterior cruciate ligament (ACL) Medial collateral ligament (MCL) Lateral collateral ligament (LCL) Posterior cruciate ligament (PCL) Acculign version only: Conditions requiring unicompartmental load control

FEATURES AND BENEFITS

- Key features
- Multi-purpose length-adjustable upper frame accommodates the user's height, helping to ensure the perfect fit, and enabling adjustment for varus/valgus alignment
- Össur Hinge alignment guide provides easy & accurate re-fitting for users, ensuring correct placement of the brace relative to the knee
- Lightweight aircraft aluminum frame keeps a low profile
- Doeskin liners for comfort & no-slip Activegrip™ strap pad
- Trimmable straps with tactile strap tabs
- Low-profile polycentric hinge with stop options for stability & range-of-motion restriction
- Step-through or Anterior frame design
- Suitable for low-high impact and sliding sports
- Optional PCL kit accessory for multi-ligament support
- Available in pre-fabricated sizes XS-3XL

REQUIRED MEASUREMENTS

- Knee caliper
- Thigh circumference 15 cm above mid patella

REBOUND® DUAL BASIC

ORDERING INFORMATION

Size	Caliper	Thigh circumference	Right	Left
XS	7.6 - 8.9cm	38.1 - 41.9cm	B-242510001B	B-242610001B
S	8.9 - 10.2cm	41.9 - 47cm	B-242510002B	B-242610002B
M	10.2 - 11.4cm	47 - 52.1cm	B-242510003B	B-242610003B
L	11.4 - 12.7cm	52.1 - 57.2cm	B-242510004B	B-242610004B
XL	12.7 - 14cm	57.2 - 62.2cm	B-242510005B	B-242610005B
XXL	14 - 15.2cm	62.2 - 69.9cm	B-242510006B	B-242610006B
3XL	15.2 - 16.5cm	69.9 - 76cm	B-242510007B	B-242610007B

LINER KITS

Measuring	Basic Liner Part #
XS	B-701090110
S	B-701090111
M	B-701090112
L	B-701090113
XL	B-701090114
XXL	B-701090115
3XL	B-701090116

REPLACEMENT PARTS

Size	Part #
RDUAL Basic Strap Kit XS-L	B-701090105
RDUAL Basic Strap Kit XL-3XL	B-701090106
Condyle Pad Kit	B-701090009

ACCESSORIES

Size	Part #
RDUAL Basic ROM Stop Kit	B-701090109
RDUAL Basic PCL Strap Kit XS-L	B-701090057
RDUAL Basic PCL Strap Kit XL-3XL	B-701090058

ROM STOP KIT INCLUDES:

Flexion stops: 0°/10°/20°/30°/45°/60°/75°/90°

Extension stops: 0°/5°/10°/20°/30°/45°

REBOUND® POST OP KNEE

The simple, versatile post-operative knee bracing solution that is easy for everyone. Whether adjusting ROM, setting the drop lock, or altering the brace length, you'll appreciate the simplicity of the Rebound Post-op knee brace.

INDICATIONS FOR USE

- ACL, PCL, MCL, LCL repairs/injuries
- Meniscal repairs
- Tibial plateau fractures
- Patellar tendon repairs
- Osteochondral repairs
- Condylar fractures
- Sprains/strains of the knee
- HTOs (High Tibial Osteotomy)

FEATURES AND BENEFITS

- Tool-free design and multi-purpose Smart-Fit™ packaging
- Available in universal, left or right contour and pediatric
- Telescoping frame from 18.5"-27" (Adult) and 15"-21" (Pediatric)
- OneTouch™ easy to adjust ROM hinge with lock-out capability
- Color-coding and numbered frame for patient education
- Quick-fit, gripping buckles for easy application



REQUIRED MEASUREMENTS

Circumference: 15 cm above mid-patella

ORDERING INFORMATION

REBOUND POST-OP KNEE

Description	Thigh circumference	Cool	Full Foam
Pediatric	30.5 cm – 63.5 cm	B-8381470	
Universal, Left contoured	41 cm – 81 cm	B-8381680	B-8382680
Universal, Right contoured	41 cm – 81 cm	B-8381580	B-8382580
Universal	41 cm – 81 cm	B-8381480	B-8382480
Universal, no box	41 cm – 81 cm	B-8385480	
Universal XL	41 cm – 96.5 cm	B-8381490	B-8382490

REBOUND® POST OP KNEE

ORDERING INFORMATION

ACCESSORIES

Description	Part #
ROM stop locks	B-8380005
Alligator hook and loop kit	B-8380006
Adhesive hook and loop kit pediatric	B-8380008
Adhesive hook and loop kit universal / universal XL	B-8380007
Anti-migration Strap kit pediatric	B-8381093
Anti-migration Strap kit Universal*	B-8381094
Anti-migration Strap kit XL	B-8381095
Standard buckle & pad kit	B-8381096
Incision pad kit 10pk	B-7031805

REPLACEMENT SOFTGOODS

Description	Cool	Full Foam
Pediatric	B-8381072	
Universal (*For universal, left contoured, right contoured)	B-8381082	B-8382082
Universal XL	B-8381092	B-8382092

FORMFIT® POST-OP KNEE

Formfit Post-Op Knee provides range of motion control and immobilisation following injury or surgery. Length adjustment is tool free and easy. The low profile ROM hinge can be adjusted with a simple pull, rotate and release motion. Wide range of flexion and extension settings allow you to choose the optimal range of motion.

INDICATIONS FOR USE

- Post-op/post-injury ROM control or immobilisation. Indicated for ligament and fracture-related surgeries and when controlled range of motion is necessary

FEATURES AND BENEFITS

- Breakaway struts for easy, tool free length adjustment
- OneTouch™ easy to adjust ROM hinge with lock-out capability from -10° to 30° (10° incremental)
- Extension control from -10° to 90° (10° incremental)
- Flexion control from 0° to 120° (10° incremental)
- Available in 3 models, 3 lengths, and cool or full foam



REQUIRED MEASUREMENTS

- Thigh circumference 15 cm above mid patella

ORDERING INFORMATION

Size	Thigh circumference	Cool	Full Foam
Knee PO 24in DL	up to 69 cm	FPOK403100	FPOK403200
Knee PO XL 24in DL	69-89 cm	FPOK403102	FPOK403202
Knee PO 28in DL	up to 69 cm	FPOK403101	FPOK403201
Knee PO XL 28in DL	69-89 cm	FPOK403103	FPOK403203
Knee PO 16in DL	up to 69 cm	-	FPOK403300
Knee PO XL 16in DL	69-89 cm	-	FPOK403301

SOFT GOODS

Description	Cool	Full Foam
Replacement Softgood, Regular	403175	403275
Replacement Softgood, XL	403176	403277
Replacement Softgood, Regular Long	403276	403278

REPLACEMENT SOFTGOODS

Size	Part#
ROM stop locks (set of 20 lock clips)	B-8380005
Strap Reduction Kit Cool	403199
Strap Reduction Kit Full Foam	403299

FORMFIT® KNEE ROM

The Formfit® Knee brace is for patients suffering from mild to moderate strains, sprains or ligament tears (ACL/PCL/MCL/LCL). It provides excellent stability, protection and compression during injury recovery. Made from strong and breathable materials it offers comfort while remaining highly durable. The open popliteal area reduces material bulging and potential irritation, while the malleable rigid hinge arms provide an intimate fit.

The Formfit Knee brace comes in either a wrap or a sleeve version.



INDICATIONS FOR USE

- Mild to moderate ACL and PCL tears
- MCL and LCL tears
- Combined mild knee sprains and strains
- Instabilities of the knee

FEATURES AND BENEFITS

- Breathable material
- Easy pull-on
- Malleable hinges
- Open popliteal
- Flexion stops: 0°, 45°, 75°, 90°
- Extension stops: 0°, 20°, 30°, 40°, lockout at full extension

REQUIRED MEASUREMENTS

- Measure thigh circumference 15 cm above mid-patella

ORDERING INFORMATION

Size	Circumference	Sleeve		Wrap	
		Long ROM Sleeve	Short ROM Sleeve	Long ROM Wrap	Short ROM Wrap
XSmall	33 – 39 cm	508352	508252	507352	507252
Small	39 – 47 cm	508353	508253	507353	507253
Medium	47 – 53 cm	508355	508255	507355	507255
Large	53 – 60 cm	508357	508257	507357	507257
XLarge	60 – 67 cm	508358	508258	507358	507258
XXLarge	67 – 75 cm	508359	508259	507359	507259
XXXLarge	75 – 81 cm			507360	507260

ACCESSORIES

Description	Part#
Universal Buttress	503000

REBOUND® KNEE IMMOBILISER UNIVERSAL

Rebound® Knee Immobiliser Universal features a universal 3-panel design that can be adjusted to accommodate most leg circumferences. With an MR* Conditional rating, patients can be safely scanned in an MR system, while maintaining immobilisation.

*See IFU for details.

INDICATIONS FOR USE

- Emergency immobilisation
- Pre and post-surgical knee immobilisation
- Traumatic knee injuries

FEATURES AND BENEFITS

- Adjustable medial/lateral stays and rigid posterior stays provide motion restriction of the knee
- Universal 3-piece design simplifies inventory needs
- MR* Conditional: safely scan patients in an MR system while maintaining immobilisation
- Min thigh circumference: 26 cm
- Max thigh circumference: 67.31 cm



ORDERING INFORMATION

Length	Part#
30 cm	210012
35 cm	210014
40 cm	210016
45 cm	210018
50 cm	210020
55 cm	210022
60 cm	210024

ACCESSORIES

Description	Part#
Hot/Cold Therapy Gel Pad	GB7

EXOFORM® KNEE IMMOBILISER

INDICATIONS FOR USE

- Post-injury
- Post-trauma

FEATURES AND BENEFITS

- Slide-to-size adjustment enables you to provide the right fit for the largest variety of patient lengths and widths
- Provides superior fit and limits migration
- Cool foam version can be worn directly against the skin for greater comfort on warm days
- Full foam version provides warmth and compression



REQUIRED MEASUREMENTS

- Universal – fits leg inseams of 46-61 cm

ORDERING INFORMATION

Description	Leg circumference	Part#
Cool	up to 71.1 cm	222100
Full foam	up to 71.1 cm	222000
Full foam XXXL	up to 96.5 cm	222050

ACCESSORIES

Description	Part#
Replacement softgood, Cool	222101
Replacement softgood, Full Foam	222001
Replacement softgood, Full Foam, 3XL	222051

REBOUND® KNEE BRACE

With six variations to choose from, Össur's Rebound hinged knee brace product line is a comprehensive portfolio of high-quality braces designed to serve the needs of a broad range of patients.

INDICATIONS FOR USE

- ROM Hinge Models: Mild to moderate ACL and / or PCL, MCL and LCL instabilities, sprains and strains
- Non-ROM Hinge Models: Mild medial / lateral knee instabilities

FEATURES AND BENEFITS

- Highly breathable aerospacer material reduces heat retention and improves comfort
- Low profile hinges with extension and flexion stop kit options (ROM version) and malleable arms to provide a custom fit along with knee movement control
- Contoured pre-tensed Cooltech™ fabric designed to reduce irritation and material bulging in the popliteal area
- Removable straps allow application from right or left side, and strap pull-tab allows for consistent donning by patient
- Universal buttresses are included with all sleeve versions for customisable patella support



REQUIRED MEASUREMENTS

- Thigh Circumference: 15 cm above mid-patella

ORDERING INFORMATION

REBOUND ROM KNEE

Size	Circumference	Sleeve Long	Sleeve Short	Wrap Long	Wrap Short
X-Small	29 – 35 cm	708152	708052	707152	707052
Small	35 – 41 cm	708153	708053	707153	707053
Medium	41 – 46 cm	708155	708055	707155	707055
Large	46 – 52 cm	708157	708057	707157	707057
X-Large	52 – 57 cm	708158	708058	707158	707058
XX-Large	57 – 63 cm	708159	708059	707159	707059
XXX-Large	63 – 72 cm	708160	708060	707160	707060
XXXX-Large	72 – 81 cm			707161	707061

REBOUND KNEE

Size	Circumference	Sleeve Short	Wrap Short
X-Small	29 – 35 cm	704052	703052
Small	35 – 41 cm	704053	703053
Medium	41 – 46 cm	704055	703055
Large	46 – 52 cm	704057	703057
X-Large	52 – 57 cm	704058	703058
XX-Large	57 – 63 cm	704059	703059
XXX-Large	63 – 72 cm	704060	703060
XXXX-Large	72 – 81 cm		703061

FORMFIT® KNEE, HINGED

The Formfit® Knee brace is for patients suffering from mild to moderate strains, sprains or ligament tears (ACL/PCL/MCL/LCL). It provides excellent stability, protection and compression during injury recovery. Made from strong and breathable materials it offers comfort while remaining highly durable. The open popliteal area reduces material bulging and potential irritation, while the malleable rigid hinge arms provide an intimate fit.

The Formfit Knee brace comes in either a wrap or a sleeve version.



INDICATIONS FOR USE

- Mild to moderate ACL and PCL tears
- MCL and LCL tears
- Combined mild knee sprains and strains
- Instabilities of the knee

FEATURES AND BENEFITS

- Breathable material
- Easy pull-on
- Malleable hinges
- Universal buttress
- Open popliteal

REQUIRED MEASUREMENTS

- Measure thigh circumference 15cm above mid-patella

ORDERING INFORMATION

Size	Circumference	Sleeve	Wrap
XSmall	33 – 39 cm	504252	503252
Small	39 – 47 cm	504253	503253
Medium	47 – 53 cm	504255	503255
Large	53 – 60 cm	504257	503257
XLarge	60 – 67 cm	504258	503258
XXLarge	67 – 75 cm	504259	503259
XXXLarge	75 – 81 cm		503260

ACCESSORIES

Description	Part#
Formfit Knee Brace Buttress (Sleeve only)	503000

FORMFIT® TRACKER

The Formfit® Tracker with micro-adjustable ReflexWing™ buttress realigns the patella, providing dynamic lateral support for common patella conditions. Its FlyWeight™ fabric delivers lightweight breathability and a low-profile design that feels great against the skin and minimises migration. PowerLock™ straps work in tandem with heat moldable CustomFit™ hinges to provide a customisable fit for your patient.



INDICATIONS FOR USE


For knee conditions that may benefit from realignment of the patella and/or patients that would benefit from an added lateral support of the patella, such as:

- lateral patellar subluxation or dislocation
- lateral maltracking


FEATURES AND BENEFITS

- CustomFit™ hinges for customised fit and support
- Anti-Migration Tech™ minimises migration
- PowerLock™ straps for secure lateral support
- ReflexWing™ buttress for comfortable patella realignment
- FlyWeight™ fabric for soft feel and breathability
- Durable and machine washable

REQUIRED MEASUREMENTS

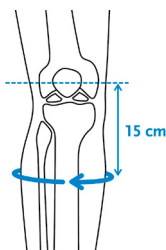

Measure calf circumference 15 cm below mid-patella. Leg in extension.

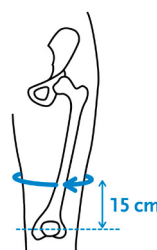

Measure thigh circumference 15 cm above mid-patella. Leg in extension.

FORMFIT® TRACKER

ORDERING INFORMATION

Size	Calf circumference 15 cm below mid patella	Thigh circumference 15 cm above mid patella	Left	Right
XS	25 - 30	33 - 39	719162	719152
S	30 - 35	39 - 47	719163	719153
M	35 - 40	47 - 53	719164	719154
L	40 - 46	53 - 60	719165	719155
XL	46 - 51	60 - 67	719166	719156
XXL	51 - 56	67 - 75	719167	719157
3XL	56 - 61	75 - 81	719168	719158

ACCESSORIES

Description	Part#
ReflexWing™ Butress	719100

NEOPRENE HINGED KNEE SUPPORT

INDICATIONS FOR USE

- Moderate ligament instabilities

FEATURES AND BENEFITS

- Accordion molding in the popliteal area reduces bunching behind the knee
- Contoured fit for additional comfort and protection
- The hinges are molded into the brace to maintain a low profile

REQUIRED MEASUREMENTS

- Knee circumference at mid-patella



ORDERING INFORMATION

Size	Knee circumference	Part#
Small	29 – 33 cm	302153BLK
Medium	33 – 37 cm	302155BLK
Large	37 – 41 cm	302157BLK
X-Large	41 – 44 cm	302158BLK
XX-Large	44 – 48 cm	302159BLK

NEOPRENE KNEE SUPPORT WITH STABILISED PATELLA

INDICATIONS FOR USE

- For patients suffering from knee conditions that may benefit from additional knee and patella support such as patellar tendonitis, subluxation dislocation, post-operation rehabilitation, post arthroscopic surgery, condromalacia, bursitis, arthritis and tendonitis

FEATURES AND BENEFITS

- Molded hardware to maintain a low profile
- Accordion molding in the popliteal area eliminates bunching
- Soft cushioned liner to offer unequaled comfort
- Varied areas of thickness give additional protection
- Lycra® liner offers a breathable contact area for comfortable and extensive use

REQUIRED MEASUREMENTS

- Knee circumference at mid-patella

ORDERING INFORMATION

Size	Knee circumference	Part#
Small	29 – 33 cm	302563BLK
Medium	33 – 37 cm	302565BLK
Large	37 – 41 cm	302567BLK
X-Large	41 – 44 cm	302568BLK
XX-Large	44 – 48 cm	302569BLK



FORMFIT® PRO KNEE

Össur Formfit® Pro Knee is a compressive knee sleeve that features a SmartPad™ buttress for targeted support and cushioning. Our patented design realigns the patella, suitable for conditions such as Patella Tendinitis (Jumper's Knee) and Chondromalacia Patella. The dynamic, 3D MotionTech™ knit is supportive, meets certified medical grade compression requirements and has been proven to be superior to the leading brands at moisture wicking and breathability, keeping your knee cool during wear. Lightweight, removable spiral stays ensure optimum shape and fit, while distinct patella graphics make donning intuitive.



MotionTech™



CoolVent™



StayBilizer™



SmartPad™



INDICATIONS FOR USE

For knee conditions that may benefit from compression, such as:

- Patella tendinitis
- Chondromalacia Patella
- Mild sprains and strains
- Knee pain
- Feeling of instability


CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit with superior moisture wicking and breathability, keeping your knee cool during wear
- Certified medical grade compression (20-30 mmHg)
- Lightweight CoolVent™ knit in popliteal area for added breathability and comfort
- Patented SmartPad™ for targeted support and cushioning
- Patella graphics for intuitive placement
- Patented FoldLock™ system for easier and more secure application
- Lightweight, removable spiral steel stays

REQUIRED MEASUREMENTS

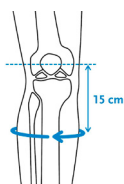

Measure calf circumference 15 cm below mid-patella. Leg in extension.



Measure thigh circumference 15 cm above mid-patella. Leg in extension.



Size	Calf circumference 15 cm below mid patella	Thigh circumference 15 cm above mid patella	Black
	cm	cm	Part#
XS	26 - 31	36 - 41	FP100001
S	31 - 36	41 - 46	FP100002
M	36 - 41	46 - 51	FP100003
L	41 - 46	51 - 56	FP100004
XL	46 - 50	56 - 60	FP100005

FOOT AND ANKLE



REBOUND® AIR WALKER

The Rebound Air Walker features a host of details designed to provide added protection, superior compression, simplified application and usage, improved comfort and, ultimately, successful patient outcomes.



INDICATIONS FOR USE

REBOUND AIR WALKER

Foot and ankle conditions that may benefit from motion restriction, such as:

- Grade 2 and 3 ankle sprains
- Stable fractures
- Tendon or ligamentous sprain
- During postoperative or rehab phase. If used with the Achilles Wedge Kit, Achilles ruptures can be treated as well

REBOUND AIR WALKER LOW TOP

Foot and ankle conditions that may benefit from motion restriction, such as:

- Stable foot fractures
- Tendon or ligamentous sprain
- During postoperative or rehab phase

FEATURES AND BENEFITS

- Full circumferential shell with additional semi-soft toe bumper for frontal toe protection
- Soft Flex Edge™ reduces pressure points
- Full rocker bottom is especially engineered to be low and wide to help promote a natural gait
- Stretch-to-fit shell for optimal fit
- Shell can be trimmed for additional stretch in the calf
- Cushioning and breathable softgood provides comfort
- Linked medial/lateral bladders for easy inflation and equal compression
- Integrated pump and air release valve for easy access

REQUIRED MEASUREMENTS

- Shoe size



REBOUND® AIR WALKER

ORDERING INFORMATION

REBOUND AIR WALKER

Size	Shoe Size UK	Standard	Low Top	Standard Open Toe	Low Top Open Toe
X-Small	1.5 - 3.5	B-242900001E	B-242900061E	B-242900001OT	B-242900061OT
Small	4 - 6.5	B-242900002E	B-242900062E	B-242900002OT	B-242900062OT
Medium	7 - 10	B-242900003E	B-242900063E	B-242900003OT	B-242900063OT
Large	10.5 - 12	B-242900004E	B-242900064E	B-242900004OT	B-242900064OT
X-Large	12.5 - 14.5	B-242900005E	B-242900065E	B-242900005OT	B-242900065OT

ACCESSORIES

REPLACEMENT SOFTGOODS

Size	Slim Liner / High Top	Slim Liner / Low Top
X-Small	B-418030101	B-418030161
Small	B-418030102	B-418030162
Medium	B-418030103	B-418030163
Large	B-418030104	B-418030164
X-Large	B-418030105	B-418030165

REBOUND ACHILLES WEDGE KIT

Size	Part#
Small	AWK0001
Medium	AWK0005
Large	AWK0009
Universal*	AWK00015

* Only fits sizes M to XL

WALKER COVERS

Size	Hygiene covers (2 pcs)	Durable weather cover	Disposable weather cover (20 pcs)
Small	B-418500602	B-418500502	B-418500512
Medium	B-418500603	B-418500503	
Large	B-418500604	B-418500504	B-418500514
X-Large	B-418500605	B-418500505	

OTHER ACCESSORIES

Description	Part#
Adhesive Insole for Wedge	WHW-W0107
Strap Extension	B-502100101
Rebound Hightop Socks SM/MD	B-418500703
Rebound Hightop Sock LG/XL	B-418500704

FORMFIT® WALKER, AIR

INDICATIONS FOR USE

- Soft tissue injuries
- Grade 2 and 3 sprains
- Stable fractures
- Post-operative, trauma or rehab use

FEATURES AND BENEFITS

- High and low top versions available
- Integrated pneumatic system
- Low, wide rocker bottom promotes a stable, natural gait
- Shock absorbing sole for reduced impact at heel strike
- Breathable, seamless liner



REQUIRED MEASUREMENTS

- Shoe size

ORDERING INFORMATION

Size	Shoe Size UK	Standard	Low Top
X-Small	1.5 – 3.5	A-W0200BLK	A-W0300BLK
Small	4 – 6.5	A-W0400BLK	A-W0500BLK
Medium	7 – 10	A-W0600BLK	A-W0700BLK
Large	10.5 – 12	A-W0800BLK	A-W0900BLK
X-Large	12.5 – 14.5	A-W1000BLK	A-W1100BLK

ACCESSORIES

REPLACEMENT SOFTGOODS

Size	Standard	Low Top
X-Small	12349	12356
Small	10675	12357
Medium	10678	12358
Large	10681	12359
X-Large	12347	12360

FORMFIT® WALKER

INDICATIONS FOR USE

- Soft tissue injuries
- Grade 2 and 3 sprains
- Stable fractures
- Post-operative, trauma or rehab use

FEATURES AND BENEFITS

- High and low top versions available
- Shock absorbing sole reduces impact during heel strike
- Low, wide rocker bottom promotes a stable, natural gait
- Breathable, seamless liner



REQUIRED MEASUREMENTS

- Shoe size

ORDERING INFORMATION

Size	Shoe Size UK	Standard	Low Top
X-Small	1.5 – 3.5	W0200BLK	W0300BLK
Small	4 – 6.5	W0400BLK	W0500BLK
Medium	7 – 10	W0600BLK	W0700BLK
Large	10.5 – 12	W0800BLK	W0900BLK
X-Large	12.5 – 14.5	W1000BLK	W1100BLK

ACCESSORIES

REPLACEMENT SOFTGOODS

Size	Standard	Low Top
X-Small	WKSG200BLK	WKSG0300BLK
Small	WKSG400BLK	WKSG500BLK
Medium	WKSG600BLK	WKSG700BLK
Large	WKSG800BLK	WKSG900BLK
X-Large	WKSG1000BLK	WKSG1100BLK

OTHER ACCESSORIES

Description	Part#
10 cm softgood width extension	WKSG-1BLK
Tibia and calf plates for use when additional splinting is required	WKSP100
Hot/Cold therapy insert for safe and reusable cryotherapy	WKGEL100
Heel wedge, Small	AWK0001
Heel wedge, Medium	AWK0005
Heel wedge, Large	AWK0009
Adhesive insole for heel wedge	WHW-W0107
Walker Strap Extender	10882

ROM WALKER

INDICATIONS FOR USE

Foot and ankle conditions that may benefit from motion restriction or controlled range of motion, such as:


- Grade 2 and 3 ankle sprains
- Stable fractures
- Achilles ruptures
- During postoperative or rehab phase

FEATURES AND BENEFITS

- Lightweight and low profile aluminum side struts
- Rocker bottom design assists gait
- Decreased heel-strike force on calcaneus
- Simple to apply and adjust by circumferential straps
- Dial lock hinges with adjustable ROM limitation
- ROM control from 0-45° of dorsi/plantar flexion (7.5° interval)
- Available in Standard and Air version



ORDERING INFORMATION

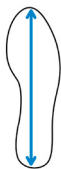

Shoe size.

Size		Standard	Air
	Shoe Size UK	Part#	Part#
XS	up to 3.5	RY-WK002-XS	RY-WK003-XS
S	4 - 6.5	RY-WK002-SM	RY-WK003-SM
M	7 - 9.5	RY-WK002-MD	RY-WK003-MD
L	10 - 12	RY-WK002-LG	RY-WK003-LG
XL	12 +	RY-WK002-XL	RY-WK003-XL

REBOUND® DIABETIC WALKER

Offloading has proven to be a key factor in foot ulcer treatment and entails reducing pressure of the plantar area in general and the ulcer site in particular. The Rebound Diabetic Walker is uniquely designed for effective treatment of plantar foot ulcers, with easy access and re-fitting during wound care, while improving the patient's mobility during the healing process.

The Rebound Diabetic Walker has major advantages over diabetic shoes and shoe inserts in its healing effectiveness and over total contact casting due to its ease of use, speed of application and low cost.



INDICATIONS FOR USE

- Plantar ulcers needing offloading

FEATURES AND BENEFITS

- Effective tri-laminate insole with removable shock-absorbing hexagons that promote ulcer offloading and a Plastazote, heat-moldable top layer that evenly distributes the pressure and minimises friction
- Unique pressure relief valve for preventing over-inflation, while balancing compression with customised fit
- Positioning Portals allow the clinician to check the fit of the walker at the heel bottom, in order to ensure appropriate positioning of the foot in the walker
- Anatomically designed shell that immobilises the foot and ankle at 90 degrees while providing protection
- Flex Edge™ technology for relief of pressure points
- Rocker Bottom outsole for a smooth roll over while allowing for individual natural toe out
- Soft and breathable, low friction, machine washable liner for hygiene promotion and prevention of infection
- Tamper evident strapping for compliance monitoring

REQUIRED MEASUREMENTS

- Shoe size



REBOUND® DIABETIC WALKER

ORDERING INFORMATION

REBOUND DIABETIC WALKER

Size	Shoe size UK	Part#
S	4 – 6.5	B-242500002
M	7 – 10	B-242500003
L	10.5 – 12	B-242500004
XL	12.5 – 14.5	B-242500005

ACCESSORIES

REPLACEMENT SOFTGOODS

Size	Liners	Insoles
S	B-418500102	B-418500202
M	B-418500103	B-418500203
L	B-418500104	B-418500204
XL	B-418500105	B-418500205

WALKER COVERS

Size	Hygiene covers (2 pcs)	Durable weather cover	Disposable weather cover (20 pcs)
S	B-418500602	B-418500502	B-418500512
M	B-418500603	B-418500503	
L	B-418500604	B-418500504	B-418500514
XL	B-418500605	B-418500505	

DIABETIC SOCKS (2 PCS)

Size	Part#
S, M	B-418500703
L, XL	B-418500704



HYGIENE COVER



DURABLE
WEATHER COVER



DISPOSABLE
WEATHER COVER



INSOLE

DH OFFLOADING WALKER®

INDICATIONS FOR USE

- Treatment of plantar ulcers

FEATURES AND BENEFITS

- Shear force reduction by stabilising foot and ankle at 90°
- Wide low profile rocker bottom provides a smooth roll-through and increased stability for altered gaits
- The softgood's patented inner bootie prevents foot from migrating on the insole
- Designed around Össur's unique Active Offloading Insole
- Provides a safe environment for the simultaneous use of wound care material
- Easy to apply and modify



REQUIRED MEASUREMENTS

- Shoe size

ORDERING INFORMATION

DH OFFLOADING WALKER

Size	Shoe Size UK	Part#
X-Small	up to 3.5	DH0200BLK
Small	4 - 6.5	DH0400BLK
Medium	7 - 9.5	DH0600BLK
Large	10 - 12	DH0800BLK
X-Large	12 +	DH1000BLK

ACCESSORIES

REPLACEMENT PARTS

Size	Softgoods	Insoles
X-Small	10958	11742C
Small	10958	PP04C
Medium	10959	PP06C
Large	10960	11740C
X-Large	10960	11741C

DH OFFLOADING POST-OP STYLE SHOE

INDICATIONS FOR USE

- Plantar wounds

FEATURES AND BENEFITS

- Designed to accommodate either the left or the right foot
- Designed around Össur's patented Active Offloading Insole
- Designed to accommodate bulky dressings
- Super-soft nylon upper with reinforced heel counter provides extra comfort and stability
- Patented removable toe piece aides in wound accessibility for daily wound site inspection



REQUIRED MEASUREMENTS

- Shoe size

ORDERING INFORMATION

DH POST-OP SHOE

Size	Shoe Size UK	Part#
Small	4 - 6.5	10340
Medium	7 - 9.5	10341
Large	10 - 12	10342
X-Large	12 +	10343

ACCESSORIES

REPLACEMENT PARTS

Size	Insoles
Small	12365
Medium	12366
Large	12367
X-Large	12368

FORMFIT® ANKLE BRACE (SPEED BRACE)

The Formfit Ankle brace provides stability and protection for the ankle – whether used prophylactically, for chronic instability or following an injury. A hybrid between a rigid ankle stirrup and a soft ankle support, the Formfit Ankle ensures a close fit which limits inversion, eversion, flexion and extension movements. The low profile design fits easily into shoes and the quick lace up feature makes it easy to don and doff. Breathable material ensures it can be comfortably worn all day. The Formfit Ankle with Figure 8 strap version, offers an even greater level of support, by further limiting the range of motion of the ankle.



INDICATIONS FOR USE


- Acute sport injuries
- Post-cast support
- Injury prevention
- Post-operative stabilisation of the ankle

FEATURES AND BENEFITS

- Breathable material
- Quick lace up
- Inversion/Eversion control
- Figure 8 strap version

REQUIRED MEASUREMENTS

- Measure ankle circumference diagonally over the base of the heel and across the ankle bone

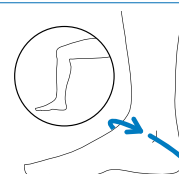

ORDERING INFORMATION

Size	Ankle circumference*	Formfit Ankle Brace
X-Small	28 – 30.5cm	B-212000011
Small	30.5 – 33cm	B-212000012
Medium	33 – 35.5cm	B-212000013
Large	35.5 – 38cm	B-212000014
X-Large	38 – 40.5cm	B-212000015

FORMFIT® ANKLE WITH SPEEDLACE

The Formfit® Ankle Brace with Speedlace provides stability and protection for the ankle – whether used prophylactically, for chronic instability or following an injury. Made from durable materials, its low profile, quick-lace design is comfortable and fits easily into shoes. Removable semi-rigid stays provide additional support and the Figure 8 straps are designed for inversion/eversion control.



INDICATIONS FOR USE

- Grade I-II sprains and strains
- Chronic instability
- Prophylactic use

FEATURES AND BENEFITS

- Durable materials
- Quick-Pull closure straps
- Figure 8 straps
- Removable stays
- Circumferential cuff

REQUIRED MEASUREMENTS

- Measure ankle circumference diagonally over the base of the heel and across the ankle bone



ORDERING INFORMATION

Size	Ankle circumference*	Part#
XXSmall	23 cm – 25.5 cm	W-10621
XSmall	25.5 cm – 28 cm	W-10622
Small	28 cm – 30.5 cm	W-10623
Medium	30.5 cm – 33 cm	W-10625
Large	33 cm – 35.5 cm	W-10627
XLarge	35.5 cm – 38 cm	W-10628
XXLarge	38 cm – 40.5 cm	W-10629

*Measure ankle circumference diagonally over the base of the heel and across the ankle bone.

FORMFIT® ANKLE STIRRUPS

Formfit Ankle Stirrups are lightweight and sturdy. Whether worn to accommodate an acute injury, chronic instability, or post cast healing, Formfit Ankle Stirrups stabilise injured ankles by restricting the inversion/eversion motion. The universal design has been carefully engineered to accommodate the malleoli on both sides and ensures a great, comfortable fit. The slip resistant heel strap can be adjusted to accommodate most ankles. Our signature Flex-Edge™ overmold surrounds the stirrup's outer shell protecting patient's skin from any hard edges, further adding to a comfortable fit. The universal stirrup design is available with three different liner options: inflatable air, gel, or foam, offering each patient a solution that best fits their needs.



INDICATIONS FOR USE

Situations requiring inversion/eversion restriction. These may include:

- Mild to severe ankle sprains
- Chronic ankle instability
- Post-cast and post walker support
- Ankle injury rehabilitation and/or post-surgical use

FEATURES AND BENEFITS

- Universal shell, carefully crafted to fit both right and left ankles
- Soft Flex-Edge overmold for increased comfort
- Adjustable heel strap to optimise patient fit and comfort
- Slip-resistant silicone dots to reduce slipping and stabilise the foot inside of shoe
- 2 bidirectional hook and loop straps ensure a secure and stable fit
- Choice of inflatable, foam, and gel liners to suit your needs
- Available in Adult and Youth sizes

REQUIRED MEASUREMENTS

- Patient's height

ORDERING INFORMATION



Size	Patient's Height	Formfit Air Ankle Stirrup	Formfit Gel Ankle Stirrup	Formfit Foam Ankle Stirrup
Adult	160 cm	70350	70351	70352
Youth	160 cm	71350	71351	71352

FORMFIT® PRO ANKLE

Össur Formfit® Pro Ankle is a compressive ankle sleeve with 2 integrated malleoli pads, designed to be worn after mild ankle injuries. The dynamic 3D MotionTech™ knit is supportive and meets certified medical grade compression requirements. Proven superior to leading brands for moisture wicking and breathability, your ankle stays cool during wear. CoolVent™ technology in the area in front of the ankle further enhances breathability and comfort.



MotionTech™



CoolVent™



SmartPad™



FoldLock™



INDICATIONS FOR USE

For Ankle conditions that may benefit from compression, such as:

- Ankle injury rehabilitation
- Mild sprains and strains
- Mild instability


CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated

FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit with superior moisture wicking and breathability, keeping your ankle cool during wear
- Certified medical grade compression (20-30 mmHg)
- CoolVent™ technology in the area in front of the ankle for added breathability and comfort
- 2 integrated malleoli SmartPads™ for targeted support and cushioning
- Patented FoldLock™ system for easier and more secure application

REQUIRED MEASUREMENTS

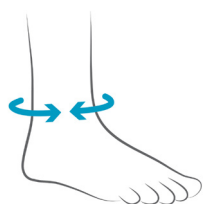

Measure ankle circumference.

ORDERING INFORMATION

Size	Ankle circumference	Part#	
		Left	Right
XS	18 - 20	FP200201	FP200101
S	20 - 23	FP200202	FP200102
M	23 - 26	FP200203	FP200103
L	26 - 29	FP200204	FP200104
XL	29 - 32	FP200205	FP200105

FORMFIT® NIGHT SPLINT

INDICATIONS FOR USE

- Plantar fasciitis
- Lower extremity overuse injuries
- Achilles tendinopathy

FEATURES AND BENEFITS

- Straps adjust from 0° – 10° of dorsiflexion
- Contoured for a comfortable fit
- Soft foam interior is washable and breathable
- Quick release buckles for easy application
- Slip resistant tread



REQUIRED MEASUREMENTS

- Shoe size

ORDERING INFORMATION

	Shoe Size UK		Part#
Size	Male	Female	Standard
Small	Up to 6.5	Up to 5	50023
Medium	7 – 9.5	5.5 – 8	50025
Large	10 and up	8.5 and up	50027

ACCESSORIES

Description	Size	Part#
Foam Wedge		10700
Replacement Liner	Small	11760
Replacement Liner	Medium	11761
Replacement Liner	Large	11762

EXOFORM® DORSAL NIGHT SPLINT

INDICATIONS FOR USE

- Plantar fasciitis
- Post-static pain
- General heel pain
- Drop foot
- Achilles tendonitis
- Lower extremity overuse injuries

FEATURES AND BENEFITS

- Adjustable aluminum stays can be positioned to maintain flexion between 85° – 90°
- Soft, Flex edge over molding improves patient comfort
- Full calf and heel strapping prevent brace from rotating
- Slip resistant padding



REQUIRED MEASUREMENTS

- Shoe size

ORDERING INFORMATION

Size	Shoe Size UK	Part#
Medium	4.5 – 8	W-50085
Large	8.5 – 13.5	W-50087

AIRFORM® NIGHT SPLINT

INDICATIONS FOR USE

- Plantar Fasciitis
- Achilles Tendonitis
- Drop foot
- Post-static pain

FEATURES AND BENEFITS

- Light weight, low profile design
- Easy to adjust

REQUIRED MEASUREMENTS

- Shoe size

ORDERING INFORMATION

	Shoe Size UK		Part#
Size	Male	Female	Standard
Medium	4.5 – 8.5	3.5 – 7.5	50045
Large	9 – 13.5	8 – 12	50047



AFO DYNAMIC

The AFO Dynamic provides a strong, lightweight solution for patients with varying degrees of drop foot secondary to a variety of neurological conditions. Össur has taken the carbon fiber technology from its Flex-Foot® product and incorporated it into the new AFO Dynamic to provide increased support over traditional ankle foot orthoses. The energy storing properties of carbon fiber make it an ideal material for this dynamic ankle foot orthosis because it provides real energy return. Best for those people leading busy lives, the AFO Dynamic will ensure a more efficient walk for the user.



INDICATIONS FOR USE

Drop foot secondary to:

- Cerebro Vascular Accident (CVA)
- Other neurological conditions
- Multiple Sclerosis
- Mild knee instability
- Charcot Marie Tooth disease (HMSN)

CONTRAINDICATIONS

- Severe ankle – foot deformities
- Fluctuating edema
- Severe spasticity

FEATURES AND BENEFITS

- Carbon fiber design provides excellent strength-to-weight ratio and dynamic motion
- The tapered carbon heel stores energy and manages forces at heel strike
- Full-length toe lever provides full support while foot is loaded, as well as reduced shock to the contralateral limb
- Creates medial-lateral stability at mid-stance
- Reduces energy consumption by reducing hip hike
- Medial strut makes it less visible in shoe
- Open anterior tibia shell provides superior anatomical fit, and reduces heat and perspiration
- Weight limit: 120kg

REQUIRED MEASUREMENTS

- Shoe size
- Calf height at thickest part of the calf

AFO DYNAMIC

ORDERING INFORMATION

AFO DYNAMIC

		Shoe Size UK			
Size	Calf Height	Male	Female	Left	Right
X-Small	30 – 34 cm	2.5 – 4.5	< 4	O-102021	O-102011
Small	32 – 36 cm	5 – 7	4.5 – 6.5	O-102022	O-102012
Medium	34 – 38 cm	7.5 – 10	7 – 9.5	O-102024	O-102014
Large	36 – 40 cm	10.5 – 13.5		O-102026	O-102016

ACCESSORIES

ACCESSORY KITS

Size	Part#
X-Small	O-102101
Small	O-102102
Medium	O-102104
Large	O-102106

Note: Accessory kits come with foam padding, upper and lower Velcro straps, and Velcro dots.



AFO LIGHT

AFO Light provides support only where it is needed, eliminating unnecessary bulk and providing a cosmetically appealing look. Using the best of Flex-Foot® technology and the energy storing properties of carbon fiber, AFO Light offers a strong and lightweight solution for people with mild to moderate Drop Foot.



INDICATIONS FOR USE

- Mild to moderate drop foot

CONTRAINDICATIONS

- Severe ankle – foot deformities
- Medial-lateral instability
- Severe spasticity
- Extreme activity
- Fluctuating edema

FEATURES AND BENEFITS

- Energy return and propulsion assistance of the limb at toe off
- Supports the foot for ground clearance during swing phase
- Carbon fiber design provides excellent strength-to-weight ratio and dynamic motion
- Gradual plantar flexion at initial contact
- Stiffness gradient at heel and the toe correspond to ground reaction forces as they are applied to the foot
- Easy to don and doff
- Weight limit: 100kg

REQUIRED MEASUREMENTS

- Shoe size
- Calf height at thickest part of the calf

ORDERING INFORMATION

AFO LIGHT

		Shoe Size UK			
Size	Calf height	Male	Female	Left	Right
Small	32 – 36 cm	< 7	4.5 – 6.5	O-121022	O-121012
Medium	34 – 38 cm	7.5 – 10	7 – 9.5	O-121024	O-121014
Large	36 – 40 cm	10.5 – 13.5		O-121026	O-121016
X-Large	38 – 42 cm	11.5 – 14.5		O-121028	O-121018

ACCESSORIES

Description	Part#
Replacement Kit	O-122101

AFO LEAF SPRING

AFO Leaf Spring is a pre-fabricated polypropylene ankle-foot orthosis designed to support flaccid drop foot. It provides a semirigid section for toe clearance and support, yet has a thin, flexible foot plate for easy trimming. The absence of a heel section makes the Leaf Spring more comfortable to wear and provides a better fit in shoes.



INDICATIONS FOR USE

- Drop foot
- Mild drop foot secondary to other neurological pathologies

FEATURES AND BENEFITS

- Lightweight injection-molded polypropylene
- Thin, flexible foot part which is easy to trim with scissors
- Absence of heel section for improved comfort

REQUIRED MEASUREMENTS

- Shoe size

ORDERING INFORMATION

Size	Shoe Size UK	Height	Left	Right	Foot Plate Length
Small – Medium	3.5 – 6.5	27 cm	O-110227	O-110127	24 cm
Large	7 – 10.5	30 cm	O-110230	O-110130	28 cm

REBOUND FOOT-UP®

Rebound Foot-Up is a lightweight ankle/foot orthosis (AFO) that offers dynamic support for drop-foot or related disorders that require dorsiflexion support. The Rebound Foot-Up supports the ankle dorsiflexion during swing phase. Dynamic support is provided by an elastic strap connected to the shoe by either a transparent plastic inlay placed between shoelaces, or by a nut and bolt fastened through the eyelets. The positioning of the fastening allows for individualising the support by providing the option of positioning the dorsiflexion support on the medial or lateral side. When using the plastic inlay, the dorsiflexion support becomes neutral. The Rebound Foot-Up Ankle Cuff has a smart clamp that holds the cuff in place and facilitates one-handed donning and doffing.



INDICATIONS FOR USE

- Drop foot
- Disorders that require dorsiflexion support

FEATURES AND BENEFITS

- Dynamic and discreet support for drop-foot or related disorders that require dorsiflexion support
- Improved function, ease of use, comfort, and quality
- Allows for voluntary plantar flexion
- The elastic strap is adjustable and easily replaceable
- The Rebound Foot-Up Ankle Cuff and Rebound Foot-Up Foot Wrap are made from breathable material that can be worn comfortably for long periods of time
- The Rebound Foot-Up is hand-washable
- Positioned medial or lateral support
- Contoured to the shape of leg
- Adjustable strap length
- Smart clamp facilitates one-handed donning / doffing
- Optional Rebound Foot-Up Foot Wrap can be used without shoes or in open shoes or sandals

REQUIRED MEASUREMENTS

- Circumference of ankle, measured around the malleoli

ORDERING INFORMATION

ANKLE CUFF

Size	Ankle circumference	Part#
Small/Medium	18 - 21 cm	RFU50001
Large/X-Large	26 - 33 cm	RFU50002

FOOT WRAP

Size	Foot circumference	Part#
One Size	20 - 28 cm	RFU50101

*Shoeless version item comprises of foot wrap only and must be ordered with the Foot-Up unless the patient already has one.

ACCESSORIES

Description	Part#
Rebound Foot-Up Attachment Kit (includes plastic inlay, elastic strap, and 1 x nut and screw)	RFU50201
Rebound Foot-Up Replacement Nuts And Bolts (5 nuts included: 2 short and 3 long screws)	RFU50202

UPPER EXTREMITY



FORMFIT® SHOULDER BRACE

The Formfit Shoulder Brace is designed to provide immobilisation following shoulder injury/surgery. With its streamlined fit, sporty look and breathability it offers excellent comfort and ease of use. Same as its predecessor, the SmartSling®, the Formfit Shoulder Brace also provides abduction of the shoulder (when the abduction pillow is attached), or external rotation of the shoulder (when the external rotation pillow is attached).

Optional axilla pillow helps distribute weight over a larger surface area and decreases pressure on the ulnar nerve.

INDICATIONS FOR USE

- Immobilisation for rotator cuff repairs
- Bankart Repairs
- Capsular Shift
- Gleno humeral dislocation/subluxations

FEATURES AND BENEFITS

- Breathable material
- Quick-release buckles
- Offloading strap
- Extra padding
- Universal thumb loop

REQUIRED MEASUREMENTS

- Olecranon to end of closed fist

ORDERING INFORMATION

Size	Fits Length	Description	Part
Small	up to 29 cm	Formfit Shoulder Brace - sling only	SSS-SO
Medium	29-40 cm	Formfit Shoulder Brace - sling only	SSR-SO
Large	40-44.5 cm	Formfit Shoulder Brace - sling only	SSL-SO
Small	up to 29 cm	Formfit Shoulder Brace with Abduction	SSS
Medium	29-40 cm	Formfit Shoulder Brace with Abduction	SSR
Large	40-44.5 cm	Formfit Shoulder Brace with Abduction	SSL
Small	up to 29 cm	Formfit Shoulder Brace with External Rotation	SSS-ER
Medium	29-40 cm	Formfit Shoulder Brace with External Rotation	SSR-ER
Large	40-44.5 cm	Formfit Shoulder Brace with External Rotation	SSL-ER

ACCESSORIES

Description	Part
Axilla Pillow	SSAXP
External Rotation Pillow	SSE



REBOUND® POST-OP ELBOW

The Rebound® Post-Op Elbow brace by Össur combines intuitive features for fitters and users with a design that aims to improve the post-op experience. The industry's only elbow immobiliser that combines a drop-lock hinge and four telescoping paddles, individual post-op protocols are easily catered to. The Rebound Post-Op Elbow, lightest of its kind, promotes user comfort with straps that stay in place, anatomical left/right fit with moldable stability cuffs, and blue patient touch-points for easy education.



INDICATIONS FOR USE

Indications requiring elbow immobilisation and/or range of motion restriction. This may include:

- Stable fractures of the elbow, distal humerus, proximal radius or ulna
- Tendon and ligament injuries or repairs
- Collateral ligament reconstructions
- Elbow hyperextension
- Triceps tendon repair
- Chronic elbow injuries
- Tennis elbow release

FEATURES AND BENEFITS

- Designed to be the most lightweight brace on the market
- Telescoping paddles relocate to individualise length and keep incision sites clear
- Easy education for users with intuitive patient touch-points, including Blue is You™ buckles and numbered strapping system
- Drop-Lock hinge enables secure range of motion with bump-resistant pull adjustment stops and optional ROM lock-outs
- Moldable stability cuffs designed to be individually customised for improved fit and support
- Strap guides maintain strap location for added security and comfort
- Universal fit for both left/right elbows

REQUIRED MEASUREMENTS

- Universal, one size

REBOUND® POST-OP ELBOW

ORDERING INFORMATION

Side	Part#
Left	B-8391680
Right	B-8391580

ACCESSORIES

ACCESSORY KITS

Description	Part#
Elbow Liner Kit	B-8390002
Elbow Wrist Control Kit	B-8390003
Elbow Sling Kit	B-8390004
Elbow ROM Locks Kit	B-8390005
Strap Tab Kit	B-8390006
Adhesive Hook Sticker Kit	B-8390007
Anti-Migration Pad Kit	B-8390008
Elbow Strap Kit	B-8390009

FORMFIT® TENNIS ELBOW

INDICATIONS FOR USE

- Epicondylitis

FEATURES AND BENEFITS

- Patented ergonomically-shaped plastic eliminates tourniquet effect associated with traditional designs
- Twin blue cushions provide counterforce, reducing traction on the tendons and thus inflammation
- Ribbed surfaces reduce vibration to the tendon, increasing comfort
- Durable design and materials enable everyday use for both work and athletic activities



REQUIRED MEASUREMENTS

- Circumference of the elbow

ORDERING INFORMATION

Size	Circumference	Part#
Small	22-26 cm	B-247000002
Medium	27-32 cm	B-247000003
Large	33-38 cm	B-247000004

AIRFORM® TENNIS ELBOW SUPPORT

INDICATIONS FOR USE

- Epicondylitis

FEATURES AND BENEFITS

- Focused pneumatic compression
- Conforms and comfortably cushions sensitive injured anatomy
- Easy to apply, simple design that fits most, reduces inventory needs



REQUIRED MEASUREMENTS

- Universal: Fits elbow circumference 20 – 36 cm

ORDERING INFORMATION

Description	Size	Part#
AirForm Tennis Elbow Support	Universal (fits elbow circumference 20 - 36 cm)	320000

FORMFIT® PRO ELBOW

Össur Formfit® Pro Elbow provides relief from elbow pain. This 3D compressive elbow sleeve is manufactured using our MotionTech™ technology and features two integrated epicondyle SmartPads™ for more targeted support and cushioning. The RadialStrap™ provides additional compression and support.



MotionTech™



CoolVent™



SmartPad™



FoldLock™



INDICATIONS FOR USE

For elbow conditions that may benefit from compression, such as:

- Medial and lateral epicondylitis
- Elbow pain and swelling


CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

FEATURES AND BENEFITS

- 3D MotionTech™ for compression and dynamic fit keep
- Epicondyle SmartPads™ for targeted support and cushioning
- Patented FoldLock™ system for easier and more secure application
- RadialStrap™ for increased compression and support

REQUIRED MEASUREMENTS

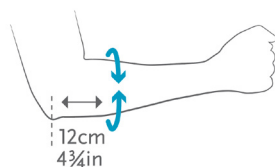

Measure circumference
12cm (43/4in) below elbow.

Size	Elbow circumference	Black
	cm	Part#
XS	17 - 20	FP300001
S	20 - 23	FP300002
M	23 - 26	FP300003
L	26 - 29	FP300004
XL	29 - 32	FP300005

FORMFIT® WRIST / FORMFIT® WRIST & FOREARM

Formfit Wrist, and Formfit Wrist & Forearm, provides immobilisation and protection for patients suffering from tendonitis, select soft tissue injuries – including mild to moderate sprains and strains, as well as for post-cast healing. The lightweight, breathable material and contour fit make this comfortable product an excellent choice.



INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Tendonitis
- Post-cast healing
- Other selected soft tissue injuries

FEATURES AND BENEFITS

- Breathable material
- Contour fit
- Contact Closure Straps
- Customisable Stays

REQUIRED MEASUREMENTS

- Wrist circumference

ORDERING INFORMATION

FORMFIT WRIST & FOREARM

Size	Wrist circumference	8"		10"	
		Left	Right	Left	Right
XSmall	13 – 16 cm	317082	317072	617082	617072
Small	16 – 18.5 cm	317083	317073	617083	617073
Medium	18.5 – 20.5 cm	317085	317075	617085	617075
Large	20.5 – 23.5 cm	317087	317077	617087	617077
XLarge	23.5 – 25.5 cm	317088	317078	617088	617078

ACCESSORIES

GEL PAD FOR HOT/COLD THERAPY

Description	Left	Right
Formfit Wrist (8" version)*	GB20L	GB20R
Formfit Wrist & Forearm (10" version)	GB22L	GB22R



* Fits XSmall/Small. For Medium-XLarge, please use Form Forearm 10" part numbers

FORMFIT® WRIST & THUMB, UNIVERSAL

Formfit Universal Wrist and Thumb products are designed to restrict motion of the wrist and the thumb. Providing immobilisation and protection from select soft tissue injuries, mild to moderate strains and sprains, including gamekeeper's thumb and post cast healing. The lightweight, universal design also reduces inventory needs.



INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Tendonitis
- Post-cast healing
- Other selected soft tissue injuries

FEATURES AND BENEFITS

- Quick-Pull close strap
- Positional dorsal stay
- Customisable thumb stay
- Malleable palmar stay

REQUIRED MEASUREMENTS

- Wrist circumference

ORDERING INFORMATION

FORMFIT WRIST

Size	Wrist circumference	Left	Right
Pediatric	9 – 13 cm	B-252603301	B-252603300
8" Universal	11 – 24 cm	B-252603109	B-252503109
8" X-Large	19 – 28 cm	B-252603106	B-252603105

FORMFIT WRIST & FOREARM UNIVERSAL

Size	Wrist circumference	Left	Right
10" Universal	11 – 24 cm	B-252603210	B-252603209

FORMFIT THUMB SPICA UNIVERSAL

Size	Wrist circumference	Left	Right
Pediatric	9 – 13 cm	B-253603301	B-253603300
8" Universal	11 – 24 cm	B-253603109	B-253503109
8" X-Large	19 – 28 cm	B-253603106	B-253603105

SPECTRA WRIST BRACE

INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Carpal tunnel and other repetitive stress injuries
- Post cast healing

FEATURES AND BENEFITS

- Lace closure provides easy one-pull application
- Adjustable palmar stay allows for fine-tuning when needed

REQUIRED MEASUREMENTS

- Wrist circumference

ORDERING INFORMATION

Size	Wrist Circumference	Left	Right
X-Small	12 - 14cm	6602210	6602315
Small	14 - 16cm	6602422	6602535
Medium	16 - 18cm	6602624	6602734
Large	18 - 20cm	6602755	6602793
X-Large	20 - 22cm	6602826	6602932



EXOFORM® CARPAL TUNNEL WRIST

INDICATIONS FOR USE

- Carpal tunnel syndrome
- Tendonitis

FEATURES AND BENEFITS

- Allows for full MCP flexion
- Contoured for anatomical fit
- Provides radial and ulnar styloid relief
- Low profile
- Preshaped into a neutral position for comfortable limited motion/ flexibility



REQUIRED MEASUREMENTS

- Wrist and palm circumference

ORDERING INFORMATION

Size	Wrist circumference	Palm circumference	Left	Right
Small	13 – 16.5 cm	17 – 20.5 cm	517083	517073
Medium	16.5 – 19 cm	20.5 – 23.5 cm	517085	517075
Large	19 – 25.5 cm	23.5 – 26 cm	517087	517077

FORMFIT® THUMB SPICA

The Formfit® Thumb Spica is designed to restrict wrist and thumb motion. It provides immobilisation and protection for patients suffering from tendonitis (e.g. DeQuervain's Tendonitis), select soft tissue injuries – including mild to moderate sprains and strains and Gamekeeper's Thumb –, as well as for post-cast healing. The lightweight, breathable material and contour fit make this comfortable product an excellent choice.



INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Tendonitis
 - e.g., DeQuervain's Tendonitis
- Post-cast healing
- Other selected soft tissue injuries
 - e.g., Gamekeeper's Thumb

FEATURES AND BENEFITS

- Breathable material
- Contour fit
- Contact Closure Straps
- Customisable Stays

REQUIRED MEASUREMENTS

- Wrist circumference

ORDERING INFORMATION

Size	Wrist circumference	Left	Right
XSmall	13 – 16 cm	3020	3120
Small	16 – 18.5 cm	3030	3130
Medium	18.5 – 20.5 cm	3050	3150
Large	20.5 – 23.5 cm	3070	3170
XLarge	23.5 – 25.5 cm	3080	3180

ACCESSORIES

GEL PAD FOR HOT/COLD THERAPY

Description	Left	Right
Formfit Wrist (8" version)*	GB20L	GB20R
Formfit Wrist & Forearm (10" version)	GB22L	GB22R



* Fits XSmall/Small. For Medium-XLarge, please use Form Forearm 10" part numbers

UNIVERSAL THUMB SPLINT

INDICATIONS FOR USE

- Gamekeeper's thumb
- Torn collateral ligaments
- DeQuervain Tendonitis

FEATURES AND BENEFITS

- Elastic contact closures
- Restricts movement of the MP/CMC
- Light weight and breathable

REQUIRED MEASUREMENTS

- Universal, one size

ORDERING INFORMATION

Description	Part#
Universal Thumb Splint	30000



FORMFIT® PRO WRIST

Össur Formfit Pro Wrist is constructed using our superior signature 3D MotionTech knit, and ideal for injuries that may benefit from compression. Malleable aluminum stays can be adjusted to conform to an individual's anatomy. CoolVent™ technology in the basal thumb area enhances breathability and comfort.



MotionTech™



CoolVent™



StayBilizer™



FoldLock™



INDICATIONS FOR USE

For wrist conditions that may benefit from compression, such as:

- Rheumatic Arthritis (RA) and other chronic pain conditions
- Wrist sprain and strains
- Overuse and pain conditions
- Wrist tendinitis


CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit has been proven to be superior to the leading brands at moisture wicking and breathability, keeping patients cool during wear
- Malleable aluminum stay for a customised fit
- CoolVent™ technology in thumb base area for extra breathability and comfort

REQUIRED MEASUREMENTS

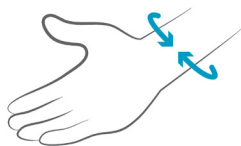

Measure wrist circumference.

Size	Wrist circumference	Left Part#	Right Part#
XS	14 - 15 cm	FP400201	FP400101
S	15 - 16 cm	FP400202	FP400102
M	16 - 17 cm	FP400203	FP400103
L	17 - 18 cm	FP400204	FP400104
XL	18 - 19 cm	FP400205	FP400105
XXL	19 - 20 cm	FP400206	FP400106

CASTING



TECHFORM® PREMIUM

INDICATIONS FOR USE

- Immobilization

FEATURES AND BENEFITS

- Thinner material is lightweight and easy to cut
- Soft, conformable rolls allow for easy handling
- Excellent lamination ensures a strong and durable cast
- Smooth finished cast is non-abrasive and will not snag on clothing
- High radiotranslucence allows for clearer imaging through the cast
- Finger holes in the pouch allow for easy opening



ORDERING INFORMATION

Width	WH	BK	OR	RD	PK	HP	LB	DB	PR	GN	RB
1.5"	154WH										
2"	204WH	204BK	204OR	204RD	204PK	204HP	204LB	204DB	204PR	204GN	204RB
3"	304WH	304BK	304OR	304RD	304PK	304HP	304LB	304DB	304PR	304GN	304RB
4"	404WH	404BK	404OR	404RD	404PK	404HP	404LB	404DB	404PR	404GN	404RB
5"	504WH										

Note: Packed in boxes of 10

White (WH)	Black (BK)	Orange (OR)	Red (RD)	Pink (PK)
Hot Pink (HP)	Lt Blue (LB)	Dk Blue (DB)	Purple (PR)	Green (GN)



WWW.OSSUR.COM/EN-ZA

Össur South Africa
Unit 1, Stonewood Business Park
9 Tanzanite Crescent
Kraaifontein
Cape Town
7550

T 0800 996 220
F 0800 981 744
E ossursa@ossur.com

