



**ÖSSUR**<sup>®</sup>  
LIFE WITHOUT LIMITATIONS

# Navii<sup>®</sup>

Expand Your World

# Transform the Patient Experience

As a dedicated prosthetist, your goal is to provide your patients with the best possible solutions to enhance their mobility and quality of life. With Navii, the latest innovation in prosthetic technology, your patients can now experience the new enhanced stability feature, greater freedom, and a new level of empowerment - all in one remarkable device.



## **Revolutionary Stability and Confidence**

Imagine a prosthetic knee that ensures confidence with every step. Navii delivers improved consistency and predictability, especially during challenging activities like descending stairs or ramps.

## **Unmatched Freedom and Versatility**

Navii's cutting-edge, waterproof design breaks down the barriers that traditionally limit prosthetic users. Now, your patients can dive into life fully, whether it's swimming in the ocean or the pool, enjoying a day at the beach, or simply not having to worry about taking a shower. Navii allows them to live without limitations, offering true freedom to participate in all the activities they love.

## **Empowerment Through Advanced Training**

Navii isn't just a prosthetic knee - it's a tool for empowerment. With expanded functional training programs, Navii encourages your patients to explore the full potential of their prosthesis. By actively engaging with their device, they can take control of their mobility journey, making every step toward greater capability and satisfaction.





### Seamless Experience for You

At Össur, we understand the importance of providing a seamless experience for both you and your patients. Our dedicated clinical support, comprehensive training programs, and responsive customer service team are here to assist you every step of the way— from ordering to maintaining the Navii prosthetic knee. We are committed to ensuring you have all the tools and resources needed to deliver the best possible performance and outcomes for your patients.




# Discover the Features that Make Navii Stand Out

## Strong, Powerful Actuator

Navii's powerful actuator delivers exceptional support, especially at high flexion angles. The advanced yielding control provides your patients with improved consistency and predictability, particularly when

descending stairs or ramps. This enhanced stability allows them to feel equal to or even more secure, ensuring a reliable mobility experience.

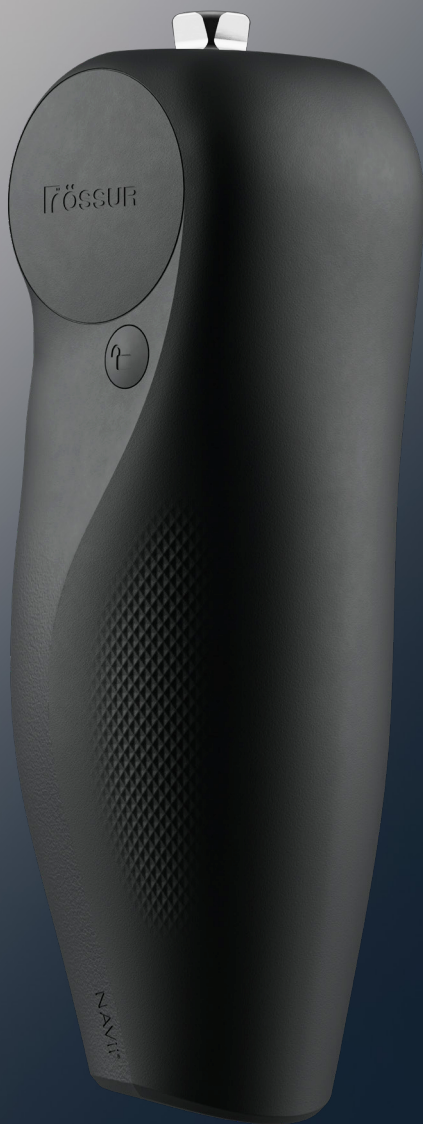

Variable Position  
Mechanical Lock



Strong, Powerful Actuator



Waterproof Design



Removable Protective  
Covers & Natural Sitting



Össur Logic Training Exercises  
and Advanced Settings



Advanced Gait Reports



Improved User Interface



High Activity Features



### **Waterproof Design**

Navii is designed with a robust waterproof build (IP68 rated), making it perfect for patients who want to engage in water activities. Whether it's a day at the pool, a beach outing, or walking through sand, Navii stands up to the challenge. It can handle occasional submersion in fresh, salt, and chlorinated water—just a quick rinse with fresh water afterward keeps it in top condition.

### **Removable Protective Covers & Natural Sitting**

Navii combines functionality and aesthetics with its removable protective covers and options for customisation. Available in three lengths—standard, medium, and long—Navii is designed to accommodate varying cosmetic preferences, including different colours and finishes. Its more anatomical shape and high-quality finish enhance the natural look and feel, while the knee naturally flexes when sitting, ensuring comfort without unwanted extension. This thoughtful, user-centric design means your patients get a prosthetic knee that not only performs exceptionally but also looks and feels great.

### **Variable Position Mechanical Lock**

Give your patients the comfort they deserve with Navii's innovative Variable Position Mechanical Lock. This feature allows for secure, flexed standing positions with a mechanically locked knee at 20°, 10°, or fully extended at 0°. This adaptability offers a comfortable and stable stance.

### **Össur Logic Training Exercises and Advanced Settings**

Navii is equipped with Össur Logic Training Exercises, including eight functional training exercises that engage users and motivate them to improve their prosthetic use. The Set-up Wizard simplifies the fitting process, while the powerful auto-adjustment feature minimises the need for manual adjustments, making the process easier for both the prosthetist and the user. Customisable settings for walking, sitting, and descent activities allow for easily adjustable support tailored to user preferences and abilities, promoting a more personalised experience.

### **Advanced Gait Reports**





Navii includes advanced gait reporting features that track the user's progress over time, encouraging continuous engagement and motivation. By monitoring their own progress, users can stay motivated and see the tangible results of their efforts, fostering a positive feedback loop that supports ongoing improvement.

### **Simple User Interface**


Navii's improved user interface ensures ease of use for both prosthetists and users. The unified interface, coherent with other Össur products, provides clear operational status and error messages, simplifying the fitting process through Össur Logic's auto-adjustment feature. Additionally, the simple and robust charging system allows for seamless integration with cosmetic covers, enhancing both functionality and appearance.

### **High Activity Features**

For patients leading active lifestyles, Navii includes high activity features designed to support a range of activities. These include step-over-step stair ascent, automatic cycling recognition, and a seamless transition to running. These features enable users to maintain an active lifestyle, participate in weekend exercises, and enjoy a wide range of physical activities without limitations.

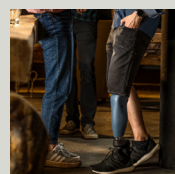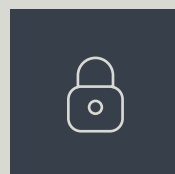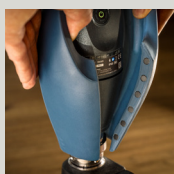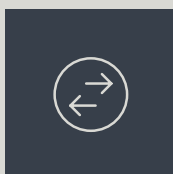

# The Essence of Iceland in Every Cover

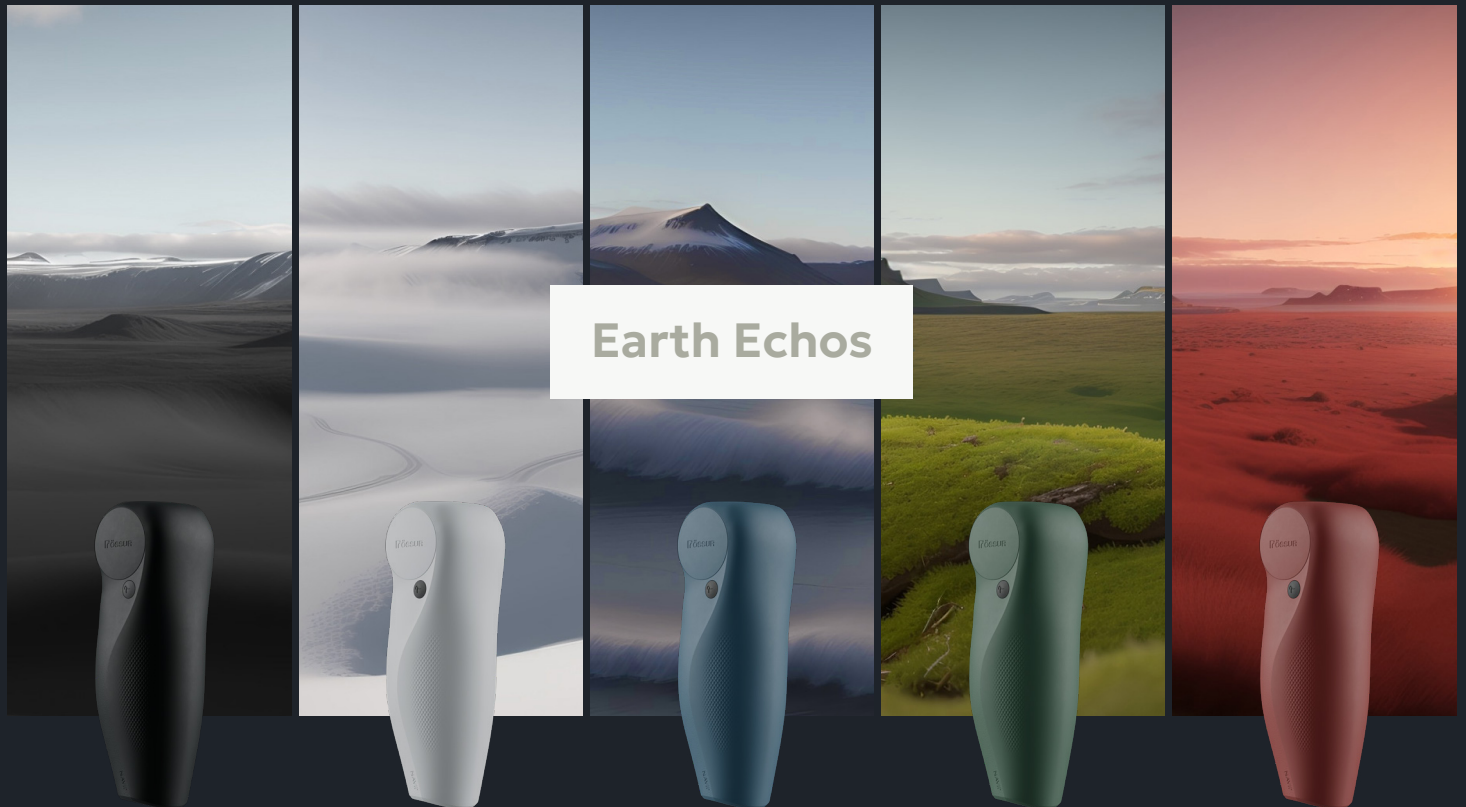

## Navii Protective Covers Inspired by Iceland's Natural Beauty

At Össur, our Icelandic heritage is woven into everything we create. With our headquarters nestled in Reykjavík, we draw inspiration from Iceland's stunning landscapes and vibrant colours to craft products that not only perform exceptionally but also resonate emotionally. Our new Navii Protective Covers, part of the Earth Echos collection, are a tribute to this beautiful land, echoing the serene landscapes, raw elements, and vivid hues of Icelandic nature.

Iceland is renowned for its pristine and untouched beauty - a land where powerful natural forces shape breathtaking vistas. This compelling environment not only captivates the eye but also symbolises purity and quality. By incorporating these elements into our Navii Protective Covers, we aim to evoke a deep emotional connection to our products, ensuring that every choice reflects both aesthetic appeal and superior quality.

# Explore the Earth Echos Collection



## Onyx Black

Inspired by the elegance and power of Iceland's black sand beaches, the Onyx Black Navii Protective Cover features a timeless, deep black hue that complements any style. This versatile cover seamlessly blends with any outfit, occasion, or mood, making it a practical yet stylish choice. With its quick magnetic snap-on feature, switching to Onyx Black is effortless, providing both elegance and adaptability for every journey. Available in 3 lengths.



## Wave Blue

Drawing inspiration from the vastness of Iceland's sky and ocean, the Wave Blue Navii Protective Cover brings a serene, calming presence with its blue hue. This cover captures the essence of tranquility and openness, making it ideal for those who embrace the expansive spirit of nature. The interchangeable magnetic snap-on design allows for easy customisation, offering a seamless way to match any outfit or occasion.



## Mosi Green

Inspired by the refreshing and calming effect of Icelandic moss, the Mosi Green Navii Protective Cover features a vibrant green color that embodies renewal and growth. Perfect for those who feel a deep connection to nature, this cover adds a touch of life and vitality to the prosthetic. With its easy-to-use magnetic snap-on system, Mosi Green makes it simple to refresh and adapt, reflecting the dynamic nature of everyday life.



## Adobe Red

Inspired by the raw, powerful beauty of Icelandic lava stone, the Adobe Red Navii Protective Cover symbolises strength and a deep connection to the earth. The rich red ochre color exudes confidence and passion, reminiscent of the fiery landscapes of Iceland's volcanic terrain. This bold cover is perfect for making a statement, and with its convenient magnetic snap-on feature, it allows for quick and easy customisation to match any mood or occasion.



## Luna Grey

Echoing the simplicity and calm of Icelandic landscapes, the Luna Grey Navii Protective Cover embodies a light grey color that serves as a balanced backdrop, enhancing other hues and adding a touch of understated elegance. This versatile cover fits effortlessly into any setting, whether it's a formal event or a casual day out. Thanks to its magnetic snap-on system, Luna Grey ensures that every change is quick and convenient, adapting perfectly to your patients' needs.



### Navii with Proprio Foot®

Synchronous knee and ankle function provides additional safety and stability with active dorsiflexion and terrain adaption.

### Navii with Pro-Flex® Terra

Össur's premiere transfemoral solution for K3/ K4 patients living an active lifestyle and looking for maximum dynamics.

## Specifications

### Without protective cover

Built Height	23.6 cm
Knee center to top	3.5 cm
Width	8.0 cm
Weight	1.8kg
Flexion range	-2 to 120°

### With protective cover

Width	8.7 cm
Weight	2.0 kg



[WWW.OSSUR.COM/EN-ZA](http://WWW.OSSUR.COM/EN-ZA)

Össur South Africa  
Unit 1, Stonewood Business Park  
9 Tanzanite Crescent  
Kraaifontein  
Cape Town  
7550

T 0800 996 220  
F 0800 981 744  
E [ossursa@ossur.com](mailto:ossursa@ossur.com)

**ÖSSUR**  
LIFE WITHOUT LIMITATIONS