
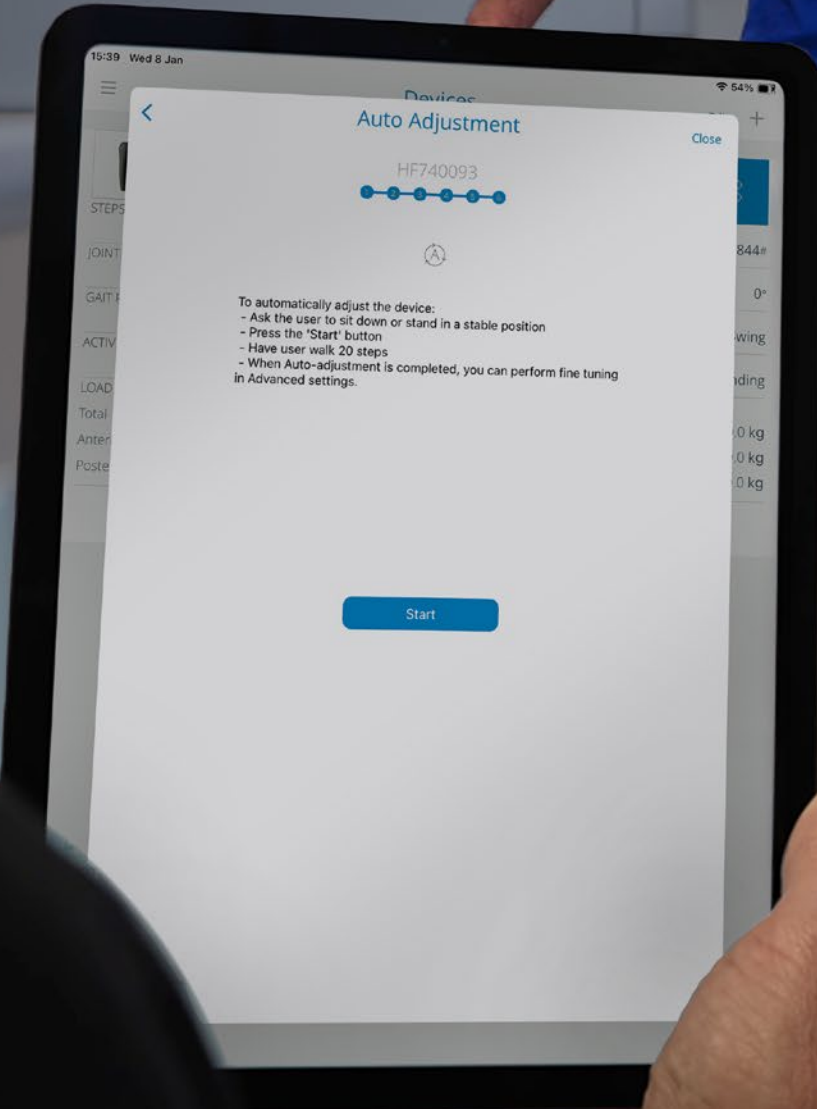


Navii[®]

Connection guide Clinician

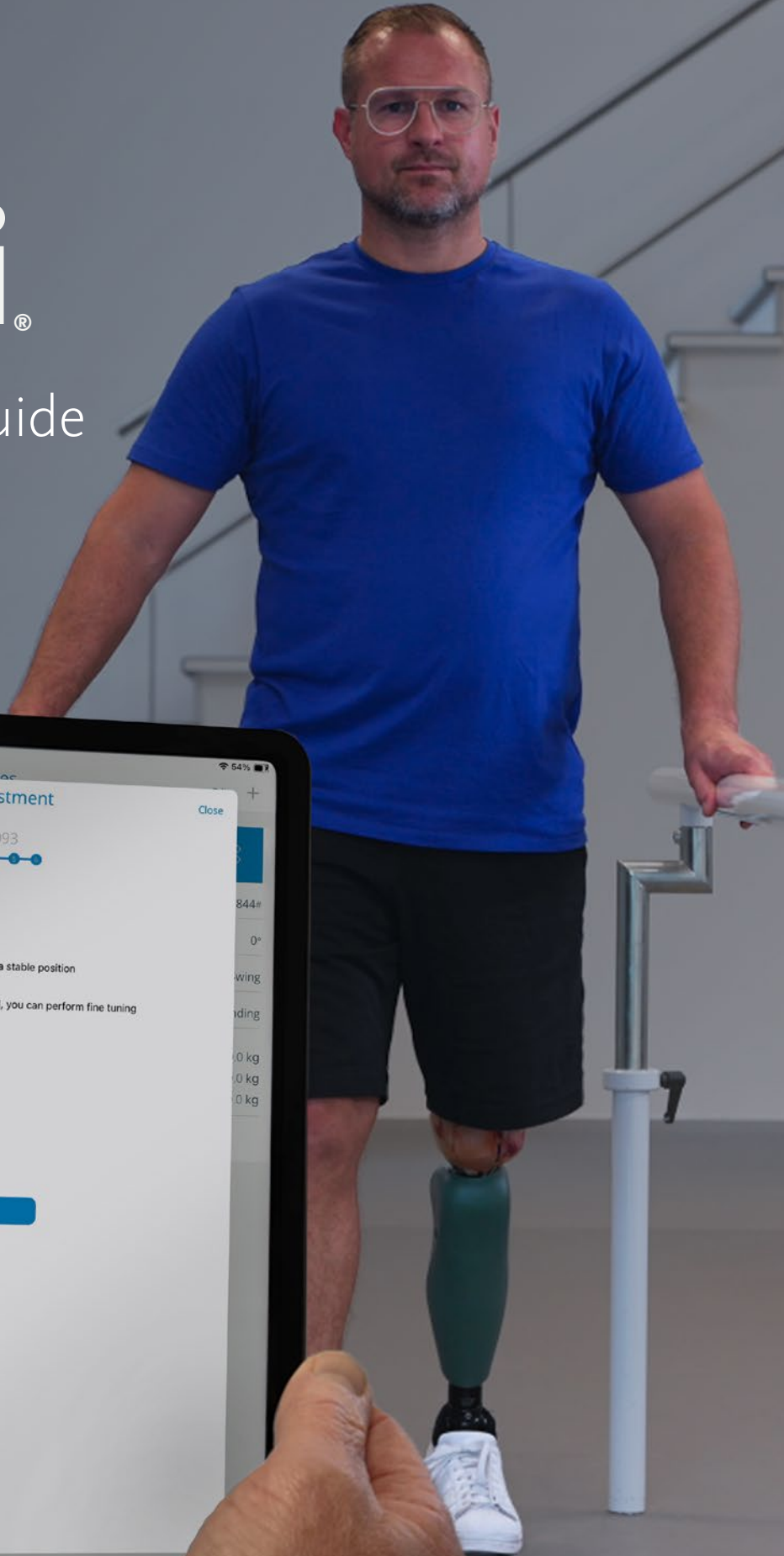



To automatically adjust the device:

- Ask the user to sit down or stand in a stable position
- Press the 'Start' button
- Have user walk 20 steps
- When Auto-adjustment is completed, you can perform fine tuning in Advanced settings.

Start

1 Download the App

Please note you need to be connected to the internet to sign up to Össur Logic.

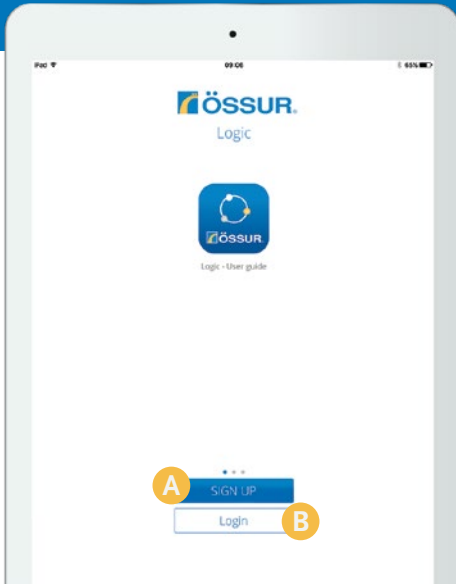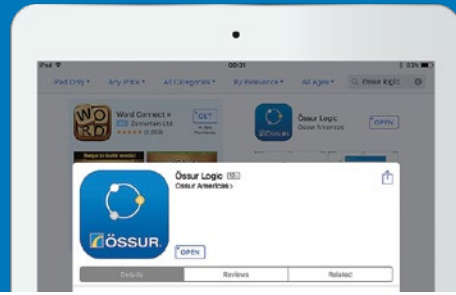

2 A Click 'Sign up'

if you have never used Össur Logic app before.

B Click 'Login'



if you have created an account already. Log in with your email address and password.

3 A Create Login

Make your own login credentials and password.

- For a Navii user (no access to settings), please click Register button and continue to step 5.
- For clinician access to settings, move on to step 4.



Please note after signing up to Össur Logic, you will need to confirm your email address by clicking the link emailed to you.

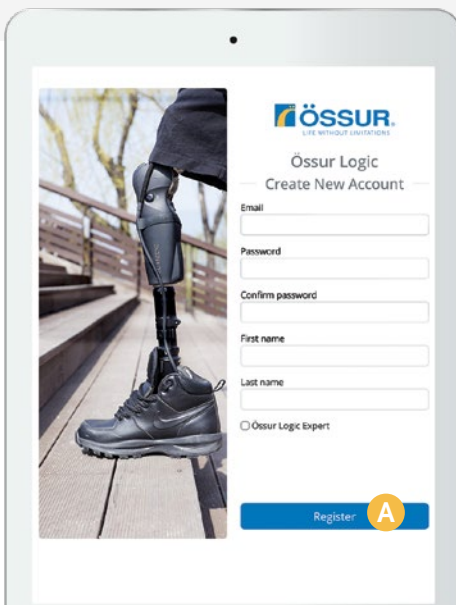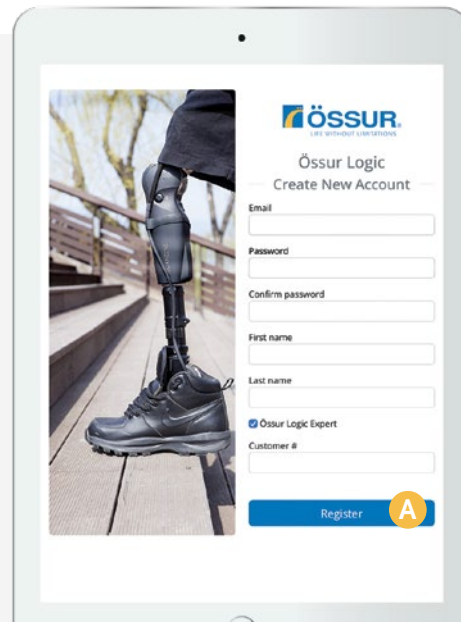

4 A Össur Logic Expert

Once you have entered your details, ensure you tick the Össur Logic Expert box and enter your customer number.

- 5 **A** Click 'Add device' to add a Navii.

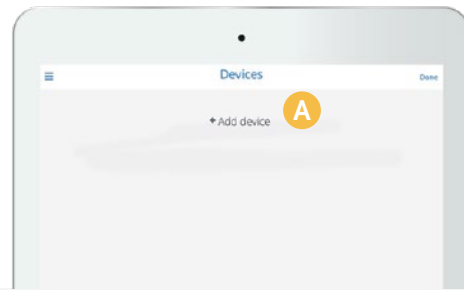
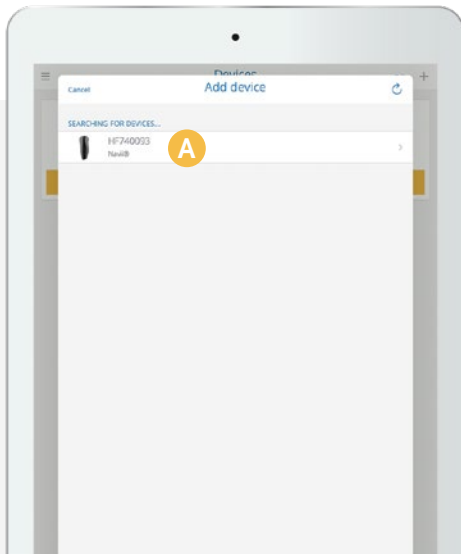


- 6 **A** Connect with Navii

Össur Logic will search for all the knees in range, to connect to it click on the Navii you want to connect to.

- 7 **A** Fill in 'pairing code'

Össur Logic will then ask for the pairing code of the knee, this is the PIN on the back of the knee. Enter the code and click 'Pair'.

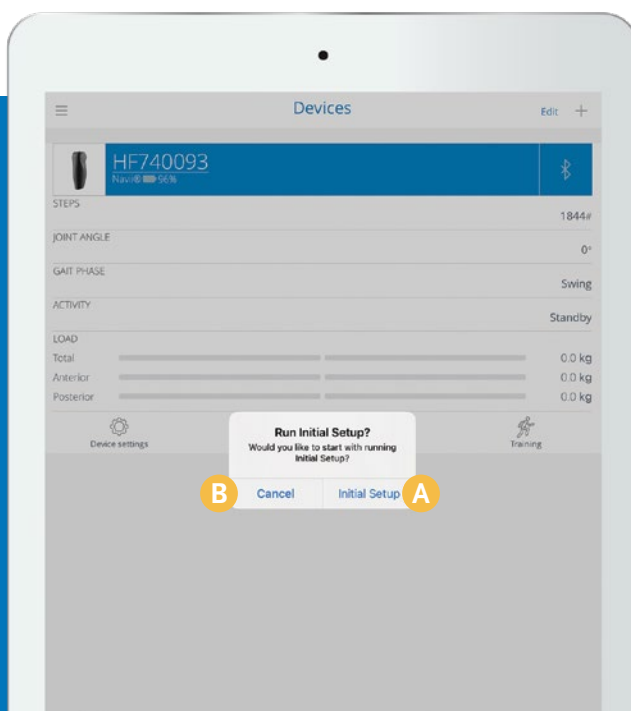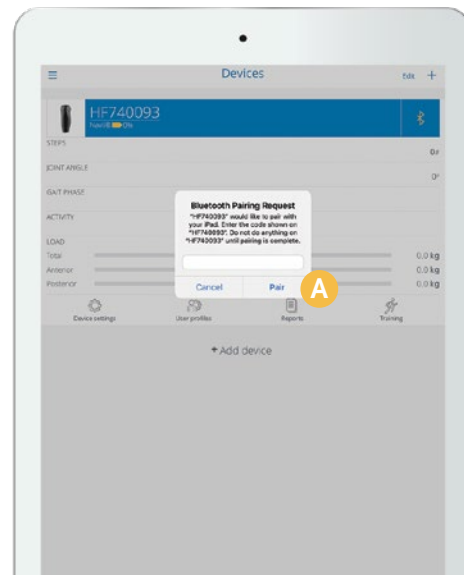


- 8 **Ready for use**

The knee is now connected to Össur Logic. Once connected, it will always ask if you want to 'Run Initial Setup'. You can press Initial Setup **A** or Cancel **B**.


9 Initial Setup

Click 'Continue' on 'Run Initial Setup' and follow the steps for 'calibration', 'bench alignment', 'static alignment', 'dynamic alignment', and 'helpful hints' as instructed.

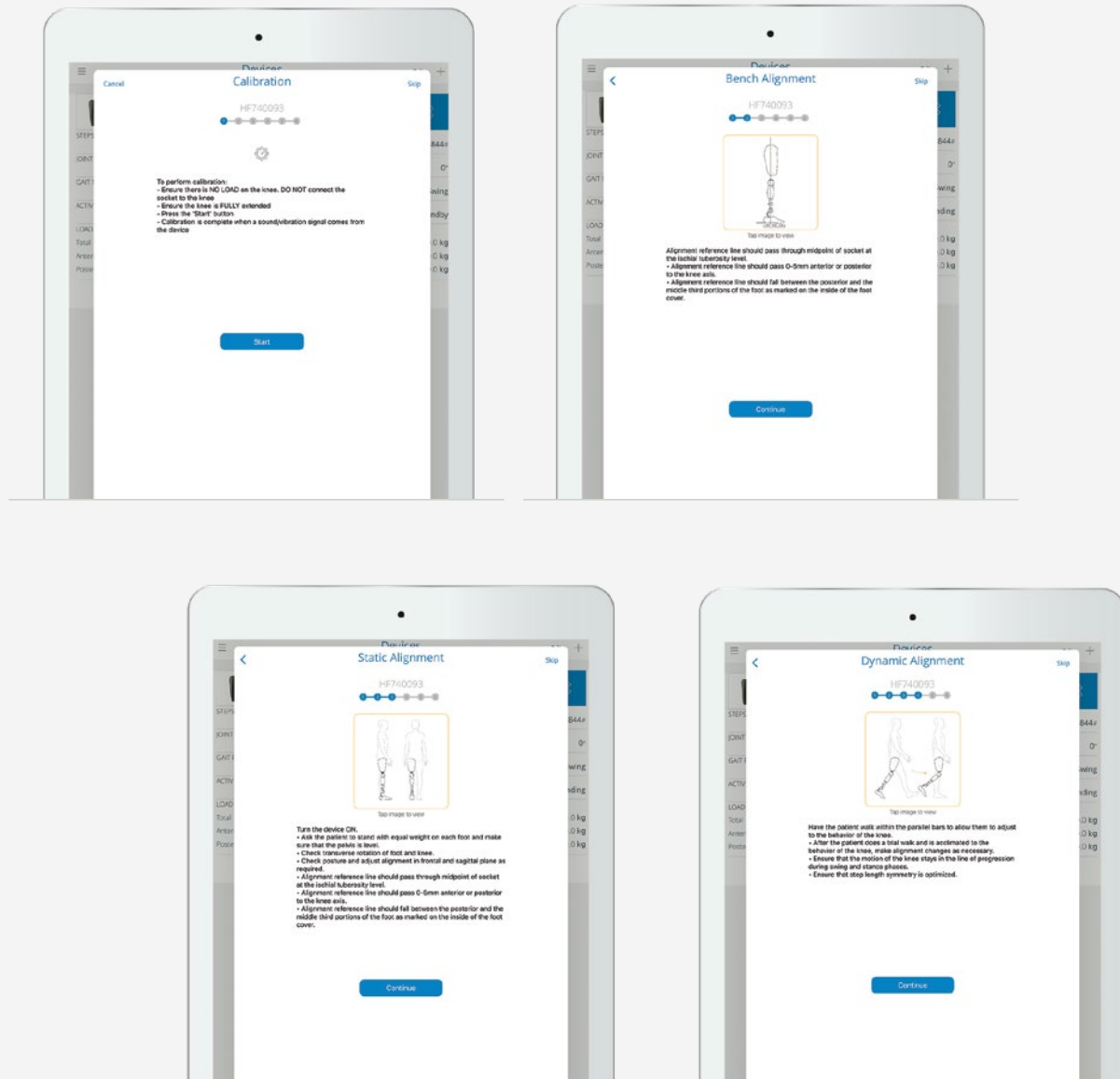


10 A Auto Adjustment

The final step of Initial Setup is Auto Adjustment. Ask the user to sit down, click the 'Start' button and ask the user to walk 20 steps at their usual walking speed. After 20 steps, Auto Adjustment will turn off and the Navii will be set up.

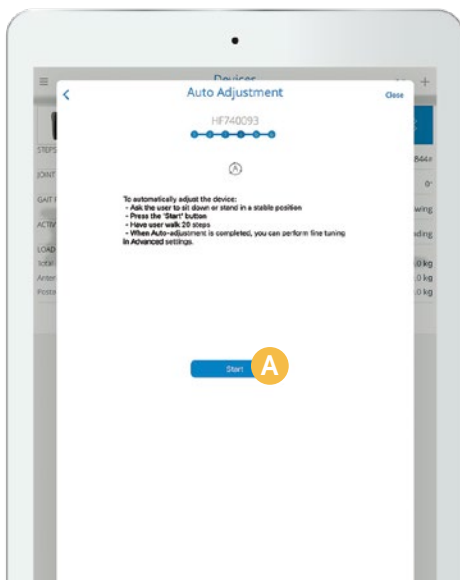

11 Device settings

If you click 'Cancel' on 'Run Initial Setup', or have finished the Initial Setup process, you can access and adjust the Advanced Settings manually by clicking on 'Device settings'.

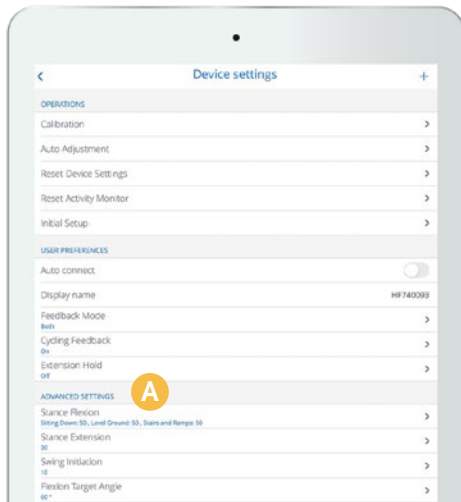

A Manual adjustments

If required, manual adjustments can be made using the Advanced Settings options.

i Watch the Navii Fitting Instructions video


Scan the QR code to see the step by step process.




i Download the Össur Logic app

Scan the QR code to find the app in the App store



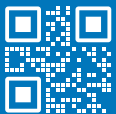

Instagram

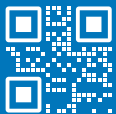

Facebook

USA (800) 233-6263
WWW.OSSUR.COM
CANADA (800) 663-5982
WWW.OSSUR.CA

© ÖSSUR, 02.2025. CRE-20766

