



Season's Greetings from Össur



As we approach the end of another year, we would like to take this opportunity to express our gratitude to you - our valued customers, partners, and supporters. Your contribution to our business and your continued partnership in improving mobility and delivering a Life Without Limitations are truly appreciated.

Thank you once again for your continued support. From the entire Össur team, we wish you all a safe and restful holiday season.

Fitting Tips for the Össur Miami TLSO

Preparation:

- Ensure the brace is clean and free of any obstructions before fitting.
- Gather the necessary tools for any adjustments (e.g. scissors, marker).

Lateral Panel Placement:

- Position the lateral panels in a horizontal orientation, keeping them close to the posterior panel.

- Avoid overlap between the lateral and posterior panels to ensure proper stability and comfort.
- Center the lateral panels on the midline of the belt height-wise. This alignment is crucial for effective support.

Midline Alignment:

- When wearing the brace, ensure that the midline of the lateral panels aligns with the midline of the patient's body. This is typically located on the side of the torso.
- Adjust the positioning as necessary to achieve this alignment while maintaining comfort.

Strapping Technique:

- Fasten the straps snugly but not too tight; there should be no excessive pressure points.
- Start with the abdominal strap, then secure the lateral straps, and finish with the posterior strap for even distribution of pressure.

Comfort Checks:

- After putting on the brace, perform a comfort check. Ask the patient to take a few steps to ensure the brace remains secure and comfortable.
- Adjust as needed based on feedback regarding pressure points or movement restrictions.


Regular Monitoring:

- Periodically check the fit and alignment of the brace, especially after extended wear or any changes in body posture or weight.
- Make any necessary adjustments to maintain optimal support and comfort



[Learn More](#)



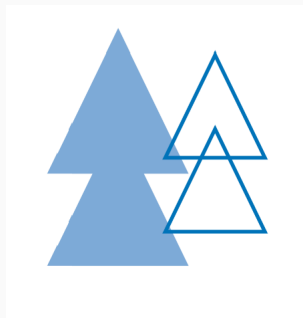

We are pleased to announce we will remain open through the Christmas and New Year season, excluding Public Holidays and Friday, 27th December. In order to receive your order before Christmas, please refer to the pre-Christmas cut-off dates provided below.

PRE-CHRISTMAS DELIVERY CUT-OFF DATES:

- NT, QLD (Far North), WA, TAS: Friday 13 December 2024
- QLD, VIC, NSW, ACT, SA, NZ: Monday 16 December 2024

[View More](#)

Price Changes Effective January 1, 2025

From 1 January 2025, Össur ANZ will implement a price increase across our range of non-contracted products. If you have any questions about this price change or would like a copy of our price list, please feel free to reach out to your local Össur representative or contact Customer Care.

[View Notification](#)

Customer Care

Contact the Customer Care team today!

[Email](#)[Meet the team](#)

Össur ANZ

Sydney: 26 Ross St, North Parramatta, NSW 2151 | 1300 123 268

Melbourne: 9 Market Dr, Bayswater North, VIC 3153 | 1300 123 268

Auckland: 6/80 Westpoint Dr, Hobsonville, Auckland, 0618 | 0800 369 524

www.ossur.com.au | www.ossur.co.nz

© 2025 Össur

[Unsubscribe](#)

Want to change the types of emails you receive from Össur?

Selecting your preferences helps us tailor content to your interests,
ensuring emails are relevant and engaging to you.

[Update your preferences](#)